

Daylight Saving Time (DST) Plan for Hologic Products

Why do I need to update my system?

Beginning March 11, 2007 a new U.S. Government Policy regarding Daylight Saving Time will take effect in the United States and Canada. This policy states that Daylight Saving Time will begin on the second Sunday in March, and end on the first Sunday of November. In accordance with the change, modifications will be required on some Hologic products on or before March 11, 2007. If the required action is not taken, certain products could display or use the incorrect time beginning on March 11, 2007.

What if I don't update my system?

If you choose not to take any action, and you live in the U.S. or Canada, your system clock, and calendar will no longer automatically adjust for Daylight Saving Time at the proper time each year. This means that the system time will be adjusted 3 weeks late in the spring, and 1 week early in the fall.

When should I manually change the time?

Under the old rules for DST, the time would have changed on April 1st. So if the system is not updated, then the time has to be changed manually. In the spring the time needs to be set forward one hour after March 11 and then set back one hour after April 1st. In the fall the time needs to be forward one hour after October 28th and then set back one hour after November 4th.

Hologic, Inc.
36 Apple Ridge Road, Danbury, CT 06810 USA
Main: 203.207.4500 Fax: 203.207.4596

How do I update my Hologic Product?

Hologic has created the following DST Plan for each Hologic product to assist our customers:

Product: QDR XP

Hologic recommends the IT administrator download and install the appropriate Microsoft patch <http://support.microsoft.com/kb/931836>. This patch will be applied to all new products starting March 1, 2007.

Product: InSight

Hologic recommends the IT administrator download and install the appropriate Microsoft patch <http://support.microsoft.com/kb/931836>. This patch will be applied to all new products starting March 1, 2007.

Product: Selenia

Hologic intends to remotely upgrade all customers with a Field Modification Instruction (FMI) kit. Tech Support will contact customers with VPN connections to schedule the installation of the patch. Field Service Engineers will be dispatched to all non-VPN sites. This patch will be applied to all new products starting March 1 and built into Selenia v3.3.

Hologic, Inc.
36 Apple Ridge Road, Danbury, CT 06810 USA
Main: 203.207.4500 Fax: 203.207.4596

Product: SecurView_{DX}

Hologic recommends for v3-x that the IT administrator download and install the appropriate Microsoft patch <http://support.microsoft.com/kb/931836>. This patch will be applied to all new products starting March 1, 2007.

Hologic recommends for v1-x and v2-x that the IT administrator download and install the appropriate Microsoft tool TZEDIT <http://support.microsoft.com/kb/914387>. This patch will be applied to all new products starting March 1.

Product: SecurView_{RT}

Hologic recommends for v2-0-10 that the IT administrator download and install the appropriate Microsoft patch <http://support.microsoft.com/kb/931836>. This patch will be applied to all new products starting March 1, 2007.

Hologic recommends for v2-0-4 that the IT administrator download and install the appropriate Microsoft tool TZEDIT <http://support.microsoft.com/kb/914387>. This patch will be applied to all new products starting March 1.

Product: MIMS-*plus*

Hologic recommends the IT administrator download and install the appropriate Microsoft patch <http://support.microsoft.com/kb/931836>. This patch will be applied to all new products starting March 1, 2007.

Hologic, Inc.

36 Apple Ridge Road, Danbury, CT 06810 USA
Main: 203.207.4500 Fax: 203.207.4596

Product: MIMS

Hologic recommends for MIMS with MIMS Impax 4.5 and Windows 2003 Server that the IT administrator download and install the appropriate Microsoft patch <http://support.microsoft.com/kb/931836>. This patch will be applied to all new products starting March 1, 2007.

Hologic recommends for MIMS with MIMS Impax 4.1 and Windows 2000 that the IT administrator download and install the appropriate Microsoft tool TZEDIT <http://support.microsoft.com/kb/914387>. This patch will be applied to all new products starting March 1.

Product: ImageChecker D, DM, Dmax, LS

Hologic intends to provide a patch at the request of customers. An RPM will be available on our FTP site or the site IT administrator can download it from the Internet directly. If the customer is under warranty or contract and could not do the install themselves, we will send out an FE to install the patch for them. The patch is only applicable to ImageChecker version 8.0 or greater.

Product: DigitalNow

Hologic intends to provide a patch at the request of customers. An RPM will be available on our FTP site or the site IT administrator can download it from the Internet directly. If the customer is under warranty or contract and can not do the install themselves, we will send out an FE to install the patch for them. The patch is only applicable to DigitalNow version 8.1 or greater.

Product: CheckMate Ultra

Hologic intends to provide a patch at the request of customers. The site IT administrator can download the appropriate version of the patch from http://support.microsoft.com/gp/cp_dst. If the customer is under warranty or contract and could not do the install themselves, we will send out an FE to install the patch for them.

Product: M-IV Series

Daylight Savings Time is an optional configurable feature on the M-IV. Refer to Chapter 3 “Additional Setup Defaults” of the M-IV Operator’s Manual to enable or disable this optional feature or to make manual adjustments to the system time.

The automatic adjustment will not be corrected until the release of M-IV Console HOST v 6.4.3 which has a projected release of mid March 2007. It will be necessary to implement the manual adjustment procedure until such time that the software release becomes available.

Product: MultiCare Platinum, StereoLoc II, DSM

Hologic recommends the IT administrator download and install the appropriate Microsoft tool TZEDIT <http://support.microsoft.com/kb/914387>.

Product: Affinity Series

There is no software provision to automatically adjust the system time for DST. The date and time need to be set manually using User Defaults, refer to Chapter 3 of Affinity Operator or Service Manual.

Product: MD-3000 (Philips)

There is no software provision to automatically adjust End of Life Mammography Products for DST. The date and time need to be set manually.

Hologic, Inc.

36 Apple Ridge Road, Danbury, CT 06810 USA

Main: 203.207.4500 Fax: 203.207.4596

End of Life Products

There is no software provision to automatically adjust End of Life Products for DST. The date and time need to be set manually.

Clarity of Vision