

REV AUTHORED BY G. PETRUCCI	DATE 4/25/2018	HOLOGIC[®]	SIGNATURES ON FILE	
REV DRAFTED BY G. PETRUCCI	DATE 4/25/2018			
PROPRIETARY: This document contains proprietary data of Hologic, Inc. No disclosure, reproduction or use of any part thereof may be made except by written permission from Hologic.		TITLE PI EXUS, APTIMA URINE SPECIMEN KIT, IVD, ENGLISH	DOCUMENT NUMBER 502136EN-IFU-PI	REV 005
REV. RELEASE DATE: 06/29/18				

Aptima Urine Specimen Collection Kit for Male and Female Urine Specimens

For *in vitro* diagnostic use.

For US export only.

Intended Use

The Aptima Urine Specimen Collection Kit for Male and Female Urine Specimens is for use with Aptima assays. The Aptima Urine Specimen Collection Kit is intended to be used for the collection and transport of male or female urine specimens.

Materials Provided

50 Aptima Urine Specimen Collection Kits for Male and Female Urine Specimens (Cat. No. 301040)

Each kit contains:

Component	Quantity	Description
Pipette	1	Disposable transfer pipette.
Specimen transport tube	1	Tube containing 2.0 mL Aptima urine transport medium.

Warnings and Precautions

- A. Do not apply the transport medium directly to skin or mucous membranes or take internally.

Kit Storage Requirements

Store collection kit at room temperature (15°C to 30°C).

Urine Specimen Performance

The assay performance characteristics of the male and female urine specimens are provided in the appropriate Aptima assay package insert. The Aptima assay package inserts may be referenced online at www.hologic.com. The performance of the male urine specimen has not been established for all Aptima assays.

Specimen Collection and Handling

Note: If the tube contents are spilled, discard and replace with a new Aptima urine specimen transport tube.

1. The patient should not have urinated for at least 1 hour prior to specimen collection.
2. Direct patient to provide a first-catch urine (approximately 20 to 30 mL of the initial urine stream) into a urine collection cup free of any preservatives. Collection of larger volumes of urine may result in rRNA target dilution that may reduce test sensitivity. Female patients should not cleanse the labial area prior to providing the specimen.
3. Remove the cap and transfer 2 mL of urine into the urine specimen transport tube using the disposable pipette provided. The correct volume of urine has been added when the fluid level is between the black fill lines on the urine specimen transport tube label.
4. Re-cap the urine specimen transport tube tightly. This is now known as the processed urine specimen.

Specimen Transport and Storage

After collection, transport the processed urine specimens in the Aptima urine specimen transport tube at 2°C to 30°C and store at 2°C to 30°C until tested. Processed urine specimens should be assayed with the Aptima assay within 30 days of collection. If longer storage is needed, refer to the appropriate Aptima assay package insert.

Urine samples that are still in the primary collection container must be transported to the lab at 2°C to 30°C. Transfer the urine sample into the Aptima urine specimen transport tube within 24 hours of collection. Store at 2°C to 30°C and test within 30 days of collection.

Note: Specimens must be shipped in accordance with applicable national and international transportation regulations.

Limitations

- A. Use this collection kit only with the Aptima assays. Performance has not been established with other products.
- B. The performance of male urine specimens has not been established for the Aptima Trichomonas vaginalis assay.



Hologic, Inc.
10210 Genetic Center Drive
San Diego, CA 92121 USA

Customer Support: +1 800 442 9892
customersupport@hologic.com
Technical Support: +1 888 484 4747
molecularsupport@hologic.com

For more contact information visit www.hologic.com.



Hologic N.V.
Da Vincilaan 5
1930 Zaventem
Belgium

Hologic, Aptima, and Aptima Combo 2 are trademarks and/or registered trademarks of Hologic, Inc. and/or its subsidiaries in the United States and/or other countries.

This product may be covered by one or more U.S. patents identified at www.hologic.com/patents.

© 2001-2018 Hologic, Inc. All rights reserved.
502136EN Rev. 005
2018-03