



## CULTURE IDENTIFICATION REAGENT KIT

### Intended use

The reagents in the *AccuProbe Culture Identification Reagent Kit* are to be used with the *AccuProbe Culture Identification Reagent Tests* according to the instructions outlined in the respective Package Inserts. Reagents are provided for 200 bacterial or 100 mycobacterial or fungal determinations per kit.

### Reagents and materials provided

**Note:** For information on any hazard and precautionary statements that may be associated with reagents, refer to the Safety Data Sheet Library at [www.hologic.com/sds](http://www.hologic.com/sds).

Cat. No. 102800	200 Bacterial or 100 Mycobacterial/Fungal Tests
<b>Reagent 1</b> (Lysis Reagent). Buffered solution with 0.04% sodium azide.	1 x 10 mL
<b>Reagent 2</b> (Hybridization Buffer). Buffered solution.	1 x 10 mL
<b>Reagent 3</b> (Selection Reagent). Buffered solution.	1 x 60 mL

### Warnings and precautions

Reagents in this kit contain sodium azide, which may react with lead or copper plumbing to form potentially explosive metal azides. Upon disposal of the reagents, always dilute the material with a large volume of water to prevent azide build-up in the plumbing.

### Storage and handling requirements

The reagents contained in the *AccuProbe Culture Identification Reagent Kit* are to be stored at 2° to 25°C and are stable until the expiration date indicated.

### DO NOT FREEZE THESE REAGENTS.

**Note:** Reagent 2 (Hybridization Buffer) must be at room temperature before use.

**Hologic, Inc.**  
10210 Genetic Center Drive  
San Diego, CA 92121 USA

Customer Support: +1 844 Hologic (+1 844 465 6442)  
customersupport@hologic.com

Technical Support: +1 888 484 4747  
molecularsupport@hologic.com

For more contact information visit [www.hologic.com](http://www.hologic.com).

©1992–2016 Hologic, Inc. All rights reserved.

102848 Rev. 001  
2016-03