

## Aptima® Urine Specimen Collection Kit for Male and Female Urine Specimens

For *in vitro* diagnostic use  
Rx only

### Intended Use

The Aptima® Urine Specimen Collection Kit for Male and Female Urine Specimens is for use with Aptima assays. The Aptima Urine Specimen Collection Kit is intended to be used for the collection and transport of male or female urine specimens.

### Materials Provided

**50 Aptima Urine Specimen Collection Kits for Male and Female Urine Specimens (Cat. No. 301040)**

Each kit contains:

Component	Quantity	Description
Pipette	1	<i>Disposable transfer pipette.</i>
Specimen transport tube	1	<i>Tube containing 2.0 mL Aptima® Urine Transport Medium.</i>

### Warnings and Precautions

- A. Do not apply the transport medium directly to skin or mucous membranes or take internally.

**Note:** For information on any hazard and precautionary statements that may be associated with reagents, refer to the SDS on the Safety Data Sheet Library at [www.hologicds.com](http://www.hologicds.com). For more information on the symbols, refer to the symbol legend on [www.hologic.com/package-inserts](http://www.hologic.com/package-inserts).

### Kit Storage Requirements

Store collection kit at room temperature (15°C to 30°C).

### Urine Specimen Performance

The assay performance characteristics of the male and female urine specimens are provided in the appropriate Aptima assay package insert. The Aptima assay package inserts may be referenced online at [www.hologic.com](http://www.hologic.com). The performance of the male urine specimen has not been established for all Aptima assays.

### Specimen Collection and Handling

1. The patient should not have urinated for at least 1 hour prior to specimen collection.
2. Direct patient to provide a first-catch urine (approximately 20 to 30 mL of the initial urine stream) into a urine collection cup free of any preservatives. Collection of larger volumes of urine may result in target dilution that may reduce test sensitivity. Female patients should not cleanse the labial area prior to providing the specimen.
3. Remove the cap and transfer 2 mL of urine into the urine specimen transport tube using the disposable pipette provided. The correct volume of urine has been added when the fluid level is between the black fill lines on the urine specimen transport tube label.
4. Re-cap the urine specimen transport tube tightly. This is now known as the processed urine specimen.

### Specimen Transport and Storage

After collection, transport the processed urine specimens in the Aptima urine specimen transport tube at 2°C to 30°C and store at 2°C to 30°C until tested. Processed urine specimens should be assayed with the Aptima assay within 30 days of collection. If longer storage is needed, refer to the appropriate Aptima assay package insert.

Urine samples that are still in the primary collection container must be transported to the lab at 2°C to 30°C. Transfer the urine sample into the Aptima urine specimen transport tube within 24 hours of collection. Store at 2°C to 30°C and test within 30 days of collection.

**Note:** Specimens must be shipped in accordance with applicable national, international, and regional transportation regulations.

## Limitations

- A. Use this collection kit only with the Aptima assays. Performance has not been established with other products.



Hologic, Inc.  
10210 Genetic Center Drive  
San Diego, CA 92121 USA

Customer Support: +1 800 442 9892  
customersupport@hologic.com

Technical Support: +1 888 484 4747  
molecularsupport@hologic.com

For more contact information visit [www.hologic.com](http://www.hologic.com).

Hologic and Aptima are trademarks and/or registered trademarks of Hologic, Inc. and/or its subsidiaries in the United States and/or other countries.

© 2001–2024 Hologic, Inc. All rights reserved.  
502260 Rev. 003  
2024-06