

## Customer Technical Bulletin

**CTB-00779**

**Date:** *September 10, 2020*

**Author:** Service Engineering

**Product:** Selenia Dimensions/  
3Dimensions

**Subsystem:** AWS

**Subject:** Procedure to Add Comments to Tomo/TomoHD/Combo/ComboHD  
Images on Hologic Selenia Dimensions/3Dimensions Systems  
Operating at Version 1.10/2.1 Software with 3DQuorum  
Licensed/Enabled

---

### Purpose

To provide information on adding comments to Tomo/TomoHD/Combo/ComboHD images on Hologic's Selenia Dimensions/3Dimensions Systems Operating at Version 1.10/2.1 Software where 3DQuorum has been licensed/enabled.

### Scope

This information applies to Selenia Dimensions, 3Dimensions operating at Version 1.10/2.1 Software when acquiring Tomo/TomoHD/Combo/ComboHD views with 3DQuorum licensed/enabled.

**Note: 3DQuorum requires ClarityHD and the system configured for high resolution Tomosynthesis.**

# Customer Technical Bulletin (cont.)

## Discussion

Due to a software bug in Version 1.10/2.1 software, the procedure to add comments to the images acquired on a Selenia Dimensions/3Dimensions systems needs to be modified for it properly work.



Please follow the procedure in this Customer Technical Bulletin when adding image comments for Selenia Dimensions/3Dimensions systems operating at Version 1.10/2.1 software.

This software bug will be corrected in the next release of Selenia Dimensions/3Dimensions software.

# Customer Technical Bulletin (cont.)


## Procedure to Add Comments to Images on Selenia Dimensions/3Dimensions Operating at Version 1.10/2.1 Software.

1. Acquire the image as normal. Before accepting the image, add the desired image comment and click OK. (Example shown below)

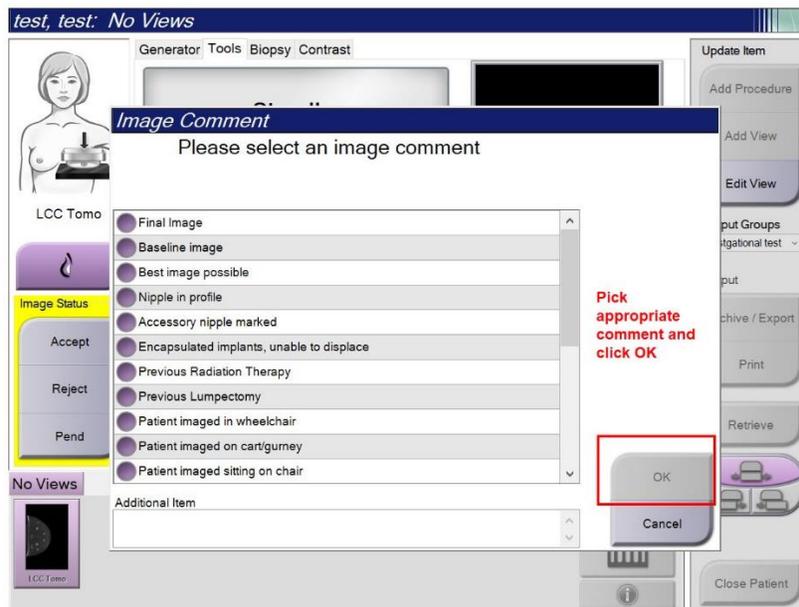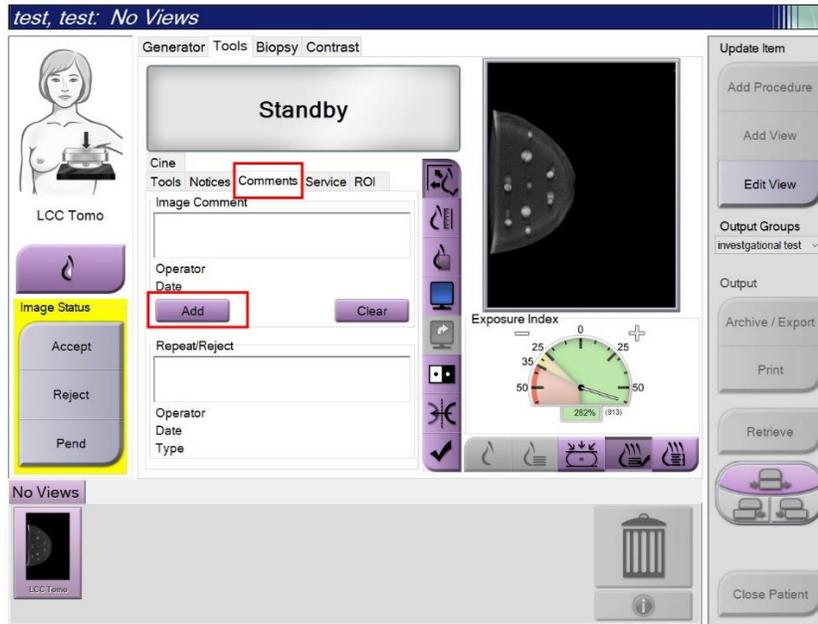


# Customer Technical Bulletin (cont.)

2. Before accepting the image and after adding the image comment, click Edit View.

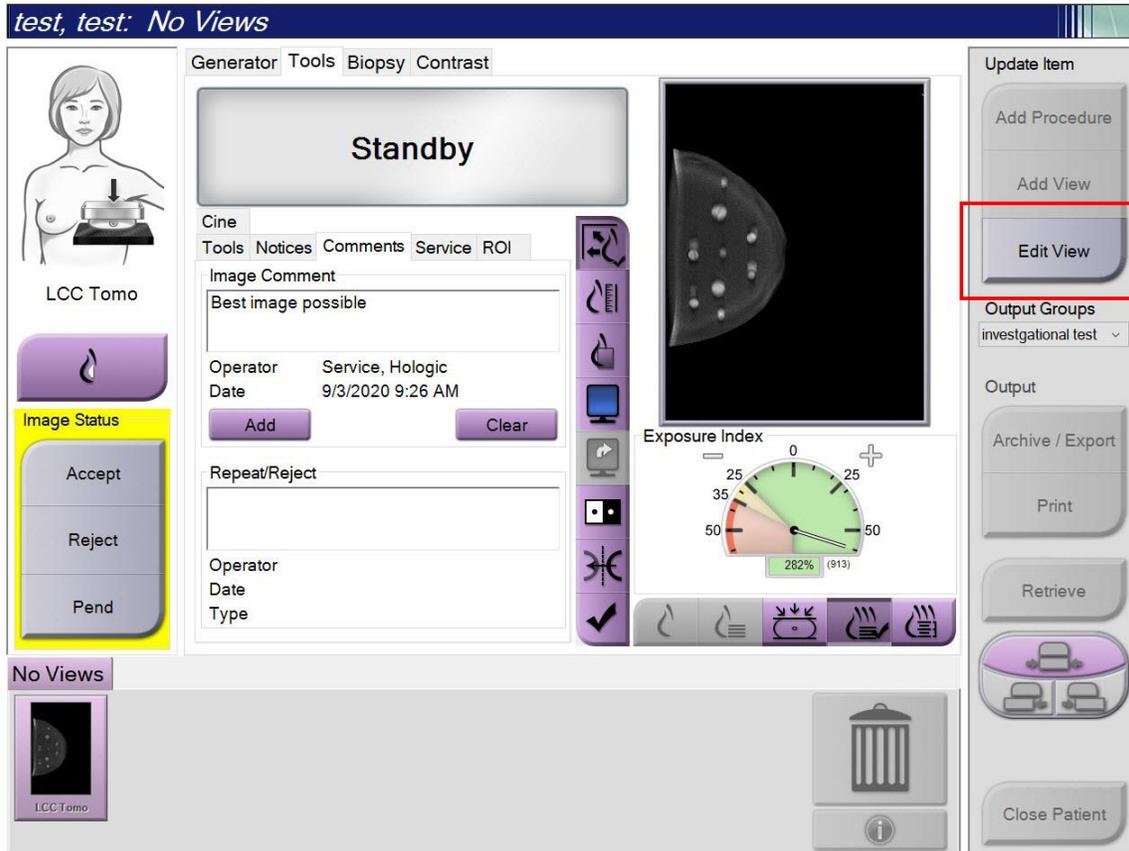


# Customer Technical Bulletin (cont.)

3. On Edit View page, click the Save button. This will regenerate the Tomosynthesis, Intelligent2D, and 3DQuorum images.

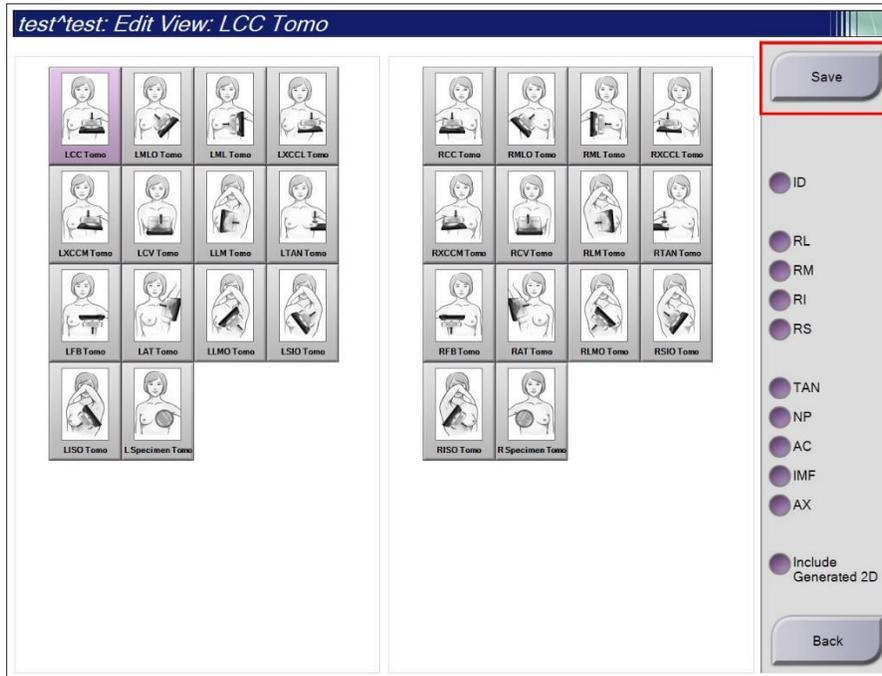


4. When the "Update Successful!" message appears, click OK.

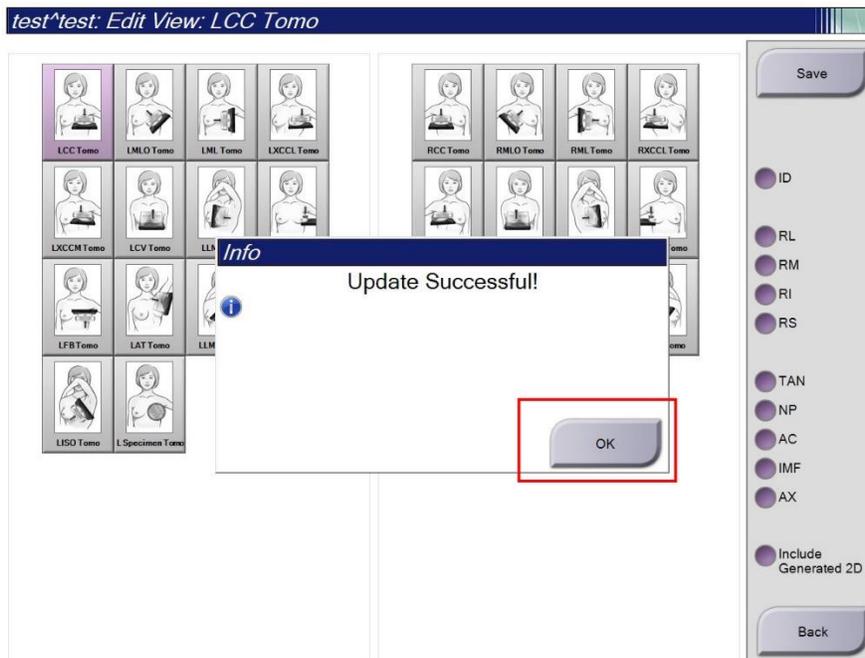

# Customer Technical Bulletin (cont.)

5. Accept the Image.

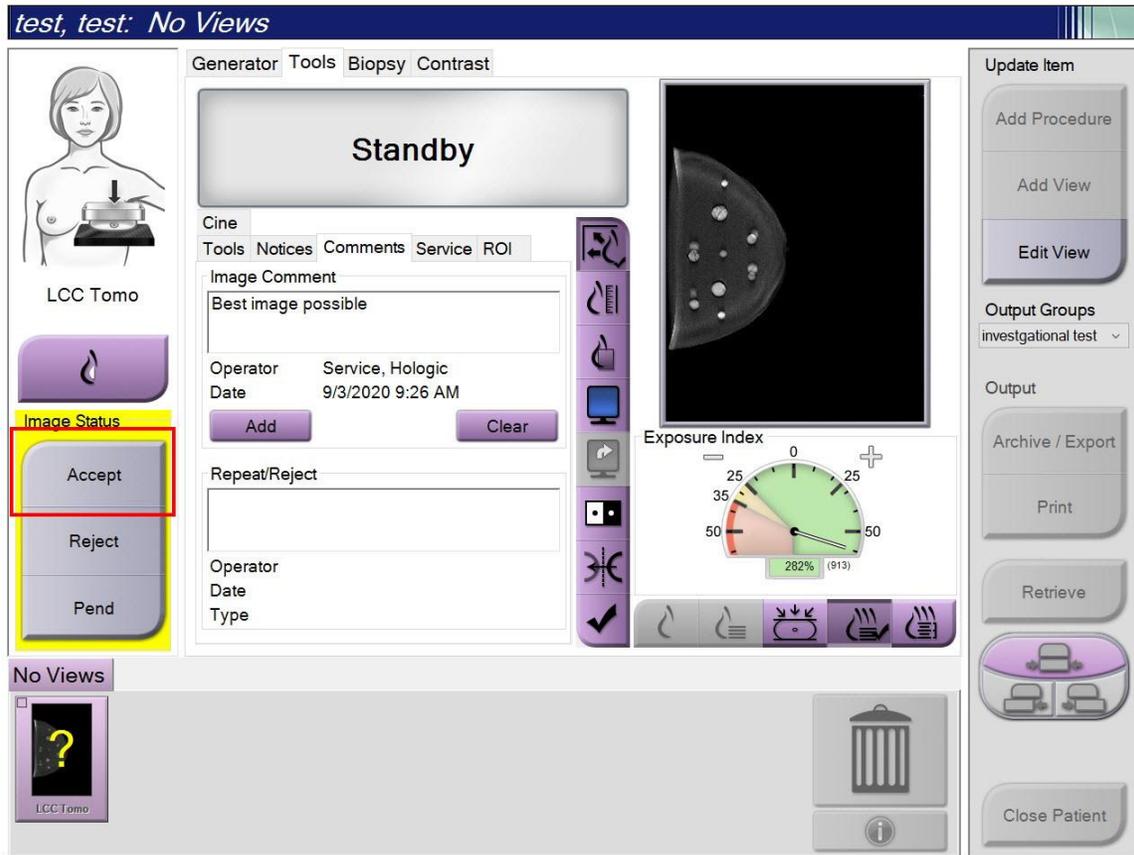

6. Repeat this procedure for every image where a comment is to be added.