
# SecurMark® Biopsy Site Marker for Eviva®

## Patient Information

Instruction Leaflet and Implant Card to be supplied to patients implanted with SecurMark Titanium Biopsy Site Marker(s).

1. Name of the patient or patient ID. To be filled by the healthcare institution or provider.
2. Date of implantation. To be filled by the healthcare institution or provider.
3. Name and address of the healthcare institution or provider. To be filled by the healthcare institution or provider.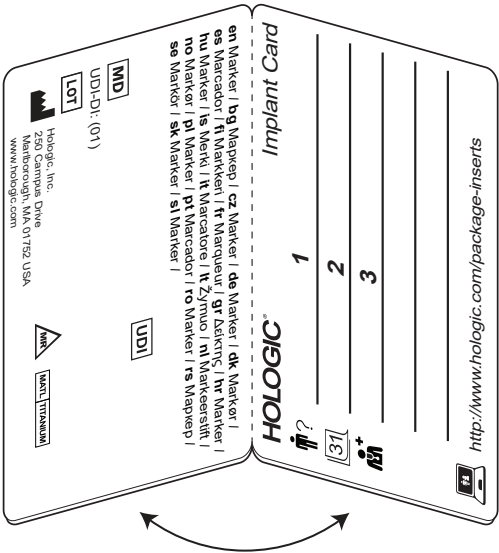

| Symbols                                                                           |                                                                       |
|-----------------------------------------------------------------------------------|-----------------------------------------------------------------------|
|  | Patient Name or Patient ID                                            |
|  | Date of Implantation                                                  |
|  | Name and Address of the implanting healthcare institution or provider |
|  | Information website for patients                                      |
|  | Medical Device                                                        |
|  | Unique Device Identifier                                              |
|  | LOT Number                                                            |
|  | Manufacturer                                                          |
|  | MR Conditional                                                        |
|   | Material Titanium Class II                                            |
|   | Material Glycoprene II                                                |

## What should you do with the implant card?

Keep your patient implant card with you. Your patient implant card has important information about your SecurMark Biopsy Site Marker(s)<sup>1</sup>. Be sure to show your implant card to any doctors that you see.

<sup>1</sup> SecurMark Biopsy Site Marker(s) will be referred to as markers throughout the entire document.

## Warnings and Precautions:

- A complete list of warnings and precautions can be found in the SecurMark Biopsy Site Marker(s) 'Instructions for Use', which is located at <http://www.hologic.com/package-inserts>.

- Contact your doctor if you think you are having side-effects related to the marker(s) or its use, or if you are concerned about risks. This document is not meant to replace a consultation with your doctor if needed.
- The marker(s) will be left in your breast after biopsy. Your tissue will grow around the marker and will stay in your body unless your doctor needs to remove the tissue around the marker(s). Your doctor may take the marker(s) out at that time.
- **MRI SAFETY INFORMATION**  
The marker(s) is MR Conditional. A patient with this device can be safely scanned in an MR system under specific conditions. Please consult your doctor for further information.

## **Possible Risks and Undesirable Effects:**

Your doctor should tell you about the benefits and possible risks of the marker(s) and the procedure. Any medical procedure has risks and unwanted effects. These are typically found by reviewing the experience of patients who have had the procedure done. Doctors and patients may also give feedback about the device. Refer to the list below for possible risks or known complications. Many factors, including your health condition and medical history, can affect the outcomes of your surgery and the complications you experience. Some complications (such as infection) may lead to additional treatment. Please contact your doctor to discuss the risks further.

### **Possible Risks & Undesirable Effects:**

- |                    |                                          |
|--------------------|------------------------------------------|
| • Pain             | • Hypersensitivity or allergic reaction  |
| • Seroma formation | • Soft tissue damage                     |
| • Inflammation     | • Misdiagnosis (due to marker migration) |
| • Bruising         | • Perforation or scar tissue             |
| • Hematoma         | • Needle stick/puncture                  |
| • Hemorrhage       | • Sepsis                                 |
| • Infections       |                                          |

## Implantable Material Information:

| Marker                  | Material/Composition                                                                                                                                                                                       |
|-------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Permanent Component     | Titanium Grade II per ASTM F67-13:<br>Nitrogen, max ..... 0.03<br>Carbon, max ..... 0.08<br>Hydrogen, max ..... 0.015<br>Iron, max ..... 0.30<br>Oxygen, max ..... 0.25<br>Titanium ..... balance (99.32%) |
| Bioabsorbable component | Glycoprene II (100%)                                                                                                                                                                                       |

In compliance with the Australian Therapeutic Goods (Medical Devices) Regulation 2002 Clause 13A.3, the following information can be found in the SecurMark Biopsy Site Marker(s) 'Instructions for Use', which is located at <http://www.hologic.com/package-inserts>.

- 2 a. The intended purpose of the device; and
- 2 b. The kind of patient on whom the device is intended to be used.
3. Any special operating instructions for the use of the device.
- 4 a. The intended performance of the device.

All other requirements of MD 2002 Clause 13A.3 are included in this Patient Information Leaflet.

Any serious incident that occurs in relation to the device should be reported to Hologic, Inc. and to the Therapeutic Goods Administration (<https://www.tga.gov.au>).

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The diagram shows a folded instruction booklet and an implant card. The instruction booklet on the left contains the following text:

Hologic, Inc.  
250 Campus Drive  
Marborough, MA 01752 USA  
www.hologic.com

MD  
UDI-DI: (01)  
LOT

AMS  
MATEL TITANIUM

UDI

The implant card on the right is titled "Implant Card" and features the "HOLOGIC®" logo. It includes a section for patient information with a question mark icon and a section for date with a calendar icon. The date section shows "31" in a box. Below these are three numbered lines for patient name, date, and provider name. The URL <http://www.hologic.com/package-inserts> is printed at the bottom right.

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