


SecurView®

Breast Imaging Workstation

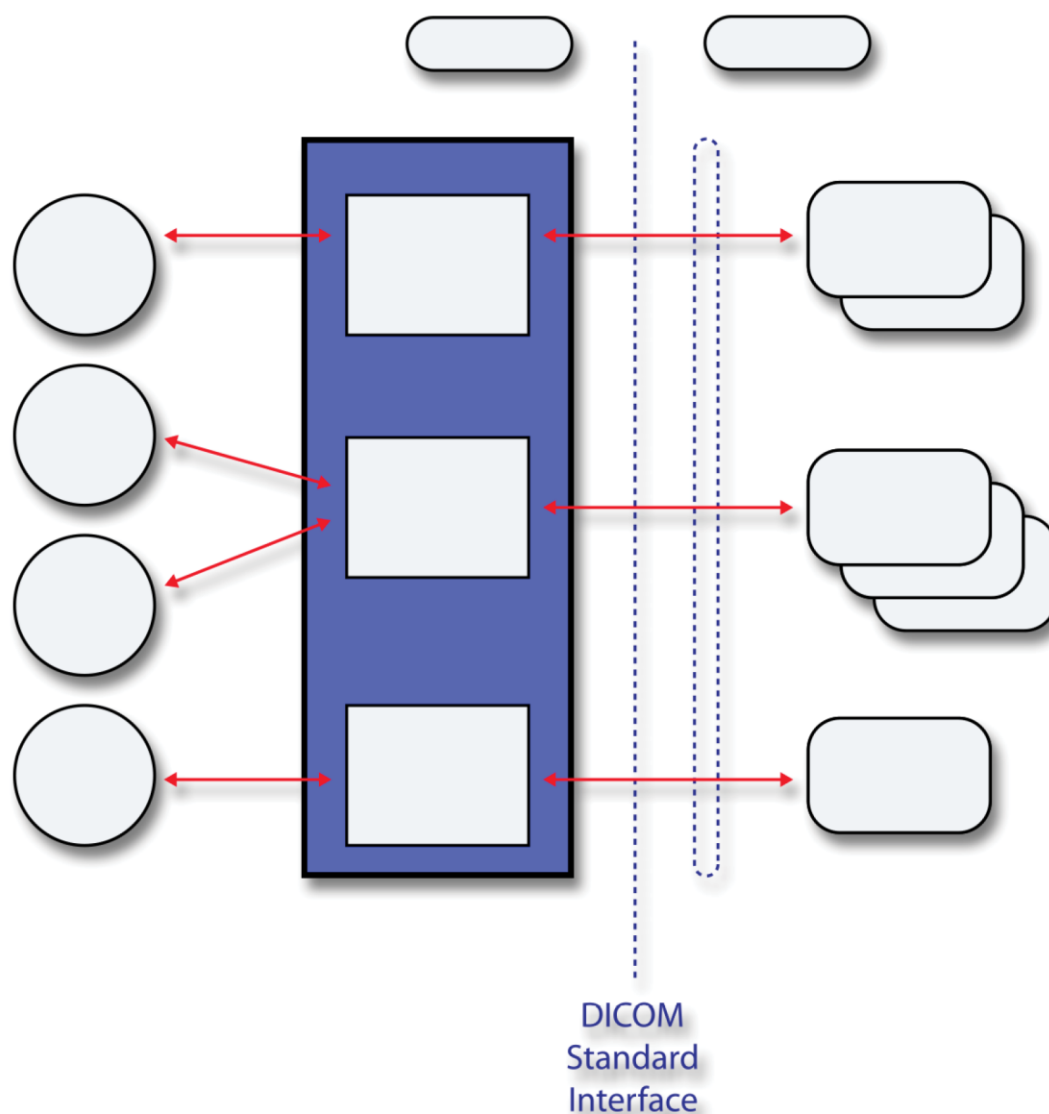

DICOM Conformance Statement

RT Diagnostic Workstation

RD-04686 Revision 002

HOLOGIC®

SecurView® RT

Technologist Workstation

DICOM Conformance Statement

For Software Version 12.0

Part Number RD-04686

Revision 002

May 2025

Product Support

| | | | |
|------------|-----------------|------------|----------------------------------|
| USA: | +1.877.371.4372 | Asia: | +852 37487700 |
| Europe: | +32 2 711 4690 | Australia: | +1 800 264 073 |
| All Other: | +1 781 999 7750 | Email: | BreastHealth.Support@hologic.com |

© 2025 Hologic, Inc. Printed in the USA. This manual was originally written in English.

Hologic, Cenova, Dimensions, Lorad, R2 Technology, SecurView, Selenia, and associated logos are trademarks and/or registered trademarks of Hologic, Inc., and/or its subsidiaries in the United States and/or other countries. All other trademarks, registered trademarks, and product names are the property of their respective owners.

This product may be protected by one or more U.S. or foreign patents as identified at www.Hologic.com/patent-information.



Hologic Inc.
600 Technology Drive
Newark, DE 19702 USA
1.800.447.1856
www.hologic.com

HOLOGIC®

1. Overview

The Hologic SecurView RT workstation is used primarily by radiologic technologists to view digital mammography images. The SecurView RT workstation implements the necessary DICOM services to receive images and related information from networked devices, query for images and related information from a networked storage device, retrieve images and related information from a networked storage device, send and receive Notices with SecurView DX workstations, and print to a networked hardcopy device.

Table 1-1 provides an overview of the network services supported by the SecurView RT workstation.

Table 1-1 – Network Services

| SOP Classes | User of Service (SCU) | Provider of Service (SCP) |
|---|-----------------------|---------------------------|
| Transfer | | |
| Computed Radiography Image Storage | No | Yes |
| Digital X-Ray Image Storage – For Presentation | No | Yes |
| Digital Mammography X-Ray Image Storage – For Presentation | No | Yes |
| Digital Mammography X-Ray Image Storage – For Processing | No | Yes |
| Breast Tomosynthesis Image Storage | No | Yes |
| Breast Projection X-Ray Image Storage – For Presentation | No | Yes |
| Breast Projection X-Ray Image Storage – For Processing | No | Option |
| Secondary Capture Image Storage | No | Yes |
| Multi-frame Grayscale Byte Secondary Capture Image Storage | No | Yes |
| Multi-frame Grayscale Word Secondary Capture Image Storage | No | Yes |
| Multi-frame True Color Secondary Capture Image Storage | No | Yes |
| Ultrasound Image Storage | No | Yes |
| Ultrasound Multi-frame Image Storage | No | Yes |
| Ultrasound Image Storage (Retired) | No | Yes |
| Ultrasound Multi-frame Image Storage (Retired) | No | Yes |
| CT Image Storage | No | Option |
| Mammography CAD SR | No | Yes |
| Grayscale Softcopy Presentation State Storage | Yes | Yes |
| Print Management | | |
| Basic Grayscale Print Management Meta SOP Class: <i>includes Basic Film Session, Basic Film Box, Basic Grayscale Image Box, Printer</i> | Yes | No |
| Presentation LUT | Option | No |
| Query/Retrieve | | |
| Study Root Query/Retrieve Information Model – FIND | Yes | No |
| Study Root Query/Retrieve Information Model – MOVE | Yes | No |

Table 1-2 provides an overview of the Media Storage Application Profiles supported by the SecurView RT workstation.

Table 1-2 – Media Services

| Media Storage Application Profile | Write Files (FSC or FSU) | Read Files (FSR) |
|-----------------------------------|--------------------------|------------------|
| Compact Disk - Recordable | | |
| General Purpose CD-R | Yes | Yes |
| DVD | | |
| General Purpose DVD-RAM | Yes | Yes |
| USB | | |
| General Purpose USB | Yes | Yes |

2. Table of Contents

| | |
|---|-----------|
| 1. Overview | 3 |
| 2. Table of Contents | 5 |
| 3. Introduction | 7 |
| 3.1. Audience | 7 |
| 3.2. Remarks | 7 |
| 3.3. Definitions, Terms and Abbreviations | 8 |
| 3.4. References | 8 |
| 4. Networking | 9 |
| 4.1. Implementation Model | 9 |
| 4.1.1. Application Data Flow | 9 |
| 4.1.2. Functional Definition of AEs | 10 |
| 4.1.3. Sequencing of Real-World Activities | 11 |
| 4.2. AE Specifications | 13 |
| 4.2.1. SCU AE | 13 |
| 4.2.2. SCP AE | 24 |
| 4.3. Network Interfaces | 29 |
| 4.3.1. Physical Network Interface | 29 |
| 4.3.2. Additional Protocols | 29 |
| 4.4. Configuration | 29 |
| 4.4.1. AE Title/Presentation Address Mapping | 29 |
| 4.4.2. Configuration Parameters | 29 |
| 5. Media Interchange | 31 |
| 5.1. Implementation Model | 31 |
| 5.1.1. Application Data Flow Diagram | 31 |
| 5.1.2. Functional Definition of AEs | 31 |
| 5.1.3. Sequencing of Real-World Activities | 31 |
| 5.1.4. File Meta Information for Implementation Class Version | 31 |
| 5.2. AE Specifications | 32 |
| 5.2.1. Media Server AE Specification | 32 |
| 5.3. Augmented and Private Application Profiles | 34 |
| 5.4. Media Configuration | 34 |
| 6. Support of Character Sets | 34 |
| 7. Security | 35 |
| 8. Annexes | 35 |
| 8.1. IOD Contents | 35 |
| 8.1.1. Created SOP Instance(s) | 35 |

8.1.2. Usage of Attributes from Received IODs 40

8.1.3. Attribute Mapping..... 44

8.1.4. Coerced/Modified Attributes 44

8.2. Data Dictionary of Private Attributes 44

8.3. Coded Terminology and Templates 44

8.4. Grayscale Image Consistency..... 44

8.5. Standard Extended/Specialized/Private SOP Classes 44

8.6. Private Transfer Syntaxes 44

3. Introduction

Hologic, Inc. develops and markets a full line of mammography products including the SecurView RT workstation, for use by radiologic technologists to facilitate easier image workflow with the SecurView DX diagnostic review workstation and the Selenia® and Selenia Dimensions® full-field digital mammography systems.

The SecurView RT workstation provides a user interface for viewing digital mammography and other breast imaging procedures, and for exchanging Notices with SecurView DX workstations. The SecurView RT workstation includes a graphical user interface (GUI) with a pointing device, keyboard, and one display.

3.1. Audience

This document contains the DICOM conformance claim for the SecurView RT workstation. This document is intended to aid in connecting the SecurView RT workstation to other components that make use of the DICOM standard for interconnecting networked diagnostic review workstation devices.

The information within this document applies to SecurView RT software version 12.x. The reader of this document should be familiar with the DICOM standard and PACS components that utilize the standard.

3.2. Remarks

A DICOM conformance statement – the structure and content of which are stipulated by the DICOM standard – is intended to aid in determining the suitability of interconnecting digital imaging devices. References to specific functionality in a conformance statement are not sufficient to guarantee interoperability between components. The following should be considered when evaluating interoperability:

- The SecurView RT workstation conformance statement provides a starting point for ascertaining whether the product can communicate with other systems.
- The only way to know for certain whether the SecurView RT workstation can interoperate with other systems is to perform connectivity testing.
- This document represents a best effort to document the functionality of commercial versions of the SecurView RT workstation and is not a functional specification of any Hologic component or product. Hologic reserves the right to make changes at any time to the functionality of the DICOM components described herein, and is committed to following the evolution of the DICOM standard.

3.3. Definitions, Terms and Abbreviations

AE: Application Entity

CAD: Computer Aided Detection

CT: Computed Tomography

DICOM: Digital Imaging and Communications in Medicine

DIMSE: DICOM Message Service Element

GRSPS: Grayscale Softcopy Presentation State

IOD: Information Object Definition

JPEG: Joint Photographic Experts Group (data compression techniques)

LUT: Lookup Table

MR: Magnetic Resonance

NEMA: National Electrical Manufacturers Association

NM: Nuclear Medicine

PACS: Picture Archive and Communications System (image management & long-term storage)

PET: Positron Emission Tomography

PDU: Protocol Data Unit

Q/R: Query/Retrieve

RLE: Run Length Encoding (data compression technique)

SC: Secondary Capture

SCP: Service Class Provider

SCU: Service Class User

SOP: Service Object Pair

SR: Structured Report

TCP/IP: Transmission Control Protocol/Internet Protocol

UID: Unique Identifier

VOI: Value of Interest

3.4. References

NEMA PS3 / ISO 12052, Digital Imaging and Communications in Medicine (DICOM) Standard, National Electrical Manufacturers Association, Rosslyn, VA, USA (available free at <http://www.dicomstandard.org/>)

4. Networking

4.1. Implementation Model

4.1.1. Application Data Flow

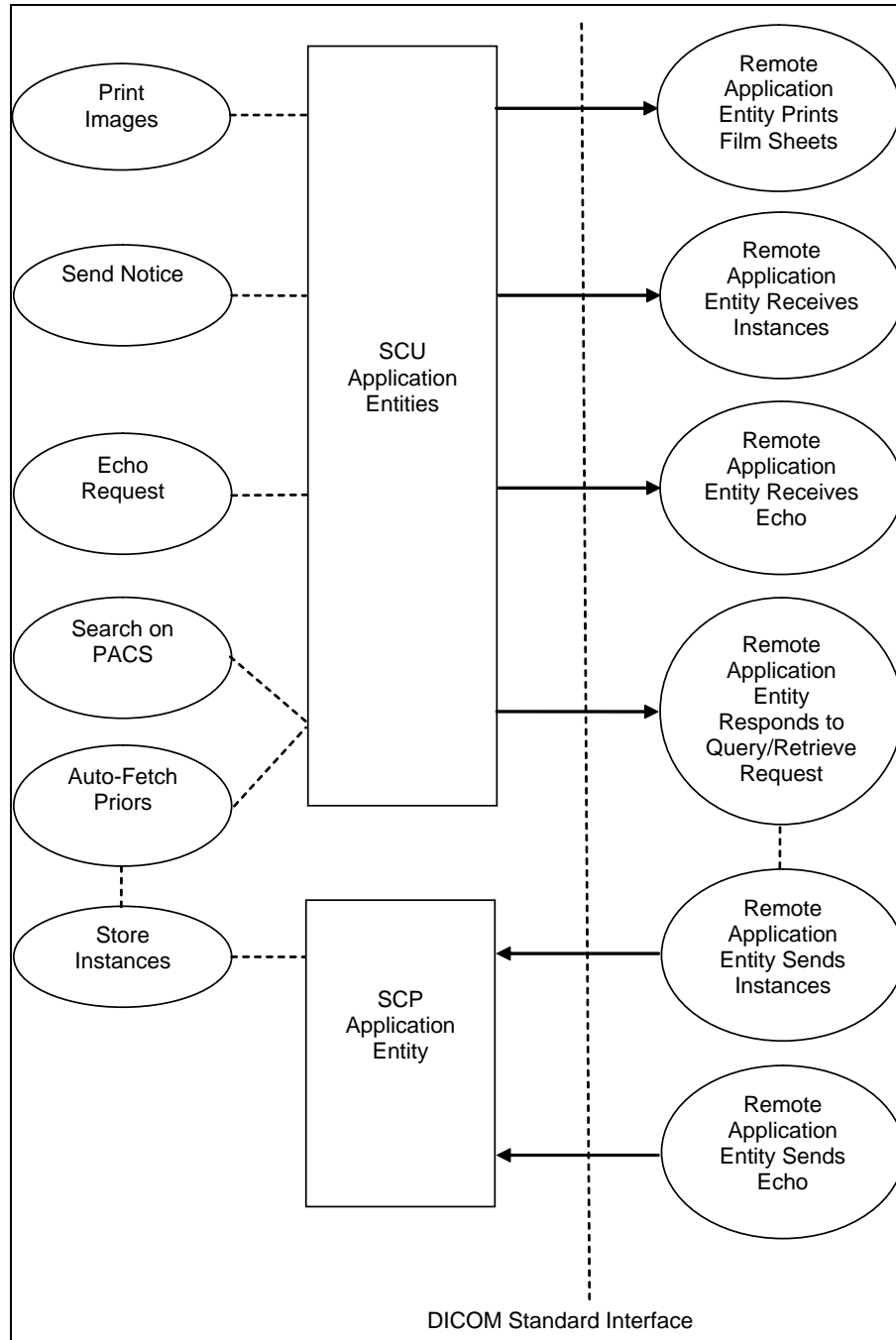


Figure 4.1-1: Functional Overview

An overview of the real-world activities and AEs is provided below:

- The **Print Images** real-world activity triggers an SCU Application Entity to print selected images to film on a selected remote AE (Printer).
- The **Send Notice** real-world activity triggers an SCU Application Entity to send a Notice (markings, annotations, measurements) created by the radiologic technologist to one or more selected remote AEs, such as SecurView DX workstations.
- The **Echo Request** real-world activity triggers an SCU Application Entity to send an echo request to a selected remote AE.
- The **Search on PACS** real-world activity triggers an SCU Application Entity to query a remote AE for patient studies, series, or instances (based on configured query level) and provides the list of matching responses. The user selects patient studies, series, or instances to be retrieved. The SCU Application Entity issues a retrieve request for each selected study, series, or instance, based on the configured level.
- The SCP Application Entity is always available to receive instances from a remote AE, and triggers the **Store Instances** real-world activity to make them available to the user.
- If configured to do so, the **Store Instances** real-world activity may trigger the **Auto-Fetch Priors** real-world activity. The **Auto-Fetch Priors** real-world activity triggers an SCU Application Entity to query and retrieve studies, series, or instances related to a received instance (e.g. prior images, images related to a Notice). The amount and types of instances to retrieve are configurable.

4.1.2. Functional Definition of AEs

The SCU Application Entities and SCP Application Entity are implemented as separate processes that run in the background. They are started automatically during startup of the SecurView RT workstation.

4.1.2.1. Functional Definition of SCU Application Entities

The SCU Application Entities initiate Print, Store, Echo, and Query/Retrieve requests as needed by the SecurView RT workstation.

4.1.2.2. Functional Definition of SCP Application Entity

The SCP Application Entity is available to accept echo and storage requests whenever the SecurView RT workstation is running. The SCP Application Entity stores received instances locally.

4.1.3. Sequencing of Real-World Activities

For Print Images, a user triggers an SCU AE to perform DICOM print by selecting one or more images, or a portion of the screen from the reviewing display to print to a selected Print Provider.

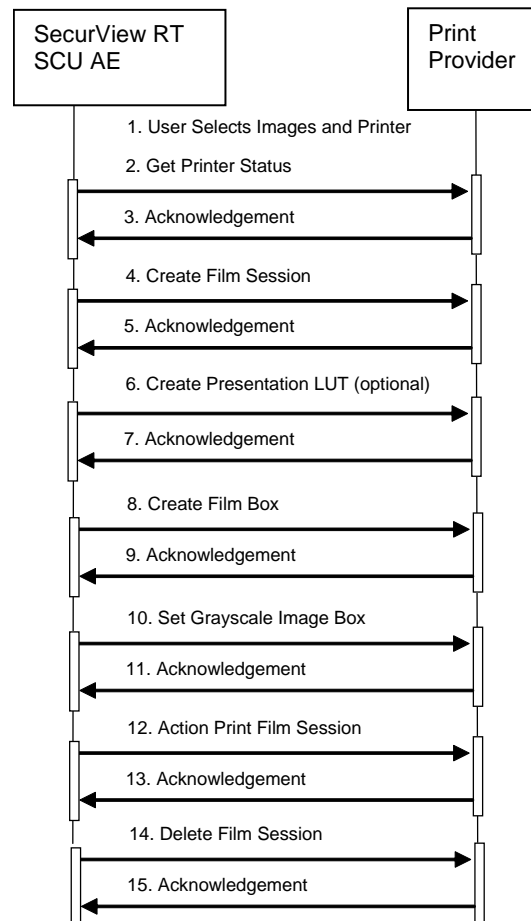


Figure 4.1-2: Print Images Sequence Diagram

For Send Notice, a user triggers an SCU AE to perform DICOM store by creating markings, annotations, and/or measurements on a displayed image and requesting Send Notice to one or more remote AEs.

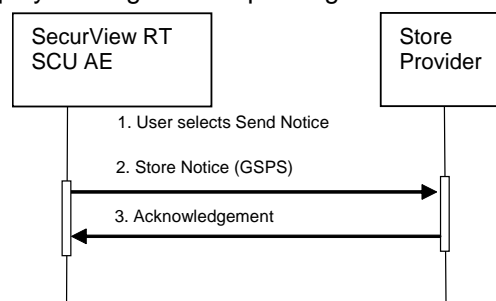


Figure 4.1-3: Send Notice Sequence Diagram

For Search on PACS, a user triggers an SCU AE to perform DICOM query/retrieve by entering search criteria and executing a query. The matching patients' studies, series, or instances are displayed to the user, based on query level configuration. The user selects one or more series to retrieve and executes a retrieve request. The SCP AE receives the instances sent by the Query/Retrieve Provider in response to the retrieve request.

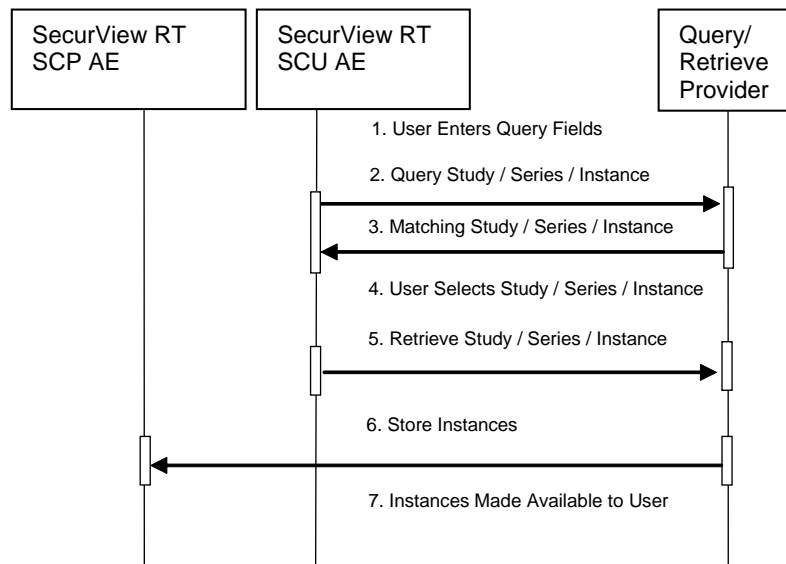


Figure 4.1-4: Search on PACS Sequence Diagram

For Auto-Fetch Priors, receiving an instance such as the first newly captured digital mammography image for a patient triggers an SCU AE to perform DICOM query/retrieve to search for studies matching the patient. Configurable auto-fetch criteria determine types of received instances that trigger auto-fetch, and the (if configured) follow-up series and instance level query content. The SCU AE retrieves the matching studies, series, or instances that meet the auto-fetch criteria. The SCP AE receives the instances sent by the Query/Retrieve Provider.

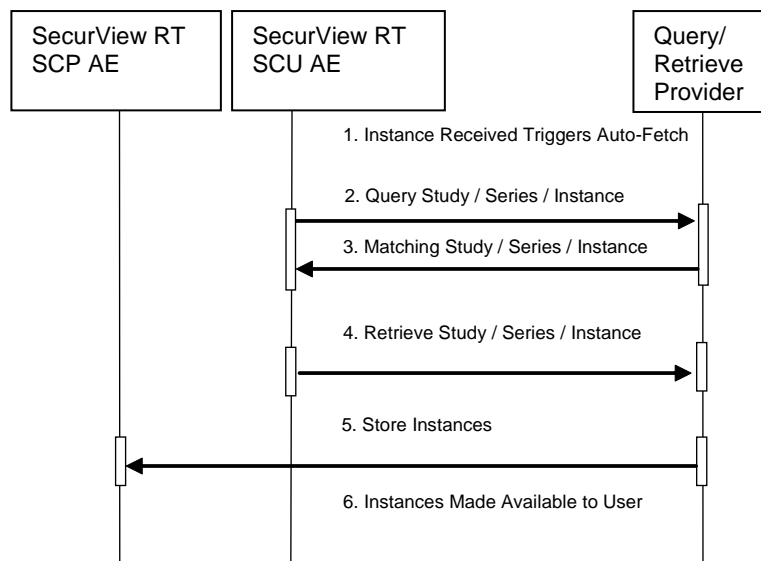


Figure 4.1-5: Auto-Fetch Priors Sequence Diagram

4.2. AE Specifications

4.2.1. SCU AE

4.2.1.1. SOP Classes

The SCU AE provides Standard Conformance to the following SOP Classes:

Table 4.2.1-1 – SOP Classes for SCU AE

| SOP Class Name | SOP Class UID | SCU | SCP |
|--|------------------------------|-----|-----|
| Basic Grayscale Print Management Meta | 1.2.840.10008.5.1.1.9 | Yes | No |
| Presentation LUT | 1.2.840.10008.5.1.1.23 | Yes | No |
| Grayscale Softcopy Presentation State Storage | 1.2.840.10008.5.1.4.1.1.11.1 | Yes | No |
| Verification | 1.2.840.10008.1.1 | Yes | No |
| Study Root Query/Retrieve Information Model – FIND | 1.2.840.10008.5.1.4.1.2.2.1 | Yes | No |
| Study Root Query/Retrieve Information Model – MOVE | 1.2.840.10008.5.1.4.1.2.2.2 | Yes | No |

4.2.1.2. Association Policies

4.2.1.2.1. General

The DICOM standard Application context shall be specified.

Table 4.2.1-2 – DICOM Application Context

| | |
|--------------------------|-----------------------|
| Application Context Name | 1.2.840.10008.3.1.1.1 |
|--------------------------|-----------------------|

4.2.1.2.2. Number of Associations

The SCU AE supports multiple associations. There are no enforced limits for the number of open associations.

4.2.1.2.3. Asynchronous Nature

The SecurView RT workstation does not support asynchronous operations (multiple outstanding transactions over a single Association).

4.2.1.2.4. Implementation Identifying Information

Table 4.2.1-3 – DICOM Implementation Class and Version for SCU AE

| | |
|-----------------------------|--|
| Implementation Class UID | 1.3.6.1.4.1.34261.273440546401752.16168.148654468 4.0.1200020 |
| Implementation Version Name | MBC_SCR_1200020 |

4.2.1.3. Association Initiation Policy

4.2.1.3.1. Print Images

4.2.1.3.1.1. Description and Sequencing of Activities

Print Images is triggered when a user selects to print selected images to a selected remote Print AE either for diagnostic print or screen capture print. The SCU AE attempts to establish an association with a remote Print AE when a request to print one or more images is received. A remote Print AE configuration is predefined for each known printer manufacturer, model, and default film size. Use of the Presentation LUT SOP Class is configurable per printer.

Print requests can be queued, in that a new print request can be initiated before previous print requests are completed. Each print request consists of Printer N-GET, Film Session N-CREATE, Presentation LUT N-CREATE (optional), Film Box N-CREATE, Grayscale Image Box N-SET, Film Session N-ACTION, and Film Session N-DELETE. One association is negotiated per print request. The association is released after the N-DELETE response is received, if not released earlier due to failure.

4.2.1.3.1.2. Proposed Presentation Contexts

The SCU AE attempts to establish associations using the following presentation contexts:

Table 4.2.1-4 – Proposed Presentation Contexts for Print Images

| Presentation Context Table | | | | | |
|---------------------------------------|----------------------|----------------------------|---------------------|------|----------------------|
| Abstract Syntax | | Transfer Syntax | | Role | Extended Negotiation |
| Name | UID | Name List | UID List | | |
| Basic Grayscale Print Management Meta | 1.2.840.10008.5.1.9 | Implicit VR, Little Endian | 1.2.840.10008.1.2 | SCU | None |
| | | Explicit VR, Little Endian | 1.2.840.10008.1.2.1 | SCU | None |
| Presentation LUT | 1.2.840.10008.5.1.23 | Implicit VR, Little Endian | 1.2.840.10008.1.2 | SCU | None |
| | | Explicit VR, Little Endian | 1.2.840.10008.1.2.1 | SCU | None |

4.2.1.3.1.3. SOP Specific Conformance for Print

The general behavior of the SCU AE during communication failure is summarized in the table below. This behavior is common to all print SOP Classes supported by the SCU AE. The number of seconds to wait for a response from the remote Print AE before timing out is configurable (default=30). The SCU AE retries a configurable number of times (default=30) after a response timeout.

Table 4.2.1-5 – Print Command Communication Failure Behavior

| Exception | Behavior |
|--|--|
| Timeout | The SCU AE releases the association, and the error is recorded in a user-accessible log. |
| Association rejected or aborted by SCP | The error is recorded in a user-accessible log. |

The SCU AE provides baseline conformance. The SCU AE uses 12-bit grayscale images for print if supported by the remote Print AE. 8-bit grayscale is used otherwise. This section describes the DIMSE services and the attributes supported for various remote Print AEs.

4.2.1.3.1.3.1. Specific Conformance to Basic Film Session SOP Class

Table 4.2.1-6 – Basic Film Session SOP Class: Supported DIMSE Operations

| Name | Description |
|----------|--------------------------|
| N-CREATE | Creates the film session |
| N-ACTION | Prints the film session |
| N-DELETE | Deletes the film session |

Table 4.2.1-7 – Basic Film Session SOP Class: Supported Attributes

| Attribute Name | Tag | Supported Values | Default Value |
|------------------|-------------|---|------------------------|
| Number Of Copies | (2000,0010) | 1 | 1 |
| Print Priority | (2000,0020) | MED | MED |
| Medium Type | (2000,0030) | BLUE FILM CLEAR FILM MAMMO BLUE FILM PAPER | Predefined per Printer |

The behavior of the SCU AE when encountering status codes in the N-CREATE, N-ACTION or N-DELETE response is summarized in the table below.

Table 4.2.1-8 – Film Session SOP Class Response Status Handling Behavior

| Service Status | Further Meaning | Error Code | Behavior |
|----------------|-----------------|------------|--|
| Success | Success | 0000 | The remote Print AE has completed the operation successfully. Proceed to next step. |
| Warning | Any | Any | If relaxed error handling is enabled, proceed to next step as if successful. If not, the SCU AE releases the association and the error is recorded in a user-accessible log. |
| Failure | Any | Any | The SCU AE releases the association, and the error is recorded in a user-accessible log. |

4.2.1.3.1.3.2. Specific Conformance to Basic Film Box SOP Class

Table 4.2.1-9 – Basic Film Box SOP Class: Supported DIMSE Operations

| Name | Description |
|----------|----------------------|
| N-CREATE | Creates the film box |

Table 4.2.1-10 – Basic Film Box SOP Class: Supported Attributes

| Attribute Name | Tag | Supported Values | Default Value |
|--------------------------------------|-------------|--|---|
| Image Display Format | (2010,0010) | STANDARD\1,1 | STANDARD\1,1 |
| Film Orientation | (2010,0040) | PORTRAIT LANDSCAPE | |
| Film Size ID | (2010,0050) | 8INX10IN 10INX12IN 10INX10IN 10INX14IN 11INX14IN 14INX14IN 14INX17IN (Configurable per printer) | Configured per each printer's requirements with image matrix size, or set to calculate automatically for true size printing |
| Magnification Type | (2010,0060) | CUBIC | CUBIC |
| Border Density | (2010,0100) | BLACK | BLACK |
| Empty Image Density | (2010,0110) | BLACK | BLACK |
| Min Density | (2010,0120) | Configurable per printer | Predefined per Printer |
| Max Density | (2010,0130) | Configurable per printer | Predefined per Printer |
| Trim | (2010,0140) | NO | NO |
| Configuration Information | (2010,0150) | Configurable per printer | Predefined per Printer |
| Illumination | (2010,015E) | Configurable per printer, sent only if Presentation LUT is sent | 5000 |
| Reflected Ambient Light | (2010,0160) | Configurable per printer, sent only if Presentation LUT is sent | 10 |
| Referenced Film Session Sequence | (2010,0500) | One sequence Item identifies the film session for this film box | |
| Referenced Presentation LUT Sequence | (2050,0500) | One sequence Item identifies the Presentation LUT for this film box, sent only if Presentation LUT is sent | |
| Referenced Image Box Sequence | (2010,0510) | Returned by the printer | |

The behavior of the SCU AE when encountering status codes in the N-CREATE response is summarized in the table below.

Table 4.2.1-11 – Film Box SOP Class Response Status Handling Behavior

| Service Status | Further Meaning | Error Code | Behavior |
|----------------|-----------------|------------|--|
| Success | Success | 0000 | The remote Print AE has completed the operation successfully. Proceed to next step. |
| Warning | Any | Any | If relaxed error handling is enabled, proceed to next step as if successful. If not, the SCU AE releases the association and the error is recorded in a user-accessible log. |
| Failure | Any | Any | The SCU AE releases the association, and the error is recorded in a user-accessible log. |

4.2.1.3.1.3.3. Specific Conformance to Basic Grayscale Image Box SOP Class

Table 4.2.1-12 – Basic Grayscale Image Box SOP Class: Supported DIMSE Operations

| Name | Description |
|-------|--|
| N-SET | Updates an image box in a previously created film box. |

Table 4.2.1-13 – Basic Grayscale Image Box SOP Class: Supported Attributes

| Attribute Name | Tag | Supported Values | Default Value |
|--------------------------------|-------------|--|------------------------|
| Image Box Position | (2020,0010) | 1 | 1 |
| Basic Grayscale Image Sequence | (2020,0110) | | |
| > Samples Per Pixel | (0028,0002) | 1 | 1 |
| > Photometric Interpretation | (0028,0004) | MONOCHROME2 | MONOCHROME2 |
| > Rows | (0028,0010) | Per printed image | |
| > Columns | (0028,0011) | Per printed image | |
| > Pixel Aspect Ratio | (0028,0034) | 1\1 | 1\1 |
| > Bits Allocated | (0028,0100) | 8 or 16 | |
| > Bits Stored | (0028,0101) | 8 or 12 | |
| > High Bit | (0028,0102) | 7 or 11 | |
| > Pixel Representation | (0028,0103) | 0 | 0 |
| > Pixel Data | (7FE0,0010) | Per printed image | |
| Configuration Information | (2010,0150) | Configurable per printer | Predefined per Printer |
| Magnification Type | (2010,0060) | CUBIC | CUBIC |
| Requested Image Size | (2020,0030) | For diagnostic print, set to desired width per printer | Predefined per Printer |

The behavior of the SCU AE when encountering status codes in the N-SET response is summarized in the table below:

Table 4.2.1-14 – Image Box SOP Class N-SET Response Status Handling Behavior

| Service Status | Further Meaning | Error Code | Behavior |
|----------------|-----------------|------------|--|
| Success | Success | 0000 | The remote Print AE has completed the operation successfully. Proceed to next step. |
| Warning | Any | Any | If relaxed error handling is enabled, proceed to next step as if successful. If not, the SCU AE releases the association and the error is recorded in a user-accessible log. |
| Failure | Any | Any | The SCU AE releases the association, and the error is recorded in a user-accessible log. |

4.2.1.3.1.3.4. Specific Conformance to Presentation LUT SOP Class

Table 4.2.1-15 – Presentation LUT SOP Class: Supported DIMSE Operations

| Name | Description |
|----------|---|
| N-CREATE | Creates a Presentation LUT to be referenced by a film box |

Table 4.2.1-16 – Presentation LUT SOP Class: Supported Attributes

| Attribute Name | Tag | Supported Values | Default Value |
|------------------------|-------------|--------------------|--------------------------|
| Presentation LUT Shape | (2050,0020) | IDENTITY LIN OD | Configurable per printer |

The behavior of the SCU AE when encountering status codes in the N-CREATE response is summarized in the table below:

Table 4.2.1-17 – Presentation LUT SOP Class N-CREATE Response Status Handling Behavior

| Service Status | Further Meaning | Error Code | Behavior |
|----------------|-----------------|------------|--|
| Success | Success | 0000 | The remote Print AE has completed the operation successfully. Proceed to next step. |
| Warning | Any | Any | If relaxed error handling is enabled, proceed to next step as if successful. If not, the SCU AE releases the association and the error is recorded in a user-accessible log. |
| Failure | Any | Any | The SCU AE releases the association, and the error is recorded in a user-accessible log. |

4.2.1.3.1.3.5. Specific Conformance to Printer SOP Class

Table 4.2.1-18 – Printer SOP Class: Supported DIMSE Operations

| Name | Description |
|----------------|--|
| N-EVENT-REPORT | Receives status notification. |
| N-GET | Retrieves an instance of a physical printer. |

- N-EVENT-REPORT requests are accepted up until the Film Session N-DELETE response is received.
- In the N-GET request, the SCU AE requests Printer Status, Printer Status Info, Printer Name, Manufacturer and Manufacturer's Model Name.
- The behavior of the SCU AE when encountering status codes in the N-GET response is summarized in the table below:

Table 4.2.1-19 – Printer SOP Class N-GET Response Status Handling Behavior

| Service Status | Further Meaning | Error Code | Behavior |
|----------------|-----------------|------------|---|
| Success | Success | 0000 | If Printer Status returned is NORMAL, proceed to next step. If not, the SCU AE releases the association and the error is recorded in a user-accessible log. |
| Warning | Any | Any | If relaxed error handling is enabled and Printer Status is NORMAL, proceed to next step as if successful. If not, the SCU AE releases the association and the error is recorded in a user-accessible log. |
| Failure | Any | Any | The SCU AE releases the association, and the error is recorded in a user-accessible log. |

4.2.1.3.2. Send Notice

4.2.1.3.2.1. Description and Sequencing of Activities

Send Notice is triggered when a radiologic technologist user creates markings, annotations, and/or measurements on one or more images and requests Send Notice, for all images or a selected image, to one or more remote AEs. The SCU AE negotiates a separate association with each selected remote Store AE to send each Notice. The SCU AE releases each association upon completion of the storage request.

4.2.1.3.2.2. Proposed Presentation Contexts

The SCU AE attempts to establish associations using the following presentation contexts:

Table 4.2.1-20 – Proposed Presentation Contexts for Send Notice

| Presentation Context Table | | | | | |
|---|------------------------------|----------------------------|---------------------|------|----------------------|
| Abstract Syntax | | Transfer Syntax | | Role | Extended Negotiation |
| Name | UID | Name List | UID List | | |
| Grayscale Softcopy Presentation State Storage | 1.2.840.10008.5.1.4.1.1.11.1 | Implicit VR, Little Endian | 1.2.840.10008.1.2 | SCU | None |
| | | Explicit VR, Little Endian | 1.2.840.10008.1.2.1 | SCU | None |

4.2.1.3.2.3. SOP Specific Conformance for Storage

The behavior of the SCU AE when encountering status codes in a C-STORE response is summarized in the table below.

Table 4.2.1-21 – C-STORE Response Status Handling Behavior

| Service Status | Further Meaning | Error Code | Behavior |
|--------------------|-----------------|------------|--|
| Success | Success | 0000 | The remote Store SCP has successfully stored the SOP Instance. Job complete. |
| Warning or Failure | Any | Any | The SCU AE releases the association, and the error is reported to the user. |

The behavior of the SCU AE during communication failure is summarized in the table below. A failed store request is automatically retried a configurable number of times before reporting failure.

Table 4.2.1-22 – Storage Communication Failure Behavior

| Exception | Behavior |
|--|---|
| Timeout | The SCU AE releases the association, and the error is reported to the user. |
| Association rejected or aborted by SCP | The error is reported to the user. |

Instance UIDs are globally unique for all SOP instances generated by the SCU AE. The UID root is '1.3.6.1.4.1.34261'.

The SCU AE provides standard conformance to the Grayscale Softcopy Presentation State Storage SOP Class. Based on which Send Notice method the user selects, a Notice GSPS instance contains the markings, annotations and measurements made by the radiologic technologist on all of the patient's images, or a selected image.

4.2.1.3.3. Echo Request

4.2.1.3.3.1. Description and Sequencing of Activities

From the DICOM configuration dialog, the service user requests an Echo to test DICOM communication with a configured remote AE. The SCU AE issues a C-ECHO request to the remote AE, and provides the response to the user interface.

4.2.1.3.3.2. Proposed Presentation Contexts

The SCU AE attempts to establish associations using the following presentation contexts:

Table 4.2.1-23 – Proposed Presentation Contexts for Echo Request

| Presentation Context Table | | | | | |
|----------------------------|-------------------|----------------------------|---------------------|------|----------------------|
| Abstract Syntax | | Transfer Syntax | | Role | Extended Negotiation |
| Name | UID | Name List | UID List | | |
| Verification | 1.2.840.10008.1.1 | Implicit VR, Little Endian | 1.2.840.10008.1.2 | SCU | None |
| | | Explicit VR, Little Endian | 1.2.840.10008.1.2.1 | SCU | None |

4.2.1.3.3.3. SOP Specific Conformance for Verification

The SCU AE provides baseline conformance. The behavior of the SCU AE when encountering status codes in a C-ECHO response is summarized in the table below.

Table 4.2.1-24 – Verification C-ECHO Response Status Handling Behavior

| Service Status | Further Meaning | Error Code | Behavior |
|----------------|-----------------|------------|----------------------------------|
| Success | Success | 0000 | Success is reported to the user. |

The behavior of the SCU AE during communication failure is summarized in the table below.

Table 4.2.1-25 – Verification Communication Failure Behavior

| Exception | Behavior |
|--|---|
| Timeout | The SCU AE releases the association, and the error is reported to the user. |
| Association rejected or aborted by remote AE | The error is reported to the user. |

4.2.1.3.4. Search on PACS / Auto-Fetch Priors

4.2.1.3.4.1. Description and Sequencing of Activities

Search on PACS is triggered by user interaction. The user may enter matching keys that the SCU AE uses to query a selected remote Query/Retrieve AE using the C-FIND operation. The SCU AE then displays a list of patient studies, series or instances returned in the C-FIND responses. The user may select one or more patient studies, series or instances from the list to retrieve, which signals the SCU AE to generate a C-MOVE request to the remote Query/Retrieve AE, specifying the SCP AE as the move destination. The instances received are made available for the user to view on the SecurView RT workstation.

The default Search on PACS behavior is for the SCU AE to send an initial Study Root, Study Level hierarchical query, followed by Study Root, Series Level hierarchical queries for each study returned. Matching responses are displayed to the user at the series level. The default retrieve behavior is to send a Study Root, Series Level hierarchical move request for each selected patient series. Alternate behavior can be configured such that only Study Level hierarchical queries and retrieves are performed. Alternate behavior can be configured such that Study, Series, and Instance Level hierarchical queries with Instance level retrieves are performed.

When configured as enabled, Auto-Fetch Priors is triggered when the first newly captured digital mammography image for a patient is received. The SCU AE performs C-FIND and C-MOVE operations in the background with a predefined remote Query/Retrieve AE. The default behavior is to send a Study Root, Study Level hierarchical query for the patient identified in the received image. Based on the configured Auto-Fetch Query/Retrieve level (Study, Series, or Image), the SCU AE may send follow-up Study Root, Series Level and Study Root, Image Level hierarchical queries to the remote Query/Retrieve AE. Based on the configured Auto-Fetch Query/Retrieve level, Studies, Series, or Instances are retrieved

one at a time via Study Root, configured level hierarchical move request. If a remote Query/Retrieve AE does not support Image Level C-FIND or C-MOVE, it is configurable to stop at the Series Level C-FIND and perform Series Level C-MOVE. If a remote Query/Retrieve AE does not support Series Level C-FIND or C-MOVE, and the remote Query/Retrieve AE supports Modalities in Study (0008,0061) as a C-FIND return key, it is configurable to stop at the Study Level C-FIND and perform Study Level C-MOVE. The default Auto-Fetch Query/Retrieve level is Series.

For each Search on PACS or Auto-Fetch Priors occurrence, the SCU AE negotiates one association within which all configured levels of C-FIND requests are sent. The SCU AE negotiates a separate consecutive association for each C-MOVE request.

4.2.1.3.4.2. Proposed Presentation Contexts

The SCU AE attempts to establish associations using the following presentation contexts:

Table 4.2.1-26 – Proposed Presentation Contexts for Search on PACS

| Presentation Context Table | | | | | |
|--|-----------------------------|----------------------------|---------------------|------|----------------------|
| Abstract Syntax | | Transfer Syntax | | Role | Extended Negotiation |
| Name | UID | Name List | UID List | | |
| Study Root Query/Retrieve Information Model – FIND | 1.2.840.10008.5.1.4.1.2.2.1 | Implicit VR, Little Endian | 1.2.840.10008.1.2 | SCU | None |
| | | Explicit VR, Little Endian | 1.2.840.10008.1.2.1 | SCU | None |
| Study Root Query/Retrieve Information Model – MOVE | 1.2.840.10008.5.1.4.1.2.2.2 | Implicit VR, Little Endian | 1.2.840.10008.1.2 | SCU | None |
| | | Explicit VR, Little Endian | 1.2.840.10008.1.2.1 | SCU | None |

4.2.1.3.4.3. SOP Specific Conformance for Query/Retrieve

The behavior of the SCU AE when encountering status codes in a C-FIND or C-MOVE response is summarized in the table below. For Auto-Fetch Priors, a failed request is retried automatically a predefined number of times before reporting failure.

Table 4.2.1-27 – C-FIND and C-MOVE Response Status Handling Behavior

| Service Status | Further Meaning | Error Code | Behavior |
|--------------------|--|------------|--|
| Success | Matching or sub-operations are complete | 0000 | The remote SCP has successfully returned all matching information, or all sub-operations are complete. For C-FIND, perform follow-up query or move, or display results to the user. For C-MOVE, success is reported to the user either in the Search on PACS dialog or the patient list. |
| Pending | Matches or sub-operations are continuing | FF00 | For C-FIND, the matching query result contained in the Identifier is collected for display or automated follow-up. For C-MOVE, the SCU AE continues to wait for additional messages. |
| Warning or Failure | Any | Any | The SCU AE releases the association. For Search on PACS C-FIND, failure is reported to the user in a status bar in the Search on PACS dialog. For Auto-Fetch Priors C-FIND, failure is reported to the user in the patient list. For C-MOVE, failure is reported to the user either in the Search on PACS dialog or the patient list. |

The behavior of the SCU AE during communication failure is summarized in the table below. For Auto-Fetch Priors, a failed request is retried automatically a predefined number of times before reporting failure.

Table 4.2.1-28 – C-FIND and C-MOVE Communication Failure Behavior

| Exception | Behavior |
|--|--|
| Timeout | The SCU AE releases the association. For Search on PACS C-FIND, failure is reported to the user in a status bar in the Search on PACS dialog. For Auto-Fetch Priors C-FIND, failure is reported to the user in the patient list. For C-MOVE, failure is reported to the user either in the Search on PACS dialog or the patient list. |
| Association rejected or aborted by SCP | For Search on PACS C-FIND, failure is reported to the user in a status bar in the Search on PACS dialog. For Auto-Fetch Priors C-FIND, failure is reported to the user in the patient list. For C-MOVE, failure is reported to the user either in the Search on PACS dialog or the patient list. |

The SCU AE provides standard conformance to the Study Root Information Model, hierarchical method only. The Relational method is not supported.

For Search on PACS, by default the SCU AE performs Study Level queries with follow-up Series Level queries as needed, and Series Level retrieves. It is possible to configure Study Level only queries and retrieves. It is possible to configure Study, Series, and Instance Level queries with Instance level retrieves.

For Auto-Fetch Priors, by default the SCU AE performs Study Level and Series Level queries, and Series Level retrieves. It is possible to configure Study Level queries and retrieves if Series Level is not supported by the remote Query/Retrieve AE. It is possible to configure Study, Series, and Instance Level queries with Instance Level retrieves.

For Search on PACS, the SCU AE supports query Matching Key Attributes as defined in the following table. The user supplies Matching Key Attribute values, either by manual entry, drop list selection, or date range selection. Wild Card matching is requested only if the user specifies a wild card character during manual entry. It is configurable to omit optional Study level matching key attributes (Modalities in Study, Referring Physician's Name, Study Description, Patient's Birth Date, Patient's Sex) individually. It is configurable to omit optional Series level matching key attributes (Series Description, Body Part Examined, Performed Procedure Step Start Date, Performed Procedure Step Start Time, Request Attributes Sequence, Scheduled Procedure Step ID, Request Attribute Sequence, Requested Procedure ID) individually. It is configurable to omit optional Image (Instance) level matching key attributes (SOP Class UID, Images in Acquisition, Number of Frames, Concept Name Code Sequence, Content Template Sequence) individually. The maximum number of displayed matches for a query is 1000. The association is released after receiving 1000 matching responses.

Specific Character Set (0008,0005) is included in C-FIND requests with the value ISO_IR 100 by default. If a language with characters outside ISO_IR 100 is used, it is possible to configure Specific Character Set (0008,0005) in C-FIND requests per remote Query/Retrieve AE to have the value ISO_IR 192 or to be omitted.

Table 4.2.1-29 – Supported C-FIND Matching Key Attributes (Search on PACS)

| Matching Key Attribute | Tag | Matching Type |
|------------------------|-------------|---|
| Study Level | | |
| Patient's Name | (0010,0010) | Wild Card or Universal Matching |
| Patient ID | (0010,0020) | Single Value, Wild Card or Universal Matching |
| Study Date | (0008,0020) | Single Value, Range or Universal Matching |
| Study Time | (0008,0030) | Range or Universal Matching |

| Matching Key Attribute | Tag | Matching Type |
|-------------------------------------|-------------|---|
| Accession Number | (0008,0050) | Single Value, Wild Card or Universal Matching |
| Modalities in Study | (0008,0061) | Single Value, Wild Card or Universal Matching; used only if alternate Study Level only query is configured (Optional) |
| Study ID | (0020,0010) | Single Value or Wild Card Matching |
| Study Instance UID | (0020,000D) | Single Value or Universal Matching |
| Referring Physician's Name | (0008,0090) | Wild Card or Universal Matching (Optional) |
| Study Description | (0008,1030) | Universal Matching (Optional) |
| Patient's Birth Date | (0010,0030) | Single Value or Universal Matching (Optional) |
| Patient's Sex | (0010,0040) | Universal Matching (Optional) |
| Series Level | | |
| Study Instance UID | (0020,000D) | Single Value Matching (from Study level result) |
| Series Instance UID | (0020,000E) | Single Value or Universal Matching |
| Series Number | (0020,0011) | Single Value or Universal Matching |
| Modality | (0008,0060) | Single Value, Wild Card or Universal Matching |
| Series Description | (0008,103E) | Universal Matching (Optional) |
| Body Part Examined | (0018,0015) | Single Value or Universal Matching (Optional) |
| Performed Procedure Step Start Date | (0040,0244) | Single Value, Range or Universal Matching (Optional) |
| Performed Procedure Step Start Time | (0040,0245) | Range or Universal Matching (Optional) |
| Request Attributes Sequence | (0040,0275) | Sequence or Universal Matching (Optional) |
| >Scheduled Procedure Step ID | (0040,0009) | Single Value or Universal Matching (Optional) |
| >Requested Procedure ID | (0040,1001) | Single Value or Universal Matching (Optional) |
| Image (Instance) Level | | |
| Study Instance UID | (0020,000D) | Single Value Matching (from Study level result) |
| Series Instance UID | (0020,000E) | Single Value Matching (from Series level result) |
| Instance Number | (0020,0013) | Single Value or Universal Matching |
| SOP Instance UID | (0008,0018) | Single Value or Universal Matching |
| SOP Class UID | (0008,0016) | Universal Matching (Optional) |
| Images in Acquisition | (0020,1002) | Universal Matching (Optional) |
| Number of Frames | (0028,0008) | Universal Matching (Optional) |
| Concept Name Code Sequence | (0040,A043) | Sequence or Universal Matching (Optional) |
| >Code Value | (0008,0100) | Universal Matching (Optional) |
| >Coding Scheme Designator | (0008,0102) | Universal Matching (Optional) |
| >Code Meaning | (0008,0104) | Universal Matching (Optional) |
| Content Template Sequence | (0040,A504) | Sequence or Universal Matching (Optional) |
| >Mapping Resource | (0008,0105) | Universal Matching (Optional) |
| >Template Identifier | (0040,DB00) | Universal Matching (Optional) |

For Auto-Fetch Priors, the SCU AE requires the remote Query/Retrieve SCP to support query Matching Key Attributes as defined in the following table.

Table 4.2.1-30 – Supported C-FIND Matching Key Attributes (Auto-Fetch Priors)

| Matching Key Attribute | Tag | Matching Type |
|------------------------|-------------|--|
| Study Level | | |
| Patient ID | (0010,0020) | Single Value Matching |
| Patient's Name | (0010,0010) | Universal Matching |
| Study Date | (0008,0020) | Universal Matching |
| Study Instance UID | (0020,000D) | Universal Matching |
| Study ID | (0020,0010) | Universal Matching |
| Modalities in Study | (0008,0061) | Universal Matching (sent only if Auto-Fetch level is set to Study) |
| Series Level | | |
| Study Instance UID | (0020,000D) | Single Value Matching (from Study level result) |
| Series Instance UID | (0020,000E) | Universal Matching |
| Series Number | (0020,0011) | Universal Matching |
| Modality | (0008,0060) | Universal Matching |
| Image Level | | |
| Study Instance UID | (0020,000D) | Single Value Matching (from Study level result) |
| Series Instance UID | (0020,000E) | Single Value Matching (from Series level result) |
| SOP Instance UID | (0008,0018) | Universal Matching |

4.2.1.4. Association Acceptance Policy

The SCU AE does not accept associations.

4.2.2. SCP AE

4.2.2.1. SOP Classes

The SCP AE provides Standard Conformance to the following SOP Classes:

Table 4.2.2-1 – SOP Classes for SCP AE

| SOP Class Name | SOP Class UID | SCU | SCP |
|--|--------------------------------|-----|-----|
| Computed Radiography Image Storage | 1.2.840.10008.5.1.4.1.1.1 | No | Yes |
| Digital X-Ray Image Storage – For Presentation | 1.2.840.10008.5.1.4.1.1.1.1 | No | Yes |
| Digital Mammography X-Ray Image Storage – For Presentation | 1.2.840.10008.5.1.4.1.1.1.2 | No | Yes |
| Digital Mammography X-Ray Image Storage – For Processing | 1.2.840.10008.5.1.4.1.1.1.2.1 | No | Yes |
| Breast Tomosynthesis Image Storage | 1.2.840.10008.5.1.4.1.1.13.1.3 | No | Yes |
| Breast Projection X-Ray Image Storage – For Presentation | 1.2.840.10008.5.1.4.1.1.13.1.4 | No | Yes |
| Breast Projection X-Ray Image Storage – For Processing | 1.2.840.10008.5.1.4.1.1.13.1.5 | No | Opt |
| Secondary Capture Image Storage | 1.2.840.10008.5.1.4.1.1.7 | No | Yes |
| Multi-frame Grayscale Byte Secondary Capture Image Storage | 1.2.840.10008.5.1.4.1.1.7.2 | No | Yes |
| Multi-frame Grayscale Word Secondary Capture Image Storage | 1.2.840.10008.5.1.4.1.1.7.3 | No | Yes |
| Multi-frame True Color Secondary Capture Image Storage | 1.2.840.10008.5.1.4.1.1.7.4 | No | Yes |
| Ultrasound Image Storage | 1.2.840.10008.5.1.4.1.1.6.1 | No | Yes |
| Ultrasound Multi-frame Image Storage | 1.2.840.10008.5.1.4.1.1.3.1 | No | Yes |
| Ultrasound Image Storage (Retired) | 1.2.840.10008.5.1.4.1.1.6 | No | Yes |

| | | | |
|--|-------------------------------|----|-----|
| Ultrasound Multi-frame Image Storage (Retired) | 1.2.840.10008.5.1.4.1.1.3 | No | Yes |
| CT Image Storage | 1.2.840.10008.5.1.4.1.1.2 | No | Yes |
| Mammography CAD SR Storage | 1.2.840.10008.5.1.4.1.1.88.50 | No | Yes |
| Grayscale Softcopy Presentation State Storage | 1.2.840.10008.5.1.4.1.1.11.1 | No | Yes |
| Verification SOP Class | 1.2.840.10008.1.1 | No | Yes |

4.2.2.2. Association Policies

4.2.2.2.1. General

The DICOM standard Application context shall be specified.

Table 4.2.2-2 – DICOM Application Context

| | |
|--------------------------|-----------------------|
| Application Context Name | 1.2.840.10008.3.1.1.1 |
|--------------------------|-----------------------|

4.2.2.2.2. Number of Associations

The SCP AE supports multiple associations. There are no enforced limits for the number of open associations.

4.2.2.2.3. Asynchronous Nature

The SecurView RT workstation does not support asynchronous operations (multiple outstanding transactions over a single Association).

4.2.2.2.4. Implementation Identifying Information

Table 4.2.2-3 – DICOM Implementation Class and Version for SCP AE

| | |
|-----------------------------|--|
| Implementation Class UID | 1.3.6.1.4.1.34261.273440546401752.16168.148654468 4.0.1200020 |
| Implementation Version Name | MBC_SCR_1200020 |

4.2.2.3. Association Initiation Policy

The SCP AE does not initiate associations.

4.2.2.4. Association Acceptance Policy

When the SCP AE accepts an association, it will respond to storage and echo requests.

4.2.2.4.1. Activity – Store Instances

4.2.2.4.1.1. Description and Sequencing of Activities

The SCP AE is always running. Store Instances is triggered when a remote Application Entity requests to store instances to the SCP AE. Expected instances include images of several accepted modalities, Mammography CAD SR, and Grayscale Softcopy Presentation State. The SCP AE stores the instances received so that they can be made available to the user. If a received instance is a duplicate of an existing stored instance it is ignored. Instances remain on the SecurView RT workstation until a configurable maximum storage period is reached, or until they are manually deleted.

4.2.2.4.1.2. Accepted Presentation Contexts

The SCP AE accepts associations using the following presentation contexts:

Table 4.2.2-4 – Acceptable Presentation Contexts for Store Instances

| Presentation Context Table | | | | | |
|--|--------------------------------|-------------------|-----|------|----------------------|
| Abstract Syntax | | Transfer Syntax | | Role | Extended Negotiation |
| Name | UID | Name | UID | | |
| Computed Radiography Image Storage | 1.2.840.10008.5.1.4.1.1.1 | See Table 4.2.2-6 | | SCP | None |
| Digital X-Ray Image Storage – For Presentation | 1.2.840.10008.5.1.4.1.1.1.1 | See Table 4.2.2-6 | | SCP | None |
| Digital Mammography X-Ray Image Storage – For Presentation | 1.2.840.10008.5.1.4.1.1.1.2 | See Table 4.2.2-6 | | SCP | None |
| Digital Mammography X-Ray Image Storage – For Processing | 1.2.840.10008.5.1.4.1.1.1.2.1 | See Table 4.2.2-6 | | SCP | None |
| Breast Tomosynthesis Image Storage | 1.2.840.10008.5.1.4.1.1.13.1.3 | See Table 4.2.2-6 | | SCP | None |
| Breast Projection X-Ray Image Storage – For Presentation | 1.2.840.10008.5.1.4.1.1.13.1.4 | See Table 4.2.2-6 | | SCP | None |
| Breast Projection X-Ray Image Storage – For Processing | 1.2.840.10008.5.1.4.1.1.13.1.5 | See Table 4.2.2-6 | | SCP | None |
| Secondary Capture Image Storage | 1.2.840.10008.5.1.4.1.1.7 | See Table 4.2.2-6 | | SCP | None |
| Multi-frame Grayscale Byte Secondary Capture Image Storage | 1.2.840.10008.5.1.4.1.1.7.2 | See Table 4.2.2-6 | | SCP | None |
| Multi-frame Grayscale Word Secondary Capture Image Storage | 1.2.840.10008.5.1.4.1.1.7.3 | See Table 4.2.2-6 | | SCP | None |
| Multi-frame True Color Secondary Capture Image Storage | 1.2.840.10008.5.1.4.1.1.7.4 | See Table 4.2.2-6 | | SCP | None |
| Ultrasound Image Storage | 1.2.840.10008.5.1.4.1.1.6.1 | See Table 4.2.2-7 | | SCP | None |
| Ultrasound Multi-frame Image Storage | 1.2.840.10008.5.1.4.1.1.3.1 | See Table 4.2.2-7 | | SCP | None |
| Ultrasound Image Storage (Retired) | 1.2.840.10008.5.1.4.1.1.6 | See Table 4.2.2-7 | | SCP | None |
| Ultrasound Multi-frame Image Storage (Retired) | 1.2.840.10008.5.1.4.1.1.3 | See Table 4.2.2-7 | | SCP | None |
| CT Image Storage | 1.2.840.10008.5.1.4.1.1.2 | See Table 4.2.2-6 | | SCP | None |
| Mammography CAD SR Storage | 1.2.840.10008.5.1.4.1.1.88.50 | See Table 4.2.2-5 | | SCP | None |
| Grayscale Softcopy Presentation State Storage | 1.2.840.10008.5.1.4.1.1.11.1 | See Table 4.2.2-5 | | SCP | None |

The SCP AE accepts presentation contexts using the following transfer syntaxes in the order listed in the tables, unless otherwise specified. Presentation contexts containing compression transfer syntaxes can be disabled if necessary:

Table 4.2.2-5 – Acceptable Transfer Syntaxes for Non-Image Instances

| Transfer Syntax Table | |
|---------------------------|---------------------|
| Name | UID |
| Implicit VR Little Endian | 1.2.840.10008.1.2 |
| Explicit VR Little Endian | 1.2.840.10008.1.2.1 |

Table 4.2.2-6 – Acceptable Transfer Syntaxes for Image Instances (Except Ultrasound)

| Transfer Syntax Table | |
|---|------------------------|
| Name | UID |
| Explicit VR Little Endian | 1.2.840.10008.1.2.1 |
| Implicit VR Little Endian | 1.2.840.10008.1.2 |
| JPEG Lossless, Non-Hierarchical, First-Order Prediction (Process 14 [Selection Value 1]): Default Transfer Syntax for Lossless JPEG Image Compression | 1.2.840.10008.1.2.4.70 |
| RLE Lossless | 1.2.840.10008.1.2.5 |
| JPEG 2000 Image Compression (Lossless Only) | 1.2.840.10008.1.2.4.90 |

The order of transfer syntaxes is different for the Breast Tomosynthesis Image Storage and Breast Projection X-Ray Image Storage SOP Classes. The JPEG lossless compression transfer syntaxes (.70, .90) are first and the uncompressed transfer syntaxes are last.

Table 4.2.2-7 – Acceptable Transfer Syntaxes for Ultrasound Image Instances

| Transfer Syntax Table | |
|---|------------------------|
| Name | UID |
| JPEG Lossless, Non-Hierarchical, First-Order Prediction (Process 14 [Selection Value 1]): Default Transfer Syntax for Lossless JPEG Image Compression | 1.2.840.10008.1.2.4.70 |
| JPEG 2000 Image Compression (Lossless Only) | 1.2.840.10008.1.2.4.90 |
| RLE Lossless | 1.2.840.10008.1.2.5 |
| Explicit VR Little Endian | 1.2.840.10008.1.2.1 |
| Implicit VR Little Endian | 1.2.840.10008.1.2 |
| JPEG Baseline (Process 1): Default Transfer Syntax for Lossy JPEG 8 Bit Image Compression | 1.2.840.10008.1.2.4.50 |

The order of transfer syntaxes is different for the Ultrasound Multi-frame Image Storage SOP Classes. The JPEG lossy compression transfer syntax (.50) is first and the JPEG 2000 lossless compression transfer syntax (.90) is last.

4.2.2.4.1.3. SOP Specific Conformance for Storage SOP Classes

The SCP AE provides Level 2 (full) conformance to all of the accepted Storage SOP Classes. All received attributes of Type 1, 2, 3, and Private Attributes are stored without modification, and may be used by the SecurView RT workstation. Instances that are stored successfully are not accessible by remote systems.

The SCP AE does not preserve or replace digital signatures.

The SecurView RT workstation does not display MR, CT, NM, or PET images.

The SecurView RT workstation displays Breast Tomosynthesis Image instances produced by any digital breast tomosynthesis manufacturer. Breast Tomosynthesis Image instances from a specific manufacturer and/or model can be configured to be accepted and discarded if SecurView does not display the reconstructed slices as expected.

The SecurView RT workstation displays Breast Projection X-Ray Image – For Presentation instances produced by any digital breast tomosynthesis manufacturer. Breast Projection X-Ray Image instances from a specific manufacturer and/or model can be configured to be accepted and discarded if SecurView does not display the tomosynthesis projection images as expected. The SecurView RT workstation can be configured to accept Breast Projection X-Ray Image – For Processing, but does not display the images.

The SecurView RT workstation supports display of breast tomosynthesis reconstructed slices received as CT Image.

The SecurView RT workstation supports display of the first DICOM 6000 overlay contained in received single-frame images only if the overlay is the same size as the image (see [8.1.2. Usage of Attributes from Received IODs](#)). The DICOM 6000 overlays in images from a specific manufacturer and/or model application can be configured to be ignored if SecurView does not display the overlay content as expected.

The SecurView RT workstation displays Mammography CAD SR instances produced by any manufacturer. Mammography CAD SR instances that contain 2D content, breast tomosynthesis reconstructed slices content, or both are displayed. Mammography CAD SR instances produced by Hologic, Inc. or iCAD, Inc. are displayed according to the manufacturer's guidelines.

The SecurView RT workstation displays the Presentation State Identification and Graphic Annotation module content of any manufacturer's Grayscale Softcopy Presentation State instances (see [8.1.2, Usage of Attributes from Received IODs](#)). All other Presentation State level module content is ignored (e.g., Shutter, Mask, Displayed Area, Graphic Layer, Softcopy VOI LUT). If a received Grayscale Softcopy Presentation State instance does not contain the Graphic Annotation module, it will be accepted and discarded. The Grayscale Softcopy Presentation State instances from a specific manufacturer and/or model application can be configured to be accepted and discarded if SecurView does not display the Graphic Annotation module content as expected.

The SCP AE will behave as described in the table below when generating the C-STORE response.

Table 4.2.2-8 – Storage C-STORE Response Status

| Service Status | Further Meaning | Error Code | Reason |
|----------------|---------------------------|------------|---|
| Success | Success | 0000 | Image is stored locally and made available to the user. |
| Failure | Refused: Out of Resources | A700 | There is not enough local storage capacity to store the instance. |

4.2.2.4.2. Activity – Receive Echo Request

4.2.2.4.2.1. Description and Sequencing of Activities

The SCP AE is always available to respond to echo requests from any remote Application Entity.

4.2.2.4.2.2. Accepted Presentation Contexts

The SCP AE accepts associations using the following presentation contexts:

Table 4.2.2-9 – Acceptable Presentation Contexts for Receive Echo Request

| Presentation Context Table | | | | | |
|----------------------------|-------------------|----------------------------|---------------------|------|----------------------|
| Abstract Syntax | | Transfer Syntax | | Role | Extended Negotiation |
| Name | UID | Name | UID | | |
| Verification SOP Class | 1.2.840.10008.1.1 | Implicit VR, Little Endian | 1.2.840.10008.1.2 | SCP | None |
| | | Explicit VR, Little Endian | 1.2.840.10008.1.2.1 | SCP | None |

4.2.2.4.2.3. SOP Specific Conformance for Verification SOP Class

The SCP AE provides standard conformance to the Verification Service Class.

4.3. Network Interfaces

4.3.1. Physical Network Interface

The SecurView RT workstation is tested and supports using 100-BaseT and 1000-BaseT Ethernet media.

4.3.2. Additional Protocols

None.

4.4. Configuration

4.4.1. AE Title/Presentation Address Mapping

A service user configures the SecurView RT workstation's SCU AE and SCP AE, and remote SCP AEs during site installation, by entering the AE Title, IP Address, and TCP/IP port number for each. When configuring a remote Print AE, a predefined printer specific configuration is also selected. There are no dedicated ports associated with dedicated services.

4.4.2. Configuration Parameters

The following are some of the relevant configurable items for the local and remote AEs:

Table 4.4-1 – Configuration Parameters

| Parameter | Configurable (Yes/No) | Default Value |
|---|-----------------------|-------------------|
| Remote Print AE | | |
| Printer name (manufacturer, model, default film size) | Yes | Per known printer |
| Configuration info | Yes | Per known printer |
| Dmin | Yes | Per known printer |
| Dmax | Yes | Per known printer |
| Portrait matrix (n x m) | Yes | Per known printer |
| Landscape matrix (n x m) | Yes | Per known printer |
| Portrait pixel spacing (n x m) | Yes | Per known printer |
| Landscape pixel spacing (n x m) | Yes | Per known printer |

| Parameter | Configurable (Yes/No) | Default Value |
|--|-----------------------|-------------------|
| Bit depth (12 or 8) | Yes | Per known printer |
| Medium Type (blue film, clear film, mammo blue film, paper) | Yes | Per known printer |
| Medium Size (8inx10in ,10inx10in, 10inx12in, 10inx14in, 11inx14in, 14inx14in, 14inx17in) | Yes | Per known printer |
| Printer LUT | Yes | P_to_LIN_at_5000 |
| Print Mode (Standard, Presentation LUT) | Yes | Standard |
| Luminescence (Presentation LUT mode only) | Yes | 5000 |
| Reflected ambient light (Presentation LUT mode only) | Yes | 10 |
| Presentation LUT Shape (Presentation LUT mode only) | Yes | IDENTITY |
| General Parameters | | |
| Number of seconds to wait for association request or release | Yes | 30 |
| Number of seconds to wait for reply to association request | Yes | 15 |
| Number of seconds to wait for reply to association release | Yes | 15 |
| Maximum PDU size the local AE can receive | Yes | 28672 |
| Modules that Invoke SCU AE | | |
| Font size for patient info in diagnostic printing | Yes | 12 |
| Font size for all other text in diagnostic printing | Yes | 10 |
| Font size for annotations in screen capture printing | Yes | 18 |
| Relaxed error handling (for print) | Yes | On |
| Destination(s) for GSPS Notice instances | Yes | None |
| Number of automatic retries on store before reporting failure | Yes | 3 |
| Auto-Fetch Digital Mammography images (On/Off) | Yes | On |
| Auto-Fetch Hologic Screen Capture images (On/Off) | Yes | Off |
| Number of priors to auto-fetch | Yes | 1 |
| Number of years of priors to auto-fetch | Yes | 2 |
| Number of automatic retries on auto-fetch before reporting failure | No | 5 |
| Auto-Fetch Q/R Level (Study, Series, Image) | Yes | Series |
| Search on PACS Q/R Level (Study, Series, Image) | Yes | Series |
| Search on PACS sends optional Study level attributes (On/Off) | Yes | On |
| Search on PACS optional Study level attributes to send (On/Off) | Yes | Some On, some Off |
| Search on PACS sends optional Series level attributes (On/Off) | Yes | On |
| Search on PACS optional Series level attributes to send (On/Off) | Yes | Some On, some Off |
| Search on PACS sends optional Image level attributes (On/Off) | Yes | On |
| Search on PACS optional Image level attributes to send (On/Off) | Yes | Some On, some Off |
| (Search on PACS) Number of minutes to wait for C-FIND / C-MOVE response | Yes | 10 |
| (Auto-Fetch) Number of minutes to wait for C-MOVE response | Yes | 11 |
| SCP AE | | |
| Auto-Fetch trigger (On/Off) | Yes | Off |
| Received Mammography CAD SR triggers auto-fetch (On/Off) | Yes | Off |
| Received GSPS triggers auto-fetch (On/Off) | Yes | Off |

5. Media Interchange

5.1. Implementation Model

5.1.1. Application Data Flow Diagram

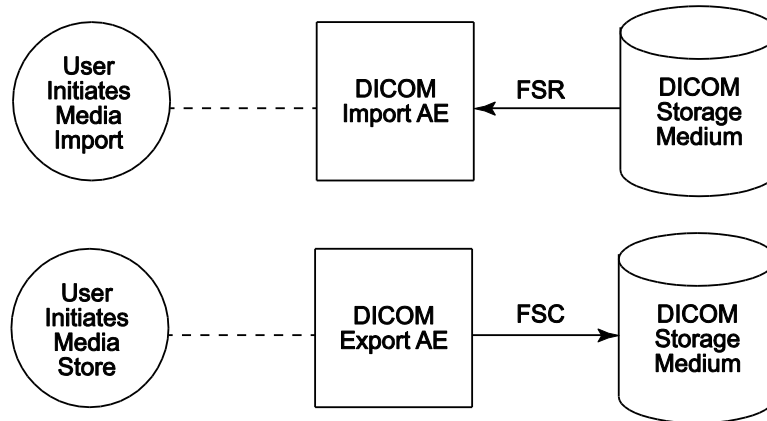


Figure 5.1-1: Application Data Flow Diagram for Media Storage

The DICOM Export AE and DICOM Import AE provide a means to export and import patient images using DICOM Part 10: Media Storage and File Format for Media Interchange. Patient images can be recorded to or read from removable media. These features are provided as a matter of convenience for users to move images from one system to another via removable media. The export feature is not intended to serve as an archive for permanently storing images.

5.1.2. Functional Definition of AEs

The DICOM Export AE can store patient images to removable media.

The DICOM Import AE can read patient images from removable media.

The SecurView RT workstation supports DICOM Media Storage as FSC and FSR.

5.1.3. Sequencing of Real-World Activities

The SecurView RT workstation Patient List provides a context menu item to export all images and related objects for selected patients to the selected removable media drive. The export is performed in the foreground, so the user must wait for completion (or cancel) before initiating another activity in the application.

The SecurView RT workstation Patient List has an Import button to select the drive from which to import all detected DICOM files from removable media. The DICOMDIR file is not read. The import is performed in the background, so the user may continue to use the application while the import is in progress (or cancel).

5.1.4. File Meta Information for Implementation Class Version

The implementation information written to the File Meta Header in each file is:

Table 5.1-1 – DICOM Implementation Class and Version for Media Storage

| | |
|-----------------------------|--|
| Implementation Class UID | 1.3.6.1.4.1.34261.273440546401752.16168.1486544684.0.1200020 |
| Implementation Version Name | MBC_SCR_1200020 |

5.2. AE Specifications

5.2.1. Media Server AE Specification

The DICOM Export AE provides standard conformance to the DICOM Interchange Option of the Media Storage Service Class (SC). The Application Profiles and roles are listed below:

Table 5.2-1 – AE Related Application Profiles, Real-World Activities, and Roles

| Supported Application Profile | Real-World Activity | Roles |
|-------------------------------|-----------------------------|----------|
| STD-GEN-CD | User Initiates Media Store | FSC |
| | User Initiates Media Import | FSR |
| STD-GEN-DVD-RAM | User Initiates Media Store | FSC |
| STD-GEN-DVD-JPEG | User Initiates Media Import | FSC, FSR |
| STD-GEN-USB | User Initiates Media Store | FSC |
| STD-GEN-USB-JPEG | User Initiates Media Import | FSC, FSR |

5.2.1.1. File Meta Information for Media Server AE

The Source Application Entity Title included in the File Meta Header is the local AE Title of the SecurView RT workstation.

5.2.1.2. Real-World Activities

5.2.1.2.1. Activity – User Initiates Media Store

The DICOM Export AE acts as an FSC when requested to export images to removable media.

To store images to removable media, the user selects one or more patients from the Patient List, and then selects the <Export to media> context menu item. All Image, Mammography CAD SR, and Grayscale Softcopy Presentation State Instances that are available locally for the selected patients are stored to the selected removable media drive. Images can be recorded to any removable media, provided the capacity is available to record all Instances for the selected patients.

5.2.1.2.1.1. Media Storage Application Profile

The DICOM Export AE supports the STD-GEN-CD Application Profile. An empty CD is required for each export. Multi-session writes are not supported.

The DICOM Export AE supports the STD-GEN-DVD-RAM and STD-GEN-DVD-JPEG Application Profiles. An empty DVD is required for each export. Only ISO 9660 is supported, without multi-session writes. UDF is not supported.

The DICOM Export AE supports the STD-GEN-USB and STD-GEN-USB-JPEG Application Profiles. If the user selects a folder other than the root for USB export, it is possible for more than one DICOMDIR file to be written to a USB.

5.2.1.2.1.1.1. Options

The DICOM Export AE supports the SOP Classes and Transfer Syntaxes listed in the table below:

Table 5.2-2 – SOP Classes and Transfer Syntaxes for Media Server

| SOP Class Name | SOP Class UID | Transfer Syntax | Transfer Syntax UID |
|--|-----------------------------|------------------------------|---------------------|
| Media Storage Directory Storage | 1.2.840.10008.1.3.10 | Explicit VR Little Endian | 1.2.840.10008.1.2.1 |
| Computed Radiography Image Storage | 1.2.840.10008.5.1.4.1.1.1 | See Table 5.2-3 | |
| Digital X-Ray Image Storage – For Presentation | 1.2.840.10008.5.1.4.1.1.1.1 | See Table 5.2-3 | |

| SOP Class Name | SOP Class UID | Transfer Syntax | Transfer Syntax UID |
|--|--------------------------------|---------------------------|---------------------|
| Digital Mammography X-Ray Image Storage – For Presentation | 1.2.840.10008.5.1.4.1.1.1.2 | See Table 5.2-3 | |
| Digital Mammography X-Ray Image Storage – For Processing | 1.2.840.10008.5.1.4.1.1.1.2.1 | See Table 5.2-3 | |
| Breast Tomosynthesis Image Storage | 1.2.840.10008.5.1.4.1.1.13.1.3 | See Table 5.2-3 | |
| Breast Projection X-Ray Image Storage – For Presentation | 1.2.840.10008.5.1.4.1.1.13.1.4 | See Table 5.2-3 | |
| Secondary Capture Image Storage | 1.2.840.10008.5.1.4.1.1.7 | See Table 5.2-3 | |
| Multi-frame Grayscale Byte Secondary Capture Image Storage | 1.2.840.10008.5.1.4.1.1.7.2 | See Table 5.2-3 | |
| Multi-frame Grayscale Word Secondary Capture Image Storage | 1.2.840.10008.5.1.4.1.1.7.3 | See Table 5.2-3 | |
| Multi-frame True Color Secondary Capture Image Storage | 1.2.840.10008.5.1.4.1.1.7.4 | See Table 5.2-3 | |
| Ultrasound Image Storage | 1.2.840.10008.5.1.4.1.1.6.1 | See Table 5.2-3 | |
| Ultrasound Multi-frame Image Storage | 1.2.840.10008.5.1.4.1.1.3.1 | See Table 5.2-3 | |
| Ultrasound Image Storage (Retired) | 1.2.840.10008.5.1.4.1.1.6 | See Table 5.2-3 | |
| Ultrasound Multi-frame Image Storage (Retired) | 1.2.840.10008.5.1.4.1.1.3 | See Table 5.2-3 | |
| CT Image Storage | 1.2.840.10008.5.1.5.1.1.2 | See Table 5.2-3 | |
| Mammography CAD SR Storage | 1.2.840.10008.5.1.4.1.1.88.50 | Explicit VR Little Endian | 1.2.840.10008.1.2.1 |
| Grayscale Softcopy Presentation State Storage | 1.2.840.10008.5.1.4.1.1.11.1 | Explicit VR Little Endian | 1.2.840.10008.1.2.1 |

Table 5.2-3 – Transfer Syntaxes for Media Server

| Transfer Syntax Table | | |
|----------------------------|---|------------------------|
| Photometric Interpretation | Name | UID |
| All other values (color) | Explicit VR Little Endian | 1.2.840.10008.1.2.1 |
| MONOCHROME1 or MONOCHROME2 | JPEG Lossless, Non-Hierarchical, First-Order Prediction (Process 14 [Selection Value 1]): Default Transfer Syntax for Lossless JPEG Image Compression | 1.2.840.10008.1.2.4.70 |

5.2.1.2.2. Activity – User Initiates Media Import

The DICOM Import AE acts as an FSR when requested to import images from removable media.

To load images from removable media the user selects the Import button on the Patient List, and then selects a removable media drive. All Image, Mammography CAD SR, and Grayscale Softcopy Presentation State Instances that are detected on the selected removable media drive as DICOM Part 10 files are read and made available to the application except Breast Projection X-Ray Image – For Processing, which if configured to be accepted are discarded.

5.2.1.2.2.1. Media Storage Application Profile

The DICOM Import AE supports the STD-GEN-CD, STD-GEN-DVD-JPEG and STD-GEN-USB-JPEG Application Profiles.

5.2.1.2.2.1.1. Options

The DICOM Import AE supports the SOP Classes and Transfer Syntaxes listed in the table below:

Table 5.2-4 – SOP Classes and Transfer Syntaxes for Media Server

| SOP Class Name | SOP Class UID | Transfer Syntax | Transfer Syntax UID |
|--|--------------------------------|------------------------------|---------------------|
| Media Storage Directory Storage | 1.2.840.10008.1.3.10 | Explicit VR Little Endian | 1.2.840.10008.1.2.1 |
| Computed Radiography Image Storage | 1.2.840.10008.5.1.4.1.1.1 | See Table 4.2.2-6 | |
| Digital X-Ray Image Storage – For Presentation | 1.2.840.10008.5.1.4.1.1.1.1 | See Table 4.2.2-6 | |
| Digital Mammography X-Ray Image Storage – For Presentation | 1.2.840.10008.5.1.4.1.1.1.2 | See Table 4.2.2-6 | |
| Digital Mammography X-Ray Image Storage – For Processing | 1.2.840.10008.5.1.4.1.1.1.2.1 | See Table 4.2.2-6 | |
| Breast Tomosynthesis Image Storage | 1.2.840.10008.5.1.4.1.1.13.1.3 | See Table 4.2.2-6 | |
| Breast Projection X-Ray Image Storage – For Presentation | 1.2.840.10008.5.1.4.1.1.13.1.4 | See Table 4.2.2-6 | |
| Breast Projection X-Ray Image Storage – For Processing | 1.2.840.10008.5.1.4.1.1.13.1.5 | See Table 4.2.2-6 | |
| Secondary Capture Image Storage | 1.2.840.10008.5.1.4.1.1.7 | See Table 4.2.2-6 | |
| Multi-frame Grayscale Byte Secondary Capture Image Storage | 1.2.840.10008.5.1.4.1.1.7.2 | See Table 4.2.2-6 | |
| Multi-frame Grayscale Word Secondary Capture Image Storage | 1.2.840.10008.5.1.4.1.1.7.3 | See Table 4.2.2-6 | |
| Multi-frame True Color Secondary Capture Image Storage | 1.2.840.10008.5.1.4.1.1.7.4 | See Table 4.2.2-6 | |
| Ultrasound Image Storage | 1.2.840.10008.5.1.4.1.1.6.1 | See Table 4.2.2-7 | |
| Ultrasound Multi-frame Image Storage | 1.2.840.10008.5.1.4.1.1.3.1 | See Table 4.2.2-7 | |
| Ultrasound Image Storage (Retired) | 1.2.840.10008.5.1.4.1.1.6 | See Table 4.2.2-7 | |
| Ultrasound Multi-frame Image Storage (Retired) | 1.2.840.10008.5.1.4.1.1.3 | See Table 4.2.2-7 | |
| CT Image Storage | 1.2.840.10008.5.1.4.1.1.2 | See Table 4.2.2-6 | |
| Mammography CAD SR Storage | 1.2.840.10008.5.1.4.1.1.88.50 | Explicit VR Little Endian | 1.2.840.10008.1.2.1 |
| Grayscale Softcopy Presentation State Storage | 1.2.840.10008.5.1.4.1.1.11.1 | Explicit VR Little Endian | 1.2.840.10008.1.2.1 |

5.3. Augmented and Private Application Profiles

None.

5.4. Media Configuration

None. The Source AE Title used by the DICOM Export AE is 'MBC SCR'.

6. Support of Character Sets

The SecurView RT workstation supports the following character sets:

- Default: ISO-IR 6
- Latin Alphabet No. 1: ISO-IR 100
- Unicode in UTF-8: ISO IR 192

7. Security

The SecurView RT workstation does not support any specific DICOM security measures.

The SecurView RT workstation is expected to be used within a secured environment. For additional information about security policies, refer to the SecurView Cybersecurity Product Report.

8. Annexes

8.1. IOD Contents

8.1.1. Created SOP Instance(s)

Abbreviations for Presence of Value:

| | |
|--------|---|
| ALWAYS | Attribute always present with a value |
| ANAP | Attribute not always present |
| EMPTY | Attribute is sent without a value (zero length) |
| VNAP | Attribute value not always present (zero length if no value is present) |

Abbreviations for Source:

| | |
|--------|---|
| AUTO | Attribute value is generated automatically |
| CONFIG | Attribute value source is a configurable parameter |
| MWL | Attribute value is the same as the value received using Modality Worklist |
| USER | Attribute value source is from User input |

8.1.1.1. Grayscale Softcopy Presentation State IOD

The attributes that are contained in created GSPS Instances are listed in the following tables.

Table 8.1.1-1 – IOD of Created GSPS SOP Instances

| IE | Module | Reference | Presence of Module |
|--------------------|---------------------------|----------------|--------------------|
| Patient | Patient | Table 8.1.1-2 | ALWAYS |
| Study | General Study | Table 8.1.1-3 | ALWAYS |
| | Patient Study | Table 8.1.1-4 | ALWAYS |
| Series | General Series | Table 8.1.1-5 | ALWAYS |
| | Presentation Series | Table 8.1.1-5 | ALWAYS |
| Equipment | General Equipment | Table 8.1.1-6 | ALWAYS |
| Presentation State | Presentation State | Table 8.1.1-7 | ALWAYS |
| | Displayed Area | Table 8.1.1-8 | ALWAYS |
| | Graphic Annotation | Table 8.1.1-9 | ANAP |
| | Graphic Layer | Table 8.1.1-10 | ANAP |
| | Softcopy VOI LUT | Table 8.1.1-11 | ALWAYS |
| | Softcopy Presentation LUT | Table 8.1.1-12 | ALWAYS |
| | SOP Common | Table 8.1.1-13 | ALWAYS |

8.1.1.2. Grayscale Softcopy Presentation State Modules

Table 8.1.1-2 – Patient Module of Created GSPS Instances

| Attribute Name | Tag | Value | Presence of Value | Source |
|------------------------------|-------------|-----------------------------|-------------------|--------|
| Patient's Name | (0010,0010) | Copied from current patient | ALWAYS | AUTO |
| Patient ID | (0010,0020) | Copied from current patient | ALWAYS | AUTO |
| Patient's Birth Date | (0010,0030) | Copied from current patient | VNAP | AUTO |
| Patient's Sex | (0010,0040) | Copied from current patient | VNAP | AUTO |
| All other Patient Attributes | | Copied from current patient | ANAP | AUTO |

Table 8.1.1-3 – General Study Module of Created GSPS Instances

| Attribute Name | Tag | Value | Presence of Value | Source |
|----------------------------|-------------|---------------------------|-------------------|--------|
| Study Date | (0008,0020) | Copied from current study | ALWAYS | AUTO |
| Study Time | (0008,0030) | Copied from current study | ALWAYS | AUTO |
| Accession Number | (0008,0050) | Copied from current study | VNAP | AUTO |
| Referring Physician's Name | (0008,0090) | Copied from current study | VNAP | AUTO |
| Study Description | (0008,1030) | Copied from current study | ANAP | AUTO |
| Study Instance UID | (0020,000D) | Copied from current study | ALWAYS | AUTO |
| Study ID | (0020,0010) | Copied from current study | VNAP | AUTO |
| All other Study Attributes | | Copied from current study | ANAP | AUTO |

Table 8.1.1-4 – Patient Study Module of Created GSPS Instances

| Attribute Name | Tag | Value | Presence of Value | Source |
|------------------------------------|-------------|--------------------------------------|-------------------|--------|
| Patient's Age | (0010,1010) | Copied from current patient | ANAP | AUTO |
| All other Patient Study Attributes | | Copied from current patient or study | ANAP | AUTO |

Table 8.1.1-5 – General & Presentation Series Modules of Created GSPS Instances

| Attribute Name | Tag | Value | Presence of Value | Source |
|---------------------|-------------|-------------------------------|-------------------|--------|
| Modality | (0008,0060) | PR | ALWAYS | AUTO |
| Series Instance UID | (0020,000E) | Unique worldwide | ALWAYS | AUTO |
| Series Number | (0020,0011) | 1 | ALWAYS | AUTO |
| Operators' Name | (0008,1070) | Copied from referenced series | ANAP | AUTO |
| Series Description | (0008,103E) | SecurView GSPS | ALWAYS | AUTO |

Table 8.1.1-6 – General Equipment Module of Created GSPS Instances

| Attribute Name | Tag | Value | Presence of Value | Source |
|---------------------------|-------------|------------------------------|-------------------|--------|
| Manufacturer | (0008,0070) | HOLOGIC, Inc. | ALWAYS | AUTO |
| Institution Name | (0008,0080) | | ALWAYS | CONFIG |
| Institution Address | (0008,0081) | | ANAP | CONFIG |
| Station Name | (0008,1010) | Local SCU AE Title | ALWAYS | CONFIG |
| Manufacturer's Model Name | (0008,1090) | SecurView | ALWAYS | AUTO |
| Software Versions | (0018,1020) | The current software version | ALWAYS | AUTO |

Table 8.1.1-7 – Presentation State Module of Created GSPS Instances

| Attribute Name | Tag | Value | Presence of Value | Source |
|-------------------------------|-------------|---|-------------------|--------|
| Referenced Series Sequence | (0008,1115) | One Item per series on which markings, annotations, measurements were made for the Notice | ALWAYS | AUTO |
| >Series Instance UID | (0020,000E) | | ALWAYS | AUTO |
| >Referenced Image Sequence | (0008,1140) | One Item per image in referenced series | ALWAYS | AUTO |
| >>Referenced SOP Class UID | (0008,1150) | | ALWAYS | AUTO |
| >>Referenced SOP Instance UID | (0008,1155) | | ALWAYS | AUTO |
| Instance Number | (0020,0013) | 1 | ALWAYS | AUTO |
| Content Label | (0070,0080) | HOLOGIC NOTE | ALWAYS | AUTO |
| Content Description | (0070,0081) | Username, Date, Time | ALWAYS | AUTO |
| Presentation Creation Date | (0070,0082) | | ALWAYS | AUTO |
| Presentation Creation Time | (0070,0083) | | ALWAYS | AUTO |
| Content Creator's Name | (0070,0084) | Current username | ALWAYS | AUTO |

Table 8.1.1-8 – Displayed Area Module of Created GSPS Instances

| Attribute Name | Tag | Value | Presence of Value | Source |
|--|-------------|--|-------------------|--------|
| Displayed Area Selection Sequence | (0070,005A) | One Item per image in Referenced Series Sequence | ALWAYS | AUTO |
| >Referenced Image Sequence | (0008,1140) | One Item per image in referenced series | ALWAYS | AUTO |
| >>Referenced SOP Class UID | (0008,1150) | | ALWAYS | AUTO |
| >>Referenced SOP Instance UID | (0008,1155) | | ALWAYS | AUTO |
| >Displayed Area Top Left Hand Corner | (0070,0052) | 0\0 | ALWAYS | AUTO |
| >Displayed Area Bottom Right Hand Corner | (0070,0053) | Columns\Rows | ALWAYS | AUTO |
| >Presentation Size Mode | (0070,0100) | SCALE TO FIT | ALWAYS | AUTO |
| >Presentation Pixel Aspect Ratio | (0070,0102) | 1\1 | ALWAYS | AUTO |

Table 8.1.1-9 – Graphic Annotation Module of Created GSPS Instances

| Attribute Name | Tag | Value | Presence of Value | Source |
|---------------------------------|-------------|---|-------------------|--------|
| Graphic Annotation Sequence | (0070,0001) | Zero or more Items | ANAP | USER |
| >Referenced Image Sequence | (0008,1140) | One Item | ALWAYS | AUTO |
| >>Referenced SOP Class UID | (0008,1150) | | ALWAYS | AUTO |
| >>>Referenced SOP Instance UID | (0008,1155) | | ALWAYS | AUTO |
| >>Referenced Frame Number | (0008,1160) | Only if referenced SOP Class is multi-frame | ANAP | AUTO |
| >Graphic Layer | (0070,0002) | LAYER0 | ALWAYS | AUTO |
| >Text Object Sequence | (0070,0008) | Zero or more Items | ANAP | USER |
| >>Anchor Point Annotation Units | (0070,0004) | PIXEL | ALWAYS | AUTO |
| >>Unformatted Text Value | (0070,0006) | | ALWAYS | USER |
| >>Anchor Point | (0070,0014) | | ALWAYS | AUTO |
| >>Anchor Point Visibility | (0070,0015) | N | ALWAYS | AUTO |
| >Graphic Object Sequence | (0070,0009) | Zero or more Items | ANAP | USER |
| >>Graphic Annotation Units | (0070,0005) | PIXEL | ALWAYS | AUTO |
| >>Graphic Dimensions | (0070,0020) | 2 | ALWAYS | AUTO |
| >>Number of Graphic Points | (0070,0021) | Depends on Graphic Type | ALWAYS | AUTO |
| >>Graphic Data | (0070,0022) | Depends on Graphic Type | ALWAYS | USER |
| >>Graphic Type | (0070,0023) | ELLIPSE or POLYLINE | ALWAYS | USER |
| >>Graphic Filled | (0070,0024) | N | ALWAYS | AUTO |

Table 8.1.1-10 – Graphic Layer Module of Created GSPS Instances

| Attribute Name | Tag | Value | Presence of Value | Source |
|--|-------------|---|-------------------|--------|
| Graphic Layer Sequence | (0070,0060) | One Item, if Graphic Annotation Sequence is present | ANAP | AUTO |
| >Graphic Layer | (0070,0002) | LAYER0 | ALWAYS | AUTO |
| >Graphic Layer Order | (0070,0062) | 1 | ALWAYS | AUTO |
| >Graphic Layer Recommended Display Grayscale Value | (0070,0066) | FFFFH | ALWAYS | AUTO |

Table 8.1.1-11 – Softcopy VOI LUT Module of Created GSPS Instances

| Attribute Name | Tag | Value | Presence of Value | Source |
|-------------------------------|-------------|--|-------------------|--------|
| Softcopy VOI LUT Sequence | (0028,3110) | One Item per image in Referenced Series Sequence. Item will contain either one VOI LUT Sequence Item or one pair of WC/WW values | ANAP | AUTO |
| >Referenced Image Sequence | (0008,1140) | One Item per image in referenced series | ALWAYS | AUTO |
| >>Referenced SOP Class UID | (0008,1150) | | ALWAYS | AUTO |
| >>Referenced SOP Instance UID | (0008,1155) | | ALWAYS | AUTO |
| >Window Center | (0028,1050) | Single value, if present | ANAP | USER |
| >Window Width | (0028,1051) | Single value, if present | ANAP | USER |
| >VOI LUT Function | (0028,1056) | LINEAR or SIGMOID, if present | ANAP | USER |
| >VOI LUT Sequence | (0028,3010) | One Item, if present | ANAP | USER |
| >>LUT Descriptor | (0028,3002) | | ANAP | USER |
| >>LUT Data | (0028,3006) | | ANAP | USER |

Table 8.1.1-12 – Softcopy Presentation LUT Module of Created GSPS Instances

| Attribute Name | Tag | Value | Presence of Value | Source |
|------------------------|-------------|---------------------|-------------------|--------|
| Presentation LUT Shape | (2050,0020) | IDENTITY or INVERSE | ALWAYS | AUTO |

Table 8.1.1-13 – SOP Common Module of Created GSPS Instances

| Attribute Name | Tag | Value | Presence of Value | Source |
|------------------------|-------------|--|-------------------|--------|
| Specific Character Set | (0008,0005) | Copied from referenced images, or ISO_IR 192 if GSPS specific string attribute characters are outside the referenced character set | ALWAYS | AUTO |
| Instance Creation Date | (0008,0012) | Current date | ALWAYS | AUTO |
| Instance Creation Time | (0008,0013) | Current time | ALWAYS | AUTO |
| SOP Class UID | (0008,0016) | Grayscale Softcopy Presentation State Storage | ALWAYS | AUTO |
| SOP Instance UID | (0008,0018) | Unique worldwide | ALWAYS | AUTO |

8.1.2. Usage of Attributes from Received IODs

Attributes with incorrect values may cause undesirable behaviors, or disable certain SecurView RT workstation functionality. In most instances, it is not possible for the SecurView RT workstation software to detect incorrect attribute values.

The following attributes and values are expected for successful interpretation of received SOP Instances.

Table 8.1.2-1 – Expected Attributes in All Received Instances

| Attribute Name | Tag | Expected Value |
|----------------------|-------------|---|
| SOP Class UID | (0008,0016) | Matches the Pixel Data |
| SOP Instance UID | (0008,0018) | Unique worldwide |
| Study Date | (0008,0020) | Non-zero length |
| Modality | (0008,0060) | Matches the SOP Class UID when enumerated values are expected |
| Patient's Name | (0010,0010) | Non-zero length |
| Patient ID | (0010,0020) | Non-zero length |
| Patient's Birth Date | (0010,0030) | Zero length accepted |
| Study Instance UID | (0020,000D) | Unique worldwide |
| Series Instance UID | (0020,000E) | Unique worldwide |

Table 8.1.2-2 – Expected Attributes in All Received Image Instances

| Attribute Name | Tag | Expected Value |
|--------------------------------|-------------|--|
| Recommended Display Frame Rate | (0008,2144) | Used to determine cine speed for ultrasound images, if present |
| Cine Rate | (0018,0040) | Used to determine cine speed for ultrasound images, if present |
| Protocol Name | (0018,1030) | Used in image overlay for ultrasound images, if present |
| Frame Time | (0018,1063) | Used to determine cine speed for ultrasound images, if present |
| Frame Time Vector | (0018,1065) | Used to determine cine speed for ultrasound images, if present |
| Sequence of Ultrasound Regions | (0018,6011) | One or more Items required to convert measurement units to cm for ultrasound images |
| Patient Orientation | (0020,0020) | Required for automatic hanging of mammography images |
| Laterality | (0020,0060) | If the anatomy is a paired body part, required for automatic hanging: 'L', 'R', or 'B' |
| Image Laterality | (0020,0062) | |
| Samples per Pixel | (0028,0002) | 1 or 3 |
| Photometric Interpretation | (0028,0004) | MONOCHROME1, MONOCHROME2, RGB, PALETTE COLOR, YBR_FULL, YBR_FULL_422, or YBR_PARTIAL_422 |
| Planar Configuration | (0028,0006) | Expected with Samples per Pixel > 1 |
| Number of Frames | (0028,0008) | Expected in multi-frame images |
| Frame Increment Pointer | (0028,0009) | Identified attribute used to determine cine speed for ultrasound images, if present |
| Rows | (0028,0010) | Matches the Pixel Data |
| Columns | (0028,0011) | Matches the Pixel Data |
| Pixel Spacing | (0028,0030) | Required to convert measurement units to mm for mammography images, if Imager Pixel Spacing (0018,1164) is not present |
| Bits Allocated | (0028,0100) | Up to 16, matches Pixel Data |

| Attribute Name | Tag | Expected Value |
|------------------------------|-------------|---|
| Bits Stored | (0028,0101) | Up to 16, matches Pixel Data |
| View Code Sequence | (0054,0220) | Required for automatic hanging of mammography images |
| >View Modifier Code Sequence | (0054,0222) | Values factored into automatic hanging of mammography images when present |
| Overlay Rows | (6000,0010) | If present must be same as Rows (0028,0010) value |
| Overlay Columns | (6000,0011) | If present must be same as Columns (0028,0011) value |
| Overlay Type | (6000,0040) | If present only G (Graphics) is supported |
| Overlay Origin | (6000,0050) | If present only 1\1 is supported |
| Overlay Bits Allocated | (6000,0100) | If present only 1 is permitted |
| Overlay Bit Position | (6000,0102) | If present only 0 is permitted |
| Overlay Data | (6000,3000) | If present must have Rows*Columns pixels |
| Pixel Data | (7FE0,0010) | Rows*Columns*(Bits Allocated / 8)*Samples per Pixel |

Table 8.1.2-3 – Expected Attributes in Digital Mammography X-Ray Images

| Attribute Name | Tag | Expected Value |
|---|-------------|--|
| Presentation Intent Type | (0008,0068) | FOR PRESENTATION or FOR PROCESSING |
| Manufacturer | (0008,0070) | Manufacturer of the equipment used to create the image |
| Source Image Sequence | (0008,2112) | FOR PRESENTATION images only, to match with corresponding Mammography CAD SR |
| >Referenced SOP Class UID | (0008,1150) | Digital Mammography X-Ray Image – For Processing |
| >Referenced SOP Instance UID | (0008,1155) | SOP Instance UID |
| Estimated Radiographic Magnification Factor | (0018,1114) | Used in sizing images for display and to convert measurement units to mm |
| Imager Pixel Spacing | (0018,1164) | Used in sizing images for display and to convert measurement units to mm |
| Pixel Padding Value | (0028,0120) | Used to maintain black background during W/L interaction |
| Pixel Padding Range Limit | (0028,0121) | Used to maintain black background during W/L interaction |
| Window Center | (0028,1050) | Valid values if present |
| Window Width | (0028,1051) | Valid values if present |
| VOI LUT Function | (0028,1052) | Used to interpret WC/WC values; default to LINEAR if not present |
| Rescale Intercept | (0028,1052) | 0 |
| Rescale Slope | (0028,1053) | 1 |
| Rescale Type | (0028,1054) | US |
| VOI LUT Sequence | (0028,3010) | Valid values in Items if present |

Note: In addition, all other attributes that the IHE Radiology Mammography Image profile requires to be made available for display, if present.

Table 8.1.2-4 – Expected Attributes in Hologic Selenia Digital Mammography X-Ray Images

| Attribute Name | Tag | Expected Value |
|--------------------|-------------|----------------------------|
| Manufacturer | (0008,0070) | 'LORAD' or 'HOLOGIC, Inc.' |
| Private Attributes | (0019,xxxx) | Creator ID = LORAD Selenia |

Table 8.1.2-5 – Expected Attributes in Mammography CAD SR Instances

| Attribute Name | Tag | Expected Value |
|-------------------|-------------|---|
| Manufacturer | (0008,0070) | 'HOLOGIC, Inc.', 'R2 Technology, Inc.', 'CADx', 'iCAD, Inc.': to display appropriate logo and to display CAD marks according to manufacturer preference |
| Software Versions | (0018,1020) | Non-zero length value |

Table 8.1.2-6 – Supported Content Items in Any Mammography CAD SR Instance

| Template | Rows | Expected Value |
|--|--------------------|---|
| TID 4000, Mammography CAD Document Root | 1-9 | All mandatory content items |
| TID 4001, Mammography CAD Overall Impression/Recommendation | 1-3 | All mandatory content items |
| TID 4002, Mammography CAD Impression/Recommendation Body | 5, 10, 11 | |
| TID 4003, Mammography CAD Individual Impression/Recommendation | 1, 2, 4, 5 | All mandatory content items |
| TID 4004, Mammography CAD Composite Feature | 1, 2, 4, 6 | |
| TID 4005, Mammography CAD Composite Feature Body | 1-4, 10 | |
| TID 4006, Mammography CAD Single Image Finding | 1-2, 5-6, 8, 22-25 | |
| TID 4015, CAD Detections Performed | 1-4 | All mandatory content items |
| TID 4016, CAD Analyses Performed | 1-4 | All mandatory content items |
| TID 4017, CAD Detection Performed | 1, 2, 4 | |
| TID 4018, CAD Analysis Performed | 1, 2, 4 | |
| TID 4019, CAD Algorithm Performed | 1-3 | |
| TID 4020, CAD Image Library Entry | 1-12, 16-17 | All mandatory content items |
| TID 4021, Mammography CAD Geometry | 1-4 | Row 3, Outline: POLYLINE, CIRCLE, ELLIPSE |

Note: Additional Mammography CAD SR Content Items are supported from the Hologic Cenova server and Selenia Dimensions Acquisition Workstation. For additional information, refer to the respective DICOM Conformance Statements.

Table 8.1.2-7 – Expected Attributes in Grayscale Softcopy Presentation State Instances

| Attribute Name | Tag | Expected Value |
|--|-------------|--|
| Referenced Series Sequence | (0008,1115) | One or more Items |
| Graphic Annotation Sequence | (0070,0001) | One or more Items, otherwise GSPS is discarded |
| >Referenced Image Sequence | (0008,1140) | One or more Items if annotations do not apply to all referenced images |
| >Graphic Layer | (0070,0002) | Ignored, all graphic and text objects are displayed at once |
| >Text Object Sequence | (0070,0008) | One or more items in Text Object Sequence and/or Graphic Object Sequence |
| >> Bounding Box Annotation Units | (0070,0003) | PIXEL (DISPLAY is not supported) |
| >>Anchor Point Annotation Units | (0070,0004) | PIXEL (DISPLAY is not supported) |
| >>Unformatted Text Value | (0070,0006) | Text is interpreted as ISO-IR 100 |
| >>Bounding Box Top Left Hand Corner | (0070,0010) | |
| >>Bounding Box Bottom Right Hand Corner | (0070,0011) | |
| >>Bounding Box Text Horizontal Justification | (0070,0012) | Text will be LEFT aligned (RIGHT and CENTER are not supported) |
| >>Anchor Point | (0070,0014) | |
| >>Anchor Point Visibility | (0070,0015) | N (Y is not supported) |
| >Graphic Object Sequence | (0070,0009) | One or more items in Text Object Sequence and/or Graphic Object Sequence |
| >>Graphic Annotation Units | (0070,0005) | PIXEL (DISPLAY is not supported) |
| >>Graphic Dimensions | (0070,0020) | 2 |
| >>Number of Graphic Points | (0070,0021) | According to Graphic Type |
| >>Graphic Data | (0070,0022) | According to Graphic Type |
| >>Graphic Type | (0070,0023) | POINT, POLYLINE, CIRCLE, ELLIPSE is displayed with enclosing circle if major and minor axes are not horizontal and vertical INTERPOLATED is displayed the same as POLYLINE |
| >>Graphic Filled | (0070,0024) | If Y, then the filling is grayscale value 160 from the range 0..255 |
| Presentation Creation Date | (0070,0082) | Used to identify annotations on display |
| Presentation Creation Time | (0070,0083) | Used to identify annotations on display |
| Content Creator's Name | (0070,0084) | Used to identify annotations on display |

8.1.3. Attribute Mapping

Not applicable.

8.1.4. Coerced/Modified Attributes

Regarding received Instances, the SecurView RT workstation will attempt to fill in missing attributes and correct invalid attribute values that are required for internal use. All Patient and Study attributes and Referenced SOP Instance UUIDs that are copied from received instances to create new instances use the unmodified values as they were received.

If the Referenced SOP Instance UUID (0008,1155) within a Source Image Sequence (0008,2112) Item of a Hologic Selenia Digital Mammography – For Presentation image is detected to be incorrect, it is corrected for use within the SecurView RT workstation.

8.2. Data Dictionary of Private Attributes

Private Attributes contained in created Grayscale Softcopy Presentation State SOP Instances do not include patient identification information.

8.3. Coded Terminology and Templates

Not applicable.

8.4. Grayscale Image Consistency

Display calibrations are performed according to the DICOM Grayscale Standard Display Function.

8.5. Standard Extended/Specialized/Private SOP Classes

None.

8.6. Private Transfer Syntaxes

None.