



## SecurView® DX/RT Workstation User Guide

**Software Version 9.0**

MAN-04427 Revision 001

SecurView®  
Breast Imaging Workstation



**HOLOGIC®**



---

**SecurView® DX/RT 9.0 Workstation**  
User Guide

---

MAN-04427 Rev 001

# HOLOGIC®

Copyright © 2015, Hologic Inc. All rights reserved. Duplication or distribution without written permission is prohibited. Issued May 2015.

Patents: <http://hologic.com/patents>

Hologic, the Hologic logo, BACS, Citra, C-View, EmphaSize, ImageChecker, Mammography Prior Enhancement, MultiView, PeerView, RightOn, Quantra, SecurView, Selenia, and TechMate are trademarks or registered trademarks of Hologic and/or its subsidiaries in the United States and/or other countries. All other trademarks, registered trademarks, and product names are the property of their respective owners.



**Hologic Inc.**  
35 Crosby Drive  
Bedford, MA 01730-1401 USA  
Tel: +1.781.999.7300  
Sales: +1.781.999.7453  
Fax: +1.781.280.0668



**Hologic N.V.**  
(EU Representative)  
Leuvensesteenweg 250A  
1800 Vilvoorde, Belgium  
Tel: +32.2.711.4680  
Fax: +32.2.725.2087

For more information about  
Hologic products and services,  
visit [www.Hologic.com](http://www.Hologic.com).

**CE 0044**



Softcopy reading software © 2002–2015 MeVis Medical Solutions AG.

All rights reserved. This product and related documentation are protected by copyright and are distributed under licenses restricting their use, copying, distribution, and decompilation. No part of this product or related documentation may be reproduced in any form by any means without prior written authorization of MeVis Medical Solutions AG and its licensors, if any.

FlowBack, FlowNext, MammoNavigator, and ReportFlow are trademarks of MeVis BreastCare GmbH & Co. KG.

This product may be protected by one or more of the following patents: 7,283,857, 6,891,920.

## Libraries

Libtiff library © 1988-1997 Sam Leffler, 1991-1997 Silicon Graphics, Inc.

Jpeglib: This software is based in part on the work of the independent JPEG Group

OFFIS\_DCMTK © 1994–2005, OFFIS

Xerces © 1999–2010 The Apache Software Foundation. All rights reserved.



---

## Contents

<b>Chapter 1: Introduction .....</b>	<b>9</b>
1.1. Overview .....	10
1.2. Intended Use .....	11
1.2.1. SecurView DX Intended Use.....	11
1.2.2. SecurView RT Intended Use .....	11
1.3. Using This Guide .....	11
1.4. Resources Available.....	12
1.5. Warnings and Precautions .....	13
1.5.1. System Operation .....	13
1.5.2. Installation and Maintenance.....	14
1.6. Product Complaints .....	15
<b>Chapter 2: Workstation Description .....</b>	<b>17</b>
2.1. Workstation Overview .....	18
2.2. SecurView DX Diagnostic Workstation.....	19
2.2.1. SecurView DX Standalone Systems.....	20
2.2.2. SecurView DX Multiworkstation Systems.....	21
2.3. SecurView RT Technologist Workstation .....	23
2.3.1. SecurView RT Standalone Systems.....	24
2.3.2. SecurView RT Multiworkstation Systems .....	25
2.4. Functional Division in Multiworkstation Configurations .....	27
2.5. User Groups and Passwords .....	28
2.6. Startup and Shutdown.....	29
2.7. Logging Into SecurView .....	30
<b>Chapter 3: Patient Manager .....</b>	<b>31</b>
3.1. Opening the Patient Manager .....	32
3.2. Using the Patient List .....	33
3.2.1. Selecting Patients .....	33
3.2.2. Patient List Buttons .....	34
3.2.3. Patient List Columns .....	35
3.2.4. Reading States.....	37
3.2.5. Auto-Fetching Patient Data .....	38
3.2.6. Using the Shortcut Menu .....	38
3.2.7. Merging Patient Data .....	39

3.2.8. Searching for Patients .....	40
3.3. Creating Sessions .....	42
3.4. Importing DICOM Images.....	43
3.5. Synchronizing Patient List with MultiView.....	43
<b>Chapter 4: Reviewing Patients .....</b>	<b>45</b>
4.1. Displaying Patient Studies.....	46
4.1.1. Patient List Worklists.....	46
4.1.2. Automatically Generated Worklists.....	47
4.1.3. Session Worklists .....	47
4.1.4. MG Viewer .....	48
4.2. Displaying Patient Images.....	50
4.2.1. Navigating Patients .....	50
4.2.2. Using the Keypad .....	51
4.2.3. Using the Pie Menu .....	52
4.2.4. Using a ReportFlow .....	54
4.2.5. Patient Reading and Lock States During Review .....	54
4.2.6. Panning Images.....	55
4.2.7. Image Hangings.....	56
4.2.8. Intelligent Roaming.....	57
4.2.9. Scaling Modes .....	59
4.2.10. Pixel Meter .....	60
4.2.11. Stack and Timepoint Indicators.....	61
4.2.12. MammoNavigator.....	61
4.2.13. Image Information .....	63
4.2.14. Patient Information Overlays .....	64
4.2.15. MG Secondary Captures and MM ScreenCaptures.....	65
4.3. Visualizing Image Details .....	65
4.3.1. Magnifier and Inverted Magnifier.....	68
4.3.2. AIE and the Magnifier Toolbar .....	69
4.3.3. Continuous Zoom.....	70
4.3.4. Window/Level and Gamma Adjustments.....	71
4.3.5. Applying VOI LUTs .....	72
4.3.6. MPE Images .....	72
4.3.7. DICOM 6000 Overlays.....	74
4.3.8. CLAHE Image Enhancement .....	75
4.4. Using CAD .....	75

4.4.1.	Displaying CAD Information.....	75
4.4.2.	Hologic ImageChecker CAD.....	76
4.4.3.	Hologic Imaging Biomarkers.....	79
4.4.4.	Switching Between Multiple Mammography CAD SRs .....	81
4.5.	Creating and Viewing Annotations.....	82
4.5.1.	Marking an Image.....	82
4.5.2.	Describing a Region of Interest .....	84
4.5.3.	Viewing Annotations .....	85
4.6.	Sending and Viewing Notices .....	87
4.6.1.	Sending Notices.....	87
4.6.2.	Viewing Notices .....	88
4.7.	Closing a Study .....	89
4.7.1.	Closing a Study as a Radiologist.....	89
4.7.2.	Closing a Study as a Technologist.....	90
4.8.	Printing Options.....	92
4.9.	Patient Synchronization with an External Application.....	94
4.9.1.	Manual Synchronization .....	94
4.9.2.	Automatic Synchronization.....	94
4.9.3.	Synchronization When You Receive a Message.....	95
<b>Chapter 5:</b>	<b>Working with Tomosynthesis Images .....</b>	<b>97</b>
5.1.	Overview of Tomosynthesis Imaging .....	98
5.2.	Navigating Tomosynthesis Images .....	99
5.2.1.	Tomosynthesis Navigation Buttons .....	99
5.2.2.	Viewing Tomosynthesis Slices .....	100
5.2.3.	Changing Slab Thickness .....	101
5.2.4.	Annotating a Tomosynthesis Image.....	101
5.2.5.	Using Cine Mode.....	102
5.2.6.	Using Local Cine Mode .....	103
5.2.7.	Focus .....	104
5.2.8.	Scrolling Through Linked Tiles.....	105
5.2.9.	Exporting a Movie.....	106
5.3.	Displaying ImageChecker 3D Calc CAD Results.....	107
5.4.	Tagging Tomosynthesis Reconstructed Slices.....	109
5.5.	Printing Tomosynthesis Reconstructed Slices .....	109
<b>Chapter 6:</b>	<b>Setting User Preferences .....</b>	<b>111</b>
6.1.	Workflow Preferences.....	112

6.2. Image Presentation Preferences .....	113
6.3. Tools and Overlays Preferences .....	114
6.4. User Profile Preferences .....	118
<b>Chapter 7: Hanging Snapshots and ReportFlows .....</b>	<b>121</b>
7.1. Viewing ReportFlows .....	122
7.2. Viewing Hanging Snapshots .....	123
7.3. Creating and Modifying Hanging Snapshots .....	124
7.3.1. Creating New Hanging Snapshots .....	124
7.3.2. Copying and Editing a Hanging Snapshot .....	126
7.3.3. Renaming a Hanging Snapshot .....	127
7.3.4. Changing a Hanging Snapshot Icon .....	128
7.4. ReportFlows .....	129
7.5. Linking a ReportFlow to a Procedure .....	131
7.6. Creating New ReportFlows .....	132
7.7. ReportFlows Preferences .....	134
<b>Chapter 8: Administrator Tasks .....</b>	<b>135</b>
8.1. Opening the Administration Module .....	136
8.2. Managing User Profiles .....	137
8.3. Administrator User Profile .....	139
8.4. Configuring System-Level Settings .....	140
8.4.1. Scheduling .....	141
8.4.2. Disk Space Monitoring and Auto-Deletion .....	141
8.4.3. Auto-Fetching/Auto-Completion Configuration .....	143
8.4.4. Active Directory Setup .....	144
8.4.5. Study List Manager (SLM) Configuration .....	146
8.4.6. Application Event Logging .....	147
8.4.7. Search on PACS .....	148
8.4.8. Configure Synchronization Interface .....	149
8.4.9. Worklists .....	150
8.4.10. Date/Time Format and Units .....	150
8.4.11. Multimodality Viewer .....	150
8.4.12. Double Reading for Studies .....	151
8.4.13. Sending Annotations, Tagged Tomo, Study State .....	151
8.4.14. Local Institution .....	152
8.4.15. Secondary Capture .....	152
8.4.16. Adoption of Institution Name and Address .....	152

8.5. Configuring System-Level Hanging Snapshots and ReportFlows .....	152
8.6. Configuring Examination Procedure Names.....	153
8.7. Configuring Image Overlays .....	155
8.7.1. MG Viewer Image Overlay .....	155
8.7.2. MammoNavigator Overlay .....	156
8.7.3. Printing Image Overlay .....	157
8.8. Maintaining the Database .....	158
8.8.1. Backing Up and Restoring the Database .....	158
8.8.2. Scheduling Database Maintenance.....	159
<b>Chapter 9: Case Administrator Tasks .....</b>	<b>161</b>
9.1. Opening the Administration Module.....	162
9.2. Deleting Patients.....	163
<b>Chapter 10: Patient and ReportFlow Files .....</b>	<b>165</b>
10.1. Exporting Currently Displayed Image Files.....	166
10.2. Exporting DICOM Files .....	167
10.3. Importing and Exporting ReportFlows .....	168
10.3.1. Importing ReportFlows from a USB drive to SecurView .....	168
10.3.2. Exporting ReportFlows from SecurView to a USB Drive.....	168
<b>Appendix A: Keyboard Shortcuts .....</b>	<b>169</b>
<b>Appendix B: Technologist's Guide.....</b>	<b>171</b>
<b>Index .....</b>	<b>173</b>





---

## Chapter 1: Introduction

- ▶ 1.1. Overview
- ▶ 1.2. Intended Use
- ▶ 1.3. Using This Guide
- ▶ 1.4. Resources Available
- ▶ 1.5. Warnings and Precautions
- ▶ 1.6. Product Complaints

This chapter provides an overview of Hologic SecurView® DX and RT workstations, with information about this guide, product support resources, and safety precautions.

## 1.1. Overview

This guide provides instructions for operating the SecurView workstations:

- SecurView DX Diagnostic Workstation
- SecurView RT Technologist Workstation

In addition, this guide provides instructions for working with the following supplemental Hologic software applications:

- ImageChecker® Computer Aided Detection
- ImageChecker® 3D Calc CAD
- Quantra™ Breast Density Assessment
- BACS™ Breast Arterial Calcification Scoring
- Application Synchronization
- Study List Manager

For information regarding the Advanced Multimodality Option, refer to the *SecurView Advanced Multimodality Option User Guide*.

SecurView workstations provide a dedicated softcopy review environment for diagnostic and screening mammography. The system user interface and workflow are optimized to support experienced mammography reviewers in high-volume reading. Efficiency and reading quality are supported by specialized features including:

- Predefined Hanging Snapshots
- Best matching ReportFlow
- Workflow keypad
- Patient barcode reader
- Individual user login and user preference settings
- Automatically generated worklists
- Predefined standard views
- Support for double reading

SecurView provides access to additional patient data:

- MammoNavigator to support easy access to non-standardized image material such as additional views, mosaics, and scanned documents
- Mammography CAD Structured Report (CAD SR) integration
- Automatic synchronization with external applications
- Application event logging to support compliance with patient privacy policies
- Study List Manager adds non-local patients to patient list to facilitate automatic synchronization

**⚠️ Only** United States federal law restricts this device to use by, or on the order of, a physician.

## 1.2. Intended Use

### 1.2.1. SecurView DX Intended Use

The Hologic SecurView DX device is intended for selection, display, manipulation, filming, and media interchange of multimodality images from a variety of different modality systems. It also interfaces to various image storage and printing devices using DICOM or similar interface standards. The device used with FDA-cleared monitors may be used by a trained physician for display, manipulation and interpretation of lossless compressed or non-compressed mammographic images for screening and diagnostic mammography and digital breast tomosynthesis, as well as any other DICOM multimodality image. SecurView DX is typically used by trained professionals, including, but not limited to physicians, radiologists, nurses, medical technicians and assistants.

### 1.2.2. SecurView RT Intended Use

The SecurView RT Technologist Workstation is a softcopy display system intended for viewing only and does not support diagnostic reading of mammography. It is capable of retrieving prior mammography images for display from PACS and other DICOM image storage systems.

## 1.3. Using This Guide

This guide is organized as follows:

- **Chapter 1: Introduction** provides background information on the system and the guides.
- **Chapter 2: Workstation Description** provides an overview of the SecurView workstations, including component descriptions. It also explains how to start up and shut down the system, and how to log in.
- **Chapter 3: Patient Manager** explains the Patient List, which includes all patients, studies, and images currently in the SecurView database. The chapter also explains how to set up reviewing Sessions.
- **Chapter 4: Reviewing Patients** describes how to open patients for viewing, how to use the viewing and annotation tools, how to close studies, and printing options.
- **Chapter 5: Working with Tomosynthesis Images** describes how to view and work with tomosynthesis images.
- **Chapter 6: Setting User Preferences** explains how to define preferences for individual users.
- **Chapter 7: Hanging Snapshots and ReportFlows** describes Hanging Snapshots and ReportFlows, how a Radiologist user can select specific ReportFlows for everyday use, and how to create new Hanging Snapshots and ReportFlows.
- **Chapter 8: Administrator Tasks** describes system administrator tasks such as managing users, configuring system-level settings, and backing up/restoring the patient database.
- **Chapter 9: Case Administrator Tasks** describes how the case administrator can delete patient data.
- **Chapter 10: Patient and ReportFlow Files** provides procedures intended to support the technologist in managing patient and ReportFlow files.

## 1.4. Resources Available

In addition to this guide, the following resources are available to assist you when working with SecurView workstations.

- **Training:** For new systems, the Hologic Applications team provides in-depth training for radiologists and technologists. For additional personalized instruction, contact your Hologic representative.
- **Hologic Training Center:** This website provides quick access to guides and training materials for Hologic products and is available at no charge to our customers under warranty or Hologic service contract. You can find the Training Center by visiting the Hologic website ([www.hologic.com](http://www.hologic.com)).
- **Additional Documentation:** In addition to this guide, Hologic supplies the following documents for the SecurView workstations:
  - *SecurView DX/RT Workstation Release Notes*
  - *SecurView DX Workstation Quality Control Manual*
  - *SecurView DX/RT Workstation Installation & Service Manual*
  - *SecurView Workstation DICOM Conformance Statements*
  - *SecurView Advanced Multimodality Option User Guide*
  - *SecurView Advanced Multimodality Option Installation & Service Manual*

The SecurView User Guide and Advanced Multimodality Option User Guide are available online by clicking the Help icon on the SecurView workstation toolbar. You can obtain additional copies of printed guides and manuals through your Hologic representative. The DICOM Conformance Statements are available at [www.hologic.com](http://www.hologic.com).

- **Technical Support and Service:** For information, see the title page of this guide.

## 1.5. Warnings and Precautions

This guide uses the following conventions to provide technical and safety information of special interest.



**⚠ WARNING!** An instruction that, if not followed, can result in a hazardous condition.

**⚠ CAUTION:** An instruction that, if not followed, can result in damage to the system.

**⚠ Important:** An instruction provided to ensure correct results and optimal performance, or to clarify limitations of the device.

**⚠ Note:** Information provided to clarify a particular step or procedure.

Before using the SecurView system, read the following warnings and precautions.

### 1.5.1. System Operation

**⚠ WARNING!** Observe the following instructions that, if not followed, can result in a hazardous condition:

- The system is intended for use only by personnel who have read this guide and received training on how to use the system. Hologic does not accept responsibility for injury or damage associated with improper or unsafe system operation.
- If Administrative users access the operating system or file system other than as instructed in this guide or by Hologic trained personnel, system corruption or changes that render the system unusable could result.
- Use the system only outside the patient environment. The system is rated for use only in an office environment.
- The symbol next to the power connector indicates a potential shock hazard. To reduce the likelihood of electrical shock or fire hazard, connect the computer only to a power receptacle that is grounded properly and provides voltage and current within the specifications of the system.
- Do not place liquid containers on the device. In the event of a spill, shut down power to all components prior to cleaning to minimize the possibility of electrical shock. If internal components are exposed to liquid, do not operate the device – contact your service representative.



**⚠ CAUTION:** Always shut down the computer according to the procedures provided in this guide. Improper system shutdown can cause loss of data or damage to the computer operating system.

**⚠ Important:** Observe the following instructions that are provided to ensure correct results and optimal performance, or to clarify limitations of the device.

- For proper system operation, the technical quality of the images (e.g., contrast) should be acceptable to the mammographer and meet the requirements of the Mammography Quality Standards Act (MQSA) or the appropriate national standards.
- The device design and mode of operation are consistent with current standard mammography clinical practices, as governed by the MQSA. Users are advised to comply with the MQSA for the United States, or the appropriate national standards, when implementing the SecurView application in clinical protocols.
- When reviewing images from a full-field digital mammography (FFDM) device, adhere to the guidelines of the American College of Radiologists (ACR). To ensure optimum display of diagnostic information, review each image with the View Actual Pixels scaling mode. When displaying images using a fractional Image Size zoom factor larger than one, SecurView enlarges the original images by interpolating pixel values between the source pixels. The pixel characteristics in the enlarged images may differ from the original image pixels. For information on the scaling modes, see **4.2.9. Scaling Modes**.



- To assure optimal performance of SecurView high-resolution displays, follow the recommended quality control procedures. Perform all quality control tests at the prescribed frequencies as required under national regulations.
- This equipment has been tested and found to comply with the limits for a Class A digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a commercial environment. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instruction guide, may cause harmful interference to radio communications. Operation of this equipment in a residential area is likely to cause harmful interference, in which case the user will be required to correct the interference at the user's expense.
- SecurView workstations are intended for use as image reviewing workstations only. The workstations are not intended to be used for archiving image data or to be used as an image archive. For permanent archiving of patient images and records, use a PACS.

### 1.5.2. Installation and Maintenance



**⚠ WARNING!** Observe the following instructions that, if not followed, can result in a hazardous condition:

- To connect primary power to the system, always use a three-conductor grounded power cable that meets local regulatory standards. Use of a two-prong adapter disconnects the utility ground and creates a severe shock hazard.
- SecurView workstation computers are heavy! If you have concerns about lifting or positioning the computer, request assistance.
- Before cleaning the system components, always shut down the system according to the procedures in this guide and disconnect power cords to prevent electrical shock. Never use alcohol, benzene, thinner, or other flammable cleaning agents.

**⚠ CAUTION:** This product contains no user-serviceable parts. To prevent damage to the system:

- Maintain equipment in a well-ventilated, air-conditioned environment that meets the temperature and humidity specifications provided in the product service manual.
- Do not install or repair the SecurView system. Only trained personnel, authorized by Hologic, are qualified to install or repair the system.
- Disconnect power cord before moving or servicing.
- Install the system behind the institution firewall for network security and antivirus protection. Hologic provides no computer virus protection or network security for the system. If antivirus software is required, visit the Hologic website for information on Hologic's cybersecurity program and antivirus software installation guidance.



## 1.6. Product Complaints

Any healthcare professional who has complaints or is dissatisfied with the quality, durability, reliability, safety, effectiveness, and/or performance of this product should notify Hologic.

If there is reason to believe the device caused or contributed to a serious injury of a patient, notify Hologic immediately by telephone, fax, or written correspondence.



---

## Chapter 2: Workstation Description


- ▶ 2.1. Workstation Overview
- ▶ 2.2. SecurView DX Diagnostic Workstation
- ▶ 2.3. SecurView RT Technologist Workstation
- ▶ 2.4. Functional Division in Multiworkstation Configurations
- ▶ 2.5. User Groups and Passwords
- ▶ 2.6. Startup and Shutdown
- ▶ 2.7. Logging Into SecurView

This chapter describes the workstation models, how they interact with each other, and how they interact with other devices in the clinical environment. It also explains how to start up and shut down the system, and how to log in.

## 2.1. Workstation Overview

Hologic provides the following SecurView workstations:

- **SecurView DX Diagnostic Workstation** – Used by radiologists to review medical images for screening and diagnostic mammography and digital breast tomosynthesis. SecurView DX workstations can be installed as Standalone workstations or several of them can be configured as Client workstations that share a central database on a SecurView DX Manager.
- **SecurView DX Manager** – Provides central database access and performs the image preparation for all connected SecurView DX Client workstations.
- **SecurView RT Technologist Workstation** – Used by technologists to review images for reference purposes. SecurView RT workstations can be installed as Standalone workstations or several of them can be configured as Client workstations that share a central database on a SecurView RT Manager. SecurView RT workstations are used primarily in sites with Hologic Selenia FFDM systems.

 **Note:** *SecurView RT TechMate is no longer supported for version 9.0 or later.*

- **SecurView RT Manager** – Provides central database access and performs the image preparation for all connected SecurView RT Client workstations.

## 2.2. SecurView DX Diagnostic Workstation

Physicians use SecurView DX diagnostic workstations to review medical images for screening and diagnostic mammography and digital breast tomosynthesis.

### ► 2.2.1. SecurView DX Standalone Systems

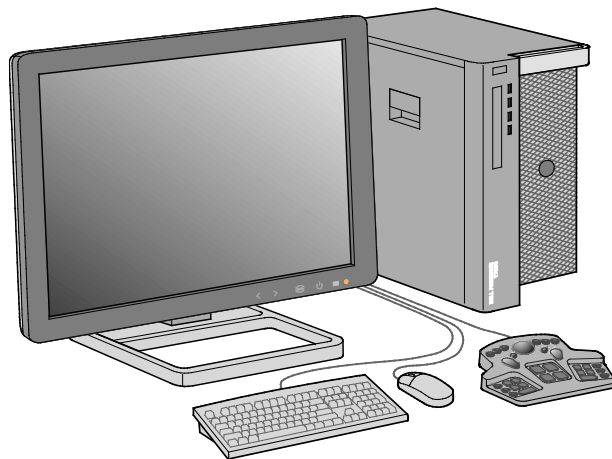
### ► 2.2.2. SecurView DX Multiworkstation Systems

SecurView DX provides a patient browser with query/retrieve functionality for access to patient data, review and annotation of patient images, display of Mammography CAD SR results, and workstation administration and user settings.

A SecurView DX Standalone or Client workstation consists of:

- A wide-screen diagnostic grayscale or color display (or two portrait displays)
- One or two optional lower-resolution displays for the Advanced Multimodality Option, MultiView™ MM software option, Patient Manager, and/or external applications
- Computer with Microsoft Windows, keyboard, and mouse
- SecurView workflow keypad with trackball (optional)
- Barcode reader (optional, not shown)
- Uninterruptible Power Supply (UPS) (optional, not shown)

**Note:** The Advanced Multimodality and MultiView MM Options enhance the softcopy reading environment of the SecurView DX workstation. You can use one or two additional displays to display the Advanced Multimodality Option, the MultiView MM option, the Patient List, and/or an approved external application (with Application Synchronization installed).




**SecurView DX Diagnostic Workstation**

### 2.2.1. SecurView DX Standalone Systems

When installed as a Standalone system, each SecurView DX workstation provides a full complement of workstation functions:

- Temporary storage of current patients
- Patient Manager for browsing patients
- Reviewing and annotating patient images
- Query and retrieval of patient studies from PACS
- DICOM connectivity
- Database with configurable system settings and user preferences
- Images prepared for display
- Storage and display of Mammography CAD SRs

The following diagram shows how images originating from the acquisition workstation flow to the Standalone SecurView DX workstation. After reviewing images at the SecurView DX workstation, any Annotations (physician markings with descriptions) are automatically stored in the SecurView database and sent to all configured destinations.

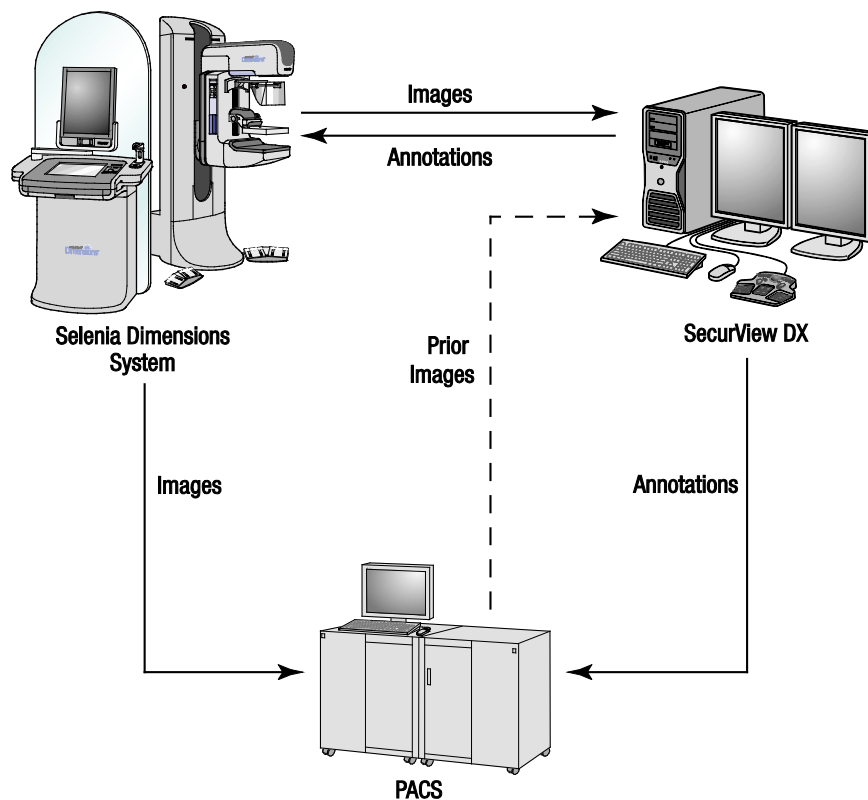

**Information Flow with a Standalone SecurView DX Workstation**

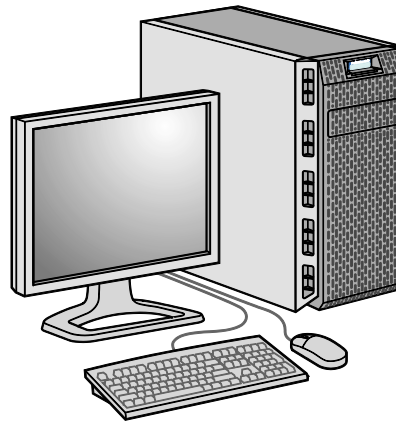


### 2.2.2. SecurView DX Multiworkstation Systems

One SecurView DX Manager supports two or more SecurView DX Client workstations as needed, depending on patient volume and workflow requirements. The Manager consists of:


- Landscape display
- Computer with Microsoft Windows, keyboard, and mouse
- UPS (not shown)

The SecurView DX Manager provides the central SecurView database for its cluster and performs image preparation for all connected SecurView DX Client workstations. The Manager acts as the DICOM entity for a multiworkstation cluster and is responsible for receiving DICOM images. A multiworkstation cluster acts as a single DICOM entity.



**SecurView DX Manager**

The following diagram shows how images originating from the acquisition workstation flow to the SecurView DX Manager. After reviewing images at the SecurView DX workstation, any Annotations (physician markings with descriptions) are automatically stored in the Manager database and sent to all configured destinations.

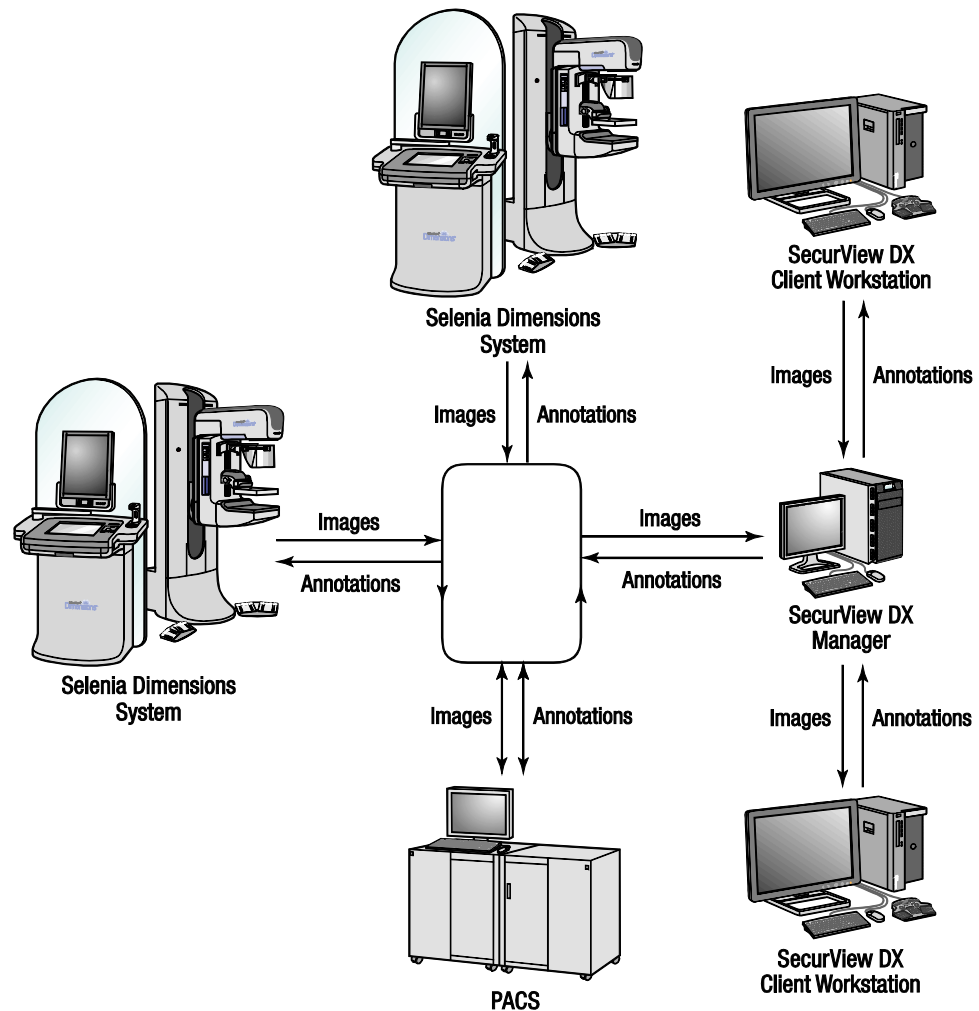

**Information Flow in a SecurView DX Multiworkstation System**

## 2.3. SecurView RT Technologist Workstation

Technologists use SecurView RT workstations to check new images, review prior images and radiologist Annotations, and for administrative tasks.

### ► 2.3.1. SecurView RT Standalone Systems

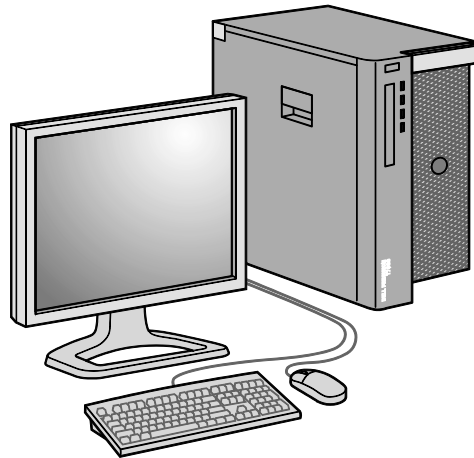
### ► 2.3.2. SecurView RT Multiworkstation Systems

SecurView RT workstations provide low-resolution display of patient images and so are not suitable for diagnostic review. However, SecurView RT provides many SecurView DX workstation features such as the patient browser, query/retrieve functionality for access to patient data, and workstation administration and user settings.

SecurView RT workstations are used primarily in sites with Hologic Selenia FFDM systems.

A SecurView RT Standalone or Client workstation consists of:

- Landscape display
- Computer with Microsoft Windows, keyboard, and mouse
- UPS (not shown)




**SecurView RT Technologist Workstation**

### 2.3.1. SecurView RT Standalone Systems

When installed as a Standalone system, each SecurView RT workstation provides a range of functions for the Technologist:

- Check new studies and images as they arrive
- Review prior images and radiologist Annotations
- Create and manage ReportFlows and Hanging Snapshots
- Print, import, and export mammography image files

The following diagram shows how images originating from the acquisition workstation flow to Standalone SecurView RT and DX workstations. After reviewing images at the SecurView DX workstation, any Annotations (physician markings with descriptions) are automatically stored in the SecurView database and sent to all configured destinations, including any SecurView RT workstations.

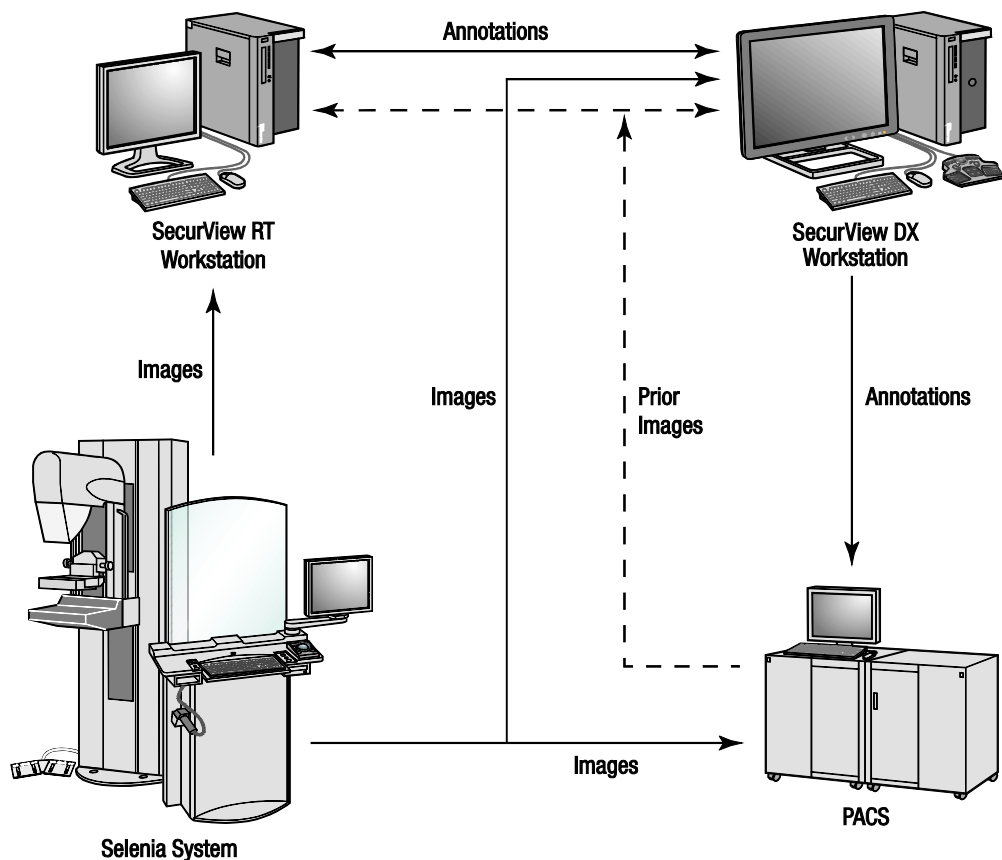


**Information Flow with Standalone SecurView DX and RT Workstations**

### 2.3.2. SecurView RT Multiworkstation Systems

One SecurView RT Manager supports two or more SecurView RT Client workstations as needed, depending on patient volume and workflow requirements. As with the SecurView DX Manager, the SecurView RT Manager provides the central SecurView database for its cluster and performs image preparation for all connected SecurView RT Client workstations.


The following diagram shows how images originating from the acquisition workstation flow to SecurView RT and DX workstations.

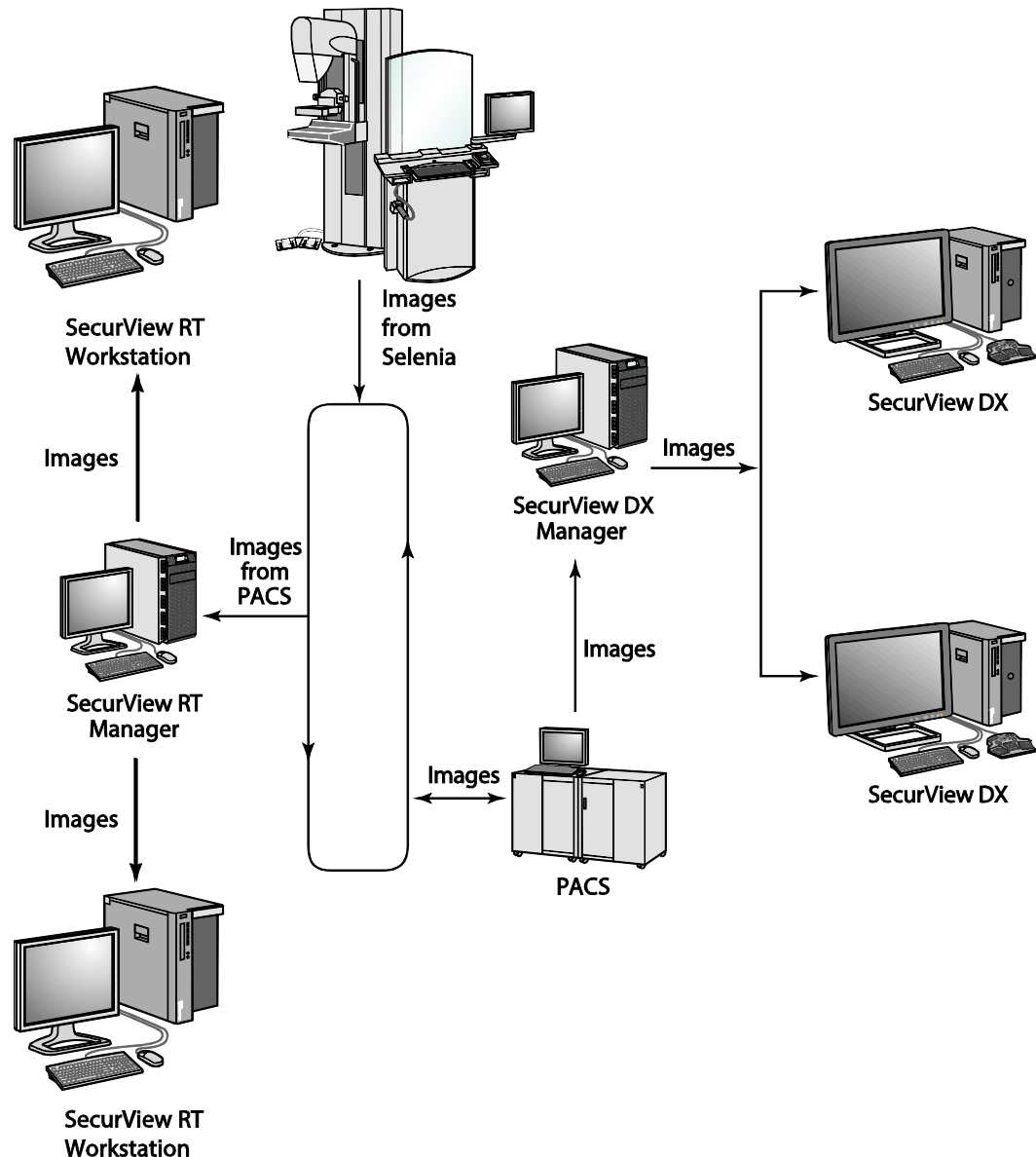

**Image Flow in a SecurView RT Multiworkstation Installation**

After reviewing images at the SecurView DX workstation, any Annotations (physician markings with descriptions) created on the Client workstation are automatically stored in the SecurView DX Manager database and sent by the Manager to all configured destinations outside the Client cluster, including any SecurView RT Standalone or Manager workstations.

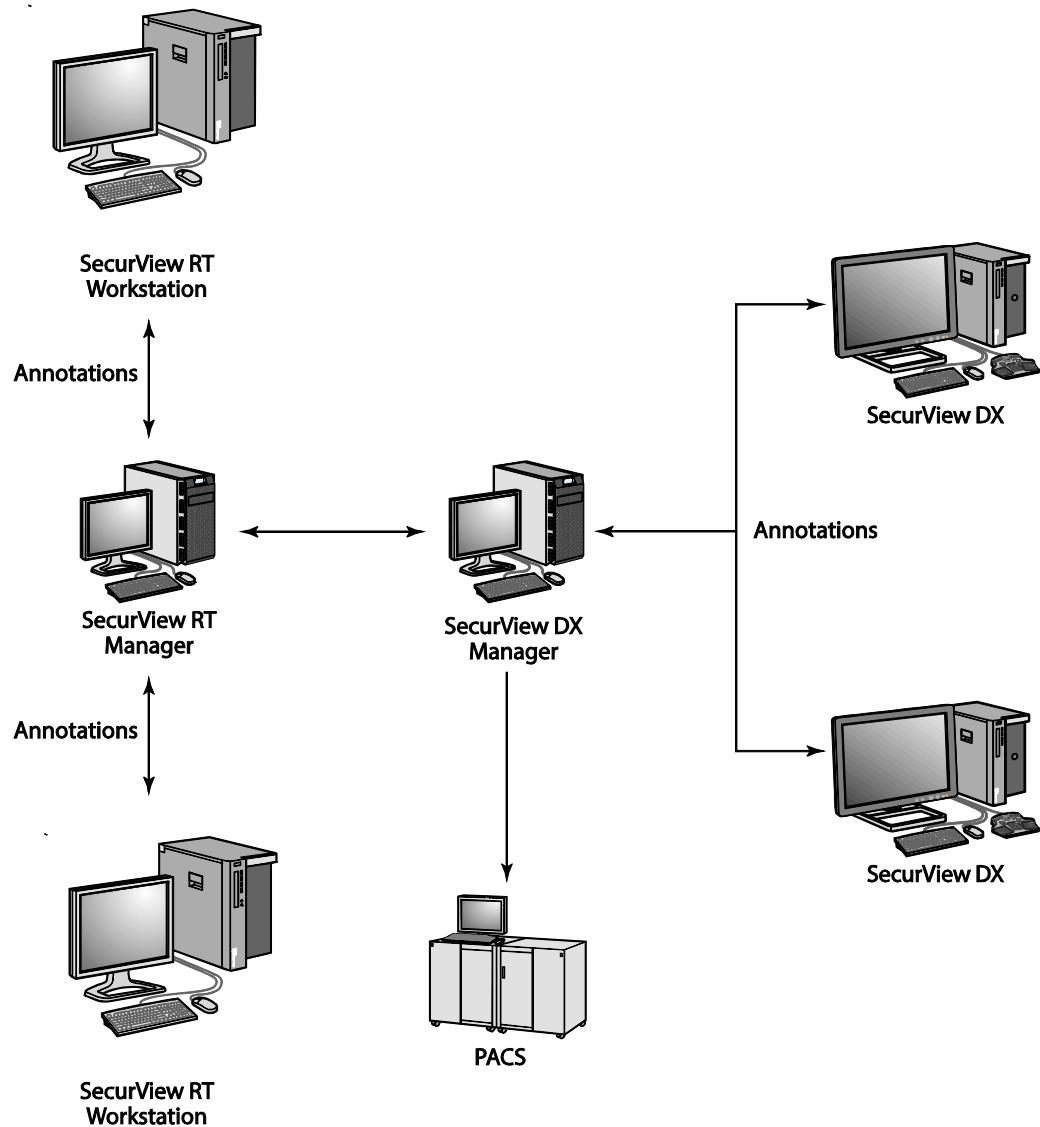The receiving SecurView RT Manager distributes Annotations to all connected Clients. As a result, any physician Annotations are available for technologists at the SecurView RT Client workstations.



**Flow of Physician Annotations in a SecurView RT Multiworkstation Installation**



## 2.4. Functional Division in Multiworkstation Configurations

The following table shows how functionality is split between Manager and Client workstations.

Function	SecurView DX or RT Manager	SecurView DX Client	SecurView RT Client
Patient Manager for browsing studies		X	X
Diagnostic review of patients		X	
Annotating patient images		X	X
Display of CAD SRs		X	X
Low-resolution display of patient images and Annotations			X
Query and retrieval of patient studies from PACS		X	X
Temporary storage of current studies and CAD SRs	X		
DICOM connectivity	X		
Database with configurable system settings and user preferences	X		

## 2.5. User Groups and Passwords

A SecurView system Administrator must register each user by setting up a user profile that includes:

- **User name** – A name that identifies the user to the system.
- **Password** – Use to log in securely.
- **First & Last Name** – The user's actual name.
- **User Group(s)** – Radiologist, Technologist, Administrator, Case Administrator, or Service (see below).
- **Rights** – A set of permissions to review images and configure system settings.


Each user has access to specific program modules depending upon the assigned group(s) and rights (see [8.2. Managing User Profiles](#)).

User Group	Rights – These users can . . .
Radiologist	<ul style="list-style-type: none"> <li>• Configure personalized workflow preferences (e.g., default tools, Hanging Snapshots, ReportFlows)</li> <li>• Perform diagnostic review of patient studies</li> <li>• Enter and view Annotations, merge multiple records for a single patient, track study reading states</li> <li>• Search for patients on PACS</li> <li>• Close studies</li> </ul>
Technologist	<ul style="list-style-type: none"> <li>• Set up worklists for screening and diagnostic sessions (on SecurView DX)</li> <li>• View patient studies and Annotations</li> <li>• Merge multiple records for a single patient, track study reading states (on SecurView DX)</li> <li>• Search for patients on PACS</li> </ul>
Case Administrator	<ul style="list-style-type: none"> <li>• Delete images and patients from database</li> <li>• Merge multiple records for a single patient</li> </ul>
Administrator	<ul style="list-style-type: none"> <li>• Set up new users and assign user rights</li> <li>• Configure system-level workflow preferences (e.g., Hanging Snapshots, ReportFlows)</li> <li>• Configure other system-level settings such as disk space monitoring, auto-fetching, and synchronization with an external application</li> <li>• Back up and restore database</li> <li>• Configure Image Overlay and MammoNavigator</li> </ul>
Service	<ul style="list-style-type: none"> <li>• Configure network and DICOM settings</li> <li>• Configure workstation, image routing, delete patients from database, and manage licenses</li> <li>• Access all Administrator settings</li> </ul>

## 2.6. Startup and Shutdown

Normally you can leave the workstations switched on. Use these procedures if someone switched off the system or if you need to shut it down (for example, if you are going to move the computer, or you know of an upcoming power outage).

► **To start up a SecurView workstation:**

 **Note:** In a multiworkstation environment, start the Manager before starting the Client workstations.

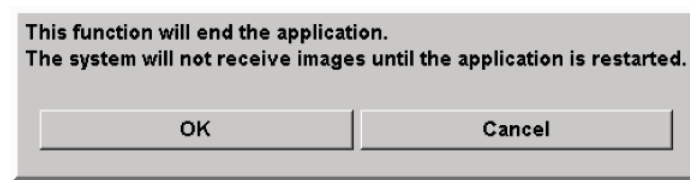
- 1 Switch on peripheral devices (first the UPS, then the displays).
- 2 Switch on the SecurView workstation computer. The SecurView application starts automatically and displays the log-in window.

A dark-themed login window with two text input fields. The first field is labeled 'User name:' and the second is labeled 'Password:'. To the right of the password field is a 'Login' button.


For information on logging in, see [2.7. Logging Into SecurView](#).

► **To shut down a SecurView workstation:**

- 1 On the Application Selector, click **Shutdown**.



- 2 Click **OK**. The SecurView application closes and the computer shuts down.
- 3 Shut off peripheral devices (first the displays, then the UPS).


 **Important:** When shut down, the SecurView workstation cannot receive images. Also, shutting down the application cancels all active print jobs.

## 2.7. Logging Into SecurView

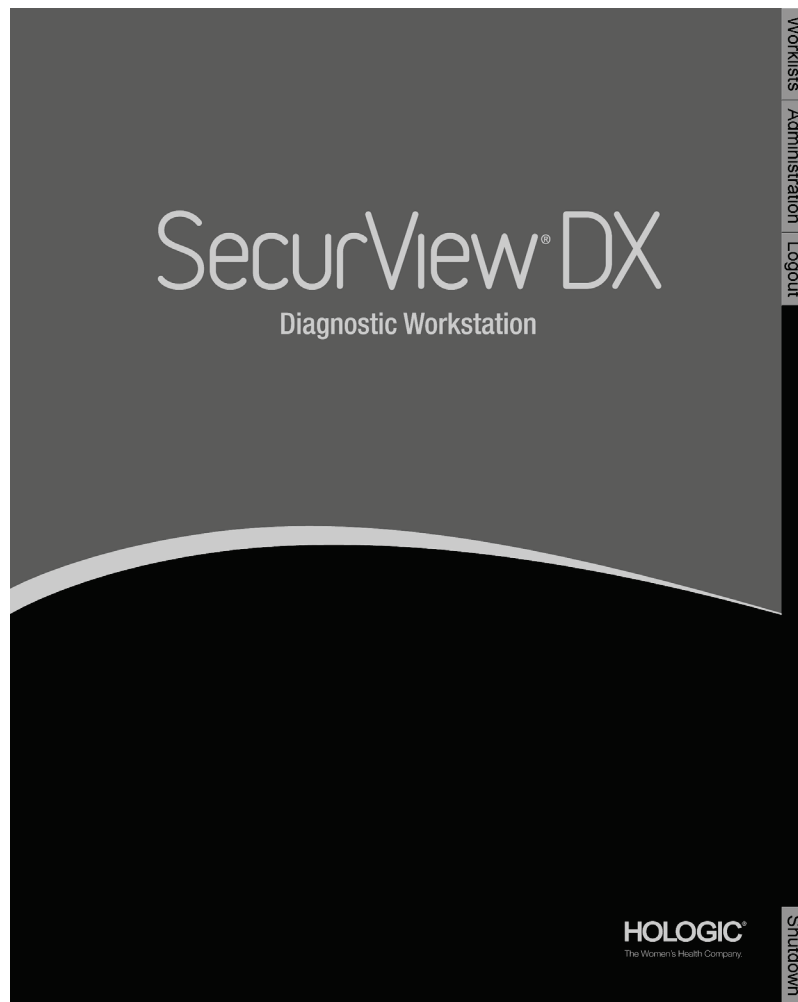Every user must log in with a distinct user name and password. A system Administrator sets up each user account and assigns the user to one or more groups (Radiologist, Technologist, Case Administrator, Administrator, Service). Each group has a set of access privileges to specific program modules. See [2.5. User Groups and Passwords](#).

► **To log into the SecurView application:**

In the log-in window, enter your user name and password, then click **Login** to display the SecurView startup screen.



Access to the application is through the tabs on the right side of the display:

- **Worklists:** Provides access to automatic worklists and user-defined Sessions. This tab is available only to Radiologist users on SecurView DX. See [4.1. Displaying Patient Studies](#).
- **Administration:** Provides access to the Administration module, which allows you to select patients for review (see [3.1. Opening the Patient Manager](#)) and set up user preferences (see [Chapter 6: Setting User Preferences](#)).
- **Logout:** Logs you out of SecurView and displays the log-in window.
- **Shutdown:** Shuts down SecurView and switches off the computer. See [2.6. Startup and Shutdown](#).

## Chapter 3: Patient Manager

- ▶ **3.1. Opening the Patient Manager**
- ▶ **3.2. Using the Patient List**
- ▶ **3.3. Creating Sessions**
- ▶ **3.4. Importing DICOM Images**
- ▶ **3.5. Synchronizing Patient List with MultiView**

The Patient Manager provides the Patient List with all patients, their studies, and image series currently in the database, and non-local patient studies received via Study List Manager. Using the Patient Manager, you can:

- Select patients for viewing
- Search for patient studies
- Create Sessions (patient worklists set up in advance for reviewing)
- Track the reading states of studies (SecurView DX only)
- Import DICOM images into SecurView
- Synchronize with an external Study List Manager

### 3.1. Opening the Patient Manager

On the Application Selector, click **Administration**. The Patient Manager tab and the Patient List appear. This window is the main window to start viewing patient images.

**Patient Manager** | **User Preferences** | **About**

**Patient List** | **Sessions** | **Log**

Reset Columns | Resend | Notices | Cancel Editing | Create Session | Review | Clear

Update Patient List | Merge Patients | Cancel Import | Import... | Suspend And Review | Reload

Study Date	Name	Patient ID	Date of Birth	Modality	State	Note	Type	CAD	Notice	P	Radiologist(s)	Re: ▲
08-21-2009	00300618	00300618	01-01-2007	MG+	Old			+				
08-21-2009	00300626	00300626	01-01-2007	MG+	Old			+				
09-07-2010	03_Implants_008	03_Implants_008	08-25-1945	MG	Old			+				
09-07-2010	03_Implants_009	03_Implants_009	12-16-1953	MG	Old			+				
09-07-2010	05_Selenia_DigitalNow	05_Selenia_DigitalNow_003	01-10-1955	MG	Read			+			Joe Smith	
09-07-2010	05_Selenia_DigitalNow	05_Selenia_DigitalNow_005	01-10-1954	MG	Old			+				
09-07-2010	05_Selenia_DigitalNow	05_Selenia_DigitalNow_007	01-10-1949	MG	Old			+				
04-14-2009	21400011	21400011	01-01-1959	MG+	Old			+				
05-20-2009	21400012	21400012	01-01-1940	MG+	Old			+				
05-05-2009	21400014	21400014	01-01-1944	MG+	Old			+				
05-05-2009	21400016	21400016	01-01-1956	MG+	Old			+				
04-28-2009	21400017	21400017	01-01-1941	MG+	Old			+				
03-12-2009	21400019	21400019	01-01-1962	MG+	Old			+				
05-07-2009	21400027	21400027	01-01-1957	MG+	Old			+				
09-14-2009	21400046	21400046	01-01-1954	MG+	Old			+				
09-15-2009	21400049	21400049	01-01-1954	MG+	Old			+				
11-24-2009	21400057	21400057	01-01-1940	MG+	Old			+				
12-02-2009	21400059	21400059	01-01-1957	MG+	Old			+				
06-29-2009	21900074	21900074	01-01-1944	MG+	Old			+				
06-30-2009	21900075	21900075	01-01-1957	MG+	Old			+				

Local Search | Search on PACS | 0 of 90 patient(s) selected

Help | 04-06-2011 02:38:20 pm User Name: mpatil | OK

The Patient Manager provides three tabs:

- **Patient List:** Displays all patients that are available on the system, and provides options for searching, selecting, and reviewing patients.
- **Sessions:** Provides an overview of existing Sessions and the patients in the Sessions. You can also edit the Sessions to change the sort order or to add or remove patients.
- **Log:** Logs failed print jobs, Auto-Fetch failures, and other events related to information exchange with DICOM devices.

## 3.2. Using the Patient List

The Patient List provides all patients, studies, and series currently in the database.

- ▶ **3.2.1. Selecting Patients**
- ▶ **3.2.2. Patient List Buttons**
- ▶ **3.2.3. Patient List Columns**
- ▶ **3.2.4. Reading States**
- ▶ **3.2.5. Auto-Fetching Patient Data**
- ▶ **3.2.6. Using the Shortcut Menu**
- ▶ **3.2.7. Merging Patient Data**
- ▶ **3.2.8. Searching for Patients**

### 3.2.1. Selecting Patients

You can select one or more patients as follows:

- Type the first few letters of the patient name to advance in the Patient List.
- Select a single patient by clicking the patient. Click **+** to display the studies and image series associated with each study.

Patient List											
Study Date	Name	Patient ID	Modality	State	Note	Type	CAD	Notice	P	Radiologist	Date of Birth
2010-07-21	87800027	87800027	MG	Read						Joe Smith	1957-01-15
2010-07-21	87800031	87800031	MG	Not Read						Joe Smith	1956-11-25
2010-07-21			MG	Read						Joe Smith	
	L CC (1)		MG								
	R CC (1)		MG								
	L MLO (1)		MG								
	R MLO (1)		MG								
2010-07-21			MG	Not Read							
	L CC (1)		MG								
	L MLO (1)		MG								
2010-07-21	87800032	87800032	MG	Read						Joe Smith	1952-08-22
2010-07-20	87800008	87800008	MG	Read						Joe Smith	1957-11-19
2010-07-21	87800029	87800029	MG	Not Read							1964-12-24

- Add additional patients one at a time by clicking them.
  - Select a block of patients by clicking the first patient, holding the mouse button, and moving the mouse pointer to the last patient you want to select (above or below). Then release the mouse button.
- ▶ **To begin reviewing:**
- Select one or more patients (up to 100) and click **Review**, or
  - Double-click a patient (to open a single patient), or
  - Use the barcode reader to read a patient barcode.

SecurView closes the Patient List, opens the Mammography (MG) Viewer to the first patient and displays the images using the ReportFlow set up in your user preferences.

### 3.2.2. Patient List Buttons

The Patient Manager provides buttons to assist in managing patients.

Patient List	Sessions	Log				
Reset Columns	Resend	Notices	Cancel Editing	Create Session	Review	Clear
Update Patient List	Merge Patients		Cancel Import	Import...	Suspend And Review	Reload

The buttons have the following functions:

- **Reset Columns** – Arrange all columns to default settings.
- **Resend** – Manually resend DICOM objects (GSPS Notices, GSPS Reports, MG Secondary Capture images, and/or MM ScreenCapture images) to all configured destinations to which a previous attempt failed (SecurView DX only). This button is available if at least one selected patient is in the 'Read\*' state (see [3.2.4. Reading States](#)) and/or there is a '\*' in the Notice column of the Patient List (see page 35).
- **Notices** – Resorts Patient List so patients with Notices appear at top.
- **Review** – Begin reviewing one or more selected patients on the MG Viewer. See [3.2.1. Selecting Patients](#).
- **Clear** – Clear selected patients.
- **Update Patient List** – Add any new studies to the Patient List. If an acquisition workstation or PACS sends images while the Patient List is open, the items appear automatically in the Patient List.
- **Merge Patients** – Manually merge two patient records. See [3.2.7. Merging Patient Data](#).
- **Suspend And Review** – Available during patient review. Click to suspend the current patient and review a newly selected patient. When you finish reviewing the new patient, SecurView presents the suspended patient in the last state. Also see [3.2.6. Using the Shortcut Menu](#).
- **Import...** – Import patient DICOM files into the SecurView database. See [3.4. Importing DICOM Images](#).
- **Cancel Import** – Enabled while you are importing DICOM files. Click to stop the import process. SecurView retains images that were imported.

There are three buttons used for creating and editing Sessions (see [3.3. Creating Sessions](#)):

- **Create Session** – Create a list of patients for reviewing.
- **Reload** – Enabled when you are editing patients in a Session. Click to undo any changes you have made.
- **Cancel Editing** – Enabled when you are editing patients in a Session. Click to deselect any highlighted patients.

Below the Patient List are two search buttons. See [3.2.8. Searching for Patients](#).

Local Search	Search on PACS
--------------	----------------

- **Local Search** – Search for patients in the local SecurView database.
- **Search on PACS** – Search for patients and retrieve them from a PACS.



### 3.2.3. Patient List Columns




The Patient List provides column headings that you can use to sort and manage patients. For example, you can click any column heading to sort patients by study date, name, reading state, etc. Sorting is possible using two sorting criteria. When clicking on a column it immediately becomes the primary sorting criterion, and the previous primary criterion automatically becomes the secondary sorting criterion. When you sort any column, SecurView collapses all patient studies and series and sorts the entries on the patient level.

You can also modify the column widths and positions:

- Drag a column header to the desired position in the patient list.
- Drag the right column border (in the header) to modify the width.

When you exit the Administration module, SecurView stores the column arrangement as a user preference.

This section describes the columns, indicating what appears at the patient level unless otherwise stated.


- **Study Date** – Acquisition date of the newest study. For non-local studies, 'non-local' is displayed at the series level in place of the series description.
- **Name** – Patient name (last name, first name), up to 100 characters.  
 **Note:** Characters in a patient name that are not supported by SecurView are represented using the question mark character ('?').
- **Patient ID** – Patient identification number, up to 70 characters. An asterisk (\*) identifies a merged or combined patient.  
 **Important:** Be careful when entering IDs on the acquisition workstation. The system uses the IDs (and the date of birth) to assign new data to existing patient data. Patient IDs are unique to the facility where the images are acquired. SecurView cannot identify incorrect data caused by incorrect entries at the acquisition workstation.
- **Date of Birth** – Patient's birth date.
- **Modality** – Comma-separated list of all modalities of each patient study. Tomosynthesis studies and series appear with modality 'MG +'. Non-local studies are likely to be of US and MR modality. On study level, each series appears with its laterality, view, image type, and number of images.
- **State** – Indicates the reading state for the patient and each patient study (SecurView DX only). See [3.2.4. Reading States](#).  
 **Note:** Non-local studies do not have a state. The State column is blank for non-local studies.
- **Note** – Indicates an intermediate status for the most recent study (Consultation Required, Additional Images Required, Additional Images Arrived, or Pending) (SecurView DX only). See [4.7. Closing a Study](#).
- **Type** – Indicates the most recent study type (screening, diagnostic, or undefined).
- **CAD** – Indicates whether a CAD report is available for a patient.
- **Notice** – Indicates whether one or more Notices are available for a patient (see [4.6. Sending and Viewing Notices](#)). A '+' indicates the SecurView workstation received one or more Notices. An asterisk '\*' indicates a failure occurred when the workstation attempted to send a Notice. Technologist users can mark patients with Notices as viewed (see [4.7.2. Closing a Study as a Technologist](#)).

- **AF** – Indicates the patient’s Auto-Fetching state. See [3.2.5. Auto-Fetching Patient Data](#).
- **P** – Indicates a patient protected against auto-deletion. To protect any patient, right-click the patient and click **Protect against auto-deletion**. Also see [3.2.6. Using the Shortcut Menu](#).
- **Radiologist(s)** – Name of radiologist(s) that either read a study or locked a patient as ‘Consultation Required’, ‘Additional Images Required (or Received)’, or ‘Pending’. An asterisk (\*) appears next to the name of the radiologist who locked the patient (SecurView DX only).
- **Technologist(s)** – Name of technologist who acquired the patient images.
- **Referring Physician** – Name of referring physician.
- **Accession Number** – The accession number of the most recent study when displayed on patient level.
- **Institution Name** – List of institution names, ordered by the ages of available studies.
- **# Exam** – Total number of available studies.
- **Gender** – Patient’s gender, ‘F’ or ‘M’.
- **Read Twice** – Indicates studies to be reviewed twice (SecurView DX only).
- **Viewed** – This column appears only on SecurView RT. A ‘+’ indicates that a received Notice has been viewed on SecurView RT for at least one study for the patient. Technologist users can mark patients with Notices as viewed (see [4.7.2. Closing a Study as a Technologist](#)).

### 3.2.4. Reading States

On SecurView DX, the State column of the Patient List shows the current reading state for each patient and each patient study. The State column does not appear on SecurView RT.

The reading states (Read, Not Read, etc.) can have different meanings depending upon whether the indication is on the patient level or on the study level.

 **Note:** Non-local studies do not have a state. The State column is blank for non-local studies.

Reading State	Patient Level	Study Level
Not Read	At least one study of this patient has the state 'Not Read' or 'Read Once'.	The study has not been read.
Read	At least one study of this patient has been read by the current user. All other studies have the state 'Old'.	The study has been read.
Read Once	(Not applicable)	In a double-reading environment, a study that was read by the first reader but not by the second.
Changed	At least one study of this patient has the state 'Changed'.	Additional images arrived after the study was read.
Old	All studies of this patient have the state 'Old'.	Study where SecurView received the images more than five days after the images were produced by the acquisition device (this value is configurable).

In some cases, the reading state changes automatically. For example, if a study has been 'Read' and SecurView receives new images (not more than five days after they were produced), then the reading state changes to 'Changed'.

During patient review, SecurView also indicates the reading state by placing a symbol in front of the Patient ID (see **4.2.5. Patient Reading and Lock States During Review**).

#### Read\* State

If DICOM objects (GSPS Notices, GSPS Reports, MG Secondary Capture images, and/or MM ScreenCapture images) were sent but failed to reach a configured destination, then SecurView sets the patient state to 'Read\*' and enables the **Resend** button on the Patient List. When 'Read\*' appears, verify that all DICOM destinations are configured correctly. If clicking **Resend** still does not change the state to 'Read', contact Hologic Technical Support. For more information, see **3.2.2. Patient List Buttons**.

### 3.2.5. Auto-Fetching Patient Data

When SecurView receives newly acquired images, it can automatically fetch DICOM data for the patient from an archive. In this case, SecurView retrieves prior mammography images, CAD SRs, GSPS Reports (study states with or without Annotations and tagged tomosynthesis slices), GSPS Notices, MG Secondary Captures, MM ScreenCaptures, and third-party GSPS objects that meet the Auto-Fetching criteria. A service engineer or Administrator must configure this feature (see [8.4.3. Auto-Fetching/Auto-Completion Configuration](#)). If Auto-Fetching is On, the AF column in the Patient List displays the state of patient data requested from the archive configured in the service interface as 'PACS 1'. Possible states are:

- **+** = Auto-Fetching completed successfully
- **0** = Auto-Fetching is in progress
- **D** = Auto-Fetching was postponed (or interrupted)
- **F** = Auto-Fetching failed – click the Log tab to view detailed information
- **–** = Nothing found on PACS 1 that matches the Auto-Fetching criteria
- **blank** = Auto-Fetching not triggered

### 3.2.6. Using the Shortcut Menu

If you right-click any patient, a shortcut menu opens with several options:

- **Suspend open patient and review** – closes the current patient, allows you to review a new patient, and then returns to the original patient (functions like the Suspend and Review button on the Patient List).
- **Protect against auto-deletion** – prevents unintended deletion of the patient.
- **Unlock** – allows you to unlock any patient that you locked (SecurView DX only). There are three locked states ('Consultation Required', 'Additional Images Required', or 'Pending'). See [4.7. Closing a Study](#).
- **Take over** – allows you to 'take over' a patient that was locked by another radiologist.
- **Synchronize patient** – allows you to send a synchronization request to an external application (see [4.9. Patient Synchronization with an External Application](#)).
- **Export to media** – allows you to export all images in DICOM format for the selected patient(s) to a folder or CD/DVD drive (see [10.2. Exporting DICOM Files](#)).
- **Undo merge patients** – allows you to separate two patient records merged in the SecurView database. This function undoes the action of the **Merge Patients** button on the Patient List (see [3.2.7. Merging Patient Data](#)).

### 3.2.7. Merging Patient Data

SecurView automatically merges all DICOM data with the same Patient ID and birth date. If the facility establishes that Patient IDs are unique and that certain image types (for example, prior digitized films) do not contain a birth date value, a service engineer can configure the system to merge images with a common Patient ID and no birth date (or the same birth date).

This section describes how to merge patient records with different Patient IDs that are in fact the same person. This functionality allows side-by-side review of patient images acquired at different times.

**Note:** You cannot manually merge patients that contain non-local study data from an external Study List Manager (SLM).

**Note:** Merging patient data on SecurView (for example, two or more studies) does not merge patient data stored on PACS.

► **To merge two patient records:**

- 1 When no patients are open, select two patient records in the Patient List and click **Merge Patients**. If the patients you select are not currently open on any Client in a workgroup cluster, SecurView shows the selected patient records:

- 2 Select the patient record for the primary patient: Make your selection and click **OK**. The system merges the two patients, and closes the dialog.

**Important:** If you accidentally merge the wrong patient data, then right-click the merged patient and select **Undo merge patients**. The system 'unmerges' the patient data and restores the patient records to their original forms.

After you merge patients, only the primary patient appears in the Patient List. SecurView assigns all studies and series from both patient records to the primary patient. In the Patient List, the primary merged Patient ID appears with an asterisk (\*).

- 3 Select the primary patient and click **Review**. SecurView displays all images and corresponding DICOM objects (Notices and study states with Annotations, CAD SRs, etc.) of the merged patients in the MG Viewer.

### 3.2.8. Searching for Patients

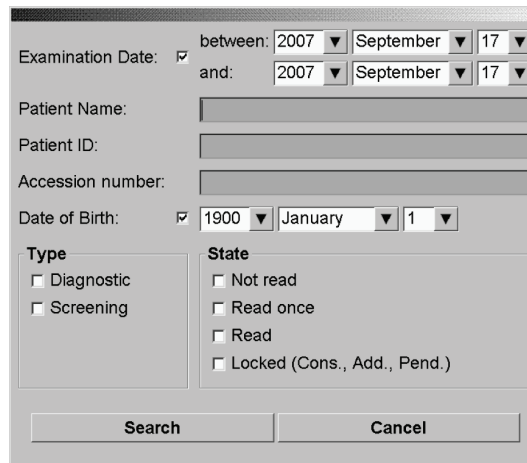
SecurView workstations offer two options for searching:

Local Search   Search on PACS

The default search field (Patient ID or Patient Name) is set according to your user preferences (see **6.1. Workflow Preferences**).

#### Local Search

This option searches for data in the local SecurView database according to the criteria shown in the following screen. (Use an asterisk (\*) as a wildcard character.)




The Local Search dialog box contains the following fields and options:

- Examination Date:** A checked checkbox followed by "between:" and a date range (2007, September, 17) and "and:" with the same date range.
- Patient Name:** A text input field.
- Patient ID:** A text input field.
- Accession number:** A text input field.
- Date of Birth:** A checked checkbox followed by a date (1900, January, 1).
- Type:**
  - ☐ Diagnostic
  - ☐ Screening
- State:**
  - ☐ Not read
  - ☐ Read once
  - ☐ Read
  - ☐ Locked (Cons., Add., Pend.)
- Buttons:** Search and Cancel.

#### Search on PACS

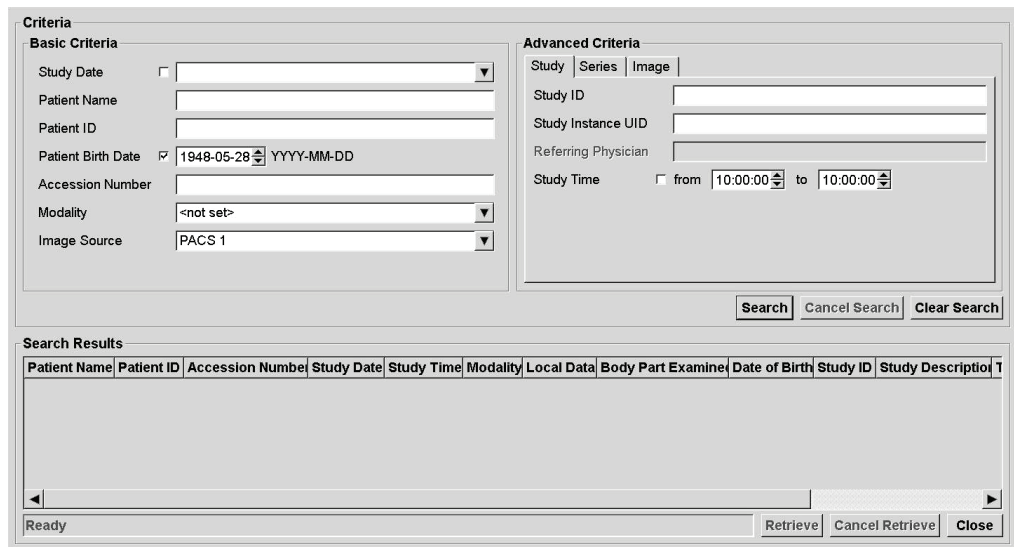You can search a configured PACS (Image Source) to retrieve DICOM data (prior images or images from other modalities). SecurView copies the retrieved images to its local database.



The Search on PACS dialog box is divided into two main sections: Criteria and Search Results.

**Criteria Section:**

- Basic Criteria:**
  - Study Date:** A dropdown menu.
  - Patient Name:** A text input field.
  - Patient ID:** A text input field.
  - Patient Birth Date:** A checked checkbox followed by a date (1948-05-28) and a format (YYYY-MM-DD).
  - Accession Number:** A text input field.
  - Modality:** A dropdown menu with "<not set>" selected.
  - Image Source:** A dropdown menu with "PACS 1" selected.
- Advanced Criteria:**
  - Study | Series | Image:** A tabbed interface.
  - Study ID:** A text input field.
  - Study Instance UID:** A text input field.
  - Referring Physician:** A text input field.
  - Study Time:** A checked checkbox followed by "from" and a time range (10:00:00 to 10:00:00).


**Buttons:** Search, Cancel Search, Clear Search.

**Search Results Section:**

A table with the following columns: Patient Name, Patient ID, Accession Number, Study Date, Study Time, Modality, Local Data, Body Part Examined, Date of Birth, Study ID, Study Description.

**Buttons:** Ready, Retrieve, Cancel Retrieve, Close.

To search, enter your criteria and click **Search**. (Use an asterisk (\*) as a wildcard character.) If the search succeeds, the matching patient data appear in the Search Results area and the Retrieve button becomes active. To transfer the data to SecurView, select one or more items in the Search Results area and click **Retrieve**.

 **Important:** *If you start a new search before the previous search is finished, only the progress of the new search is displayed.*

Your PACS may not support some tabs and fields in the Advanced Criteria area. A service engineer must configure and activate the tabs and fields.

### 3.3. Creating Sessions

A Session is a patient worklist set up in advance by a technologist or radiologist on SecurView DX. A Technologist user can set up Sessions for any radiologist. A Radiologist user can set up Sessions only for personal use.

**Note:** To use the Sessions feature, an Administrator must enable the Scheduling option (see **8.4.1. Scheduling**) and configure each user with Setup rights (see **8.2. Managing User Profiles**). Also, note that SecurView automatically sorts patients in your Session worklists according to your user preference on the Workflow tab (see **6.1. Workflow Preferences**).

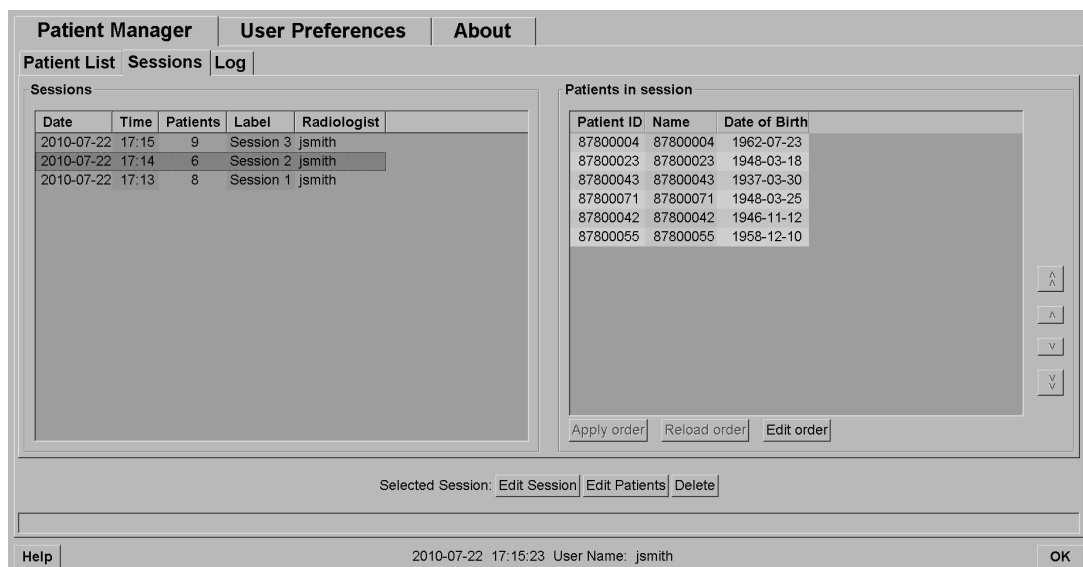
► **To create a Session:**

- 1 In the **Patient List**, select patients you want to assign to a Session. Then click **Create Session**:



Sessionlabel: Session 1  
Radiologist: jsmith  
OK Cancel

- 2 Type a Session name in the **Sessionlabel** field. If you have Technologist user rights, you can assign the Session to any radiologist using the Radiologist drop-down list.
- 3 Click **OK** to create the Session. If you are a Radiologist user, the **Sessions** tab opens automatically. (If you are a Technologist user, the Patient List opens.)



The screenshot shows the Patient Manager interface with the Sessions tab selected. The left pane displays a table of sessions, and the right pane displays a table of patients in the selected session.

Date	Time	Patients	Label	Radiologist
2010-07-22	17:15	9	Session 3	jsmith
2010-07-22	17:14	6	Session 2	jsmith
2010-07-22	17:13	8	Session 1	jsmith

Patient ID	Name	Date of Birth
87800004	87800004	1962-07-23
87800023	87800023	1948-03-18
87800043	87800043	1937-03-30
87800071	87800071	1948-03-25
87800042	87800042	1946-11-12
87800055	87800055	1958-12-10

Buttons: Apply order, Reload order, Edit order

Selected Session: Edit Session Edit Patients Delete

Help 2010-07-22 17:15:23 User Name: jsmith OK

On the Sessions tab, point to any Session on the left side and then:

- Edit the patient order by selecting a patient, clicking **Edit order**, and clicking a column heading or one of the arrow keys on the right side.
- Edit the Session by clicking **Edit Session** to reopen the Sessionlabel dialog. Then edit the entries as desired and click **OK**.
- Add or remove patients by clicking **Edit Patients** to reopen the Patient List. Then reselect (or deselect) patients as desired and click **Create Session** to reopen the Sessionlabel dialog. Edit the entries as desired and click **OK**.

If while editing patients you decide to abandon your changes, then on the Patient List, click **Reload** to restore the Session to its previous state.



## 3.4. Importing DICOM Images

Use the **Import...** button to import DICOM images from either a local folder or external media (e.g., CD, DVD, USB drive).

 **Note:** To export images, see [10.2. Exporting DICOM Files](#).

► **To import DICOM images:**

- 1 In the Patient List, click **Import...** to display the Import from dialog.
- 2 Navigate to the folder with the DICOM images you wish to import. The default path is F:\Exports\.
- 3 Select the images folder and click **OK**. SecurView imports all images in the folder that are DICOM Part 10 conformant (including those stored using DICOM compression transfer syntaxes). This may take several minutes – the DICOM files are large.
- 4 After importing the images, check to ensure all imported images appear in the Patient List.


## 3.5. Synchronizing Patient List with MultiView


If configured, the SecurView workstation can be synchronized with an external Study List Manager.

- SecurView's Patient List contains all studies with images received by SecurView (local studies) as well as the studies available on external systems that are known to the Study List Manager (non-local studies). On client workstations, only the non-local studies of an associated Study List Manager (SLM) client will be listed. On a manager, all non-local studies known to the Study List Manager will be listed. For information on configuring the associated SLM client for client workstations, see [8.4.8. Configure Synchronization Interface](#).
- SecurView sends information on local studies that are displayed in the Patient List to the Study List Manager.

Synchronization with external applications (see [4.9. Patient Synchronization with an External Application](#)) allows simultaneous patient review on SecurView and a synchronized MultiView application to be triggered from either application.

For information about configuring Study List Management, see [8.4.5. Study List Manager \(SLM\) Configuration](#).

 **Caution:** If the synchronization with the SLM fails (e.g. due to a communication error), non-local studies may not be available in the SecurView Patient List. Check the local patient list in other connected SLM client applications (e.g., MultiView) to ensure all relevant studies for a patient are reviewed.

 **Caution:** If a non-local patient is the primary patient of a merged patient, the patients will be unmerged automatically if SLM synchronization deletes the non-local patient.



---

## Chapter 4: Reviewing Patients

- ▶ 4.1. Displaying Patient Studies
- ▶ 4.2. Displaying Patient Images
- ▶ 4.3. Visualizing Image Details
- ▶ 4.4. Using CAD
- ▶ 4.5. Creating and Viewing Annotations
- ▶ 4.6. Sending and Viewing Notices
- ▶ 4.7. Closing a Study
- ▶ 4.8. Printing Options
- ▶ 4.9. Patient Synchronization with an External Application

This chapter describes how to open patients for viewing, use the viewing and annotation tools, close studies, print images, and synchronize with an external application.

## 4.1. Displaying Patient Studies

Review is often based on a patient worklist. There are three types of worklists. Once you set up a worklist, you can begin reviewing patients with the MG Viewer.

- ▶ **4.1.1. Patient List Worklists**
- ▶ **4.1.2. Automatically Generated Worklists**
- ▶ **4.1.3. Session Worklists**
- ▶ **4.1.4. MG Viewer**

### 4.1.1. Patient List Worklists

From the Patient List, you can create a temporary worklist manually by pointing to one or more patients (highlighted in dark gray below) and clicking **Review**.

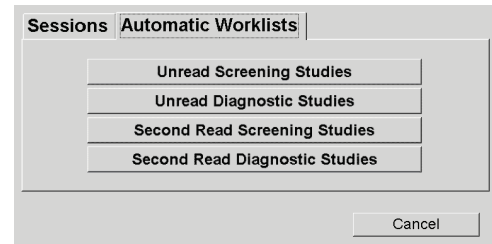
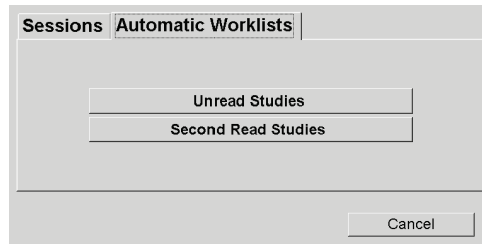
Patient Manager			User Preferences			About		
Patient List			Sessions			Log		
<div>Reset Columns</div>			<div>Resend</div>			<div>Notices</div>		
<div>Update Patient List</div>			<div>Merge Patients</div>					

### 4.1.2. Automatically Generated Worklists

SecurView DX automatically generates worklists of Unread and Second Read studies. The system can further divide the Unread and Second Read studies into worklists of Screening and Diagnostic studies. The worklist types generated depend upon settings configured by an Administrator (see [8.4.9. Worklists](#)).

► **To select an automatic worklist:**

- 1 After logging in, select the **Worklists** tab.
- 2 Select the **Automatic Worklists** tab. Depending upon the configuration settings, you may see 2, 3, or 4 buttons:



- 3 Click a button. The MG Viewer opens showing the first patient in the selected worklist, showing all patients that match the stated criteria.


The buttons are inactive if there are no unread or second read studies.

### 4.1.3. Session Worklists

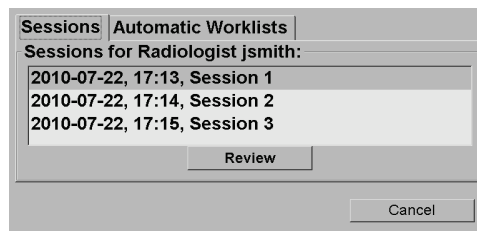If your workstation has been configured for Sessions, you can start reviewing by opening a Session. For information on setting up a Session, see [3.3. Creating Sessions](#).

► **To select a Session:**


- 1 After logging in, select the **Worklists** tab.
- 2 Select the **Sessions** tab. If you have created Sessions (or had them created for you), then they appear as shown in the example below.



- 3 Click a Session and then click **Review**. The MG Viewer opens showing the first patient in the Session worklist.

4.1.4. MG Viewer

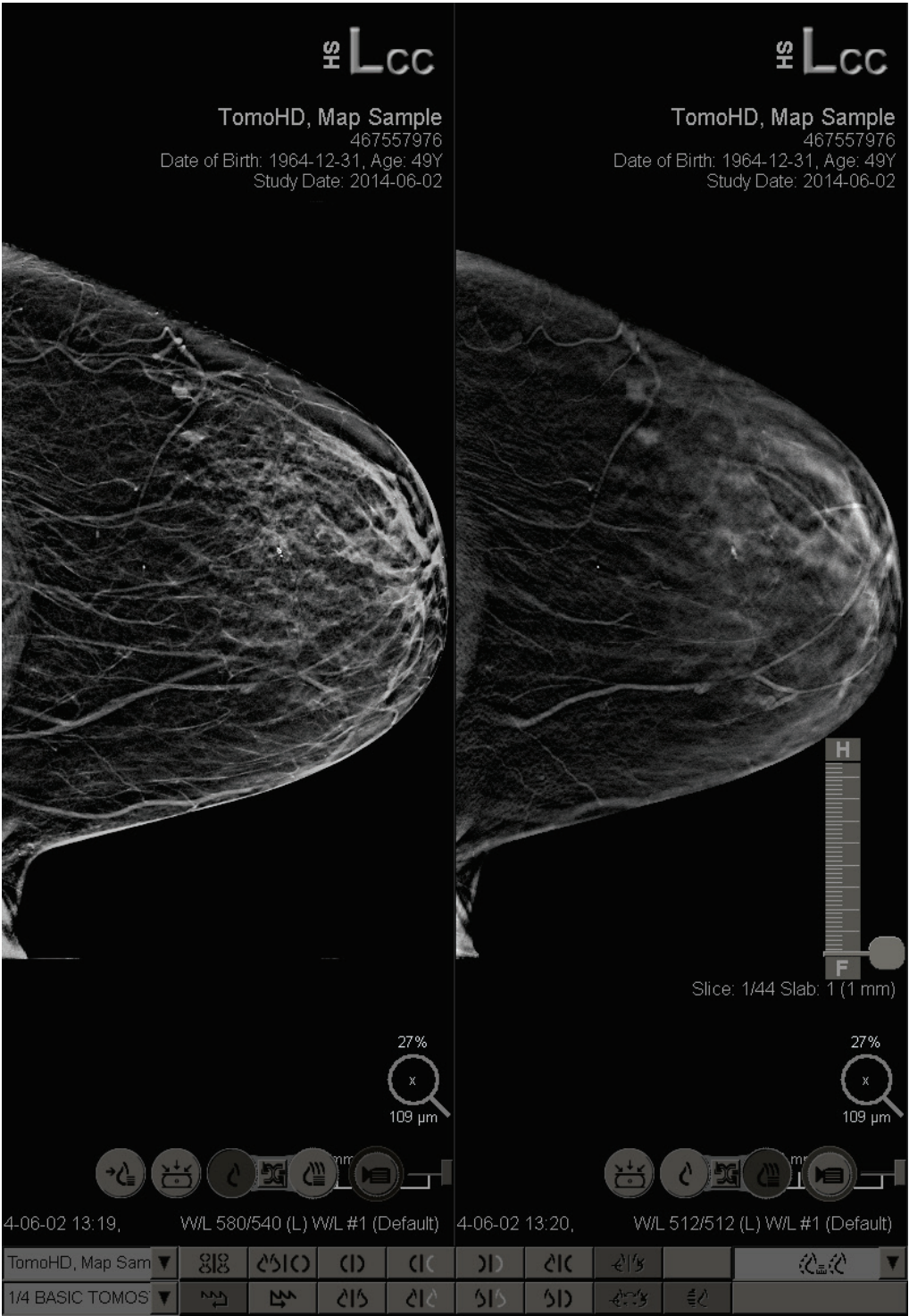
When a patient opens, the MG Viewer appears.



MG Viewer – Left Display



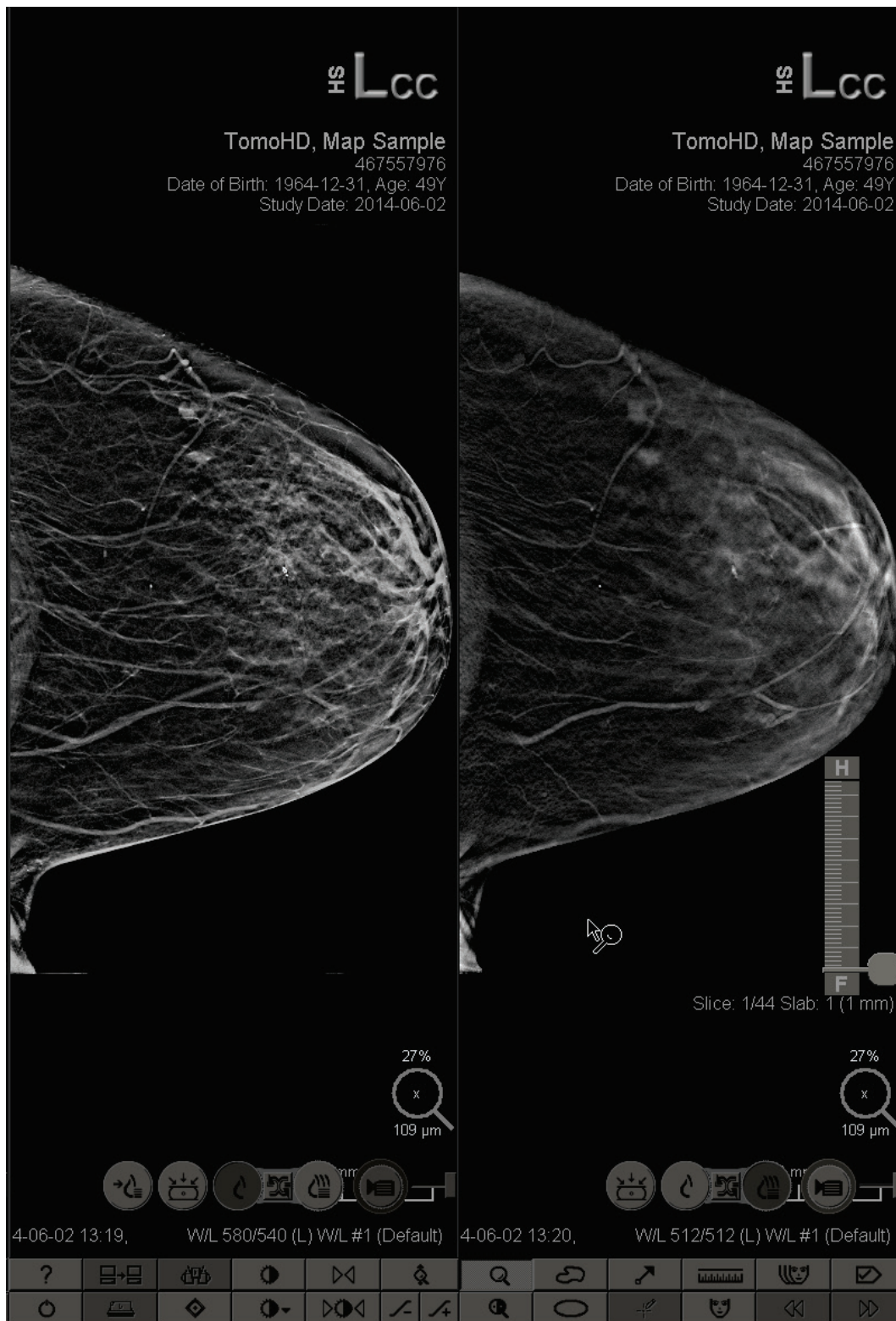Most of your interaction with the application is through the toolbar buttons at the bottom of each display or their corresponding keypad buttons.



MG Viewer – Right Display








## 4.2. Displaying Patient Images

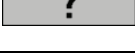This section describes tools and options for displaying patient images.

- ▶ [4.2.1. Navigating Patients](#)
- ▶ [4.2.2. Using the Keypad](#)
- ▶ [4.2.3. Using the Pie Menu](#)
- ▶ [4.2.4. Using a ReportFlow](#)
- ▶ [4.2.5. Patient Reading and Lock States During Review](#)
- ▶ [4.2.6. Panning Images](#)
- ▶ [4.2.7. Image Hangings](#)
- ▶ [4.2.8. Intelligent Roaming](#)
- ▶ [4.2.9. Scaling Modes](#)
- ▶ [4.2.10. Pixel Meter](#)
- ▶ [4.2.11. Stack and Prior Indicators](#)
- ▶ [4.2.12. MammoNavigator](#)
- ▶ [4.2.13. Image Information](#)
- ▶ [4.2.14. Patient Information Overlays](#)
- ▶ [4.2.15. MG Secondary Captures and MM ScreenCaptures](#)

### 4.2.1. Navigating Patients


The toolbar at the bottom of the right display provides tools for getting started.

Icon	Purpose
	<b>Next Patient:</b> Display the next patient in the worklist.
	<b>Previous Patient:</b> Display the previous patient in the worklist.
	<b>Patient List:</b> Display the Patient List. See <a href="#">3.2. Using the Patient List</a> .
	<b>Reset:</b> Undo changes you made and reset images for the current patient to their initial states when opened (Annotations remain).
	<b>Close Study:</b> Close the study. Active when displaying the last ReportFlow step. For more information, see <a href="#">4.7. Closing a Study</a> .
	<b>Help:</b> Open the SecurView user guides in a separate window. (System Administrators can display the guides using the Help button at the lower left of the Administration tabs.)
	<b>Exit:</b> Close the MG Viewer and displays the Administration module.



### 4.2.2. Using the Keypad

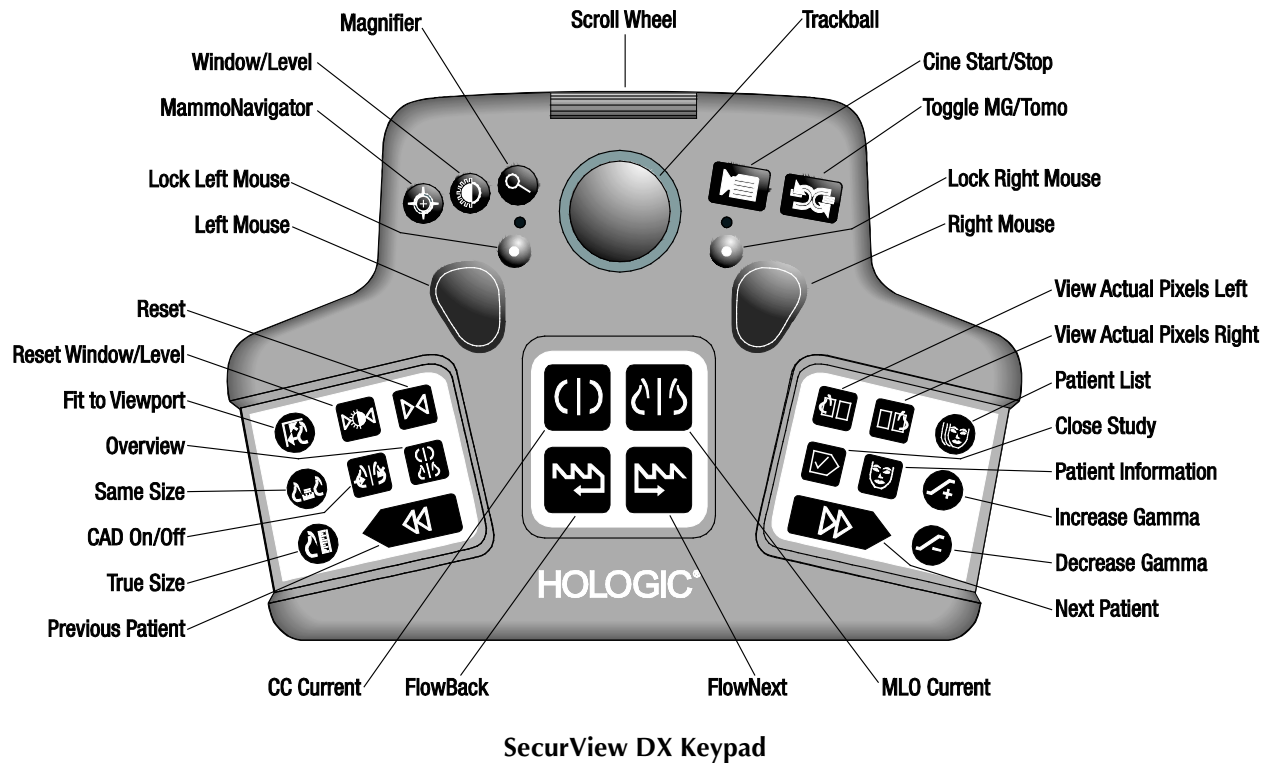The optional keypad provides quick access to most viewing options. The keypad icons correspond to similar icons that appear on the toolbar buttons. The following sections of the guide explain the function of each tool.



#### ► Troubleshooting the keypad (SecurView DX)

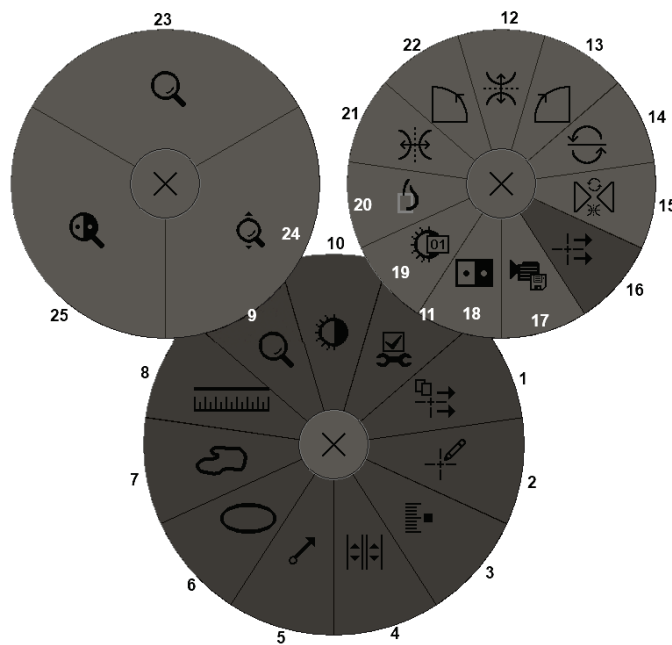
- 1 If the keypad is unresponsive to input, perform the following steps:
  - a Log into SecurView as a user in the Administrator user group (e.g. **admin**).
  - b Disconnect the keypad from the computer.
  - c Click the **Exit to Windows** tab and **OK** to confirm, then wait 5-10 seconds.
  - d Reconnect the keypad.
  - e Double-click the **SecurView** icon and log into SecurView.
  - f Confirm that the keypad functions.
- 2 If the keypad is still unresponsive, perform the following steps:
  - a Log into SecurView as any user.
  - b Disconnect the keypad from the computer.
  - c Click the **Shutdown** tab and **OK** to confirm.
  - d Reconnect the keypad.
  - e Power on the computer.
  - f Log into SecurView and confirm that the keypad functions.

### 4.2.3. Using the Pie Menu

The Pie menu provides access to additional tools for image evaluation.

► **To use the Pie menu:**

- Right-click any image and then select a tool from the menu.
- Point to **Image Tools** or **Sizing Tools** on the Pie menu to open a second Pie menu.
















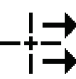







**Legend**





1. Send All Notices
2. Annotation User Filter
3. Tag Tomo Images
4. Link Tile
5. Arrow
6. Ellipse
7. Freehand
8. Measurement
9. Sizing Tools Menu
10. Window/Level
11. Image Tools Menu
12. Flip Up/Down
13. Rotate 90° Clockwise
14. Rotate 180°
15. Reset Flip/Rotate
16. Send Image Notice
17. Export Movie
18. Invert Image
19. Window/Level (numeric)
20. View Actual Pixels
21. Flip Left/Right
22. Rotate 90° Counter-Clockwise
23. Magnifier
24. Continuous Zoom/Reset
25. Inverted Magnifier

\* Note: Continuous Zoom is a toggle feature. Click it to activate Continuous Zoom. When Continuous Zoom is activated, the icon changes to Reset Continuous Zoom.

Some Pie menu tools also appear on the MG Viewer toolbar and keypad. The following table describes each tool.



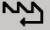

Icon	Description
	<b>Magnifier</b> – Magnify a selected area of any image. See <a href="#">4.3.1. Magnifier and Inverted Magnifier</a> .
	<b>Continuous Zoom</b> – Resizes the image around the focal point. See <a href="#">4.3.3. Continuous Zoom</a> .
	<b>Continuous Zoom Reset</b> – Resets zoomed image to the initial state.

Icon	Description
	<b>Inverted Magnifier</b> – Invert a magnified area.
	<b>Invert Image</b> – Invert an image.
	<b>Window/Level</b> – Adjust brightness and contrast for any image. See <a href="#">4.3.4. Window/Level and Gamma Adjustments</a> .
	<b>Window/Level (numeric)</b> – Use numeric entries to adjust brightness and contrast precisely for any image.
	<b>View Actual Pixels</b> – For an image in single tiling mode, display one pixel of the original image data as one pixel in the viewport. See <a href="#">4.2.9. Scaling Modes</a> .
	<b>Ellipse</b> – Draw an elliptical marking. See <a href="#">4.5.1. Marking an Image</a> .
	<b>Freehand</b> – Draw a freehand marking.
	<b>Arrow</b> – Draw an arrow marking.
	<b>Measurement</b> – Draw a line with a measured length.
	<b>Annotation User Filter</b> – View Annotations for the currently displayed images. See <a href="#">4.5.3. Viewing Annotations</a> .
	<b>Send All Notices</b> – Send Notices for all images to one or more DICOM devices. See <a href="#">4.6. Sending and Viewing Notices</a> .
	<b>Send Image Notice</b> – Send a Notice for the currently selected image.
	<b>Link Tile</b> – Link image tiles to allow simultaneous scrolling through reconstructed slices. See <a href="#">5.2.8. Scrolling Through Linked Tiles</a> .
	<b>Tag Tomo Images</b> – Tag tomosynthesis slices for printing or exporting. See <a href="#">5.4. Tagging Tomosynthesis Reconstructed Slices</a> .
	<b>Export Movie</b> – Export a scrolling movie of tomosynthesis reconstructed slices or tomosynthesis projection images. See <a href="#">5.2.9. Exporting a Movie</a> .
	<b>Image Tools Menu</b> – Open Image Tools menu.
	<b>Rotate 90° Clockwise</b> – Rotate an image clockwise.
	<b>Rotate 90° Counterclockwise</b> – Rotate an image counterclockwise.


Icon	Description
	<b>Rotate 180°</b> – Rotate an image 180°.
	<b>Flip Up/Down</b> – Flip an image on its horizontal axis (or for tomosynthesis, flip the stack of images).
	<b>Flip Left/Right</b> – Flip an image on its vertical axis (or for tomosynthesis, flip the stack of images).
	<b>Reset Flip/Rotate</b> – Reset any flipped or rotated image to its initial orientation.

#### 4.2.4. Using a ReportFlow

When you open a patient, SecurView automatically selects a ReportFlow (a series of image hangings). The ReportFlow that appears by default for a given patient depends upon a user preference (see [7.7. ReportFlows Preferences](#)). The ReportFlow name appears on the toolbar at lower left.

Patient ID →	# Patient 214			FlowBack &
ReportFlow →	3/17 SCR DIAG MLO CC P-C O-N, System			← FlowNext

- To navigate step-by-step through the ReportFlow, click **FlowNext** or **FlowBack** on the toolbar, or press **FlowNext** or **FlowBack** on the keypad.
- You can select an alternative predefined hanging from the toolbar or keypad. SecurView remembers the current ReportFlow step and, when you click **FlowNext** again, proceeds with the next ReportFlow step.
- At any time, you can select an alternative ReportFlow from the pop-up list shown above, which shows all available ReportFlows.

 **Important:** If you use a ReportFlow created by another reviewer, be aware that the ReportFlow can be changed without notice.

For more information about ReportFlows, see [Chapter 7: Hanging Snapshots and ReportFlows](#).

#### 4.2.5. Patient Reading and Lock States During Review

During patient review, SecurView indicates the patient's reading state with a symbol that appears before the patient name, as shown above and in the following examples:

This symbol...	Indicates that the reading state is...
Smith, Jane	'Not Read' or 'Changed' (no symbol appears)
# Jones, Alice	'Read', 'Read Once' (by the current user), or 'Old'
* Kumar, Revati	Locked as 'Consultation Required', 'Additional Images Required', or 'Pending' or 'Locked' – in the latter case, the patient is being read on another workstation
++ Brown, Kelly	Locked as 'Additional Images Arrived'
@ Wong, Brenda	'Notice Arrived'

For more about reading states, see [3.2.4. Reading States](#).

A Radiologist user can lock a patient from the Close Study dialog (see [4.7. Closing a Study](#)). When a radiologist locks a patient, SecurView prevents other users from closing the study and marking it as 'Read'. Other users can make and send Annotations, but SecurView does not give them access to the Close Study dialog. However, other users can unlock the patient via the Shortcut Menu (see [3.2.6. Using the Shortcut Menu](#)).


The lock state is independent of the reading state. When another user reviews a locked patient, the lock state appears in the upper outside corners of the displays.

#### **4.2.6. Panning Images**

At any time during image display, you can move an image within a tile. Simply right-click and drag the image to a new location within the tile.

### 4.2.7. Image Hangings

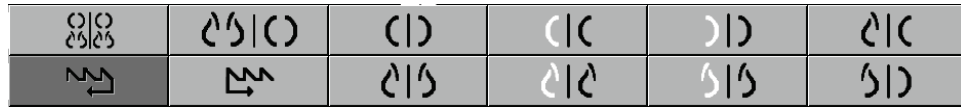
At any time during reviewing, you can select a predefined hanging from the left toolbar.



► To select a predefined image hanging:



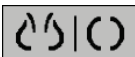

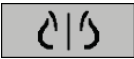
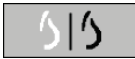

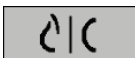
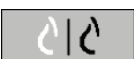
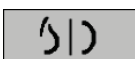
Overview



MLO CC

- Click **Overview** to display all eight images of a standard screening patient (four current and four prior images). You can customize the image hanging assigned to the Overview button with a user preference (see [7.7. ReportFlows Preferences](#)).
- Click **MLO CC** once to display the *current* MLO and CC images in double-tiling mode (two MLO images on the left display, and two CC images on the right).
- Click **MLO CC** again to display the *prior* images in the same manner.
- Double-click any image to switch to single-tiling. Double-click the image again to return to the previous tiling.

The following table describes the remaining predefined hangings.

Icon	Meaning	Icon	Meaning
	CC Current		LCC Prior Current
	MLO Current		LMLO Prior Current
	RCC Prior Current		RMLO RCC Current
	RMLO Prior Current		LMLO LCC Current

When you click the button more than once:

- The viewport will display the available prior images of the same laterality and view in reverse chronological order.
- If a prior study does not contain an image of the displayed laterality, but does contain an image of the same view, the viewport will be blank.
- If a prior study does not contain an image of the displayed view (of either laterality) then the prior study will be skipped.


The appearance of the predefined hanging icons depends upon a user preference (see [6.2. Image Presentation Preferences](#)). In the example above, the user has configured SecurView to display the right breast on the left side, prior images on the left, MLO on the left, CC on the right, with chest wall orientation the same for currents and priors. Note that the icon indicates prior images in *light gray*.



The examples at left show how two predefined hanging icons might appear when the user preference is set with the chest wall orientation back-to-back. In this case, current images are on the left, and priors are on the right. The icons indicate the left or right breast with the letter 'L' or 'R' embedded.

#### 4.2.8. Intelligent Roaming

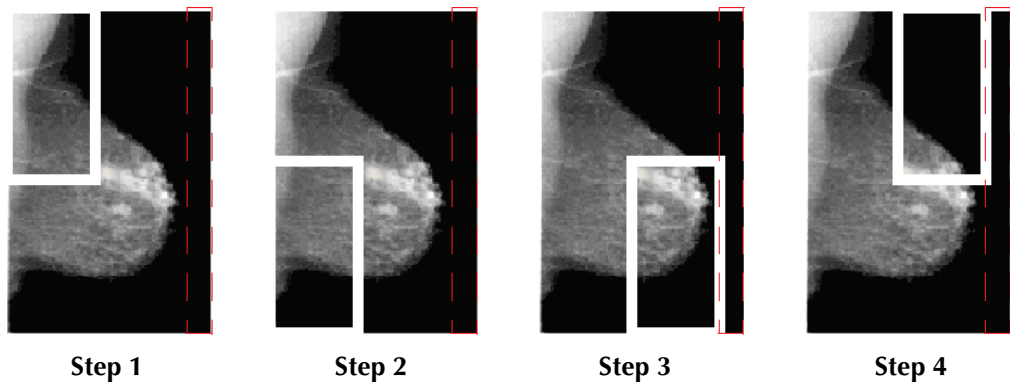Use Intelligent Roaming to pan an image displayed in View Actual Pixels mode in a predefined, step-by-step order. SecurView segments the image and displays only the breast plus a safety margin, ignoring black border areas.




SecurView may divide the breast into two or four zones depending upon the breast size. Intelligent Roaming starts in an upper corner of the image and, depending on the laterality, steps clockwise or counter-clockwise.

The Intelligent Roaming buttons are enabled on the right toolbar when the displays are in single tiling mode.

##### ► To step through the image:

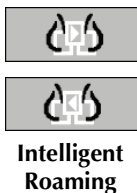Display any image in single-tiling mode. Then:

- Click **Intelligent Roaming Forward** to move one step forward.
- Click **Intelligent Roaming Backward** to move one step backward.




When you start Intelligent Roaming, the displays switch automatically to View Actual Pixels mode.

The Intelligent Roaming indicator shows a thumbnail of the image. A white bordered area within the thumbnail indicates the position of each Intelligent Roaming step.

By default the indicator disappears after 1.5 seconds. To adjust how long the indicator appears see [6.3. Tools and Overlays Preferences](#).

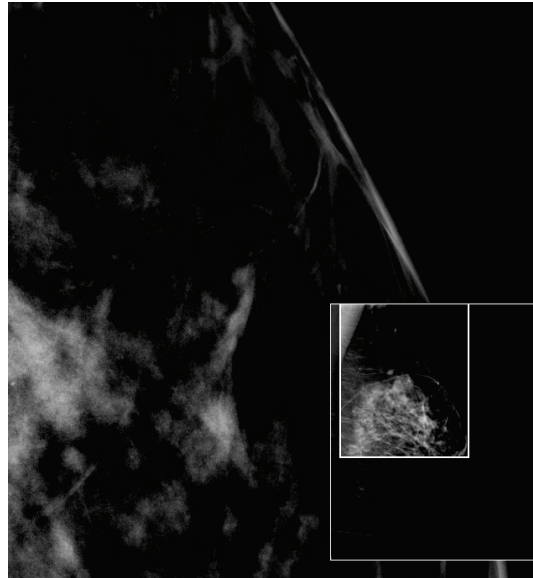**⚠ Important:** *If part of an image is not visible, right-click and drag the image to a new location within the tile. The Intelligent Roaming indicator appears to show the position.*



**Intelligent Roaming Indicator**



### 4.2.9. Scaling Modes

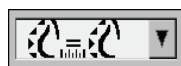







Image Scaling

Use the image scaling options to change the resolution of displayed images. When you click the **Image Scaling** icon, the scaling options appear as a pop-up. The table below describes the behavior of each icon.

Icon	Purpose / Keyboard Shortcut
	<p><b>Fit to Viewport</b> – Resize each image to fill its viewport. If the width and height dimensions of the image are smaller than the viewport, the minimum allowed expansion factor (default = 1.5) determines the behavior. If the factor is exceeded, the image will be expanded to fill the viewport. Otherwise the image will be displayed in native resolution. A service engineer can change the minimum allowed expansion factor if necessary.</p>
	<p><b>Right Size</b> – Display the images on each monitor at the same resolution based on the breast outline of the largest image with valid breast outline available for the patient.</p> <ul style="list-style-type: none"> <li>Press [Y] to toggle Right Size and Same Size.</li> </ul>
	<p><b>Same Size</b> – Display the images on each monitor at the same resolution based on the entire imaged area of the largest image available for the patient.</p> <ul style="list-style-type: none"> <li>Press [Y] to toggle Right Size and Same Size.</li> </ul>
	<p><b>True Size</b> – Display all images such that the breast tissue is in its true physical size, i.e., 1 cm of breast tissue measures 1 cm on the display.</p> <ul style="list-style-type: none"> <li>Press [X] to display images in True Size.</li> </ul>
	<p><b>View Actual Pixels</b> – Display 1 pixel of the original image as 1 pixel in the viewport (available when both displays are in single tiling mode).</p> <ul style="list-style-type: none"> <li>Press [D] or [7] to View Actual Pixels on the left display. Press again to return to the previous scaling mode.</li> <li>Press [F] or [9] to View Actual Pixels on the right display. Press again to return to the previous scaling mode.</li> </ul> <p><b>Note:</b> If part of an image is not visible, right-click the image and drag it to a new location within the tile.</p>

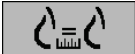

If SecurView displays an image that is not in the selected scaling mode, an icon with an 'X' appears next to the displayed image (see examples at left). For example, the icon appears when you display an image on the left monitor in View Actual Pixels mode while the right monitor is displaying four images in any of the other modes.

**Note:** To achieve a more suitable image presentation in Double Tiling, adjust the user preference **Adjust Image Size in Double Tiling**. Depending on this setting, images may not be fit into the Viewport as described in the above rules. For more information on user preferences see [6.2 Image Presentation Preferences](#).

## Displaying Diagnostic Images at Full Resolution

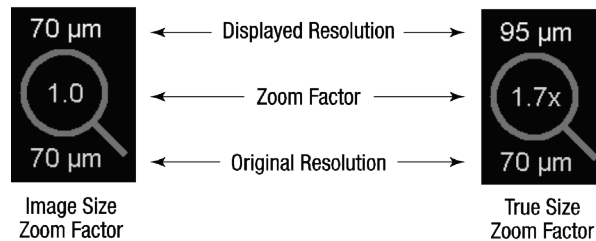
When reviewing FFDM images, adhere to ACR guidelines. To ensure optimum display of diagnostic information, review each image with the View Actual Pixels scaling mode.

When displaying images using a fractional Image Size zoom factor larger than one, SecurView enlarges the original images by interpolating pixel values between the source pixels. The pixel characteristics in the enlarged images may differ from the original image pixels.

- For more information on the zoom factor, see **4.2.10. Pixel Meter** on the next page.
- For instructions on setting View Actual Pixels as the default when creating a ReportFlow, see **7.6. Creating New ReportFlows**.

### 4.2.10. Pixel Meter

The Pixel Meters display sizing information for each image:




#### Pixel Meters

You can select the zoom factor setting with a user preference (see **6.3. Tools and Overlays Preferences**).

- **Image Size** zoom factor indicates the magnification relative to the pixel size of the original image. In the example above, the Image Size zoom factor is '1.0', which indicates that the image appears in View Actual Pixels mode.
- **True Size** zoom factor indicates the magnification relative to the true physical size of the image, displayed above as '1.7x'.

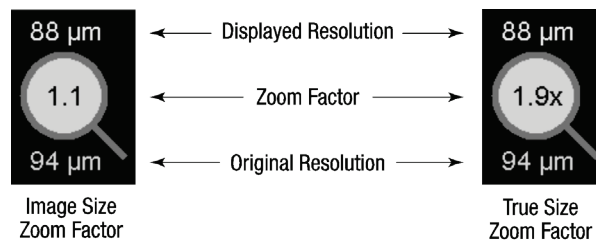When displaying images using a fractional Image Size zoom factor larger than one, SecurView enlarges the original images by interpolating pixel values between the source pixels. The pixel characteristics in the enlarged images may differ from the original image pixels. When this occurs, the background inside the Pixel Meter turns white, as shown below.



#### Pixel Meters with White Background Indicating Interpolated Pixel Values

### 4.2.11. Stack and Timepoint Indicators

When an image tile set contains more than one image, a Stack indicator appears in the corresponding tile. Either one or two indicators may appear, depending upon your user preference (see [6.3. Tools and Overlays Preferences](#)).



**Stack Indicator**



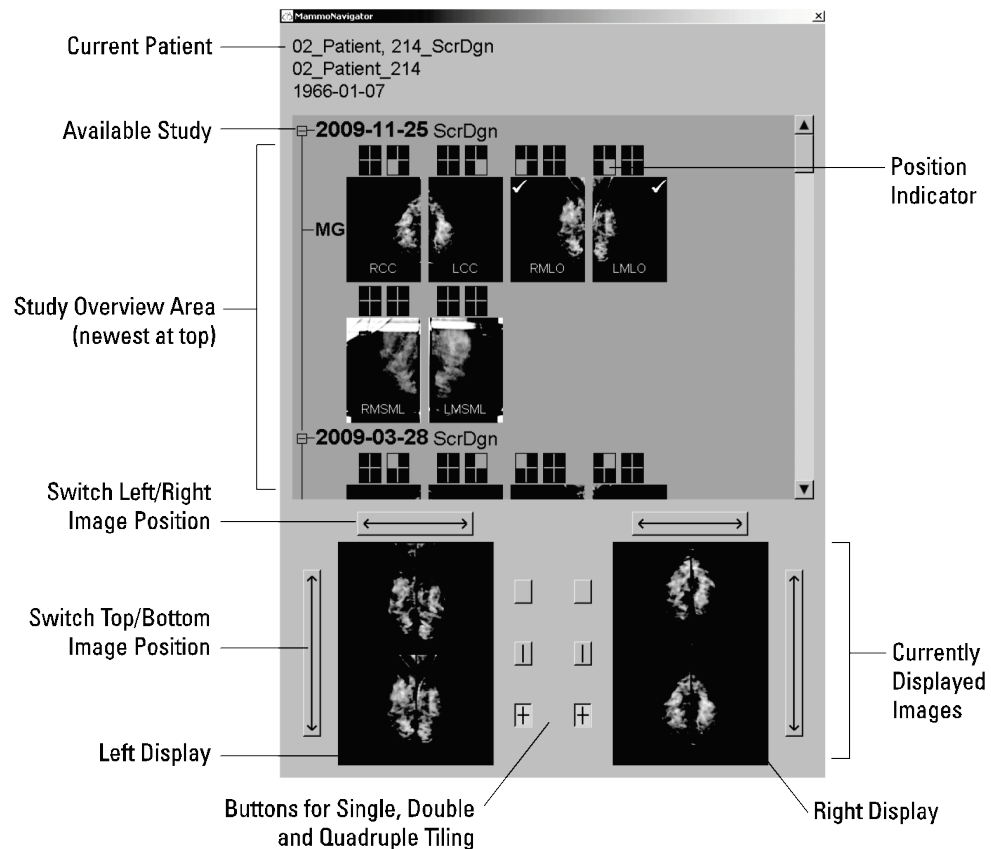
**Stack & Timepoint Indicators**

- Click the up (or down) arrow to go to the next (or previous) image in the stack.
- Click the right (or left) arrow to go to the next (or previous) timepoint.

You can also move through the images by pointing to an indicator or image and rotating the mouse wheel or the keypad scroll wheel.

### 4.2.12. MammoNavigator

The MammoNavigator provides quick access to all images and studies of the current patient.



#### ► To open the MammoNavigator:





- Click **MammoNavigator** on the right toolbar. Click again to close the window.

ReportFlows can be customized to open (or close) the MammoNavigator automatically by adding the Open (or Close) Navigator functional ReportFlow step.

At the bottom of the MammoNavigator are thumbnails of the images currently appearing on the displays. At the top is an overview of all study images for the current patient including the study date and procedure description. Also appearing for each image are the laterality, view code, and any modifiers. A check mark appears on any thumbnail after you view the corresponding image in single tiling mode.

- Drag an image from top to bottom to display the image.
- Click a tiling button to change the display tiling (single tiling, double tiling, etc.).

 **Note:** Starting with SecurView version 8.2, double tiling images are sized according to the selected sizing mode (Fit to Viewport, Right Size, Same Size, or True Size). Depending on the original image size and mode selected, the image may be displayed at a lower resolution than in previous versions.

 **Note:** Starting with SecurView version 8.4, the viewport size used to scale images presented in Double Tiling can be configured, see [6.2. Image Presentation Preferences](#). Depending on the user preference, the original image size, and the selected scaling mode, the image may not fit into the viewport.

- Click the horizontal arrow to switch left and right image positions (for double and quadruple tilings).
- Click the vertical arrow to switch top and bottom positions (quadruple tiling only).
- Double-click an image in the lower area to remove the image from the display.

Tomosynthesis thumbnails are labeled with 'M' for conventional 2D mammography images, 'C' for C-View 2D™ images, 'T' for tomosynthesis reconstructed slices, and 'P' for tomosynthesis projection images.


A service engineer can configure the order in which the thumbnails appear.

### 4.2.13. Image Information

To display detailed DICOM information for any image, open the MammoNavigator and right-click the image thumbnail that appears in the Study Overview area. The example below shows how image information might appear:

Image information	
Name	Value
Modality	MG
Laterality	Left
View Position	LMLO
Institution	HOLOGIC, Inc.
Institution Address	35 Crosby Drive, Bedford, MA 01730
Referring Physician	NWH_913874
Acquisition Date	2009-11-25
Acquisition Time	09:53
Body Part	BREAST
Compression Thick	42.0 mm
kVp	27
Half Value Layer	0.324 mm
Exposure	164 mAs
Exposure Time	1643 ms
AGD	0.00 mGy
ESD	0 mGy
Exposure Index	460
Anode Material	MOLYBDENUM
Filter Material	MOLYBDENUM
Compression Force	125 N
C-Arm Angle	45
Focal Spot	0.3 mm
Grid	HTC_IN
Paddle	18cm x 24cm fast
Exposure Control Mode	AUTO_FILTER
Exposure Control Mode Description	LORAD AUTO AEC
Manufacturer	HOLOGIC, Inc.
Unit	Selenia
Device Serial Number	H1KRHR835b72e2
Model Name	Lorad Selenia
Detector ID	MP1570
UID	1.2.840.113681.2203808482.774.3335080556.154.1
Presentation Intent	FOR PRESENTATION
Software Versions	AWS:3_1_6_0 (AWS 3_1_5_2), PXC:1.2.9.0, ARR:1.4.2.6, IP:4.5.2
Last Detector Calibration	2006-08-30
Gantry	n/a
Close	

An Administrator can configure the information that appears in the window. See [8.7.2. MammoNavigator Overlay](#).

 **Note:** To view DICOM information that is not available in the MammoNavigator overlay, ensure that the mouse cursor is placed over the image, and press **[H]**.

#### 4.2.14. Patient Information Overlays

Patient and image information appear as overlays that you can switch on and off.

► To display Patient Information overlays:



**Patient  
Information**

- Click **Patient Information** on the right toolbar to open the Patient Information overlay in 'reduced mode'.
- Click again to open the Patient Information overlay in 'extended mode'.
- Click again to close the overlays.
- Click and hold **Patient Information** for two seconds to 'Clear All' overlay information from the displays (except the study state and Stack indicators).

**Note:** For keypad users, the Clear All feature is available only on the current keypad (with a scroll wheel).



State Indicator (Old, Read, Pending etc.)

Digital Marker, Technologist & Current/Prior Identifiers (1 = Prior Study, 2 = Previous Prior Study, etc.)

Patient Information (upper part) with Patient Data

Patient Information (lower part) with DICOM Data

Stack Indicators and Pixel Meter

Ruler

Description

The elements that appear depend in part on your user preferences (see **6.3. Tools and Overlays Preferences**). In addition, an Administrator can customize the information that appears on the upper and lower Patient Information overlays (see **8.7.1. MG Viewer Image Overlay**).

A color or grayscale highlight of study date, digital marker, and stack indicator for more prominent distinction between current and prior images can be configured by a Hologic service engineer. If the highlight is activated, it is configured by default to apply only to study date.

When activated, the highlight for current and prior images can be toggled on or off using the keyboard shortcut [CTRL + t]. If the highlight is turned off, it will be turned on by default when the system is restarted.

**⚠ Important:** If Patient Information overlays hide image data, click **Patient Information** to toggle the overlays off.

**⚠ Note:** A service engineer can configure the system to display a warning message if no current studies are available for the Patient.

#### 4.2.15. MG Secondary Captures and MM ScreenCaptures



MG Secondary  
Capture

MG Secondary Capture images can be created on close of study if the 'Destinations for an MG Secondary Capture (Annotations and Tagged Tomo Slices)' setting is configured (refer to the *SecurView DX/RT Workstation Installation & Service Manual*). MG Secondary Capture images are needed when the destination PACS does not accept GSPS, or if the PACS cannot display GSPS and a user wishes to view Annotations on the PACS workstation. When SecurView retrieves an MG Secondary Capture image from a DICOM source, an icon (shown at left) appears to identify the received image.



MM ScreenCapture

Similarly, MM ScreenCapture images can be created on close of study when the 'Destinations for an MM ScreenCapture' setting is configured (refer to the *SecurView DX/RT Workstation Installation & Service Manual*). When SecurView retrieves an MM ScreenCapture image from a DICOM source, an icon (shown at left) appears to identify the received image.

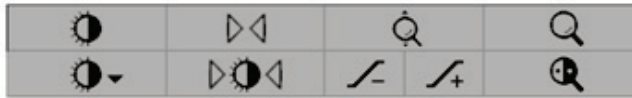
For more information, see **4.7. Closing a Study**.

### 4.3. Visualizing Image Details

This section explains tools used for visualizing image details, including magnification and brightness/contrast adjustments.


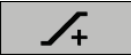
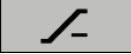







- ▶ **4.3.1. Magnifier and Inverted Magnifier**
- ▶ **4.3.2. AIE and the Magnifier Toolbar**
- ▶ **4.3.3. Continuous Zoom**
- ▶ **4.3.4. Window/Level and Gamma Adjustments**
- ▶ **4.3.5. Applying VOI LUTs**
- ▶ **4.3.6. MPE Images**
- ▶ **4.3.7. DICOM 6000 Overlays**
- ▶ **4.3.8. CLAHE Image Enhancement**

The right toolbar provides a set of image evaluation tools:





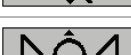The following table describes the purpose of each tool:

Icon	Purpose
	<b>Window/Level:</b> Adjust the brightness and/or contrast of any image.
 	<b>Increase / Decrease Gamma Factor:</b> Adjust the brightness and contrast of <i>all</i> images of the current patient.
	<b>Reset Window/Level &amp; Gamma:</b> Reset any changes to Window/Level and Gamma settings to their initial values.
	<b>VOI LUT:</b> Apply an alternative VOI LUT (e.g., Window/Level setting).
	<b>Reset:</b> Undo changes you made and reset images for the current patient to their initial states when opened (Annotations remain).
	<b>Magnifier:</b> Magnify a selected image area.
	<b>Inverted Magnifier:</b> Invert the magnified area.
	<b>Continuous Zoom:</b> Activates Continuous Zoom.
	<b>Reset Continuous Zoom:</b> Resets all zoomed images to the initial state.

The Pie menu provides additional image visualization tools (see [4.2.3. Using the Pie Menu](#)).

### 4.3.1. Magnifier and Inverted Magnifier

Use the Magnifier to enlarge any displayed image area by a factor of two. (For tomosynthesis data, the Magnifier uses pixel replication). Scaling marks in millimeters appear inside the magnified area.

► **To magnify an image area:**



Magnifier

- Click **Magnifier** to change the pointer to the Magnifier icon. Point to the area you want to magnify and click. Hold the mouse button and move the pointer on the image to update the magnified area dynamically. Then release the mouse button. The magnified area remains in the current position.
- Point to another image and click to place a new Magnifier on that image.

► **To invert a magnified area:**



Inverted Magnifier

- Click **Inverted Magnifier** change the pointer to the Inverted Magnifier icon. Point to the area you want to invert and click. As with the Magnifier, you can move the pointer to update the inverted area dynamically.

► **To invert an entire image:**

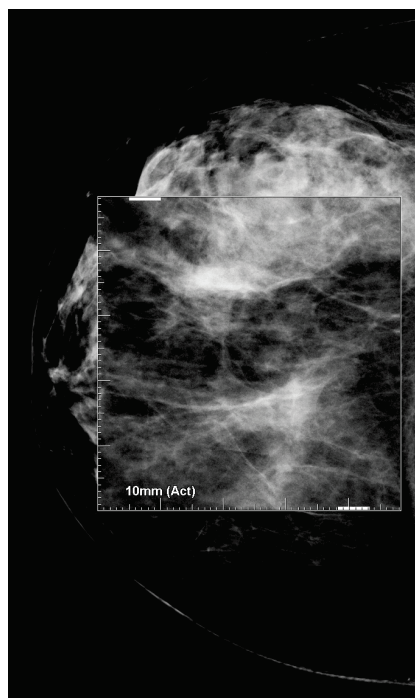


Invert Image


- Right-click an image to open the Pie menu, and point to **Image Tools** to open the second Pie menu. Then click **Invert Image**.

► **To invert all displayed images:**

- Press [I] on the keyboard.



Magnifier

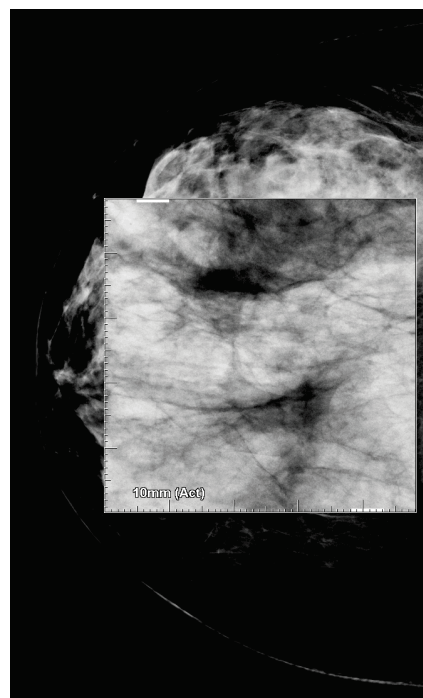

Inverted Magnifier

► **To close the Magnifiers:**






- Double-click a magnified area (a service engineer can configure the double-click to close either one or all Magnifiers), or
- Change the image tiling (or move to the next ReportFlow step).


The Magnifier or Inverted Magnifier can be the default tool when you open the MG Viewer (user preference). You can also switch the scaling marks off or on. See [6.3. Tools and Overlays Preferences](#).

### 4.3.2. AIE and the Magnifier Toolbar

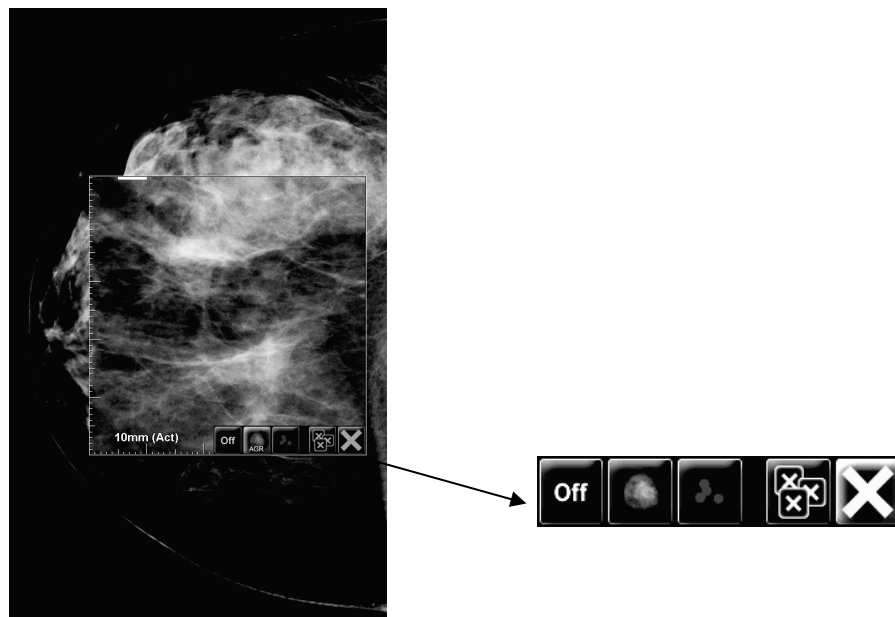Advanced Image Enhancement (AIE) filtering assists in visualizing masses or calcifications. The AIE features appear only if a service engineer has configured your workstation with a special license.

Move the pointer to the top or bottom of the magnified area to open the AIE toolbar. The following table explains the function of each AIE tool.

Icon	Purpose
	<b>AIE On/Off</b> – Switch AIE filtering on/off.
	<b>AIE Masses</b> – Switch AIE mass filtering on/off ('AGR' = aggressive).
	<b>AIE Calcs</b> – Switch AIE calcification filtering on/off ('MDR' = moderate).
	<b>Close All Magnifiers</b> – Close all open Magnifiers.
	<b>Close Magnifier</b> – Close the currently selected Magnifier.



**Magnifier with AIE Tools**

### 4.3.3. Continuous Zoom

Use Continuous Zoom to continuously increase or decrease the zoom factor of a displayed image. Continuous zooming is limited to a maximum (20x) and a minimum (50%) zoom factor. Contact Hologic Technical Support if you would like to change the default settings.

**Note:** Continuous zoom is automatically applied to all combination procedure images of the same laterality and view within a single tile. For example, if you zoom one reconstructed slice and scroll to a different reconstructed slice, the new slice is also zoomed, and if you switch to the 2D or C-View 2D image in the same tile, that is also zoomed.

**Note:** Annotations are resized when changing the zoom factor of images. If an annotation is created on a zoomed image and the zoom factor is changed it may happen that either the annotation becomes very small or it will cover parts of the image that a radiologist wants to see. Thus, annotations should not be created on zoomed images.

► **To activate Continuous Zoom:**



**Continuous Zoom**

- 1 Click **Continuous Zoom** on the toolbar or press [F7] on the keyboard. The pointer changes to the Continuous Zoom icon.
- 2 Move the pointer to the focal point on the image, click and hold the mouse button, and drag up and down to change the zoom factor:

- Dragging up – increases the zoom factor
- Dragging down – decreases the zoom factor

**Note:** To zoom out to a smaller size than initially displayed, first zoom out to the initially displayed size and release the mouse button. Then click the mouse button and drag again to view a smaller size image.

- 3 Release the mouse button when you have reached the desired size. The image remains displayed in the new size.

**Note:**

- After activating the Continuous Zoom mode, the toolbar button changes to the Reset Continuous Zoom icon.
- The Flip and Rotate tools are disabled while Continuous Zoom is active.
- Only the size of CAD mark outlines is affected by Continuous Zoom. RightOn CAD marks are not zoomed.

► **To reset Continuous Zoom:**



**Reset Continuous Zoom**

- 1 Click **Reset Continuous Zoom** on the toolbar or press [F7] on the keyboard to reset all images in all tiles to the initial state.
- 2 Click **Reset Continuous Zoom** (displayed within a tile) to reset only the image(s) in the tile to the initial state.

**Note:** All panning performed while using Continuous Zoom will be reset. Panning, flipping, and rotating performed before zooming will be maintained.

Continuous Zoom can be the preferred default tool when you open the MG Viewer (user preference). See **6.3. Tools and Overlays Preferences**.

#### 4.3.4. Window/Level and Gamma Adjustments

There are two tools used to adjust image brightness and contrast:

- Use the Window/Level tool to adjust brightness and contrast for any image. With a user preference, you can set the Window/Level tool to be the default when you open the MG Viewer (see **6.3. Tools and Overlays Preferences**).
- Use the Increase/Decrease Gamma tool to adjust brightness and contrast for all images of the current patient.

► **To adjust brightness and contrast for any image:**

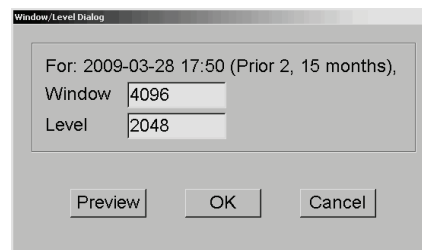


- 1 Click **Window/Level**, then, point to an image and drag.
  - Dragging left or right changes the window width (contrast) – left increases contrast, right decreases contrast.
  - Dragging up or down changes the window level (brightness) – up brightens the image, down darkens the image.
- 2 Release the mouse button when you have adjusted the image to your satisfaction. The image remains displayed with the new Window/Level values.

► **To adjust brightness and contrast by numeric entries:**




- 1 Right-click an image to open the Pie menu, and point to **Image Tools** to open the second Pie menu. Then click **Window/Level Numeric**.



- 2 Type values in the Window and Level fields.
  - Click **Preview** to test your settings.
  - Click **OK** when you are satisfied with the new settings.

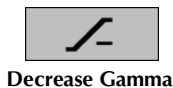► **To adjust brightness and contrast for all images of the current patient:**



- Click **Increase Gamma** to increase brightness and contrast by one step.
- Click **Decrease Gamma** to decrease brightness and contrast by one step.

► **To reset Window/Level and/or Gamma settings to their default values:**



- Click Reset Window/Level, or
- Click **Reset**, or
- Switch to another patient.

### 4.3.5. Applying VOI LUTs


An image may contain one or more Value of Interest Look-Up Tables (VOI LUTs). A VOI LUT may be a predefined Window/Level setting or a non-linear LUT. Typically, images from a single series offer the same VOI LUT, although multiple series within one study may offer different VOI LUTs. When the MG Viewer opens, SecurView applies a default VOI LUT and makes any other embedded VOI LUTs available. A service engineer can configure the default VOI LUT.

► **To change from the default to a different VOI LUT:**



Select VOI LUT

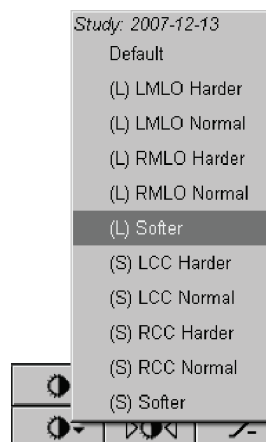- 1 Click **Select VOI LUT**. A list of VOI LUTs appears, if they are available in the images.



- 2 Select a LUT from the list to apply the new VOI LUT.

### 4.3.6. MPE Images

Mammography Prior Enhancement™ (MPE) is a software module that processes conventional two-dimensional digital mammography x-ray images. MPE is designed specifically for use with screening views from prior exams originating from GE Senographe Full-Field Digital Mammography (FFDM) systems. MPE images appear only if a service engineer has configured your workstation with a special license.

Input to the MPE module includes image pixel data, imaging information, and image processing parameters. The module performs image processing consisting of steps to improve the appearance of the image through logarithmic conversion, skinline correction, and contrast enhancement. These are standard methods used to allow optimal display and review of mammography images with minimal window/leveling operation.

#### Intended Use

Mammography Prior Enhancement is a software module intended to enhance the appearance of prior non-Hologic digital mammography x-ray images so that they more closely resemble Hologic digital mammography images. MPE-processed images are intended for comparison purposes only and cannot be used for primary diagnosis.

MPE runs on a Windows-based computer. Results can be displayed on a workstation capable of displaying mammography x-ray images, such as Hologic's SecurView DX workstation.

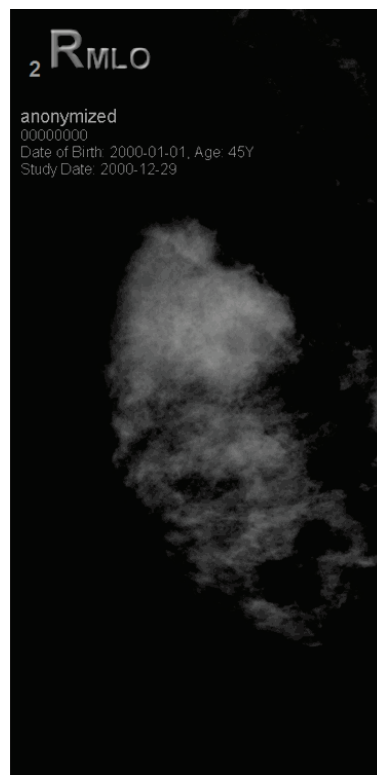
## Viewing MPE Images

The radiologist views MPE-processed images when comparing them to current digital mammography images. The SecurView DX workstation applies MPE processing automatically to all images that meet the MPE criteria, and displays the images using its standard hanging protocols as defined for prior-current comparisons.


**⚠ Important!** Do not make a clinical decision or diagnosis from MPE-processed images alone. Always base interpretation on current study images to which MPE processing has not been applied.

**⚠ Note:** In rare cases, the initial presentation of some MPE-processed images may not be optimal. Use manual window/level adjustment to improve the appearance of these images.

Shown below are images taken from the SecurView DX workstation. The image on the left shows an example of how an image appears before MPE processing. On the right is an example of the image following MPE processing.



**Without MPE Processing**

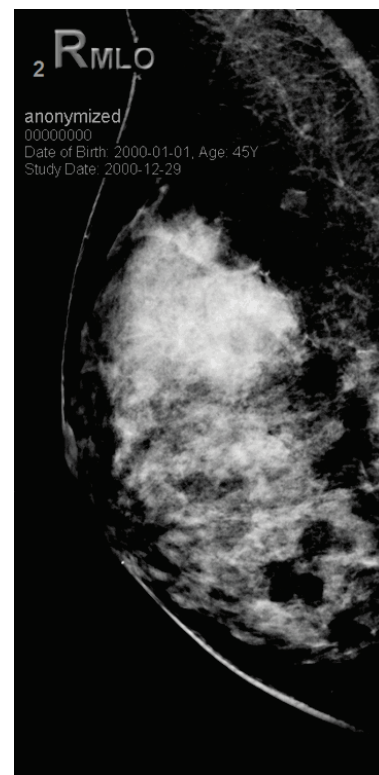

**With MPE Processing**




### 4.3.7. DICOM 6000 Overlays

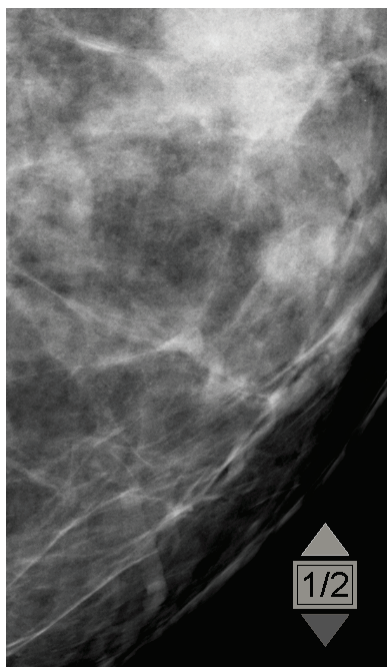
The SecurView workstation displays overlays contained in the DICOM 6000 group within an image header. For images containing a DICOM 6000 overlay, SecurView creates an internal copy of the image with the overlay burned in.

If the display of DICOM 6000 overlays is not desired a service engineer can disable the feature.


#### Viewing DICOM 6000 Overlays

The internal copy of the image with the burned-in overlay is stacked with the corresponding original image.

 **Note:** If a user does not scroll through all images in a stack, annotations contained in DICOM 6000 overlays could be missed.



**Original Image**

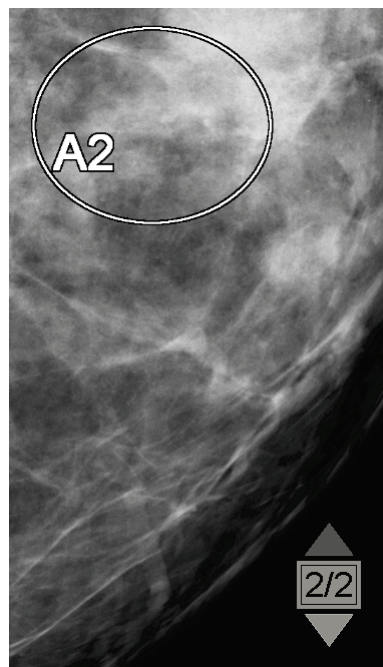

**Image with DICOM 6000 Overlay**

SecurView is limited to displaying only one DICOM 6000 group containing a graphical overlay that has the same size as the original image.

The overlays are burned in using white enclosed in a black outline. To enhance visibility of the overlays a service engineer can configure the thickness and outline width.


The SecurView user is only permitted to create markings, annotations, and measurements on the original image. It is not possible to annotate the internal copy of the image with the burned-in overlay.



### 4.3.8. CLAHE Image Enhancement

The SecurView workstation supports Contrast Limited Adaptive Histogram Equalization (CLAHE) image enhancement. Images with CLAHE enhancement display the overlay 'CLAHE.' If CLAHE enhancement is configured for an image but is not applied successfully, the original image is displayed along with a system message 'Image Processing Failed.'

A service engineer can configure CLAHE enhancement.

 **Note:** In rare cases the initial presentation of some CLAHE enhanced images may not be optimal. Use manual window/level adjustment to improve the appearance of these images.

## 4.4. Using CAD

SecurView workstations accept Mammography CAD SR objects produced by Hologic ImageChecker® CAD, Hologic Quantra™, iCAD SecondLook, and other applications. If the CAD SR includes CAD results, then SecurView can display the CAD results for each image.

CAD functionality is license-protected. The license controls access to CAD functionality by enabling the CAD button on the toolbar or keypad. For more information on each CAD application, see the CAD vendor user guide.

- ▶ **4.4.1. Displaying CAD Information**
- ▶ **4.4.2. Hologic ImageChecker CAD**
- ▶ **4.4.3. Hologic Imaging Biomarkers**
- ▶ **4.4.4. Switching Between Multiple Mammography CAD SRs**

### 4.4.1. Displaying CAD Information

If there are CAD results for a patient, a '+' symbol appears in the CAD column of the Patient List. In addition, when you review a patient with CAD results, the toolbar CAD button becomes active (not gray). CAD results can be configured to appear automatically as a step in a ReportFlow.

- ▶ **To display CAD results:**



**Computer Aided  
Detection**

While viewing a study with CAD results, click **Computer Aided Detection**. The application displays the CAD overlay. CAD marks appear if they exist for the currently displayed images.

#### 4.4.2. Hologic ImageChecker CAD




Hologic's Citra™ Core license group provides the standard ImageChecker CAD features with the Cenova™ digital mammography server. Citra Core includes:

- RightOn™ CAD Marks
- EmphaSize™ CAD Marks
- PeerView® CAD Marks
- LesionMetrics™

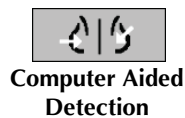
When licensed, the ImageChecker software includes CAD information in the CAD SR output.

##### RightOn CAD Marks


The ImageChecker software provides three types of RightOn CAD marks (Mass, Calc, and Malc). You can choose to display any or all of the three types of marks with a user preference. Each mark identifies a region of interest.

-  **Calc** – Marks regions suggestive of calcifications.
-  **Mass** – Marks regions suggestive of masses or architectural distortions.
-  **Malc** – Marks regions where Calc and Mass marks are coincident.

##### ImageChecker CAD Overlay



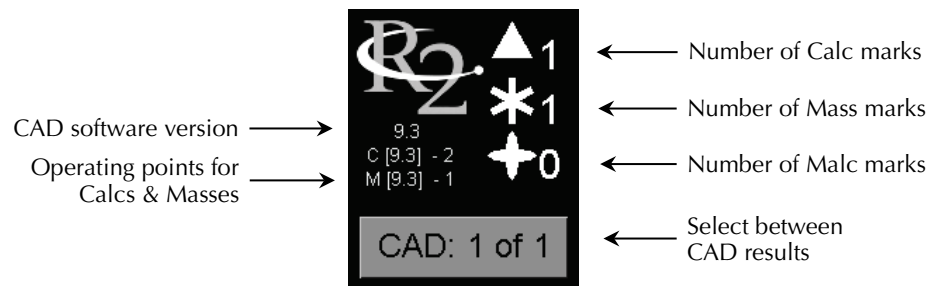When you click **Computer Aided Detection** while viewing a study with ImageChecker CAD results, SecurView displays the ImageChecker CAD overlay. CAD marks appear if they exist for the currently displayed images.




##### ImageChecker CAD Overlay

The numbers of Calc, Mass, and Malc marks appear on the right side. On the left side, SecurView displays the ImageChecker CAD algorithm version (9.3) and the operating points selected for calcifications (C) and masses (M).

If the Cenova server software is not configured to generate Calc or Mass results, then the algorithm version, CAD symbol and number of marks for the corresponding algorithm do not appear. The following example shows how SecurView indicates that the ImageChecker CAD Mass algorithm was not run.

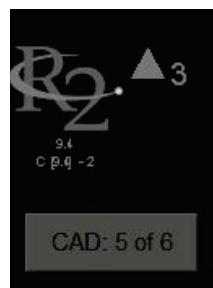ImageChecker CAD  
Mass algorithm not run




If ImageChecker CAD cannot process the image, then SecurView displays broken lines around the CAD symbols and the number of marks for the corresponding algorithm does not appear:




ImageChecker CAD  
unable to process  
the image

### EmphaSize CAD Marks

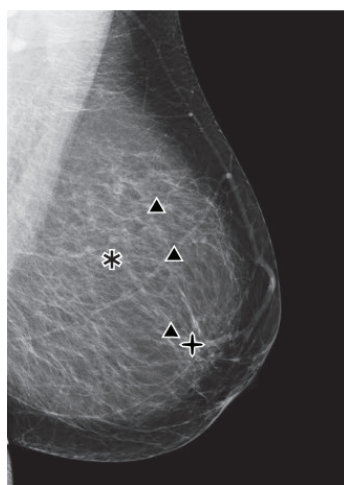This feature allows SecurView to display ImageChecker CAD marks of variable size, where the size correlates to the prominence of the features of the finding. When ImageChecker determines that a region has more prominent features, the CAD mark appears larger, indicating that the region should receive more careful review from the radiologist. SecurView systems display EmphaSize marks by default. You can switch off EmphaSize marks with a user preference (see [6.3. Tools and Overlays Preferences](#)).



CAD without EmphaSize

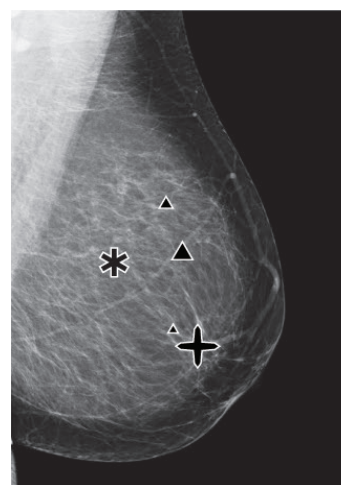


CAD with EmphaSize

## PeerView CAD Marks

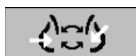
PeerView highlights anatomic findings detected by the algorithm. In the example shown below, the same region is shown with and without a PeerView Malc mark (mass with calcifications). PeerView outlines the central density of a mass, and highlights the individual calcifications of a cluster.

► **To view PeerView results:**




PeerView /  
RightOn

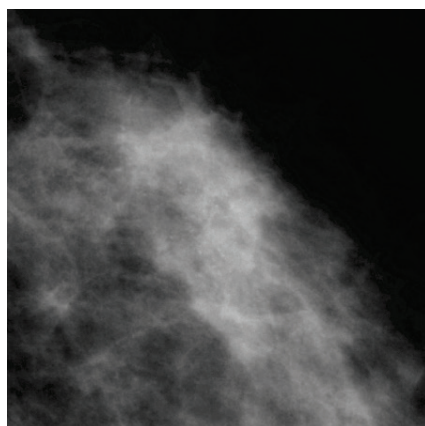Click **PeerView / RightOn**. Click again to view RightOn marks.



**PeerView Off**

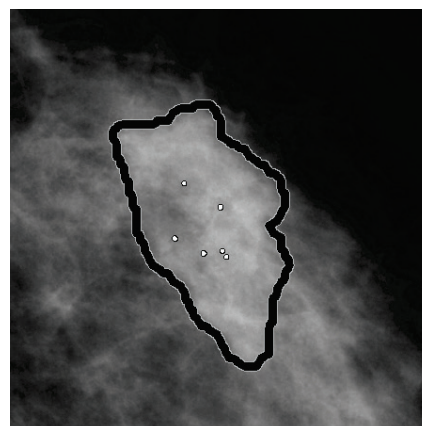

**PeerView On**

PeerView results appear only when CAD display has been activated, and when at least one displayed image shows CAD SR information. If no PeerView information is present in the current CAD SR, SecurView shows only RightOn CAD marks.

## LesionMetrics

LesionMetrics provides data computed by ImageChecker for each region of interest marked by the algorithm. Depending upon the type of lesion, the software can compute metrics such as lesion size, distance to nipple, distance to chest wall, degree of spiculation, calcification contrast, number of calcifications, and mass density.

► **To display LesionMetrics for each ImageChecker CAD finding:**

Double-click a RightOn or PeerView CAD mark. A new window opens next to the selected CAD mark:

Calcification Cluster	
Name	Value
Number of calcifications	12
Size (Long axis)	0.4 cm
Distance to nipple	10.4 cm
Distance to chest wall	4.8 cm
Contrast	9 %
CAD Operating Point	0
<div>Close All OK</div>	

Mass	
Name	Value
Size (Long axis)	2.0 cm
Distance to nipple	7.8 cm
Distance to chest wall	5.6 cm
Measure of density	36 %
Degree of spiculation	31 %
CAD Operating Point	0
<div>Close All OK</div>	

Malc	
Name	Value
Number of calcifications	17
Size (Long axis)	2.2 cm
Distance to nipple	10.7 cm
Distance to chest wall	5.3 cm
Measure of density	24 %
Degree of spiculation	40 %
Contrast	33 %
CAD Operating Point	0
<div>Close All OK</div>	

### 4.4.3. Hologic Imaging Biomarkers

**Note:** Display of biomarkers is subject to availability. Check with your local sales representative for information.

The Hologic Imaging Biomarkers algorithms analyze each image within a study and provide several measures including breast volume, area breast density, volumetric breast density, a BI-RADS®-like measurement of breast density, and calcium scoring. SecurView reports the assessments per patient, per breast, and per image. For more information, see the *Understanding Quantra* and *Understanding BACS* user guides. (Note that these products are sold separately).

► **To display Hologic Imaging Biomarkers results:**




**Biomarkers**

Click **Biomarkers**. The biomarkers results appear on three tabs according to your user settings (see **6.3. Tools and Overlays Preferences**). Using the user settings, you can configure which results will initially be displayed (per subject, per breast (default), or per image).

**Note:** The biomarkers result content may vary depending on the Hologic Imaging Biomarkers algorithm version.

Per Subject   Per Image   Per Breast		
2008-10-06		
	R	L
Quantra	2.1.1	
Vd (cm3)	98	84
Vb (cm3)	1194	1176
Vbd (%)	8.2	7.2
Vd-score	0.2	-0.1
Vbd-score	-0.7	-0.8
Abd (%)	5.2	3.3
qDC	2.0	1.8
QDC (1~a,4~d)	b	b
<b>Calcium Scoring</b>		
Abac (mm2)	2.52	0.61



Per Subject   Per Image   Per Breast		
	2008-10-06	
	Total	
Quantra	2.1.1	
Vd (cm3)	183	
Vb (cm3)	2369	
Vbd (%)	7.7	
Vd-score	0.1	
Vbd-score	-0.8	
Abd (%)	4.3	
qDC	1.9	
QDC (1~a,4~d)	b	
Calcium Scoring		
Abac (mm2)	3.13	

OK

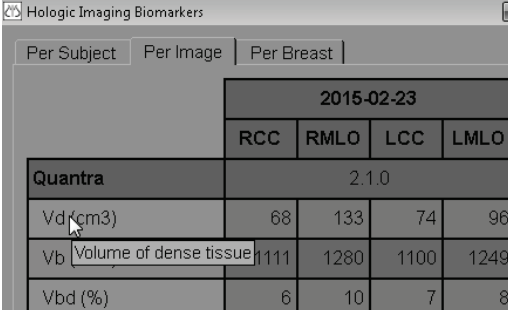
**Hologic Imaging Biomarkers Per Breast & Per Subject Tabs**

Per Subject   Per Image   Per Breast				
2008-10-06				
	RCC	RMLO	LCC	LMLO
Quantra	2.1.1			
Vd (cm3)	68	133	74	96
Vb (cm3)	1111	1278	1100	1251
Vbd (%)	6.0	10.0	7.0	8.0
Vd-score	-0.4	0.8	-0.3	0.2
Vbd-score	-1.1	-0.2	-1.0	-0.7
Abd (%)	2.0	8.0	3.0	4.0
qDC	1.6	2.3	1.7	1.9
QDC (1~a,4~d)	b	b	b	b
<b>Calcium Scoring</b>				
Abac (mm2)	1.43	2.52	0.00	0.61

**Hologic Imaging Biomarkers Per Image Tab**

Results without any value refer to an empty cell, i.e., the report does not contain a value for the desired measurement.

To display the full name of the biomarkers result, point to the abbreviated name as shown below.



The screenshot shows the 'Hologic Imaging Biomarkers' application window. It has three tabs: 'Per Subject', 'Per Image', and 'Per Breast'. The 'Per Subject' tab is selected. The table displays results for a patient on the date 2015-02-23. The columns are RCC, RMLO, LCC, and LMLO. The rows include 'Quantra' (2.1.0), 'Vd (cm3)' (68, 133, 74, 96), 'Vb Volume of dense tissue' (111, 1280, 1100, 1249), and 'Vbd (%)' (6, 10, 7, 8).

	2015-02-23			
	RCC	RMLO	LCC	LMLO
Quantra	2.1.0			
Vd (cm3)	68	133	74	96
Vb Volume of dense tissue	111	1280	1100	1249
Vbd (%)	6	10	7	8

#### 4.4.4. Switching Between Multiple Mammography CAD SRs

SecurView initially displays the marks of a specific CAD SR on each image. The default CAD SR is the newest that references the image, identified by date and time. Within a single study, there can be multiple CAD SRs, referencing different images. For example, a new CAD SR may be produced after a study is reopened at the acquisition workstation to add a new image.

If you select a CAD SR on an image, SecurView displays the contents on all images referenced by this CAD SR. The selected CAD SR is persistent for all referenced images as long as the patient is loaded.

► **To switch to a new CAD SR:**

CAD: 1 of 2  
Select between  
CAD results

- 1 On the CAD information overlay, click **Select between CAD results**. A pop-up menu displays a list of all available CAD SRs for the image. A check mark appears next to the currently displayed CAD SR.

✓	11-07-2011 16:04 R2 Technology, Inc.
	11-07-2011 16:01 R2 Technology, Inc.

- 2 Select one of the list entries to load the referenced CAD SR and display the marks on the image.

## 4.5. Creating and Viewing Annotations

An Annotation consists of a marking and an optional description of a region of interest. You can mark a lesion with an ellipse, freehand drawing, arrow, or with measurements, and then describe the region. SecurView links each Annotation to a specific image.

- ▶ **4.5.1. Marking an Image**
- ▶ **4.5.2. Describing a Region of Interest**
- ▶ **4.5.3. Viewing Annotations**

On SecurView DX the Annotation creator has exclusive revision and deletion rights. However, other users can view the Annotations (except those in second-read studies), and other users can enter their own Annotations for the patient. You can send Annotations to other Standalone workstations or multiworkstation clusters. See **4.6. Sending and Viewing Notices** and **4.7. Closing a Study**.

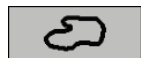
### 4.5.1. Marking an Image

Use the Ellipse, Freehand, Arrow, and/or Measurement tool to mark a region of interest.

▶ **To draw a marking:**



Ellipse



Freehand

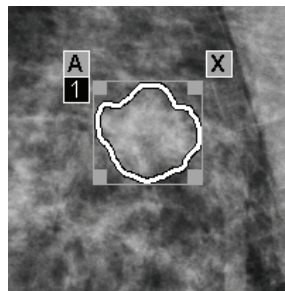


Arrow




Measurement


- 1 Click **Ellipse**, **Freehand**, **Arrow**, or **Measurement**. Click at the starting point of the marking, drag to create the shape, then release the mouse button. (For a freehand marking, SecurView automatically connects the two endpoints.) SecurView numbers each Annotation (**1**, **2**, and **3** below).




Freehand

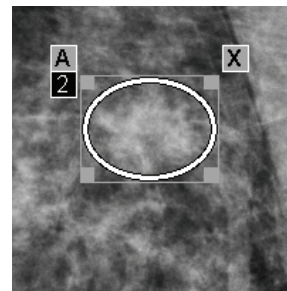

Ellipse

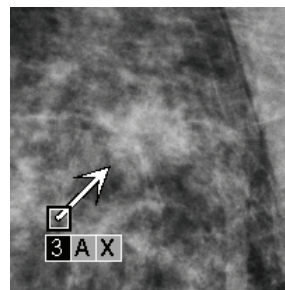

Arrow

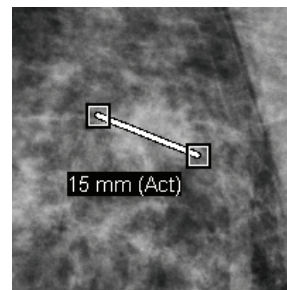


Measurement

- To change the marking size, click and drag a square handle.
- To move the marking, click and hold inside the bounding box (or click the measurement line), then drag the marking to the new location.
- To enter a description for an ellipse, freehand, or arrow marking, click **A** (see next page).




- To delete the ellipse, freehand, or arrow marking, click **X** (or click in the bounding box and press **Backspace**). To delete the measurement line, click a square handle (or press **Backspace**).
- 2 Click another tool (or move to another image) to lock the marking in place. (To unlock a marking, first click the tool used to create it: Ellipse, Freehand, Arrow, or Measurement.)

 **Important:** Use caution when making measurements on magnification views. For some manufacturers the pixel spacing conversion factors may not be encoded correctly. Take measurements on unmagnified views, if available.

► **To measure with the ruler:**

Click and drag the ruler displayed with each image. To turn the ruler 90 degrees, right-click the ruler, then drag.



 **Note:** Measurement lengths are calculated by using pixel spacing conversion factors provided by the data source. Please refer to the data source guides for accuracy claims.

### 4.5.2. Describing a Region of Interest

After marking an image, you can enter a description of the region of interest by selecting lesion attributes, entering text, and/or inserting predefined text strings.

► **To enter an Annotation description:**



Ellipse



Freehand



Arrow

- 1 Click **Ellipse**, **Freehand**, or **Arrow**, then click the marking to open the bounding box.
- 2 Click **A** (or double-click inside the bounding box) to open the Annotation dialog:

- 3 Select one or more check boxes for the lesion, and enter text (or insert a predefined text string) as desired in the lower field. When done, click **OK** to save the description.

You can configure the Annotation dialog to appear with or without the checkbox area through a user preference (see [6.3. Tools and Overlays Preferences](#)).

► **To predefine text strings for Annotation descriptions:**



New



Insert



Edit



Delete

- 1 In the Annotation dialog, click **New** to open the Enter New Text dialog.
- 2 Type your text and click **OK** to add the new text string to the drop-down list.

Once you define a text string, you can select it from the drop-down list, and then:

- Click **Insert** to add the text to the Annotation description, or
- Click **Edit** to modify the text string, or
- Click **Delete** to delete the text string.

### 4.5.3. Viewing Annotations



Annotations  
Present

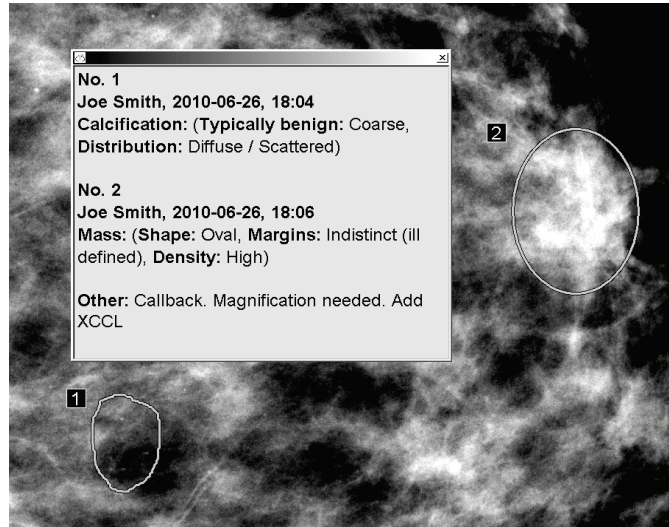
When a patient opens in the MG Viewer, SecurView hides Annotations by default. An icon (shown at left) marks any image that has one or more Annotations.

► **To view Annotations for all currently displayed images:**



Annotation  
User Filter

Click **Annotation User Filter** to display annotations for all currently displayed images.




- To close the Annotations window, click the X in the upper right corner.
- To hide the Annotations, click **Annotation User Filter** again.

► **To view Annotations for a single image:**

Right-click the image to open the Pie menu, then select **Annotation User Filter** to display or hide annotations.


► **To select reviewers:**

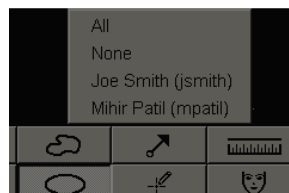If there are Annotations from multiple reviewers, a list of reviewers appears. Select the name of the Radiologist whose Annotations you want to view (or select **All** to view all Annotations). To hide the Annotations, select **None**.




► **To view third-party GSPS Annotations:**



SecurView can display third-party GSPS Annotations. If third-party GSPS Annotations are present for an image, the icon at left appears. When you click **Annotation User Filter**, SecurView marks GSPS Annotations with a  icon.



 **Note:** SecurView does not support all content of third-party GSPS. If GSPS Annotations from a specific manufacturer or device model do not display well on SecurView, a service engineer can configure SecurView not to display them.

## 4.6. Sending and Viewing Notices

A GSPS Notice is a message from another Hologic Standalone workstation or multiworkstation cluster that contains all Annotations for an image, with current window/level values, the user name, and the creation date and time (but not the study reading state). Notices based on tomosynthesis images include all Annotations from all slices of the corresponding image type (projection or reconstruction).

While viewing a patient, users can send Notices to other configured Hologic Standalone workstations or multiworkstation clusters. When received, other users can view the Annotations. In most cases, Radiologist users send Notices for viewing by Technologist users. A service engineer must configure the Notice destinations.

### ► 4.6.1. Sending Notices

### ► 4.6.2. Viewing Notices

#### 4.6.1. Sending Notices

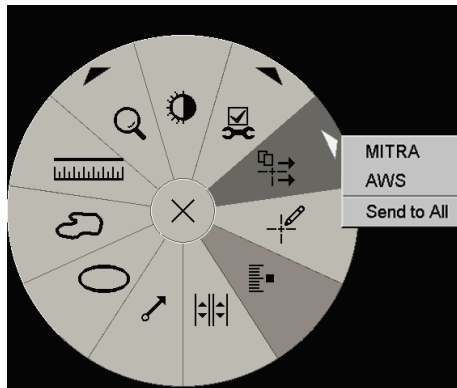
There are three ways to send Notices. You can (1) send all Notices of the current study as described below, (2) send all Notices when you close the study (see [4.7. Closing a Study](#)), or (3) send a Notice for the currently selected image (see next page).

##### ► To send all Notices:



##### 1 Right-click to open the Pie menu, then select **Send All Notices**.

- If there is one configured Notice destination, then SecurView creates the Notices and sends them immediately. The Notices contain all Annotations made by either (1) the current Radiologist user on unread studies or (2) the current Technologist user on new studies (SecurView RT).
- If there is more than one configured Notice destination, a submenu appears.




##### 2 To send the Notices, select a destination or 'Send to All'.

**Note:** 'Send All Notices' applies only to unread studies. To send Notices for Read, Old or Changed studies use 'Send Image Notice' or 'Close Study' (see [4.7. Closing a Study](#)).

► **To send an Image Notice:**



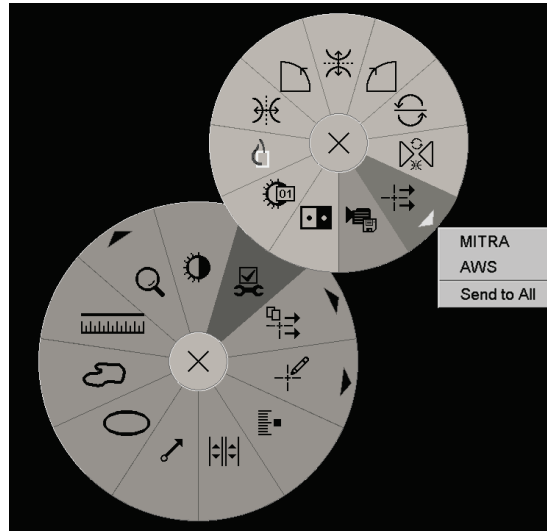Right-click an image to open the Pie menu, and point to **Image Tools** to open the second Pie menu. Then click **Send Image Notice**.




SecurView either sends the Notice immediately or, if there are multiple configured Notice destinations, opens a submenu to allow you select a destination. This Notice contains all Annotations on the selected image, regardless of the creator or study state.

#### 4.6.2. Viewing Notices




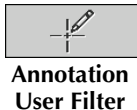When the workstation receives a Notice for a patient, a '+' appears in the **Notice** column of the Patient List (see page 35). When a patient opens in the MG Viewer, SecurView hides the Notices by default. The icon at left marks any image that has one or more Notices.

► **To view Notices for the currently displayed images:**



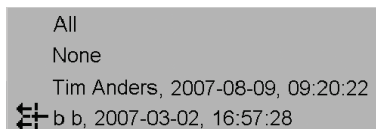- 1 Click **Annotation User Filter**. The Notice Present icon appears next to each Notice.



- 2 Select the name of the Radiologist whose Notices you want to view (or select **All** to view all Annotations). To hide the Annotations, select **None**.

## 4.7. Closing a Study

The last step in reviewing patients is usually to close the studies, i.e. to complete the review of currently loaded studies.

- ▶ **4.7.1. Closing a Study as a Radiologist**
- ▶ **4.7.2. Closing a Study as a Technologist**

### 4.7.1. Closing a Study as a Radiologist

After reviewing a patient, a Radiologist user on SecurView DX opens the Close Study dialog and changes the reading state for one or more studies, usually from 'Not Read' to 'Read'. If a study is still unread or requires a second reading, the Radiologist can instead assign lock states.

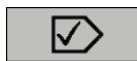
Close Study can also be used to send Notices, store annotations, tagged tomosynthesis slices and multimodality ScreenCaptures, or cancel second reading.

The Close Study dialog opens automatically when you reach the last ReportFlow step or click **Close Study**, depending upon the patient's reading state:

- The Close Study dialog is available when the reading state is currently 'Not Read', 'Read Once', or 'Changed'. The Close Study dialog is also available for new or changed annotations, measurements, or tagged tomosynthesis slices for 'Read' or 'Old' studies.
- The Close Study dialog is not available when the reading state is 'Locked', 'Read', or 'Old' without new or changed annotations, measurements, or tagged tomosynthesis slices. However, if you locked the patient, then you can unlock it – see **3.2.6. Using the Shortcut Menu**.


For more about reading states, see **3.2.4. Reading States**.

- ▶ **To close a study as a Radiologist:**




Close Study

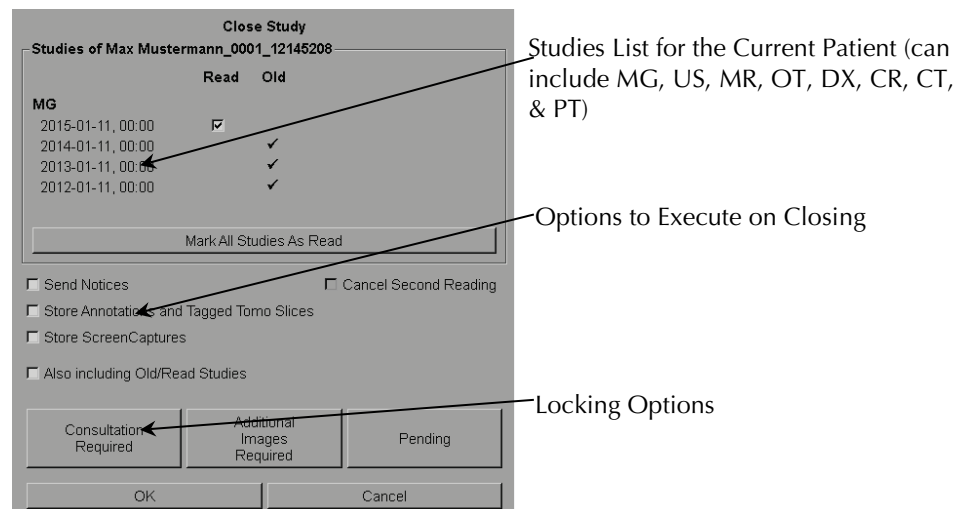- 1 Click **Close Study** (or proceed to the last ReportFlow step).





- 2 Adjust study settings as needed. SecurView can execute any of the following options when you close the dialog:
  - a In the Studies list, click one or more checkboxes to set the reading status to 'Read', leave the boxes unchecked, or click **Mark All Studies As Read**.


- b** Select options for Notices, Annotations, and ScreenCaptures, if you want to send them to configured destinations when you click OK to close the dialog. (See **6.1. Workflow Preferences** to set up default settings for these options.)
- **Send Notices** – For each unread study, when you click OK, this option sends a GSPS Notice containing markings of the current user ('Annotations and Tagged Tomo Slices') but not the reading state.
  - **Store Annotations and Tagged Tomo Slices** – When you click OK, this option sends (1) a GSPS Report containing the study reading state and current user markings ('Annotations and Tagged Tomo Slices'), and/or (2) an MG Secondary Capture image for each image with markings of the current user and for each tagged tomosynthesis slice. This option is applicable for each study marked as 'Read', and also for already 'Read' and 'Old' studies, if activated via the corresponding option below.
  - **Store ScreenCaptures** – When you click OK, this option sends multimodality ScreenCaptures. This option is applicable for each study marked as 'Read', and also for already 'Read' and 'Old' studies, if activated via the corresponding option below.
  - **Also including Old/Read Studies** – Select this option if you want to include new or changed annotations, tagged tomosynthesis slices or multimodality ScreenCaptures for studies that are 'Read' or 'Old' in **Send Notices**, **Store Annotations and Tagged Tomo Slices** or **StoreScreenCaptures**, respectively.

 **Note:** If 'Also including Old/Read Studies' is selected, only the new/changed annotations, tagged tomosynthesis slices or multimodality ScreenCaptures will be sent to the configured destinations. Previously stored/sent Notices, GSPS Reports, MG Secondary Captures or MM ScreenCaptures are not affected.

For more information, see **4.2.15. MG Secondary Captures and MM ScreenCaptures**.

- c** Click **Cancel Second Reading** to change a study state from 'Read Once' to 'Read'.
-  **Important:** This checkbox is active only if double reading is configured and you set the study to 'Read' (see step 2a). If you cancel the second reading and change the reading state to 'Read', you cannot restore the reading state to 'Not Read' or 'Read Once'.
- d** Lock the patient by selecting **Consultation Required**, **Additional Images Required**, or **Pending**. Note that clicking a lock button marks all new studies as 'Not Read'.
-  **Note:** To unlock a patient after exiting the Close Study dialog, see **3.2.6. Using the Shortcut Menu**.

- 3** To save the settings and send data to configured destinations, click **OK** or **Next Patient**.

 **Note:** You can configure the system to display a warning message if you have not viewed all images in single (or double) tiling mode when you close a study (see 'Missed View Safety Warning' in **6.1. Workflow Preferences**).


#### 4.7.2. Closing a Study as a Technologist

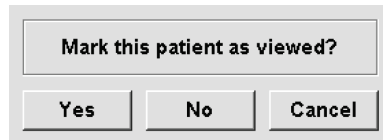SecurView RT enables the Close Study button if at least one GSPS Notice is available for the currently opened patient. SecurView indicates studies with one or more received Notices with a (+) in the **Notice** column of the Patient List (see page **35**).

If SecurView RT receives one or more Notices for the patient, a Technologist user can close a study and mark it as 'Viewed'.

If a Technologist attempts to close a patient with received Notices, SecurView displays:





- Click **Yes** to mark the patient as viewed and proceed with your next action.
- Click **No** to proceed with your next action without marking the current patient.
- Click **Cancel** to view the current patient again.

The **Viewed** column of the Patient list (see page 36) identifies patients with received Notices that have been viewed by the Technologist.

## 4.8. Printing Options

The DICOM print function is available to all users with viewing rights. You can print images, along with other data such as patient information, Annotations, etc., on a DICOM film printer.

There are two printing modes for the MG Viewer:

- **Screen Capture** mode prints the images as they appear on the left and/or right display(s). The displayed images should be in the format desired for printing, usually single tiling with one image per display. SecurView prints all supplemental information (overlays, markings, magnified areas, etc.) as it appears on the display, except for Text Annotations, which print in a box opposite the chest wall.
- **Diagnostic** mode prints all MG images of a selected study. Use this mode for printing images for ACR review. You can choose from two orientations, either dorsal (right breast chest wall on the right), or ventral (right breast chest wall on the left). Diagnostic printing is active only for patients with MG images (MG-MG, DX-MG, CR-MG, or SC-MG).

**CAUTION:** The text message “Printed in reduced resolution” of printed images indicates that the printouts are not for diagnostic use. This message may overwrite or overlap other text information.

**Note:** If a smaller film size is used for diagnostic printing than required for True Size printing, the disclaimer ‘Image not printed in True Size’ is printed on the film.

If the image to be printed is magnified (for example, includes the View modifiers magnification (M), spot-compression (S) or the ERMF value is too large), it is resized to fit and includes a disclaimer ‘Image adjusted to film size’.

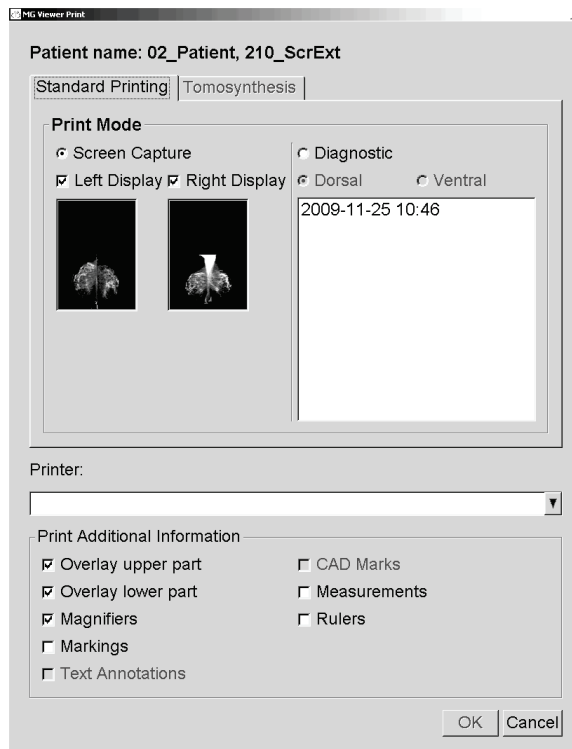
**Note:** Diagnostic printing is not supported for specimen images.

► **To print the currently viewed patient:**




DICOM Print

- 1 On the toolbar, click **DICOM Print** to display the Print dialog.



- 2 Under Print Mode, select either **Screen Capture** or **Diagnostic**.
  - For Screen Capture mode, select **Left Display**, **Right Display**, or both.
  - For Diagnostic mode, select either **Dorsal** or **Ventral**.

- 3 From the Printer drop-down list, select the printer.
  - For Screen Capture mode, you must select a printer name that includes film size.
  - For Diagnostic mode, SecurView automatically selects the film size and DICOM presentation size mode based on the image size.
- 4 In the Print Additional Information section, select one or more options.
  - For Screen Capture mode, you can include the Printing Image Overlays, Magnifiers, markings, text descriptions, CAD marks, measurements, and rulers.
  - For Diagnostic mode, you can print only the overlays.

 **Note:** To configure the overlays, see [8.7.3. Printing Image Overlay](#).

- 5 Click **OK** to print the selected image(s) and information.

If desired, click **DICOM Print** immediately to start a new print job before the previous print job completes.

## 4.9. Patient Synchronization with an External Application

If configured, your SecurView workstation can synchronize patients with an external application in several ways:

- Manual synchronization
- Automatic synchronization according to user preferences
- Automatic synchronization when SecurView receives a message

For information about configuring synchronization with an external application, see [8.4.8. Configure Synchronization Interface](#).

### 4.9.1. Manual Synchronization

You can synchronize patients manually either from the Patient List or during patient review.

- On the Patient List, right-click a patient, then from the shortcut menu, select **Synchronize**.
- During patient review, click **Synchronize** on the toolbar or press [R] on the keyboard.




Synchronize

In response, SecurView sends an Open Patient message to the external application.


### 4.9.2. Automatic Synchronization

You can synchronize patients automatically according to your ReportFlows and user preferences (see [6.4. User Profile Preferences](#)).

- **Synchronization with a ReportFlow** – Use the Synchronize step in your ReportFlows. When Synchronize becomes the current step of your workflow, SecurView sends an Open Patient message to the external application.
- **Synchronization when you open a patient** – With your user preferences, you can set SecurView to send an Open Patient message to the external application each time you open a patient in the MG Viewer.

 **Note:** If you open a patient by barcoding an accession number, by default only the study with the barcoded accession number is sent in the Open Patient message.

- **Synchronization when closing a study** – With your user preferences, you can set SecurView to send an Update Patient State message to the external application each time you close a study where at least one study is marked as 'Read'.

 **Note:** *At this time, only the Hologic MultiView workstation supports synchronization at close of study.*

#### 4.9.3. Synchronization When You Receive a Message

If you log in as a Radiologist user, SecurView can open a patient automatically in the MG Viewer when the workstation receives an Open Patient message from the external application.



---

## Chapter 5: Working with Tomosynthesis Images

- ▶ 5.1. Overview of Tomosynthesis Imaging
- ▶ 5.2. Navigating Tomosynthesis Images
- ▶ 5.3. Displaying ImageChecker 3D Calc CAD Results
- ▶ 5.4. Tagging Tomosynthesis Reconstructed Slices
- ▶ 5.5. Printing Tomosynthesis Reconstructed Slices

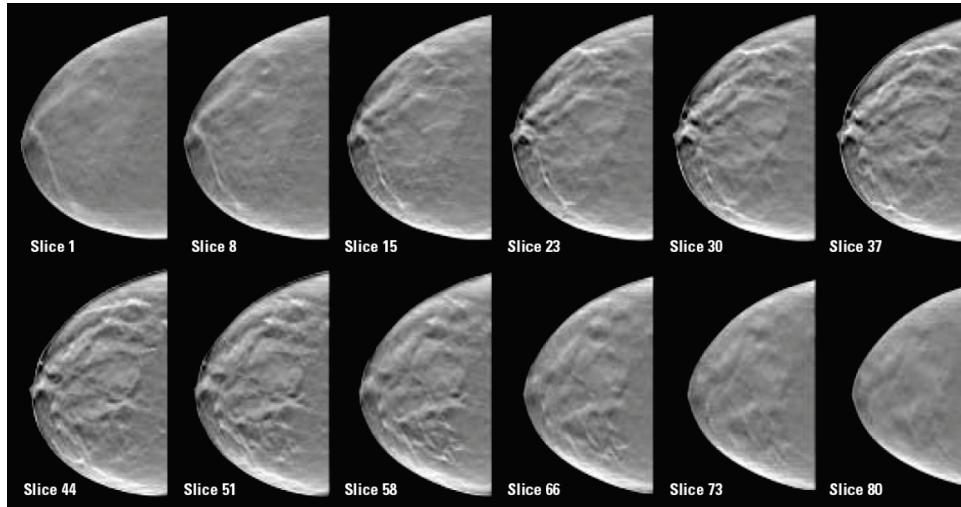
This chapter describes how to view and work with tomosynthesis images. Breast tomosynthesis is a three-dimensional imaging technology based on multiple projection images of a stationary compressed breast taken from different angles. The projection images are reconstructed into a series of thin high-resolution images (slices) that can be displayed individually or in a dynamic 'Cine' mode.

 **Note:** *Display and printing of tomosynthesis images requires a special software license.*

## 5.1. Overview of Tomosynthesis Imaging

A typical tomosynthesis combination procedure consists of the following types of images with image sets for each view:

- One or more mammography images (conventional mammography or C-View 2D image)
- A number of tomosynthesis projection images (typically 15)
- A number of tomosynthesis reconstructed slices. The quantity depends on the thickness of the compressed breast.



**Tomosynthesis: Reconstructed Slices (schematic representation)**

Tomosynthesis images appear in single, double or quadruple tilings. Projection image sets, reconstructed slice sets, and conventional mammography or C-View 2D images that belong to the same laterality and view of a combination procedure appear as a stack within one tile.

**⚠ Important:** Be sure to review the study thoroughly. Once you view at least one slice or image of a combination procedure, SecurView does not inform you that there are unviewed images (i.e., the 'Missed View Safety Warning' will not be displayed).

**⚠ Note:** SecurView accepts tomosynthesis reconstructed slices in the Hologic Secondary Capture Image (private pixel data), Breast Tomosynthesis Image, and CT image formats. If tomosynthesis reconstructed slices are already available in one format and the same reconstructed slices arrive in another format, the system is designed to discard all but the first received reconstructed slices.

**⚠ Note:** Tomosynthesis is not applied to magnification views.

**⚠ Note:** SecurView accepts C-View 2D images in the Digital Mammography X-Ray Image – For Presentation and Breast Tomosynthesis Image formats. If a C-View 2D image is already available in one format and the same C-View 2D image arrives in the other format both images will be made available for display.



## 5.2. Navigating Tomosynthesis Images

- ▶ 5.2.1. Tomosynthesis Navigation Buttons
- ▶ 5.2.2. Viewing Tomosynthesis Slices
- ▶ 5.2.3. Changing Slab Thickness
- ▶ 5.2.4. Annotating a Tomosynthesis Image
- ▶ 5.2.5. Using Cine Mode
- ▶ 5.2.6. Using Local Cine Mode
- ▶ 5.2.7. Focus
- ▶ 5.2.8. Scrolling Through Linked Tiles
- ▶ 5.2.9. Exporting a Movie

### 5.2.1. Tomosynthesis Navigation Buttons

You can select three different image types when a tomosynthesis image dataset appears within a tile or viewport:

- Projection images
- Mammography image (conventional mammography or C-View 2D image)
- Reconstructed slices



**Tomosynthesis Navigation Buttons**

The navigation buttons shown above appear with tomosynthesis images. If desired, click and drag the group of buttons to another display position.

Slabs derived from tomosynthesis reconstructed slices appear in Maximum Intensity Projection (MIP).

Icon	Purpose
	<b>Projection</b> – Displays projection images.
	<b>MG Images</b> – Displays conventional mammography or C-View 2D images. If more than one image is available, the number of images is displayed below the icon, e.g., '1/2'. Clicking this button repeatedly switches between the available mammography images.
	<b>MG / Tomosynthesis Toggle</b> – Switches between mammography images and reconstructed slices.
	<b>Reconstruction</b> – Displays reconstructed slices.
	<b>Cine</b> – Starts and stops sequential display of reconstructed slices.
	<b>Focus</b> – Activates and deactivates the Focus mode.

You can configure the display order of the conventional mammography and C-View 2D images via the Hanging Snapshot configuration (see [7.3. Creating and Modifying Hanging Snapshots](#)).

The initial slice to be used for displaying tomosynthesis reconstructed slices or projection images can be configured with a user preference (see [6.2. Image Presentation Preferences](#)).

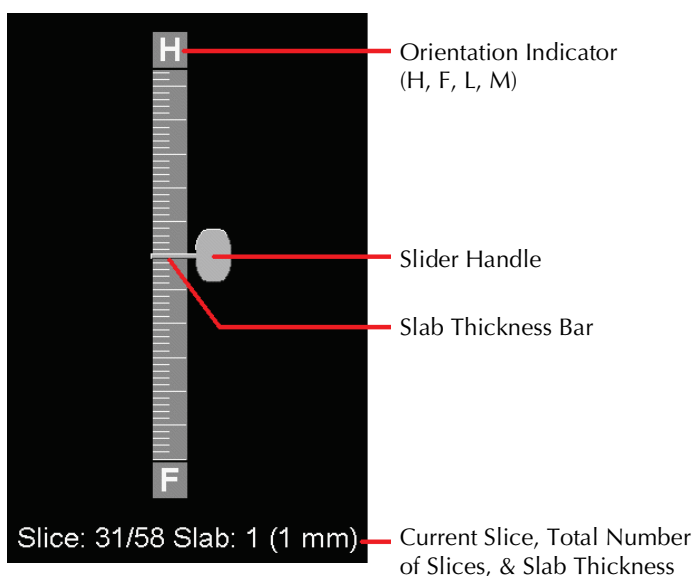
### 5.2.2. Viewing Tomosynthesis Slices

Use the slider tool to view slices or change slab thickness. Use the mouse or keypad wheel to scroll through the slices.

► **To view Tomosynthesis slices:**




Click **Reconstruction** (one of the tomosynthesis navigation buttons) to display reconstructed slices. A vertical slider tool appears.



**Slider Tool**

The orientation indicators at the top and bottom of the ruler depend on the current view direction (ML, MLO, LM, LMO, CC, SIO, ISO, XCCL, XCCM) of the image. 'H' indicates head, 'F' foot, 'M' medial, and 'L' lateral.

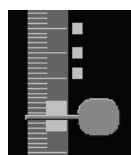If desired, move the slider tool to another display position by pointing to the ruler and dragging the tool.



**Tagged Slices**

Use the Tag Tomo Images tool to identify a reconstructed slice for printing, exporting to external media, or storing to configured destinations on close of study. A small mark next to the vertical slider tool indicates a tagged reconstructed slice. This mark is stored in the database and is visible as long as the patient remains on the SecurView workstation. See [5.4. Tagging Tomosynthesis Reconstructed Slices](#).



SecurView indicates received MG Secondary Capture tomosynthesis Annotations with the camera icon shown at left. If an MG Secondary Capture represents a tagged tomosynthesis slice, location information for the slice also appears.

► **To navigate the slices:**

- Click any location on the tool (or point to the slider handle and move the slider up and down). SecurView displays the corresponding reconstructed slice.

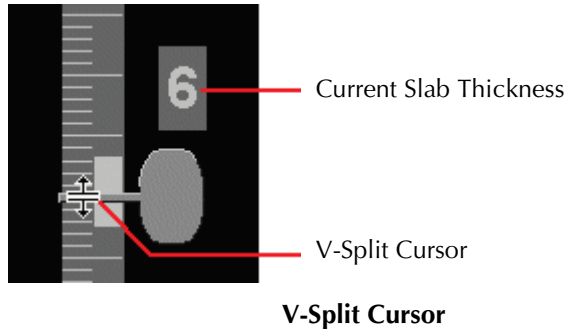
- Change the location by rotating the scroll wheel on the mouse or keypad.
- Use the slab thickness bar to adjust the number of displayed slices (see next page).

With a user preference, you can set the mouse/keypad wheel mode to either Sequential (one slice at a time) or Advanced (several slices at a time). See [6.2. Image Presentation Preferences](#).

### 5.2.3. Changing Slab Thickness

Use the slab thickness bar to change the number of displayed slices.

- 1 Point the mouse at the slab thickness bar to display the V-Split Cursor.





- 2 Click and drag the pointer up or down to increase or decrease slab thickness. The current slab thickness (number of slices) appears to the right of the slider tool.
- 3 To reset slab thickness to the default, double-click when the V-Split Cursor is active.

Set the default slab thickness with a user preference (see [6.2. Image Presentation Preferences](#)).

### 5.2.4. Annotating a Tomosynthesis Image

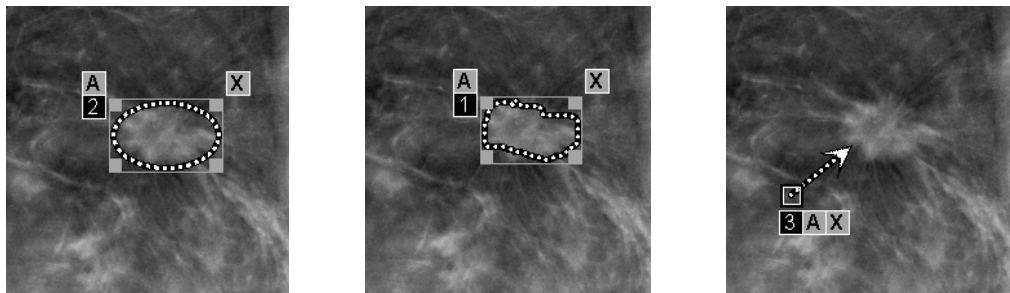
Tomosynthesis images are annotated in the same manner as conventional MG images (see [4.5. Creating and Viewing Annotations](#)). However, if you increase the slab thickness of the tomosynthesis image you are annotating, the freehand, ellipse, or arrow mark appears in the middle slice of the slab and displays as a dotted line (see below).



 A Hologic GSPS-Report or GSPS-Notice references the specific DICOM image objects on which the user created annotations. If the same reconstructed slices or C-View 2D images exist in more than one DICOM format (for example, Hologic Secondary Capture Image, Breast Tomosynthesis Image and CT Image for reconstructed slices; Digital Mammography X-Ray Image – For Presentation and Breast Tomosynthesis Image for C-View 2D), the annotations will be displayed only with the images matching the DICOM format referenced in the GSPS.

### 5.2.5. Using Cine Mode

You can 'Cine' through a set of tomosynthesis projection images or reconstructed slices in a single tile. Cine mode applies simultaneously to all linked tiles. To configure automatic activation of cine mode for tomosynthesis reconstructed slices when entering a ReportFlow step that contains a single tiling hanging snapshot, see [6.2 Image Presentation Preferences](#).


 **Note:** Cine mode only starts automatically for hanging snapshots that are part of a ReportFlow.

► **To start Cine mode:**



Cine


- 1 Click **Cine** or press **Cine** on the workflow keypad. SecurView starts with the current slice and moves through increasing slice numbers. Cine mode reverses when it reaches the last (or first) slice.

 **Note:** When ImageChecker 3D calcification marks are displayed, Cine mode sequences only through the slices of the currently selected calcification cluster.

- 2 To stop Cine mode, click **Cine** again or rotate the mouse or keypad scroll wheel.

► **To change Cine speed:**

- 1 Click the **boundary** of the Cine button to display the speed controller (if it is not visible).
- 2 Move the **slider handle** to adjust the speed from slow/left (5 frames per second) to fast/right (30 frames per second).

 **Note:** The default speed is set in the user preferences. Systems with slower processors may not be able to cine at 30 frames per second.

- 3 If desired, click the Cine button **boundary** again to hide the speed control.



Boundary

Speed Controller

**Cine Button & Speed Controller**

### 5.2.6. Using Local Cine Mode

Use Local Cine mode to view a limited range of slices in a single tile. To set the slice range, see [6.2. Image Presentation Preferences](#).

► **To use Local Cine mode:**

**1** There are three ways to start Local Cine mode:

- Press and hold **Cine** on the workflow keypad, or
- Press and hold **[F6]** on the keyboard, or
- Click and hold **Cine** with the mouse.

SecurView starts with the current slice and travels back and forth through the specified number of slices. For example, if the current slice is 25 and the range is 20, SecurView scrolls through slices 15–35.

If normal Cine mode is running when you start Local Cine mode, SecurView changes to the Local Cine range.


**2** While Local Cine mode is running, change the Local Cine center slice by rotating the mouse wheel or keypad scroll wheel forward or back – the center slice shifts up or down, but the slice range does not change.

**3** There are three ways to stop Local Cine mode:

- Press **Cine** on the workflow keypad, or
- Press **[F6]** on the keyboard, or
- Click **Cine**.

### 5.2.7. Focus

Focus can be used to easily visualize the correlation from a region of interest in a C-View 2D image to the most representative reconstructed slice. Focus is available in Single and Double tiling.


 **Note:** The Focus button is displayed in the tile with the C-View 2D image only when the C-View 2D image and the corresponding reconstructed slices hang side by side. In Double tiling they need to hang in adjacent tiles on the left or right side.

► **To use Focus**




Focus

- 1 Click **Focus** in the tile displaying the C-View 2D image or press the [V] key on the keyboard to activate Focus for the tile. The pointer changes to a crosshair to indicate Focus is active.

 **Note:** Focus is only active in the tile in which the button was clicked.

 **Note:** The Focus button will be displayed even if “Omit Tomosynthesis Tools” is configured for the tile (see 7.3.1. **Creating New Hanging Snapshots**).

 **Note:** To apply Focus in a single instance without activating the feature, press and hold [CTRL] on the keyboard while clicking the left mouse button in the C-View 2D image.


- 2 Click on a region of interest within the C-View 2D image. The associated reconstructed slice tile switches to the most representative slice.

 **Note:** If the Focus data is corrupt the displayed slice may not be the most representative slice.


**a** When the pointer is in the tile with the C-View 2D image scrolling (e.g. using the scroll wheel) is applied to the tile with the corresponding reconstructed slices.


**b** If Cine mode is applied to a focused reconstructed slice, local cine mode is applied automatically with the focused reconstructed slice as the center slice.

 **Note:** Clicking within a Magnifier does not activate Focus.

 **Note:** When clicking on the image background, no corresponding focused reconstructed slice is assigned. The currently displayed reconstructed slice will not change.


- 3 Click **Focus** in the tile displaying the C-View 2D image or press the [V] key to deactivate Focus for the tile.

 **Note:** Focus is not deactivated by selecting another image evaluation tool.

 **Note:** Focus is automatically deactivated when the current hanging changes, e.g. by going to the next or previous step in the ReportFlow, manually dragging a different image from the MammoNavigator, switching to a different image in either tile using the Tomosynthesis Navigation buttons, or clicking a predefined hanging button.

### 5.2.8. Scrolling Through Linked Tiles


When tiles are linked, scrolling through reconstructed slices or projection images in one tile automatically scrolls the reconstructed slices in all other linked tiles. Linked scrolling operates on reconstructed slices, slabs, or projection images of the same or different spacing.

 **Note:** *Linked scrolling is available only when images of the same type (tomosynthesis reconstructed slices or projections) are displayed in different tiles.*

► **To use linked scrolling:**

- 1 To initiate linked scrolling, display reconstructed slices (or projection images) on two or more tiles.
- 2 Right-click the image and select **Link Tile** from the Pie menu. Repeat for each tile to be linked. An indicator icon appears on each linked tile.



 **Note:** *You can also switch linking on and off via the Hanging Snapshot configuration.*

- 3 To perform linked scrolling:
  - Rotate the mouse wheel, or
  - Move the slider handle for a linked tile, or
  - Start Cine mode.

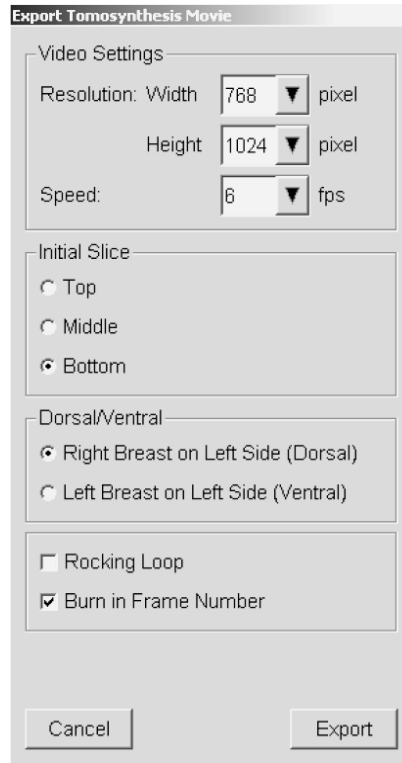
When using the mouse wheel or slider, you can switch off linked scrolling temporarily by holding down **Shift**.

### 5.2.9. Exporting a Movie

You can export a scrolling movie of tomosynthesis reconstructed slices or tomosynthesis projections.

► **To export a movie:**

- 1 Right-click an image to open the Pie menu, then point to **Image Tools** to open the second Pie menu.
- 2 Click **Export Movie** to open the Export Movie dialog. The button is available only when the viewer shows a tomosynthesis projection image or reconstructed slice.



- Under Video Settings select the Resolution (Width and Height) and the Speed (frames per second). The maximum resolution is 2048 × 2460 pixels; the maximum speed is 120 fps.
  - Select the Initial Slice for the movie. By selecting the middle slice, you also select Rocking Loop mode (below).
  - Under Dorsal/Ventral, select the movie orientation.
  - Select Rocking Loop to start the movie at the middle slice, scroll to the topmost slice, then to the lowermost slice, and up again to the middle slice.
  - Select Burn in Frame Number to indicate every slice with the slice number and the total number of slices.
- 3 Click **Export** to open a **Save as** dialog. Select the path, and enter the file name for the movie, then confirm. A progress bar appears until the movie export is finished.



## 5.3. Displaying ImageChecker 3D Calc CAD Results

 **Note:** ImageChecker 3D Calc CAD is not available in the United States.

ImageChecker 3D Calc CAD is a software algorithm that identifies regions of interest in tomosynthesis slices. To produce and view ImageChecker 3D Calc CAD results, you will need both the:

- Cenova digital mammography server with Hologic's ImageChecker 3D Calc CAD license, and the
- SecurView DX workstation (version 7.2 or later) with Hologic's Tomosynthesis CAD Display license.

If there are CAD results for a patient, a '+' symbol appears in the CAD column of the Patient List. In addition, when you review a patient with CAD results, the toolbar CAD button becomes active (not gray).

An Administrator can configure CAD results to appear automatically as a step in a ReportFlow.

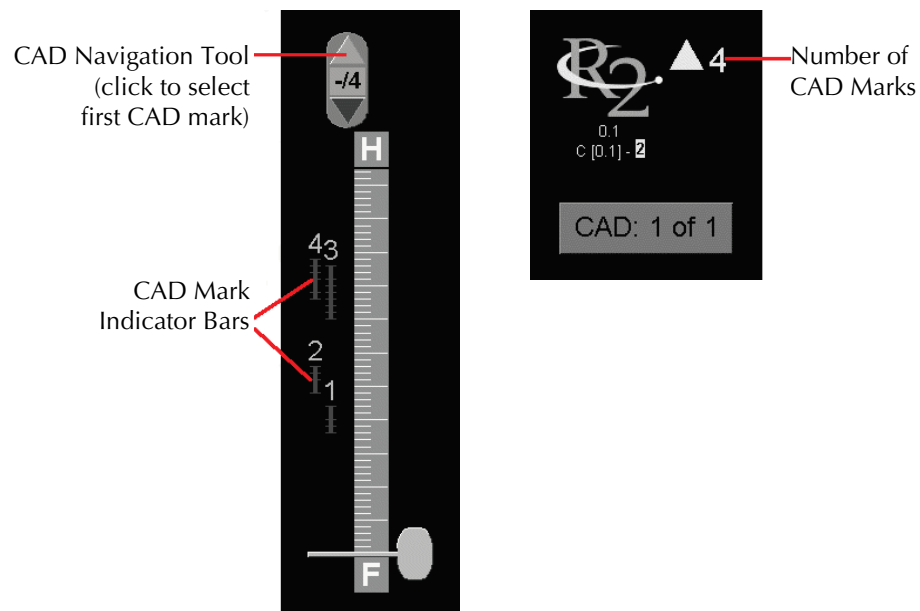
### ► To display ImageChecker 3D Calc CAD results:



**Computer  
Aided  
Detection**

- 1 While viewing tomosynthesis slices, click **Computer Aided Detection**.

If tomosynthesis CAD results are available, SecurView displays a set of CAD indicator bars next to the tomosynthesis slider tool. Each bar indicates slices that include at least one prominent calcification. The CAD overlay also appears.




### Slider Tool with ImageChecker 3D Calc CAD Indicators; R2 Logo

- 2 To display the first mark, click the **Up** arrow on the CAD navigation tool or press [W] on the keyboard.


SecurView displays the first CAD mark and the corresponding 'slice of interest', which is the slice that best represents the region of interest as a whole, generally the slice that contains the most calcifications. SecurView also highlights the selected cluster and the corresponding CAD indicator bar. Other visible clusters appear gray.

The first appearance of the CAD marks depends upon the view tiling and the default settings for each user (see [6.3. Tools and Overlays Preferences](#)).

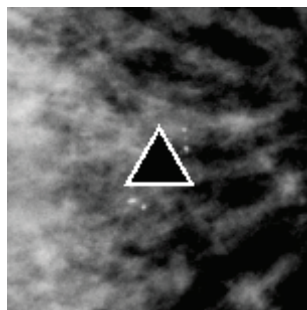
- In quadruple tiling, the results appear as RightOn CAD marks.
- In double or single tiling, each CAD mark appears as a dotted boundary line around the region of interest and/or as outlined individual calcifications.




RightOn CAD Mark

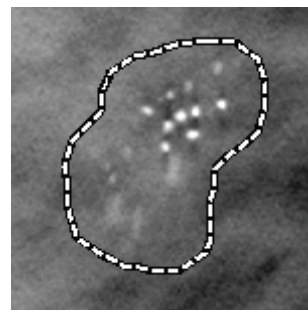

CAD Mark Boundary

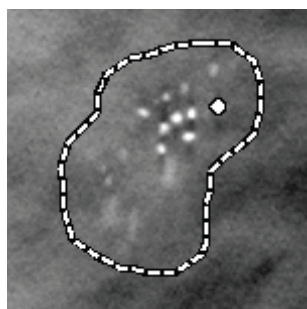

PeerView (1 slice)

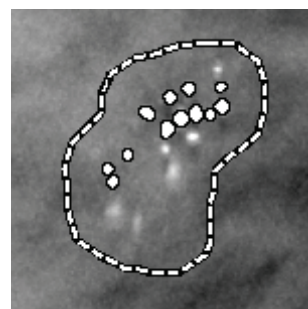

PeerView (4 slices)

#### ImageChecker 3D Calc CAD Marks

- 3 In single or double tiling, adjust the CAD mark display as follows:
  - Press **[B]** on the keyboard to toggle the boundary lines on or off.
  - Press **[C]** to toggle PeerView marks on and off.
  - Increase slab thickness to visualize the full extent of calcifications within a cluster (see [5.2.3. Changing Slab Thickness](#)).
- 4 To select another CAD mark:
  - Click the **Up** and **Down** arrows on the CAD navigation tool.
  - Press **[W]** on the keyboard to move to the next mark.
  - Press **[S]** to move to the previous mark.
  - Click any CAD indicator bar.

The slider jumps to the slice of interest of the selected mark.



- 5 To use Cine mode, select a CAD mark and click the **Cine** button (or click **Cine** on the Pie menu).

SecurView starts with the current slice and travels up through the slices that correspond to the CAD mark. Cine mode reverses when it reaches the first/last slice of the cluster.

To stop Cine mode, click **Cine** again.

## 5.4. Tagging Tomosynthesis Reconstructed Slices

Use these instructions to identify slices to be printed, exported to external media, or stored to configured destinations on close of study. After tagging the slices see:

- ▶ [5.5. Printing Tomosynthesis Reconstructed Slices](#)
- ▶ [10.1. Exporting Currently Displayed Image Files](#)

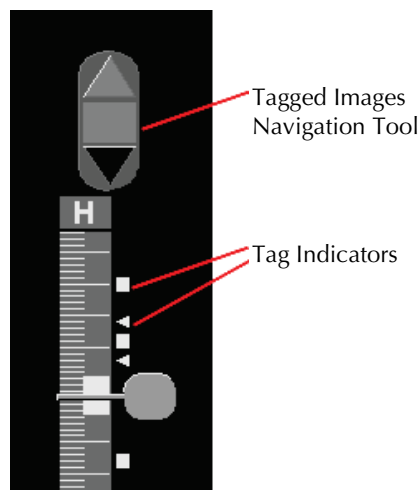
▶ **To tag tomosynthesis reconstructed slices:**

Display the slice, then:

- Click **Tag Tomo Images** on the Pie menu, or
- Press **Spacebar** on the keyboard.



SecurView tags the corresponding slice. On the right side of the slider tool, small marks indicate the tagged slices:



**Slider Tool with Tag Indicators**



The Tagged Images Navigation tool appears whenever you tag at least one reconstructed slice for printing or exporting. The icon shown at left also appears.

- To display the next or previous tagged slice, click **Arrow Up** or **Arrow Down** on the navigation tool.
- To remove a tag indicator, display the tagged slice and press **Spacebar** (or click **Tag Tomo Images** again).

▶ **Working with triangle tag indicators:**

A triangle tag indicates another reviewer's tag mark from a received GSPS object. Triangle marks cannot be edited. However, if you tag a triangle-marked slice, your tag overwrites the triangle and appears as a square tag indicator.

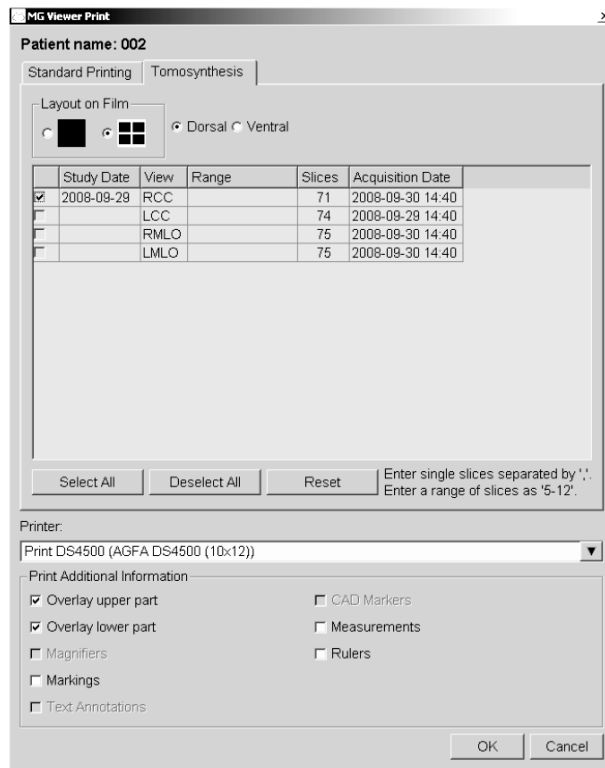
## 5.5. Printing Tomosynthesis Reconstructed Slices

To print tomosynthesis reconstructed slices, you must first identify the slices to be printed; either by tagging the slices (see [5.4. Tagging Tomosynthesis Reconstructed Slices](#)) or identifying them with the Print dialog (as explained below).



**DICOM Print**

- 1 On the toolbar, click **DICOM Print** to display the Print dialog.



- 2 Select the Layout on Film and either Dorsal or Ventral.
  - When you select quadruple tiling, the images print to fit in the corresponding quadrant according to the selected film size.
  - When you select single tiling, the image prints in true size if possible. If the film size does not allow true size printing, then it prints to fit the film area.
  - If the number of reconstructed slices to be printed exceeds one film, SecurView distributes the images on several films.
  - A new film starts for a new laterality and view.
- 3 In the first column, check the box to select **reconstructed views of a study**. The **Range** column lists the numbers of the slices tagged for printing.
- 4 Click the box and enter an individual slice number, a range of slices with a hyphen (e.g., '10-15'), or individual slices in ascending order separated by a comma (e.g., '10,12,20,25').
- 5 Click **Select All** to select all reconstructed views. Click **Deselect All** to clear all selections. Click **Reset** to reset all changes made on the Tomosynthesis tab.
- 6 On the lower area, select the Additional Information to be printed.
- 7 Click **OK** to print the images. (If desired, click **DICOM Print** immediately to start a new print job before the previous print job completes.)

---

## Chapter 6: Setting User Preferences

- ▶ **6.1. Workflow Preferences**
- ▶ **6.2. Image Presentation Preferences**
- ▶ **6.3. Tools and Overlays Preferences**
- ▶ **6.4. User Profile Preferences**

This chapter explains how to define preferences for individual users. SecurView allows each Radiologist and Technologist user to customize the interface in order to optimize workflow. Once an Administrator has added you as a new SecurView user, you can configure your own system profile.

Note that the User Preferences are default settings. When you view patients, you can change view options at any time on an as-needed basis.

- ▶ **To display the User Preferences tabs:**

On the Main screen, click **Administration**. Then click **User Preferences** to display the Workflow tab (partial view shown)

Patient Manager		User Preferences		About
Workflow	Image Presentation	Tools and Overlays	User Profile	
<b>Review and Session Worklist Sorting</b> <ul style="list-style-type: none"><li>Click Order in Patient List</li><li>Study Date</li><li>Alphabetically</li></ul> <b>Automatic Worklist Sorting</b>		<b>Sending Annotations, Tagged Tomo, ScreenCaptures</b> <p>In the Close Study Dialog, check the following options by default:</p> <ul style="list-style-type: none"><li><input checked="" type="checkbox"/> Send Notices</li><li><input checked="" type="checkbox"/> Store Annotations and Tagged Tomo Slices</li><li><input checked="" type="checkbox"/> Store ScreenCaptures</li></ul>		

 **Note:** The User Preferences tabs shown are those available to Radiologist users.

The following pages describe the four User Preferences tabs. For each tab, select the desired settings. When done, save your selections by clicking **Apply** (at the lower right corner of the window).

## 6.1. Workflow Preferences

The Workflow tab appears when you select User Preferences from the Administration module:

The screenshot shows the 'User Preferences' dialog box with the 'Workflow' tab selected. The 'Workflow' sub-tab is active, showing the following settings:

- Review and Session Worklist Sorting:** Radio buttons for 'Click Order in Patient List' (selected), 'Study Date', and 'Alphabetically'.
- Automatic Worklist Sorting:** Radio buttons for 'Study Date' (selected) and 'Alphabetically'.
- Hanging Snapshots and ReportFlows:** A 'Configure ...' button.
- Search:** A checkbox for 'Close Search Dialog on Retrieve' (unchecked).
- Multimodality Viewer:** A 'Configure ...' button.

The 'Tools and Overlays' sub-tab is also visible, showing the following settings:

- Sending Annotations, Tagged Tomo, ScreenCaptures:** Text: 'In the Close Study Dialog, check the following options by default:'; checkboxes for 'Send Notices', 'Store Annotations and Tagged Tomo Slices', 'Store ScreenCaptures', and 'Also including Old/Read Studies' (all unchecked).
- Missed View Safety Warning:** Radio buttons for 'Single Tiling' (selected), 'Single or DoubleTiling', and 'Off'.
- Receiving Notices:** A checkbox for 'Notify on Arrival of Notices' (checked).
- Patient Search Default Focus (PACS/Local):** Radio buttons for 'Focus to Patient ID' (selected) and 'Focus to Patient Name'.

After you make your selections, save your settings by clicking **Apply** (at the lower right corner of the screen).

- **Review and Session Worklist Sorting** – Sets the order in which SecurView displays patients selected manually (see [3.2. Using the Patient List](#)) or patients in a Session worklist (see [3.3. Creating Sessions](#)).
- **Automatic Worklist Sorting** – Sets the order in which SecurView displays new patients queued automatically as they arrive (see [4.1.2. Automatically Generated Worklists](#)).
- **Hanging Snapshots and ReportFlows** – Click **Configure...** to set up image hangings and ReportFlows, including your personal preferences for default ReportFlows. See [Chapter 7: Hanging Snapshots and ReportFlows](#).
- **Search** – Select to close the search dialog automatically after you click **Retrieve** (see [3.2.8. Searching for Patients](#)).
- **Multimodality Viewer** – Click **Configure...** to open the Multimodality Configuration Editor (refer to the *SecurView Advanced Multimodality Option User Guide*).
- **Sending Notices, Annotations, Tagged Tomo, ScreenCaptures, Also including Old/Read Studies** – For SecurView DX, these options apply if the service engineer has configured respective destinations. If you check any of these options then, when you open the Close Study dialog, SecurView checks them automatically. Note that you can override these settings on a per-patient basis in the Close Study dialog (see [4.7. Closing a Study](#)).

- **Missed View Safety Warning** – For SecurView DX, you can configure the system to display a warning message if you have not viewed all images in single (or double) tiling mode when you close a study (see [4.7. Closing a Study](#)).
- **Receiving Notices** – Select to have SecurView inform you when it receives a Notice from another Hologic workstation (see [4.6. Sending and Viewing Notices](#)).
- **Patient Search Default Focus (PACS/Local)** – Sets either the Patient ID or Patient Name as the default input field when performing a patient search (see [3.2.8. Searching for Patients](#)).

## 6.2. Image Presentation Preferences

The following screen appears when you select the Image Presentation tab:



After you make your selections, save the settings by clicking **Apply** (at the lower right corner of the screen).

- **Toolbar Hanging Buttons:** Use to set how you want images to be positioned, oriented, and ordered when you use the image hanging buttons. See [4.2.7. Image Hangings](#).
- **Sizing:**
  - **Default Image Size Mode** sets the default image scaling mode.
  - **Adjust Image Size in Double Tiling** configures the viewport size used to scale images presented in Double Tiling. Applies to the Scaling Modes Fit to Viewport, Right Size and Same Size. See [4.2.9. Scaling Modes](#) for more information. In order to activate the behavior shown in SecurView versions prior to 8.2, use the option **to Single Tiling Viewport**.
  - The **Same Size Roaming Magnification** sets the digital magnification factor for this image size mode (a decimal value from 1.0 to 2.0). See [4.2.9. Scaling Modes](#).
- **Tomosynthesis:** Use to configure how SecurView displays images from tomosynthesis combination studies. See [Chapter 5: Working with Tomosynthesis Images](#).
  - **Initial Slice** sets the slice that appears first when you view tomosynthesis reconstructed slices or projection images.

- **Cine Speed** sets the speed at which SecurView displays a sequence of tomosynthesis images in Cine mode. The range is 5–30 frames per second.
- **Start Cine Mode** can be used to configure automatic or manual start of cine mode for tomosynthesis reconstructed slices in single tiling ReportFlow hanging snapshots.
- **Mouse Wheel Mode** sets the behavior of the mouse wheel when you scroll through tomosynthesis slices, either Sequential (one slice at a time) or Advanced (several at a time).
- **Default Slab Thickness** sets the default number of slices to be combined into a single viewable entity during image display.
- **Local Cine Range** sets the number of slices that SecurView displays in local Cine mode. The range is 3–99 slices.

### 6.3. Tools and Overlays Preferences

The following screen appears when you select the Tools and Overlays tab:

The screenshot shows the 'User Preferences' dialog box with the 'Tools and Overlays' tab selected. The dialog has four main tabs: 'Patient Manager', 'User Preferences', 'About', and 'User Profile'. Under 'User Preferences', there are four sub-tabs: 'Workflow', 'Image Presentation', 'Tools and Overlays', and 'User Profile'. The 'Tools and Overlays' sub-tab is active, showing various settings:


- Image Overlay:** Default State (Reduced, Extended, Off), Display Ruler (in Reduced, in Extended, When Off).
- Annotations:** Full, Reduced, Only Text, Open Annotation Dialog Automatically, Always Ask Before Deleting Annotations.
- Left Mouse Button Default Function:** Magnifier, Inverted Magnifier, Window/Level, Ellipse Marking, Freehand Marking, Arrow Marking, Measurement, Continuous Zoom, Use Default Tool per Session.
- Roaming Indicator:** Hide Roaming Indicator after 1.5 (sec), Always Display Roaming Indicator.
- Toolbar:** Autohide.
- Magnification Glass Scaling:** Active.
- PixelMeter Zoom Factor:** Image Size, True Size, Off.
- Default Linking Properties:** Window/Level, Scrolling/Cine Mode.
- Scrolling through Stacked Images and Priors:** One Combined Stack Indicator, Two Separate Stack Indicators, Link Scrolling through Priors.
- TIFF Export Resolution:** Quadruple Tiling, Full Resolution.
- CAD Display:** Configure...


After you make your selections, save your settings by clicking **Apply** (at the lower right corner of the screen).

- **Image Overlay:** During review, the image overlay provides patient information for the currently viewed image. In Reduced or Extended mode, the system displays the information configured by the Administrator for that mode (see [4.2.14. Patient Information Overlays](#)).
- **Annotations:** Use to set how SecurView responds when you add a text description to an ellipse, freehand, or arrow marking, for example, by selecting from a Full or Reduced set of predefined classifications, selecting from predefined text you create, and/or providing dialog space where you can enter text (see [4.5. Creating and Viewing Annotations](#)).
- **Left Mouse Button Default Function:** Determines the default image tool for the left mouse button.
  - **Use Default Tool per Session** - when you select a new left mouse button tool during a patient review session, the chosen tool remains selected when you review the next patient.



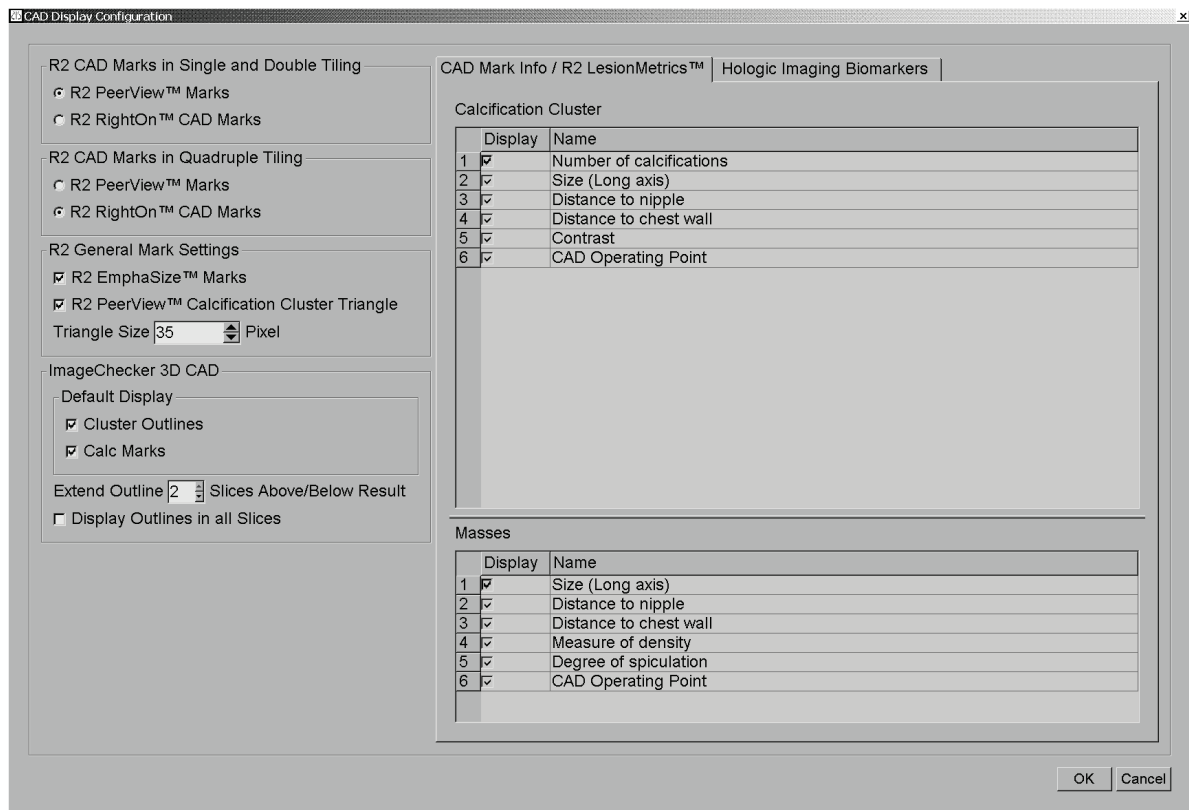- **Use Default Tool per Patient** - when you select a new left mouse button tool during a patient review session, the tool reverts to the user default when you review the next patient.
- **Roaming Indicator:** Determines the behavior of the roaming indicator. See [4.2.8. Intelligent Roaming](#).
- **Toolbar:** Determines whether the MG Viewer toolbar is visible. If you hide the toolbar, you must use the keypad and keyboard to enter commands. To display (or hide) the toolbar at any time, press [\*].
- **Magnification Glass Scaling:** Determines whether metric scaling lines appear when you use the Magnifier tool (see [4.3.1. Magnifier and Inverted Magnifier](#)).
- **Pixel Meter Zoom Factor:** Determines the zoom factor used by the Pixel Meter, either Image Size (relative to the pixel size of the original image) or True Size (relative to true physical size). You can also turn the Pixel Meter off (see [4.2.10. Pixel Meter](#)).
- **Default Linking Properties:** Sets the behavior of all tiles you designate as linked.
  - Selecting Window/Level allows you to change window/level parameters simultaneously in all linked tiles (see [4.3.4. Window/Level and Gamma Adjustments](#)).
  - Selecting Scrolling/Cine Mode allows you to cine through all linked tiles (see [5.2.5. Using Cine Mode](#)).
- **Scrolling through Stacked Images and Priors:** When a single tile contains multiple images, you can elect to display either one or two Stack indicators (see [4.2.11. Stack and Timepoint Indicators](#)). If you select **Link Scrolling through Priors**, then you can use the Stack indicator to scroll through all stacked tiles simultaneously.
- **TIFF Export Resolution:** Configures the resolution of exported Tiff images (see [10.1. Exporting Currently Displayed Image Files](#)).
- **CAD Display:** Use to specify how CAD Marks appear on the MG Viewer. When you click **Configure...**, the following screen appears:



 **Note:** The ImageChecker 3D CAD section appears only if the product license is present.

If you click the Hologic Imaging Biomarkers tab, the following screen appears:

**R2 CAD Marks in Single and Double Tiling**

- ☒ R2 PeerView™ Marks
- ☐ R2 RightOn™ CAD Marks

**R2 CAD Marks in Quadruple Tiling**

- ☐ R2 PeerView™ Marks
- ☒ R2 RightOn™ CAD Marks

**R2 General Mark Settings**

- ☒ R2 EmphaSize™ Marks
- ☒ R2 PeerView™ Calcification Cluster Triangle
- Triangle Size:  Pixel

**ImageChecker 3D CAD**

**Default Display**

- ☒ Cluster Outlines
- ☒ Calc Marks

**Extend Outline**

- Slices Above/Below Result
- ☐ Display Outlines in all Slices

**CAD Mark Info / R2 LesionMetrics™** | **Hologic Imaging Biomarkers**

**Quantra**

Display	Abbreviation	Name
<input checked="" type="checkbox"/>	Vd	Volume of dense tissue
<input checked="" type="checkbox"/>	Vb	Volume of breast
<input checked="" type="checkbox"/>	Vbd	Volumetric breast density
<input checked="" type="checkbox"/>	Vd-score	Volume of dense tissue reference score
<input checked="" type="checkbox"/>	Vbd-score	Volumetric breast density reference score
<input checked="" type="checkbox"/>	Abd	Area breast density
<input checked="" type="checkbox"/>	qDC	Quantra Breast Density Category - fractional
<input checked="" type="checkbox"/>	QDC (1~a,4~d)	Quantra Breast Density Category

**Calcium Scoring**

Display	Abbreviation	Name
<input checked="" type="checkbox"/>	Abac	Area of breast arterial calcium

Display by Default:

OK Cancel

**Note:** Display of biomarkers is subject to availability. Check with your local sales representative for information.

**Note:** The biomarkers result content may vary depending on the Hologic Imaging Biomarkers algorithm version.

## 6.4. User Profile Preferences


The following screen appears when you select the User Profile tab:

The screenshot shows the 'User Profile' tab within the 'User Preferences' section. The 'User Profile' section contains input fields for 'User Name (\*)' (jsmith), 'Last Name (\*)' (Smith), 'First Name (\*)' (John), 'E-mail Address', 'Password (\*)' (masked with asterisks), and 'Confirm Password (\*)' (masked with asterisks). The 'Auto Log-off' section at the bottom left has a dropdown for 'Auto Log-off Time' set to '10 min'. The 'Synchronization Credentials' section on the right has input fields for 'User Name (\*)' (jsmith), 'Password (\*)' (masked), and 'Confirm Password (\*)' (masked). Below this, the 'Synchronize with External Application' section has four unchecked checkboxes: 'On login and logout', 'When opening a patient', 'When closing a study', and 'Ask me to select the study to synchronize'. The 'Incoming Synchronization Requests' section has one checked checkbox: 'Notify if patient not available'. The status bar at the bottom displays 'Help', the date and time '10-18-2012 11:31:30 am', the user name 'User Name: jsmith', and 'OK', 'Cancel', and 'Apply' buttons.

When done with your settings, click **Apply** (at the lower right corner of screen).

- **User Profile:** Allows you to enter your name, password, and, if desired, an e-mail address. Note that an Administrator must set the User Name field.
- **Auto Log-off:** Sets how long the application waits for activity before automatically logging you off.
- **Synchronization Credentials:** Use to enter the reviewer's user name and password on the external application (if they differ from those used with SecurView and login/logout synchronization is supported). Active Directory users must enter the reviewer's user name and password if login/logout synchronization is supported.
- **Synchronize with External Application:** Use to set preferences for synchronizing with an external application. See [4.9. Patient Synchronization with an External Application](#).
  - **On login and logout:** This setting applies only if the external application supports receiving login and logout messages. When the box is checked and you log into (or out of) SecurView, the workstation sends a synchronization message to log you into (or out of) the external application.
  - **When opening a patient:** This setting applies only if the external application supports receiving Open Patient messages. When the box is checked and you open a patient on SecurView, the workstation sends a synchronization message to open the patient on the external application.

- **When closing a study:** This setting applies only if the external application supports receiving the Update Patient State message. When the box is checked and you close a study on SecurView, the workstation sends a synchronization message to update the patient state on the external application.

 **Note:** *At this time, only the Hologic MultiView workstation supports synchronization at close of study.*

- **Ask me to select the study to synchronize:** This setting applies only if the external application supports receiving Open Patient messages. When the box is checked and you open a patient on SecurView, the workstation opens a dialog with a list of studies available for the patient on SecurView. When you select a study from the list, the workstation sends a synchronization message to open the same study on the external application. This setting is useful when more than one unread study for a patient is a common occurrence (e.g., mammography and ultrasound), patients are not being opened by barcoding an accession number, and you want to control which study is sent.
- **Incoming Synchronization Requests:** The 'Notify if patient not available' setting applies only if the external application supports sending Open Patient messages. Check this box if you want SecurView to display an error message when it receives an Open Patient message for a patient that is not available on SecurView.



---

## Chapter 7: Hanging Snapshots and ReportFlows

- ▶ 7.1. Viewing ReportFlows
- ▶ 7.2. Viewing Hanging Snapshots
- ▶ 7.3. Creating and Modifying Hanging Snapshots
- ▶ 7.4. ReportFlows
- ▶ 7.5. Linking a ReportFlow to a Procedure
- ▶ 7.6. Creating New ReportFlows
- ▶ 7.7. ReportFlows Preferences

This chapter describes Hanging Snapshots and ReportFlows, how a Radiologist user can select specific ReportFlows for everyday use, and how to create and modify Hanging Snapshots and ReportFlows.

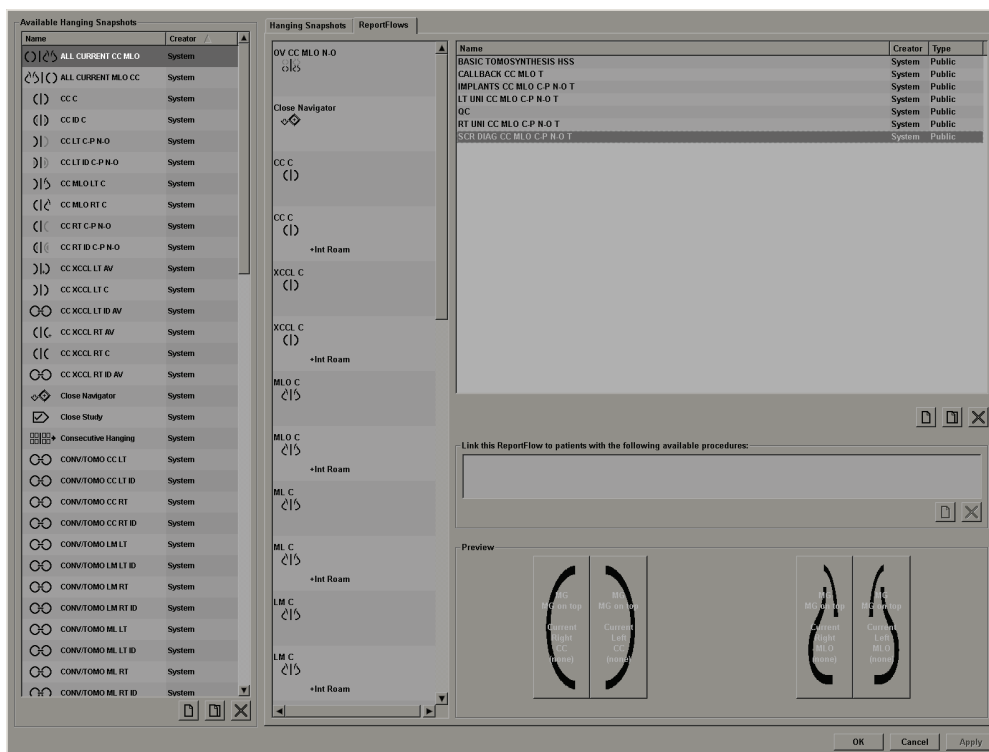
## 7.1. Viewing ReportFlows

SecurView provides a default set of ReportFlows that you can use for viewing many common study types.

► **To view available ReportFlows:**

- 1 In the Administration module, select **User Preferences** and then **Workflow**.
- 2 In the Hanging Snapshots and ReportFlows group, click **Configure...** to open the configuration window showing the ReportFlows tab, which appears by default.


At the top, there are three tabs for Hanging Snapshots, ReportFlows, and Preferences. A list of Available Hangings appears at left. At right is the current list of ReportFlows.





## 7.2. Viewing Hanging Snapshots

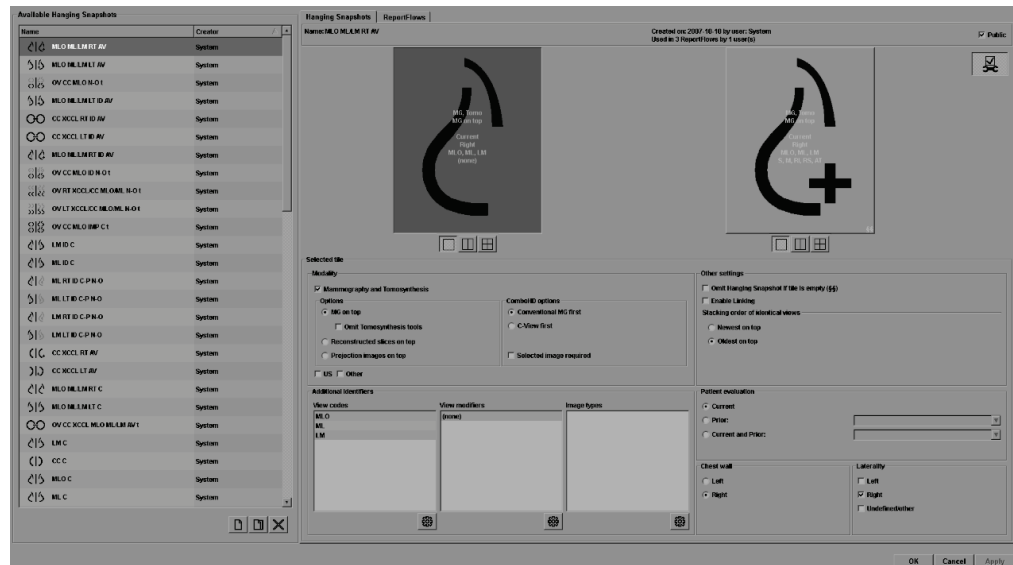A Hanging Snapshot is a set of images in a specified layout on one or more displays. When you click the Hanging Snapshots tab, the editing pane for the currently selected Hanging Snapshot appears, as shown in the example below:



For each hanging snapshot:

- The Name and Type appear at the top.
- Below are individual tilings, where each large tile represents one display. The preview shows the modalities, time points (current or prior), lateralities, view codes, view code icons, view modifiers, image types, and Hanging Snapshot modifiers.
- Properties for the selected tile appear at the bottom.

## 7.3. Creating and Modifying Hanging Snapshots

SecurView workstations provide two levels of Hanging Snapshots:

- System-level Hanging Snapshots are either delivered with the system or created by an Administrator user. These Hanging Snapshots can be modified only by an Administrator user.
- User-defined Hanging Snapshots are created by a Radiologist user, and configured either for:
  - All users - 'Public' is selected by default
  - or
  - Individual use - 'Public' must not be selected

If you create a new Hanging Snapshot, your name appears in the Creator column of the Available Hangings list.

You can create and modify Hanging Snapshots as explained in the following sections:

- ▶ [7.3.1. Creating New Hanging Snapshots](#)
- ▶ [7.3.2. Copying and Editing a Hanging Snapshot](#)
- ▶ [7.3.3. Renaming a Hanging Snapshot](#)
- ▶ [7.3.4. Changing a Hanging Snapshot Icon](#)

### 7.3.1. Creating New Hanging Snapshots

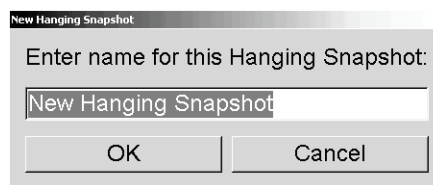
Use the New button to create a new Hanging Snapshot. Alternatively, use the Copy button to copy and modify an existing Hanging Snapshot (see [7.3.2. Copying and Editing a Hanging Snapshot](#)).

- ▶ **To create a new Hanging Snapshot:**



New

- 1 Click the **Hanging Snapshot** tab. Then click **New** below the list of Available Hangings.



- 2 Type a name for the new Hanging Snapshot and click **OK**. SecurView adds a new Hanging Snapshot icon at the very bottom of the Available Hangings list. Your user name appears to the right of the new Hanging Snapshot name.



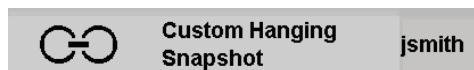
Single  
Tiling



Double  
Tiling



Quadruple  
Tiling



By default, SecurView indicates 'custom' hangings with the icon shown at left above.

**Note:** To rename your custom hanging or to assign a different icon, right-click the Hanging Snapshot name in the Available Hangings column.

- 3 Click **Single Tiling**, **Double Tiling**, or **Quadruple Tiling** to select a tiling for each display in the preview area.

- 4 Click the tile you wish to configure, and then assign properties from the **Selected tile** region:

The screenshot shows a configuration window titled 'Selected tile'. It contains several sections for configuring image display properties. The 'Modality' section includes options for Mammography and Tomosynthesis, with sub-sections for 'Options' (MG on top, Omit Tomosynthesis tools, Reconstructed slices on top, Projection images on top), 'Combined options' (Conventional MG first, C-View first, Selected image required), and 'Other settings' (Omit Hanging Snapshot if tile is empty, Enable Linking, Stacking order of identical views). The 'Additional identifiers' section includes 'View codes' (RL O, RL, LR), 'View modifiers' (none), and 'Image types'. The 'Patient evaluation' section includes 'Current', 'Prior', 'Current and Prior', 'Chest wall' (Left, Right), and 'Laterality' (Left, Right, Undefined/other). The 'Other settings' section includes 'Omit Hanging Snapshot if tile is empty', 'Enable Linking', and 'Stacking order of identical views' (Newest on top, Oldest on top).

- **Modality** – Mammography (including Tomosynthesis), Ultrasound, or Other. For Mammography images you can configure which image type is initially shown in the viewport.

- **Options** – **MG on top**, **Reconstructed slices on top**, or **Projection images on top**. Displays the configured image on top.

**Note:** If Tomosynthesis images are configured to be displayed on top, a Tomosynthesis combination procedure will only be added if the image that is configured to be on top is available. If MG images are configured to be on top and no MG image is available for a combination procedure, the combination procedure will still be added.

- **Omit Tomosynthesis tools** (only in combination with MG on top) – Tomosynthesis images and Tomosynthesis tools are not displayed. The MG images are stacked. Within the stack the MG images are grouped together according to the ComboHD options defining which image to display first.
- **ComboHD options** – **Conventional MG first**, or **C-View first**. Displays the configured MG image first.
- **Selected image required** - The tomosynthesis combination procedure is only added if the configured MG image and an additional image (MG, Projection, or Reconstruction) are available.

**Note:** Additionally, if **Omit Tomosynthesis tools** is selected, the tile is only filled with images of the selected type (i.e. if **C-View first** is configured and C-View 2D and MG Conventional are available, only the C-View 2D image is hung in the tile).

- **Patient evaluation** – Sets the time point for the image display. ‘Current’ refers to the most recent study. Select ‘Prior’ either to view images from a specific prior time point or to view all priors. ‘Current and Prior’ allows viewing current and prior images in one image stack either from newest to oldest (*Current, Prior All (Newest)*) or oldest to newest (*Prior All (Oldest), Current*).
- **Laterality** – Left, right, or undefined/other.
- **Chest Wall** – Image orientation for MG images.
- **Omit Hanging Snapshot if tile is empty** – If selected, SecurView omits this Hanging Snapshot from the ReportFlow if the tile is empty.
- **Enable Linking** – For MG images, use this setting to adjust window width/center values for linked tiles simultaneously. For Tomosynthesis images, this setting synchronizes the tile so it scrolls automatically with other linked tiles. (This setting corresponds to the Link Tile option on the Pie menu. See [5.2.8. Scrolling Through Linked Tiles](#)).

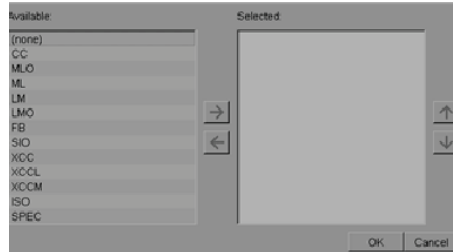
- **Stacking order of identical views** – Stacks identical views within the same tile by the time sequence in which they were taken.

5 Choose **Additional identifiers** for the tile.




Edit


- a Click **Edit** under View Codes to open the Edit Views dialog.



Edit Views

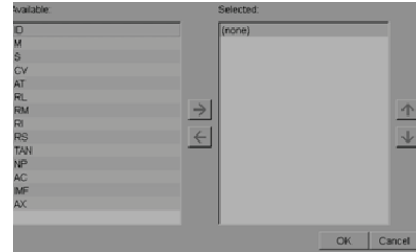

Edit View Modifiers

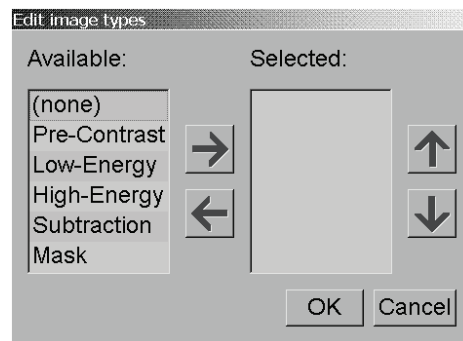

Edit Image Types


- b Click the view(s) you want in the Available column (you can click more than one). Then either drag the views to the Selected column or use the arrow to move them.
- You can stack any or all views in the tile. They stack in the order you choose.
  - Change the order by selecting a view and clicking the arrow buttons at right.
  - If you choose '(none)', the system hangs images that have no view (view modifier or image type) identifier in the tile.
- c Click **OK** to close the Edit Views dialog.
- 6 Choose **View Modifiers** for the tile by repeating the sequence in step 5.
- 7 Choose **Image Type** for the tile by repeating the sequence in step 5.
- 8 Repeat steps 4–7 for other tiles in this hanging.
- 9 When you finish defining all tiles for the hanging:
- Click **Apply** to save your new Hanging Snapshot. (Continue editing, if desired.)
  - Click **OK** to save your new Hanging Snapshot and close the Hanging Snapshot tab.

### 7.3.2. Copying and Editing a Hanging Snapshot

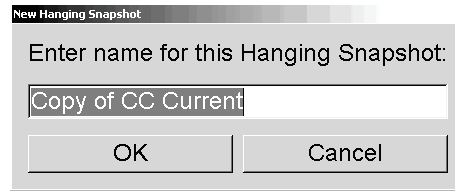Use the Copy button to create a new Hanging Snapshot from an old one.

- 1 Click any Hanging Snapshot in the Available Hangings list.
- 2 Click **Copy** and then enter a name for the copied Hanging Snapshot.





- 3 Modify any properties of the Hanging Snapshot using steps 3–9 of the previous procedure.

### 7.3.3. Renaming a Hanging Snapshot

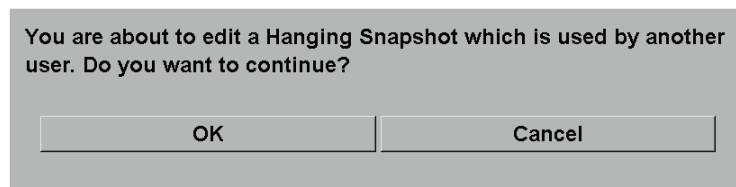
You can rename a Hanging Snapshot, with some restrictions:

- Radiologist users can rename their own custom Hanging Snapshots.
- Administrator users can rename some system-level Hanging Snapshots.
- Some system-level Hanging Snapshots cannot be renamed.

► **To rename a Hanging Snapshot:**

Rename  
Assign icon  
**Shortcut  
Menu**

- 1 In the Available Hangings list, right-click a Hanging Snapshot and select **Rename** from the shortcut menu. If you are renaming a system-level Hanging Snapshot, SecurView displays:



- 2 Click **OK** then enter the new name:



- 3 When done, click **OK**.

### 7.3.4. Changing a Hanging Snapshot Icon

Administrator users can change the icon assigned to system-level Hanging Snapshots. Radiologist users can change the icon assigned to their own custom Hanging Snapshots.

► **To change a Hanging Snapshot icon:**

Rename  
Assign icon  
Shortcut  
Menu

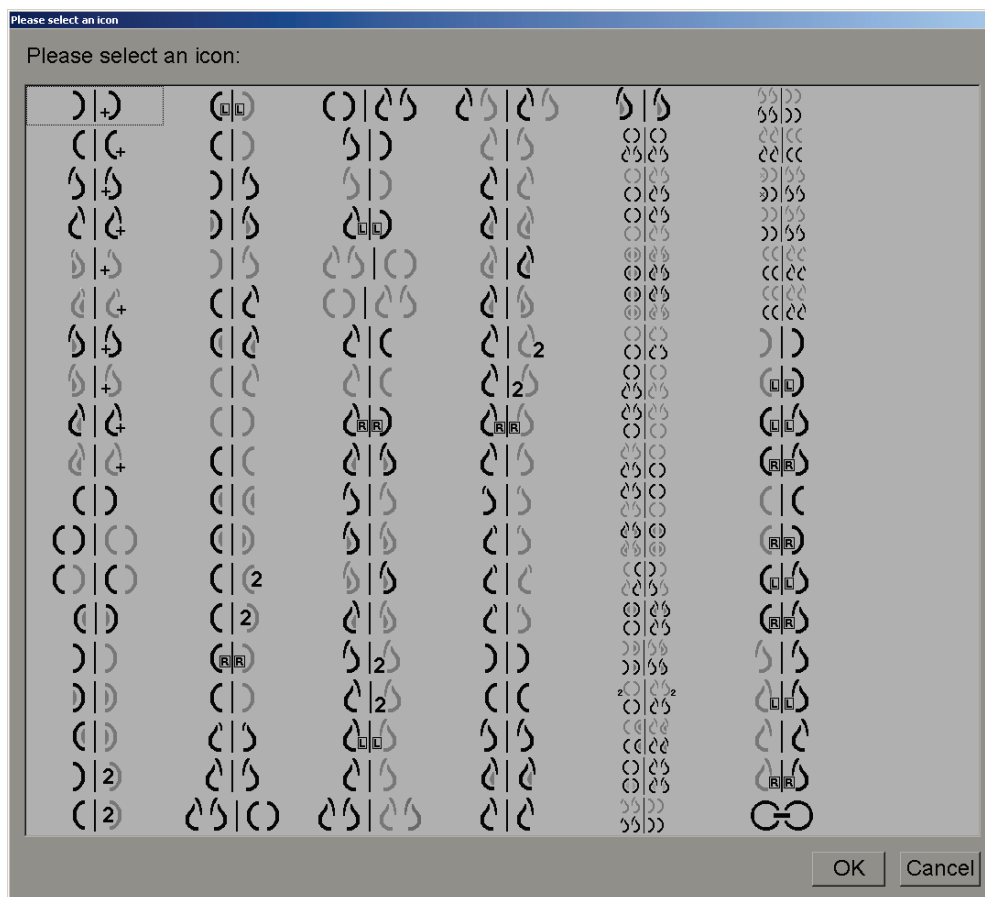
- 1 In the Available Hangings list, right-click a Hanging Snapshot and select **Assign icon** from the shortcut menu. If you are modifying a system-level Hanging Snapshot, SecurView displays:

You are about to edit a Hanging Snapshot which is used by another user. Do you want to continue?

OK

Cancel

- 2 Click **OK** then select a new icon:



- 3 When done, click **OK**.

## 7.4. ReportFlows

A **ReportFlow** is a sequence of Hanging Snapshots and reviewing steps. Shown highlighted below is an example of a ReportFlow (partial view).

Hanging Snapshots   ReportFlows   Preferences			
OV MLO CC N		Name	Enabled
Close Navigator		SCR DIAG MLO CC P-C O-N	✓
MLO C		SCR DIAG MLO CC P-C N-O	✓
		SCR DIAG MLO CC C-P O-N	✓
		SCR DIAG MLO CC C-P N-O	✓
		SCR DIAG CC MLO P-C O-N	✓
		SCR DIAG CC MLO P-C N-O (1)	✓
		SCR DIAG CC MLO P-C N-O	✓
		SCR DIAG CC MLO C-P O-N	✓
		RT UNI MLO CC P-C O-N	✓
		RT UNI MLO CC P-C N-O	✓
		RT UNI MLO CC C-P O-N	✓
		RT UNI CC MLO P-C O-N	✓
			System Public

This ReportFlow, 'SCR DIAG MLO CC C-P N-O', refers to a ReportFlow used for a Screening or Diagnostic mammogram that presents images as explained below:

- MLO images are shown at left, CC images at right
- Current images are shown first, then Prior images
- Newer images are shown at the top of the stack, Older images at bottom

When installed, SecurView includes a set of ReportFlows, available to all radiologists, which provide hangings suitable for most practices. Radiologists and Administrators can create new ReportFlows as needed (see [7.6. Creating New ReportFlows](#)). You can configure SecurView to select automatically the best matching ReportFlow when opening a patient (see [7.7. ReportFlows Preferences](#)). You can also select manually any available ReportFlow during patient review.

The ReportFlow steps appear in sequence in the column at left above. If you click any step, the details of that hanging appear in the Preview area at bottom right as shown below.

When selecting a ReportFlow step of a ReportFlow, the corresponding hanging snapshot will be selected in the list of hangings.

A private hanging of another user will appear in the list of ReportFlow steps of the ReportFlow with a lock icon. This hanging does not appear in the list of hangings. The selected hanging is not changed.



On the right side of the ReportFlow list are three columns:

Enabled	Creator	Type
✓	System	Public
✓	System	Public
✓	System	Public
✓	System	Public
✓	System	Public

- **Enabled:** When checked, indicates that this ReportFlow is available to the current Radiologist.
- **Creator:** Indicates whether the ReportFlow was defined by an Administrator ('System'), or by a user such as a Radiologist. If you create a new ReportFlow, your name appears in the Creator column next to the ReportFlow name.
- **Type:** Indicates whether the ReportFlow is available to all users ('Public'), or available only to the creator ('Private'). 'System' ReportFlows are always 'Public', that is, available to all Radiologist users.

By **right-clicking** in the Enabled column, you can (1) select and deselect ReportFlows to use or leave out, and (2) assign a ReportFlow to be either Public or Private.



## 7.5. Linking a ReportFlow to a Procedure

The **Link this ReportFlow** window appears just below the ReportFlow list.



You can link a specific ReportFlow to a procedure selected by a technologist at the mammography acquisition workstation. Each procedure corresponds to a predefined set of images associated with the study type. SecurView uses a specific ReportFlow based on information in the DICOM header(s) of the patient's images and the encoded procedure name.

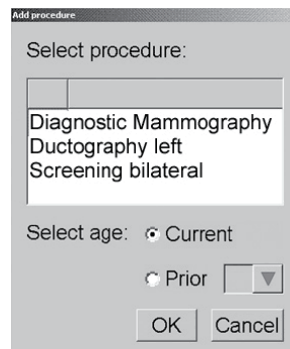
To use this feature, an Administrator user must configure SecurView with the procedure names (see [8.6. Configuring Examination Procedure Names](#)). Also, the **Select ReportFlow, based on procedure names** option must be selected on the ReportFlow Preferences screen. (See [Workflow Selection](#) on page 134.)

► **To link a ReportFlow to a procedure:**

- 1 In the Name column at top, click a ReportFlow name.
- 2 Below the Link this ReportFlow window, click **New** (shown above) to display a list of procedure names:



New



- 3 Select a procedure to link to the ReportFlow, specify either Current or Prior images, and click **OK**.

## 7.6. Creating New ReportFlows

The ReportFlows tab also allows you to create, modify, and delete ReportFlows. It is the Administrator's task to create and modify system-level ReportFlows, but every Radiologist user can create and modify ReportFlows for private or public use.

► **To create a new ReportFlow:**



New

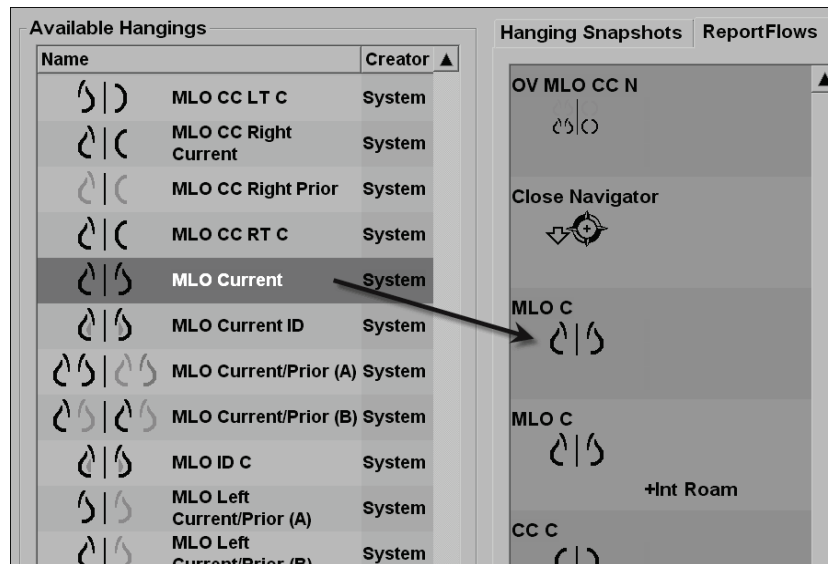
- 1 Click the **ReportFlows** tab. Then click **New** below the list of ReportFlow names.



Copy

Alternatively, copy an existing ReportFlow by pointing at the ReportFlow name and clicking **Copy**.

- 2 Type a unique name for the new ReportFlow and click **OK**. The new ReportFlow is added to the list labeled with your user name as 'Creator', 'Enabled', and 'Private'.
  - To make the ReportFlow available to other users, right-click the ReportFlow name and select **Public**.
  - To change the name of a Private ReportFlow, right-click the ReportFlow name, and select **Rename**.
- 3 Drag and drop a Hanging Snapshot from the Available Hangings list into the list of ReportFlow steps.




- 4 Repeat for each hanging needed in the ReportFlow. You can:
  - Move a Hanging Snapshot by dragging it to a new position.
  - Remove a Hanging Snapshot by dragging it back to the Available Hangings list.


The Available Hangings area also includes several functional ReportFlow steps that you can use in your ReportFlow.

Icon	ReportFlow Step
	Open MammoNavigator
	Close MammoNavigator
	Consecutive Hanging – a Hanging Snapshot that displays additional images not covered by the current ReportFlow.
	Synchronize with an external application
	Close Study
	Open Hologic Imaging Biomarkers Dialog. Changing to the next or previous ReportFlow step will close this dialog automatically.

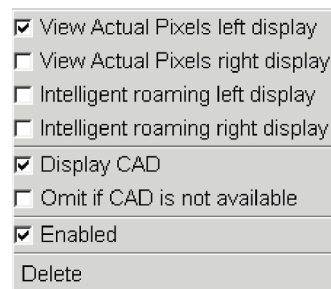- Assign additional properties as needed to the Hanging Snapshots in your ReportFlow. Right-click the Hanging Snapshot to open the shortcut menu, and make your selections:




**Shortcut Menu**

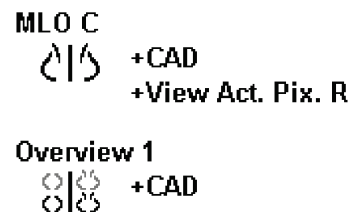

**Hanging Snapshots with Modifiers**

**Note:** Intelligent Roaming and View Actual Pixels are available if the hanging display is configured in single tilling. The hanging tilling should not be changed after enabling Intelligent Roaming or View Actual Pixels.

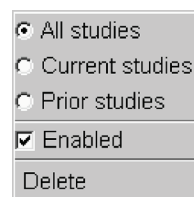- If your ReportFlow includes the Consecutive Hanging, then you can modify the step by right-clicking it to open its shortcut menu.



- After defining the steps:

- Click **Apply** to save your new ReportFlow. (Continue editing, if desired.)
- Click **OK** to save your new ReportFlow and close the ReportFlows tab.

## 7.7. ReportFlows Preferences

Use the Preferences tab to configure the system to select the best matching ReportFlows for your procedures.

### Workflow Selection

Choose one of three radio buttons. The button selected determines how SecurView chooses the ReportFlow to apply automatically when you open a patient.

- **Select ReportFlow, based on procedure names** – SecurView chooses the ReportFlow from the set of procedure descriptions that are associated with the available studies (see [7.5. Linking a ReportFlow to a Procedure](#)).
- **Select best matching ReportFlow, based on all images** – SecurView chooses the ReportFlow based on the images and views contained in the available studies.
- **Always use this ReportFlow by default** – For each procedure type listed, SecurView uses the ReportFlow you select from the drop-down list of all ReportFlows available to you.

Note that when you review a patient, you may choose a different ReportFlow as desired.

### Overview Hanging Configuration



Overview

You can select a Hanging Snapshot to serve as your personal overview display. The configured setting is linked to the Overview button on the left toolbar of the MG Viewer, and with the Overview key on the special keypad (see [4.2.7. Image Hangings](#)).

#### ► To select a personal overview display:

On the Available Hangings list, click a Hanging Snapshot and drag it to the configuration button.

---

## Chapter 8: Administrator Tasks

- ▶ 8.1. Opening the Administration Module
- ▶ 8.2. Managing User Profiles
- ▶ 8.3. Administrator User Profile
- ▶ 8.4. Configuring System-Level Settings
- ▶ 8.5. Configuring System-Level Hanging Snapshots and ReportFlows
- ▶ 8.6. Configuring Examination Procedure Names
- ▶ 8.7. Configuring Image Overlays
- ▶ 8.8. Maintaining the Database

This chapter describes how the SecurView system Administrator can manage users, configure system-level settings, and back up or restore the database.

## 8.1. Opening the Administration Module

Use the Administration module for managing user profiles, configuring system-level settings, and backing up/restoring the database.

► **To open the Administration module:**

- 1 Log into SecurView as **admin**.
- 2 Click the **Administration** tab to display the User Setup window.

Patient Manager	User Setup	User Preferences	Settings	ReportFlow	Maintenance	◀▶
User Name	Name	Groups	Rights			
admin	admin admin	Administrator	User Setup			
application	application application	Administrator	User Setup			
jsmith	jsmith jsmith	Radiologist, Technologist	Diagnostic Setup, Diagnostic Reading, Screening Setup, Screening Reading			
mpatil	mpatil mpatil	Radiologist	Diagnostic Setup, Diagnostic Reading, Screening Setup, Screening Reading			
msmith	Michael Smith	Technologist	Diagnostic Setup, Screening Setup			
r2reader	r2reader r2reader	Radiologist	Diagnostic Setup, Diagnostic Reading, Screening Setup, Screening Reading			
review	User Review	Radiologist, Technologist	Diagnostic Setup, Diagnostic Reading, Screening Setup, Screening Reading			
service	service service	Service				
tzhang	tzhang tzhang	Case Administrator				
				Add	Edit	Delete
				08-11-2011 04:23:33 pm User Name: admin		
				Help	OK	

The Administration module provides the following tabs:

- **Patient Manager** – Displays the Patient List with all patient IDs, their studies, and series currently in the database. The system Administrator does not have the option to create a new session, merge or synchronize patients.
- **User Setup** (opens by default) – Use for adding, editing, and deleting users. See [8.2. Managing User Profiles](#).
- **User Preferences** – Displays the system Administrator's user profile. See [8.3. Administrator User Profile](#).
- **Settings** – Use for configuring system settings such as disk space monitoring and synchronization with an external application. See [8.4. Configuring System-Level Settings](#).
- **ReportFlow** – Use for defining procedure names and configuring system-level image hangings for Radiologist users. See [8.5. Configuring System-Level Hanging Snapshots and ReportFlows](#). You can also use the ReportFlow tab to configure procedure names. See [8.6. Configuring Examination Procedure Names](#).
- **Maintenance** – Use for backing up, restoring, and maintaining the database. See [8.8. Maintaining the Database](#).
- **Overlay** – Use for configuring image overlay information for Radiologist users. See [8.7. Configuring Image Overlays](#).
- **About** – Displays application information. Refer to this information when contacting the Hologic Help Desk.

## 8.2. Managing User Profiles

The User Setup window displays profile information for all registered users. You can use the three buttons to add, edit, or delete user profiles.

User Setup		User Preferences		Settings	ReportFlow	Maintenance	Overlay	About ◀▶	
User Name	Name	Groups		Rights					
admin	admin admin	Administrator		User Setup					
application	application application	Administrator		User Setup					
jsmith	jsmith jsmith	Radiologist, Technologist		Diagnostic Setup, Diagnostic Reading, Screening Setup, Screening Reading					
mpatil	mpatil mpatil	Radiologist		Diagnostic Setup, Diagnostic Reading, Screening Setup, Screening Reading					
msmith	Michael Smith	Technologist		Diagnostic Setup, Screening Setup					
r2reader	r2reader r2reader	Radiologist		Diagnostic Setup, Diagnostic Reading, Screening Setup, Screening Reading					
review	User Review	Radiologist, Technologist		Diagnostic Setup, Diagnostic Reading, Screening Setup, Screening Reading					
service	service service	Service							
tzhang	tzhang tzhang	Case Administrator							
Add		Edit		Delete					
Help		08-11-2011 04:24:33 pm User Name: admin							OK

### ► To add a new user profile:

- 1 In the User Setup Window, click **Add** to display the following dialog:

User Name:

Last Name:

First Name:

Password:

Confirm Password:

Authentication  
☐ Use Active Directory

**Groups**

- ☐ Administrator
- ☐ Radiologist
- ☐ Technologist
- ☐ Service
- ☐ Case Administrator

**Rights**

**Diagnostic**

- ☐ Setup
- ☐ Reading

**Screening**

- ☐ Setup
- ☐ Reading

**User**

- ☐ Setup


OK Cancel

**Note:** On SecurView RT, the 'Rights' options do not appear.

- 2 Enter the user name, last name, and first name. Then:
  - Enter a password in the Password and Confirm Password fields, or
  - Select 'Use Active Directory' to authenticate the user via Active Directory server.

**Note:** For sites using Active Directory authentication:

- Active Directory must be configured before adding new users. See [8.4.4. Active Directory Setup](#).
- The user name must not contain a backslash (\) character.

- Use the '@' character only in the form *username@domain*, which restricts the user to authenticate against a specific domain. This is useful when different users with the same name exist in different domains.
    - There is no requirement to add the domain to the user name. If no domain is specified, the system can authenticate the user against any domain.
  - 3 Assign the user to a group. (As a special case, you can assign a user to both the Radiologist and Technologist groups.)
    - **Administrators** have rights to configure system settings, as explained in this chapter.
    - **Radiologists** have rights to review patient images on SecurView DX and set up Sessions of diagnostic and/or screening studies (see [3.3. Creating Sessions](#)).
    - **Technologists** have rights to set up Sessions of diagnostic and/or screening studies (see [3.3. Creating Sessions](#)).
    - **Service** users have rights to configure certain system settings (refer to the *SecurView Workstation Installation & Service Manual*).
    - **Case Administrators** have rights to merge multiple records for a single patient and delete patients from the Patient List (see [Chapter 9: Case Administrator Tasks](#)).
  - 4 Customize access rights (for Administrators, Radiologists, and Technologists only on SecurView DX).
    - **Diagnostic or Screening Setup:** If checked, Radiologist and Technologist users can create Sessions. See [3.3. Creating Sessions](#).
    - **Diagnostic or Screening Reading:** If checked, Radiologist users can review patient images.
    - **User Setup:** If checked, the Administrator user has rights to create and edit user profile information.
  - 5 Click **OK** to save the new user settings.
  - **To edit a user profile:**
    - 1 Do one of the following:
      - Select a user name from the User Setup window and click **Edit**, or
      - Double-click a user name that appears in the User Setup window.
    - 2 Edit the user profile and click **OK** to save.
  - **To delete a user profile:**
    - 1 Select a user name from the User Setup window and click **Delete**. SecurView displays, 'Do you want to delete this user?'
    - 2 Choose **OK** to delete the user profile.
-  **Note:** Application and Service users cannot be deleted from the User Setup list.



## 8.3. Administrator User Profile

Select the **User Preferences** tab to display the User Profile window for the Administrator user who is currently logged in. You can edit your User Profile and set the system **Auto Log-Off** time (i.e., the amount of time the application waits for activity before automatically logging you off).

The screenshot displays the 'Administrator User Profile' window with the 'User Preferences' tab selected. The window has a tabbed interface at the top with the following tabs: 'User Setup', 'User Preferences' (active), 'Settings', 'ReportFlow', 'Maintenance', 'Overlay', and 'About'. Below the tabs, the 'User Profile' section contains two main areas. The first area, titled 'User Profile', includes fields for 'User Name (\*)' (admin), 'Last Name (\*)' (admin), 'First Name (\*)' (admin), 'E-mail Address' (1), 'Password (\*)' (\*\*\*\*\*), and 'Confirm Password (\*)' (\*\*\*\*\*). The second area, titled 'Auto Log-off', includes a label 'Auto Log-off Time' and a dropdown menu set to '30 min'. At the bottom of the window, there is a status bar with a 'Help' button, the text '08-11-2011 04:25:37 pm User Name: admin', and buttons for 'OK', 'Cancel', and 'Apply'.

Tab	Content
User Setup	User Profile
User Preferences	User Profile User Name (*) admin Last Name (*) admin First Name (*) admin E-mail Address 1 Password (*) ***** Confirm Password (*) ***** Auto Log-off Auto Log-off Time 30 min
Settings	
ReportFlow	
Maintenance	
Overlay	
About	

Help 08-11-2011 04:25:37 pm User Name: admin OK Cancel Apply

## 8.4. Configuring System-Level Settings

Use the Settings window to configure system-level settings. (Note that the screen shown below is for SecurView DX. Some settings do not apply to SecurView RT.)

Adjust settings as described in the following sections. After you make your selections, save your settings by clicking **OK**. Some of the settings (e.g., Scheduling, Worklists, and Date/Time Format and Units, etc.) require a SecurView restart to save changes.

- ▶ **8.4.1. Scheduling**
- ▶ **8.4.2. Disk Space Monitoring and Auto-Deletion**
- ▶ **8.4.3. Auto-Fetching/Auto-Completion Configuration**
- ▶ **8.4.4. Active Directory Setup**
- ▶ **8.4.5. Study List Manager (SLM) Configuration**
- ▶ **8.4.6. Application Event Logging**
- ▶ **8.4.7. Search on PACS**
- ▶ **8.4.8. Configure Synchronization Interface**
- ▶ **8.4.9. Worklists**
- ▶ **8.4.10. Date/Time Format and Units**
- ▶ **8.4.11. Multimodality Viewer**
- ▶ **8.4.12. Double Reading for Studies**
- ▶ **8.4.13. Sending Annotations, Tagged Tomo, Study State**

- ▶ **8.4.14. Local Institution**
- ▶ **8.4.15. Secondary Capture**
- ▶ **8.4.16. Adoption of Institution Name and Address**

### 8.4.1. Scheduling

On SecurView DX, use this setting to activate the Create Session option (see **3.2.2. Patient List Buttons** and **3.3. Creating Sessions**).

- **Off:** The Create Session option is unavailable.
- **On:** The Create Session option is available. Users may create a Session if they have Screening or Diagnostic Setup rights.

### 8.4.2. Disk Space Monitoring and Auto-Deletion

These settings specify how long SecurView stores patients in its database and what happens when the hard disk reaches its storage capacity.

Disk Space Monitoring and Auto-Deletion		
<b>Disk Space Capacity Level</b>		
Warning level	80.00	%
Critical level	90.00	%
<b>Maximum Storage Period</b>		
Newly read studies	5	days
Unread studies	10	days
Old studies	10	days
Counting of days includes weekdays, and:		
<input type="checkbox"/> Saturdays		
<input type="checkbox"/> Sundays		
Automatic Deletion <input checked="" type="radio"/> On <input type="radio"/> Off		

#### Disk Space Capacity Level

A warning message appears when disk storage reaches each of the specified limits. Two levels of disk capacity produce alert messages:

- **Warning level:** The default threshold is 80%. Messages appear every 20 minutes until disk space becomes available.
- **Critical level:** The default threshold is 90%. The system does not accept incoming information (e.g., DICOM messages or images) until disk space becomes available.

#### Maximum Storage Period

These settings are available only when Automatic Deletion is **On**. The Maximum Storage Period is the number of days that SecurView stores patients before automatically deleting patients that meet the configured criteria. The storage period is based on the date and time SecurView receives the last image of a given study.

On SecurView RT workstations, the settings for Newly Read and Unread studies are replaced with one setting called New studies.

## Automatic Deletion

When set to **On**, SecurView automatically deletes patients (image data and other objects) according to predefined criteria. Automatic Deletion functions in two ways:

- **Ongoing automatic deletion** – SecurView deletes patients with expired storage periods. The available disk capacity has no influence. As an example on SecurView DX, when a patient's state is *Read* and the *Newly Read studies* storage period expires, SecurView deletes the patient even if the patient has *Old studies* for which the storage period has not expired.
- **Forced automatic deletion** – When available disk capacity exceeds the Warning level, SecurView deletes the oldest patients with status *Read* until the available disk capacity falls below the Warning level. The maximum storage period settings have no influence.

The following table summarizes the behavior.

Event	Automatic Deletion On	Automatic Deletion Off
Check hourly	Perform ongoing Automatic Deletion	No action
System reaches Warning level	Perform forced Automatic Deletion	Display Warning level message
System reaches Critical level	Display Critical level message. Refuse further incoming DICOM data. Perform forced Automatic Deletion when no user is logged in.	Display Critical level message. Refuse further incoming DICOM data. Perform forced Automatic Deletion when user clicks <b>Cleanup</b> in the Patient Manager window.

SecurView does not automatically delete a patient if:

- A user is currently viewing the patient.
- A user is logged in on a multiworkstation system.
- The patient has a pending store-annotations job or printing job.
- The patient is locked (e.g., Pending) and/or protected against automatic deletion (SecurView DX only).
- The patient includes an image with a Notice, and the study is not marked as 'viewed' (SecurView RT only).
- The patient has only non-local studies.

When deleting merged patients, SecurView considers the studies of both the primary and secondary patients.

Patients having both local and non-local studies can be automatically deleted. Afterward SLM synchronization may add the patient to the Patient List with just the non-local studies.

### Automatic Deletion in a Multiworkstation Configuration

In a multiworkstation configuration, the Automatic Deletion function behaves differently than on a Standalone system:

- On the **Manager**, the Maximum Storage Period settings are global. Warning and Critical level settings are local. If Automatic Deletion is **On**, the Manager performs hourly checks, ongoing patient deletions (propagated to all Clients), and cleanup procedures. During cleanup, the Manager deletes copies of prepared images from the Clients. (The Clients' Automatic Deletion settings have no influence).
- On the **Client**, Automatic Deletion settings are local; they do not influence other Clients or the Manager. If Automatic Deletion is **On** and the Warning level has been reached, the Client deletes locally stored patients. The datasets removed from the local system still reside on the Manager.

### Cleanup on Multiworkstation Clients

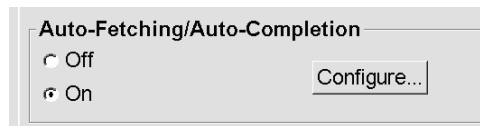
On a Client you can delete patients with the **Cleanup** button. If you click **Cleanup**, a message appears asking whether you want to start the Automatic Deletion function. If you click 'Yes', SecurView logs you off the system and carries out forced Automatic Deletion.

- For Radiologist users, the **Cleanup** button appears in the Patient Manager window (next to the OK button) whenever the Critical level is exceeded.
- For Administrators and Service users, the **Cleanup** button is always available. If the available disk capacity is below the Warning level, then the Cleanup function deletes a predetermined number of patients (default is 50).

#### 8.4.3. Auto-Fetching/Auto-Completion Configuration

The Auto-Fetching function automatically retrieves prior objects from an archive when SecurView receives new studies. The objects can be images, CAD SRs, study states with or without Annotations, and third-party GSPS objects in studies that meet the Auto-Fetching criteria. See [3.2.5. Auto-Fetching Patient Data](#).

The Auto-Completion function automatically retrieves all objects of the referenced study when SecurView receives a GSPS or CAD SR object.



If you click **Configure...**, you can select options from the following dialog:

- Mode:** Select **Auto-Fetching of prior images** if SecurView is responsible for retrieving prior studies from the archive.
  - For SecurView DX, Auto-Completion is usually unnecessary because it adds redundant network traffic.
  - Note:** If MG Secondary Capture from a first reader is sent to PACS and GSPS Report from the first reader is received before the second reading is complete, then selecting 'Auto-Completion of study on incoming GSPS' can violate a double-blind reading protocol by retrieving the MG Secondary Capture (see [8.4.13. Sending Annotations, Tagged Tomo, Study State](#)).
  - For SecurView RT, Auto-Completion may be appropriate (e.g., SecurView RT may receive a Notice for which it does not have the images, depending on the site's image routing and/or SecurView RT automatic deletion settings).
- Auto-Fetching Limit:** Select either a time interval (**years back**) or the number of **most recent studies**.
- Retrieve Modalities:** Select which modalities to retrieve. (The 'MG' setting retrieves prior digital mammography studies with associated GSPS Annotations and Mammography CAD SRs.)
- DICOM Query/Retrieve Level:** Select a level based on PACS 1 requirements, where PACS 1 is configured in DICOM Setup by the service engineer.
  - Series level** is preferred if PACS 1 supports it.
  - Study level** is also an acceptable setting. This level requires that PACS 1 supports Modalities in Study (0008,0061) correctly in DICOM queries.
- Auto-Fetching/Auto-Completion Execution:**
  - For a multiworkstation system, the Manager uses the **All the time** option. (**Not logged in** is not available.)
  - For a Standalone system, select either setting. If you select **Not logged in**, the system performs auto-fetching only when no user is logged into the system.


#### 8.4.4. Active Directory Setup

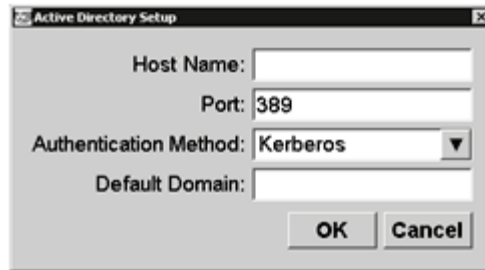Active Directory is a Microsoft Windows directory service used to help manage network security. When you add or edit a user profile, SecurView gives you the option to use Active Directory for user authentication. See [8.2. Managing User Profiles](#).

Active Directory is configurable from the Administration > Settings window:

##### ► To set up Active Directory:


- 1 Click **Configure...** to open the Active Directory Setup dialog:



- 2 Enter the fully qualified host name of the Active Directory server.
- 3 Enter the port of the Active Directory server. The default setting is 389.
- 4 Select an authentication method:
  - Kerberos – the default setting.
  - Digest – not usable in multiple domain scenarios, as different users with the same username but in separate domains cannot be distinguished (one user will not be able to log in).
  - Plaintext Password – not recommended, as passwords are transmitted without encryption.
- 5 As an option, enter the default domain to use if an Active Directory user does not specify a domain when logging in.
- 6 Click **OK** to save the settings.


### 8.4.5. Study List Manager (SLM) Configuration

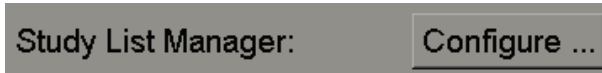Communication with a Study List Manager is configurable from the **Administration > Settings** window:



► **To set up a Study List Manager**

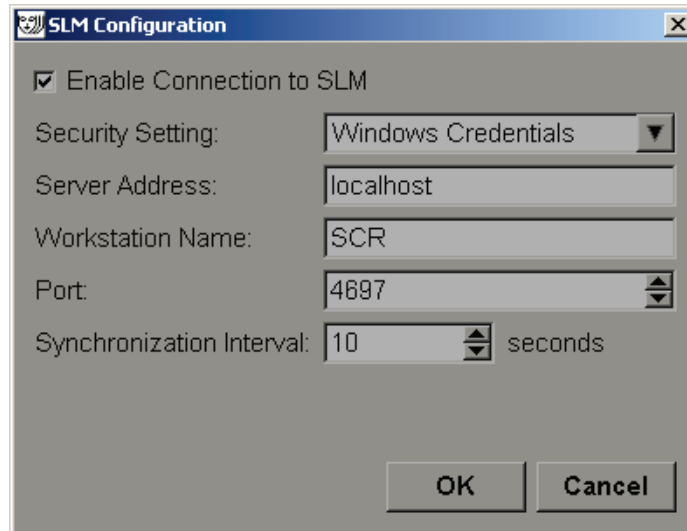- 1 Click **Configure...** to open the SLM Configuration dialog:



- 2 Select **Enable Connection to SLM** to activate connection to a Study List Manager.
- 3 Select a security setting to be used for the communication between SecurView and the Study List Manager. If you select Windows Credentials, the credentials of the currently logged in Windows user are used for connection to the Study List Manager.

Select one of the following security settings:

- a No Security
  - b HTTPS Anonymous Client
  - c Windows Credentials – default
- 4 Enter the **Server Address** (IP address or name) of the server hosting the Study List Manager. The default name is localhost, which can be used when the Study List Manager is running on a SecurView standalone or the SecurView manager in a multiworkstation configuration.
  - 5 Enter the **Workstation Name** used for the communication with the Study List Manager. The default name is the AE Title of SecurView.
  - 6 Enter the **Port** of the server hosting the Study List manager.
    - a The default port for Security Setting 'No Security' is 4699.
    - b The default port for Security Setting 'HTTPS Anonymous Client' is 4698.
    - c The default port for Security Setting 'Windows Credentials' is 4697.



- 7 Select an interval in seconds for the frequency of synchronization with the Study List Manager. The default setting is 10 seconds.
- 8 Click **OK** to save the settings.

#### 8.4.6. Application Event Logging

SecurView can create a log file that captures key application level events. Customers may access these logs to monitor activity on the system or to help demonstrate compliance with HIPAA or other patient privacy policies. An Administrator or Service user can configure application event logging through the Settings tab on the Administration screen. Configuration options allow for enabling or disabling of logging and specifying a target directory for the log file.

The application event log is in CSV format. Each line of the file represents a single event and contains comma-separated values specific to that event. The file can be easily imported into a spreadsheet to allow for detailed analysis.

The following fields are captured for events recorded in the log file. Not all fields apply to all events.

- Date and time stamp
- User Group (Radiologist, Administrator, Case Administrator, and/or Service)
- User (login name)
- Event
- Patient ID
- Study Instance UID
- Other (captures additional information specific to certain events)

The following application level events are captured in the log file:

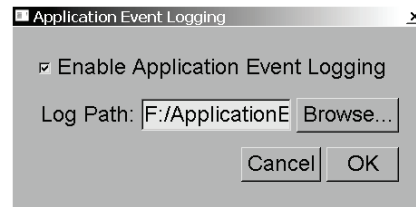
Event	Text Appearing in Event Field	Additional Information in Other Field
Failed login attempt	login failed	
Successful login	logged in	
Successful logout	logged out	
Patient deletion (either manual or automatic)	deleted	
Patient study opened for review	opened	
Patient study imported into system	imported	
Patient study exported from system	exported	type of exported content
Patient study printed	printed	
Patients merged (primary patient information)	merged as primary	secondary Patient ID
Patients merged (secondary patient information)	merged as secondary	primary Patient ID
Patients un-merged	un-merged	primary Patient ID
Password changed	password changed	If changed by Administrator user, contains Administrator user name and groups
Patient study received from external system (each received DICOM file is considered a	received	Remote AE title and IP address

single event)		
Logging disabled	logging disabled	old log file path
Logging enabled (also configuration changes)	logging enabled	new log file path

Application event logging is configurable from the **Administration > Settings** tab:

Application Event Logging: **Configure ...**

When you click **Configure...**, you can enable or disable the feature and define the log folder (the default is F:/ApplicationEventLogging).




**⚠ Important:** Hologic recommends that the Application Event Log folder be maintained on a secure drive external to the SecurView workstation.

**⚠ Note:** In a multiworkstation cluster, this configuration screen is accessible only on the Manager.

#### 8.4.7. Search on PACS

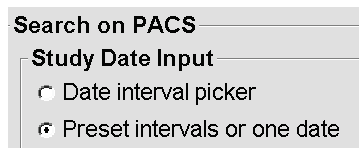Use to select the Radiologist user input for the Study Date field of the Search on PACS dialog (see [3.2.8. Searching for Patients](#)). The date format can be either a specific date range selected by the user or a preset interval/date.

- **Date interval picker:** Displays a calendar sheet, which allows the Radiologist user to select specific start and end dates for the PACS search.
- **Preset intervals or one date:** Displays a preset interval (e.g., today, last month, etc.) in the Study Date text field.




### 8.4.8. Configure Synchronization Interface

Enter the IP address and Port information to allow synchronization between the SecurView workstation and an external application. Synchronization with non-Hologic applications requires an Application Synchronization license.


On a SecurView client workstation, enter the name of a specific SLM client (e.g., MultiView client) in order to add only the named SLM client's studies to the SecurView client's patient list.

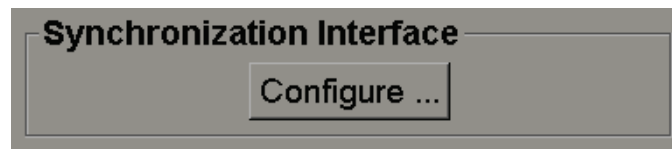The Synchronization Interface is configurable from the **Administration > Settings** window:



► **To set up the Synchronization Interface**

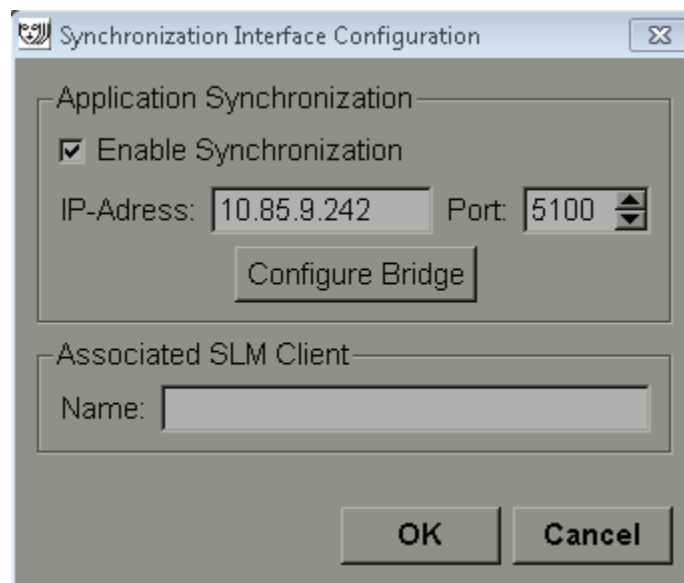- 1 Click **Configure...** to open the Synchronization Interface Configuration dialog:




- 2 Select **Enable Synchronization** to enable communication with Application Synchronization.
- 3 Enter the Application Synchronization IP address.
- 4 Enter the Application Synchronization Port. The default setting is 5100.
- 5 Click **Configure Bridge** with valid IP-Address and Port settings, then the Application Synchronization Configuration interface appears if Application Synchronization is installed on SecurView. For more information, refer to the *Application Synchronization Installation Manual*.

- 6 On a client workstation enter the **Associated SLM Client Name**.
  - Enter a name only if SLM is configured.
  - This option is available on SecurView Standalone and SecurView Client in multiworkstation configurations. This value has to be set on all workstations that have an associated SLM client (e.g., MultiView).
  - Ensure that the entered value matches the associated SLM client's name as registered with the SLM.

#### 8.4.9. Worklists

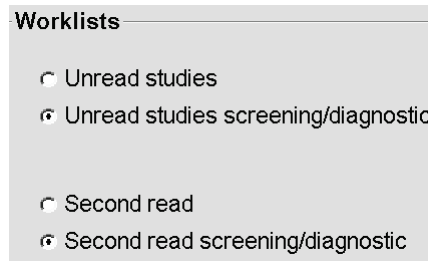On SecurView DX, use to define how SecurView generates worklists automatically.



- **Unread studies:** Generates a single worklist that includes both screening and diagnostic studies.
- **Unread studies screening/diagnostic:** Generates separate worklists for screening and diagnostic studies.
- **Second read:** Generates a single worklist that includes both screening and diagnostic studies.
- **Second read screening/diagnostic:** Generates separate worklists for second-read screening and diagnostic studies. See [4.1.2. Automatically Generated Worklists](#).

#### 8.4.10. Date/Time Format and Units

Click **Configure...** to choose settings for date, time, and force units used with the compression paddle.



#### 8.4.11. Multimodality Viewer

On SecurView DX, click **Configure...** to display the Multimodality (MM) Configuration Editor to modify MM Viewer settings. This function is only available with a valid Advanced Multimodality Option license. For more information, refer to the *SecurView Advanced Multimodality Option User Guide*.

### 8.4.12. Double Reading for Studies

On SecurView DX, use to activate automatic double reading for each type of study (Screening, Diagnostic, and Undefined MG). Default is single reading. When the double reading option is selected, two radiologist users can independently view the same study without knowledge of each other's findings.

Double Reading for Studies		
	single	double
Screening (MG)	<input checked="" type="radio"/>	<input type="radio"/>
Diagnostic (MG)	<input checked="" type="radio"/>	<input type="radio"/>
Undefined	<input checked="" type="radio"/>	<input type="radio"/>

**⚠ Important:** Once SecurView assigns a study for single reading, there is no way to reassign it for double reading. However, for double reading studies, a user can cancel the second reading at close of study. See [4.7. Closing a Study](#).

### 8.4.13. Sending Annotations, Tagged Tomo, Study State

In a double reading scenario, SecurView DX can send a GSPS Report (study reading state with or without Annotations and tagged tomosynthesis slices) and MG Secondary Capture images after each reader marks a study as 'Read'. This setting allows the GSPS mechanism to synchronize reading states between multiple Standalone systems configured for double reading.

Send Annotations, Tagged Tomo, Study State	
for studies to be double-read	
<input checked="" type="radio"/>	after each single-read event
<input type="radio"/>	after the double-read event only

- **after each single-read event:** SecurView sends the GSPS Report and MG Secondary Capture images after any reader marks the study as 'Read'.

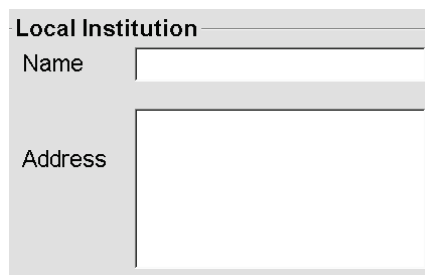
**⚠ Note:** If GSPS or MG Secondary Capture is sent to PACS after the first read, then access to the information before the second reading is complete can violate a double-blind reading protocol.

- **after the double-read event only:** SecurView sends the GSPS Report and MG Secondary Capture images only after the second reader marks the study as 'Read'.

SecurView sends the GSPS Reports and MG Secondary Capture images to destinations configured through the service interface.

#### 8.4.14. Local Institution


SecurView can include your institution name and address when it creates GSPS Reports, GSPS Notices, MG Secondary Captures, and MM ScreenCaptures. See [8.4.16. Adoption of Institution Name and Address](#). If you select this option, enter the institution name and address.



The 'Local Institution' form contains two input fields: 'Name' and 'Address'. The 'Name' field is a single-line text box, and the 'Address' field is a larger multi-line text box.

#### 8.4.15. Secondary Capture

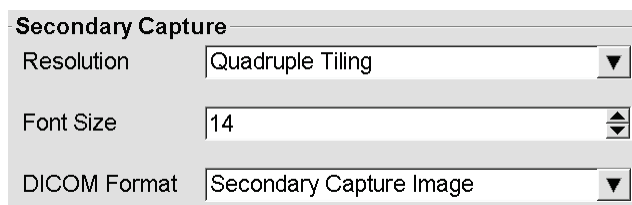On SecurView DX, use this setting to format any MG Secondary Capture images automatically sent at close of study. An MG Secondary Capture image is created only if the destination PACS does not accept GSPS or cannot display GSPS and the customer wants to view Annotations on the PACS workstation. See [4.7. Closing a Study](#).




The 'Secondary Capture' form has three settings: 'Resolution' set to 'Quadruple Tiling', 'Font Size' set to '14', and 'DICOM Format' set to 'Secondary Capture Image'. Each setting is in a dropdown menu.

**Note:** Do not change DICOM Format from the default 'Secondary Capture Image'.

#### 8.4.16. Adoption of Institution Name and Address

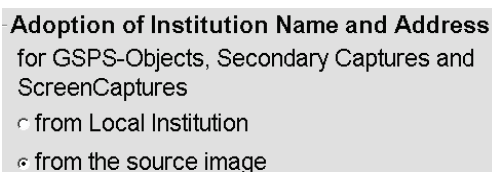Selects source of institution information for GSPS Reports, GSPS Notices, MG Secondary Captures, and MM ScreenCaptures.



The 'Adoption of Institution Name and Address' form has a title 'for GSPS-Objects, Secondary Captures and ScreenCaptures' and two radio button options: 'from Local Institution' and 'from the source image'. The 'from the source image' option is selected.

- **from Local Institution:** Information entered in the **Local Institution** area is applied to all SecurView-created GSPS objects, MG Secondary Captures, and MM ScreenCaptures.
- **from the source image:** Institution information contained in the DICOM header of the source image (from which the GSPS object, MG Secondary Capture, or MM ScreenCapture was created) is applied.

### 8.5. Configuring System-Level Hanging Snapshots and ReportFlows

Administrator users can configure system-level Hanging Snapshots and ReportFlows by selecting the ReportFlow tab and clicking **Configure**.

**Hanging Snapshots and ReportFlows**

Configure ...

Time range for discrimination between Current - Prior 1 - Prior 2 - etc:

30 days

The ReportFlows window opens with lists of Available Hanging Snapshots and ReportFlows. For more information, see [Chapter 7: Hanging Snapshots and ReportFlows](#).

### Current–Prior Time Range

Use the 'Time range' field to set the number of days a study is Current. The default setting is 30, which means a Current study changes to a Prior 1 study if it resides more than 30 days on the system.

## 8.6. Configuring Examination Procedure Names

Use the Examination Procedure Identification window to add, edit, or delete procedure names. A procedure corresponds to a predefined set of images associated with a study type. When producing images, the mammography acquisition workstation encodes the name of the examination procedure into the image DICOM headers. When SecurView receives these images, it determines which ReportFlow to select based on the procedure name. See [7.5. Linking a ReportFlow to a Procedure](#).

**Patient Manager** **User Setup** **User Preferences** **Settings** **ReportFlow** **Maintenance**

Examination Procedure Identification

Procedure Description	Procedure Type	Manufacturer	Modality	Attribute Tag	Attribute Name	Attribute Value
Diagnostic Callback	Diagnostic	Hologic	MG	(0008,1040)	Institutional Department Name	(0008,1040)
Screening-Bilateral Mammography Screening	Screening	LORAD	MG	(0008,1030)	Study Description	Screening-Bilateral Mamm

Priority

↑

↓

⏪ ⏩

⏴ ⏵ ⏶ ⏷

**Hanging Snapshots and ReportFlows**

Configure ...

Time range for discrimination between Current - Prior 1 - Prior 2 - etc:

30 days

Help 10-18-2012 09:31:02 am User Name: admin OK

► **To add a new procedure:**



**New**

- 1 Select the **ReportFlow** tab to display the Examination Procedure Identification window.
- 2 Click the **New** icon to open the New Procedure Identification dialog.

**Important:** Be sure to enter the following procedure attributes correctly. Failure to do so can lead to missing studies in the automatic worklists.

Procedure

Description:

Type:  ▼

Identification

Manufacturer:

Modality:  ▼

Attribute Tag:

Format e.g. '(0008,1030)',  
for sequences  
e.g. '(0040,0275).  
(0040,0007)'

Private Creator:   
(for private attributes only)

Attribute Name:

Attribute Value:

- 3 Enter a procedure name in the **Description** field (e.g., Screening Bi-Lateral Mammography).
- 4 Select the appropriate **Type** from the drop-down list.
- 5 Enter the **Manufacturer** name (optional).
- 6 Select the **Modality** from the drop-down list. **MG** is the default.
- 7 Enter a valid **Attribute Tag** associated with the procedure. The default is (0008,1030), which is the tag for 'Study Description'.
- 8 Enter the procedure name exactly as it appears on the acquisition workstation, including abbreviations, case, and punctuations in the **Attribute Value** field.
- 9 When done with your settings, click **OK**.

► **To edit a procedure:**



**Edit**

- 1 Select the **ReportFlow** tab to display the Examination Procedure Identification window.
- 2 Click the **Edit** icon to open the Edit Procedure Identification dialog.
- 3 When finished editing the procedure, click **OK**.

► **To delete a procedure:**



**Delete**

- 1 Select a procedure from the Procedure Identification window and click the **Delete** icon. The system checks for ReportFlows linked to the procedure you wish to delete. If a ReportFlow is linked to the procedure, the following message appears:

Warning: At least one ReportFlow is linked to this procedure description. By removing this procedure description it will be deleted from the list of linked procedures of one or more ReportFlows. Do you want to continue?

- 2 Click **OK** to delete or **Cancel**.



## 8.7. Configuring Image Overlays

Use the Overlay window to select which information appears in the MG Viewer, MammoNavigator, and Printing Image Overlays.

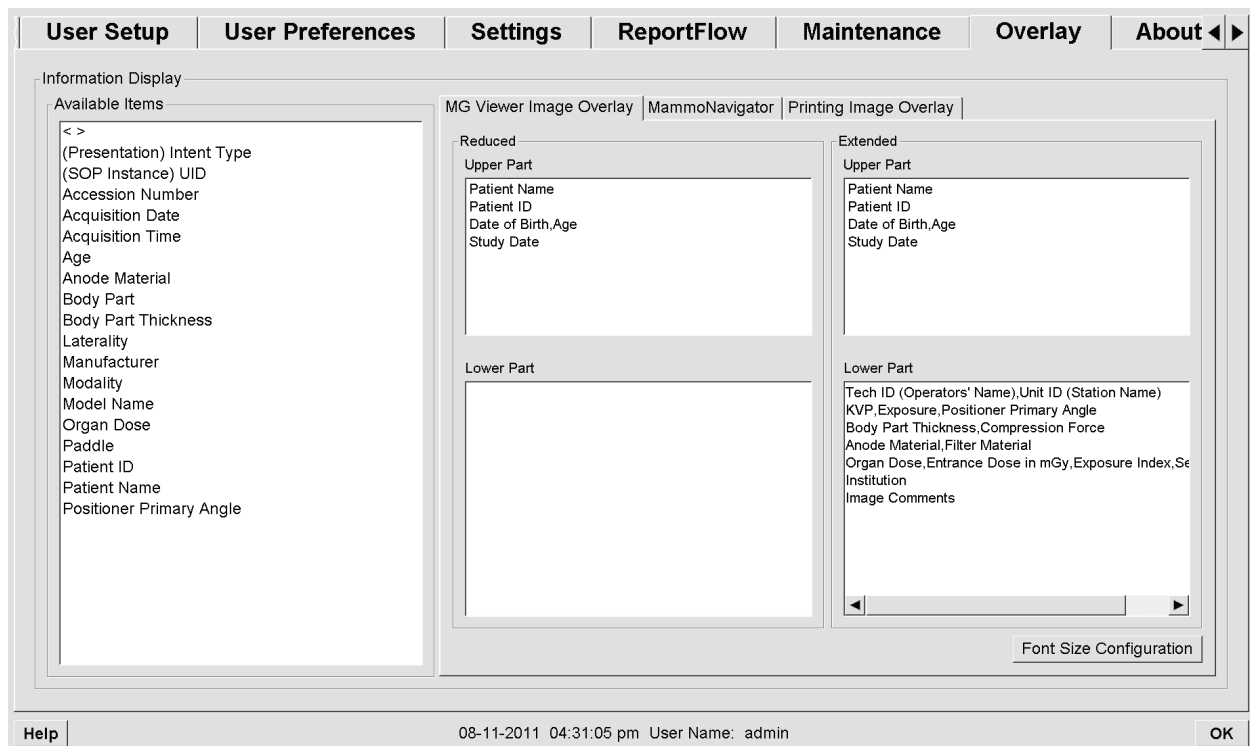
- ▶ **8.7.1. MG Viewer Image Overlay**
- ▶ **8.7.2. MammoNavigator Overlay**
- ▶ **8.7.3. Printing Image Overlay**

### 8.7.1. MG Viewer Image Overlay

The MG Viewer displays images for review by the Radiologist user. Patient, study, and image information appear as overlays that the user can switch on and off. See [4.2.14. Patient Information Overlays](#).

- ▶ **To customize the MG Viewer Image Overlay:**

- 1 In the Overlay window, select the MG Viewer Image Overlay tab.



- 2 Select an item from the Available Items list and drag into the Reduced state's Upper/Lower Part or Extended state's Upper/Lower Part windows.

**Note:** Radiologist users can set their own Image Overlay preferences to display in either Reduced or Extended mode. See [6.3. Tools and Overlays Preferences](#).


- 3 To adjust font size, click **Font Size Configuration**. Select the font sizes for each tiling and click **OK** to save your settings.
- 4 After configuring the overlay, click **OK** to save your settings.

### 8.7.2. MammoNavigator Overlay

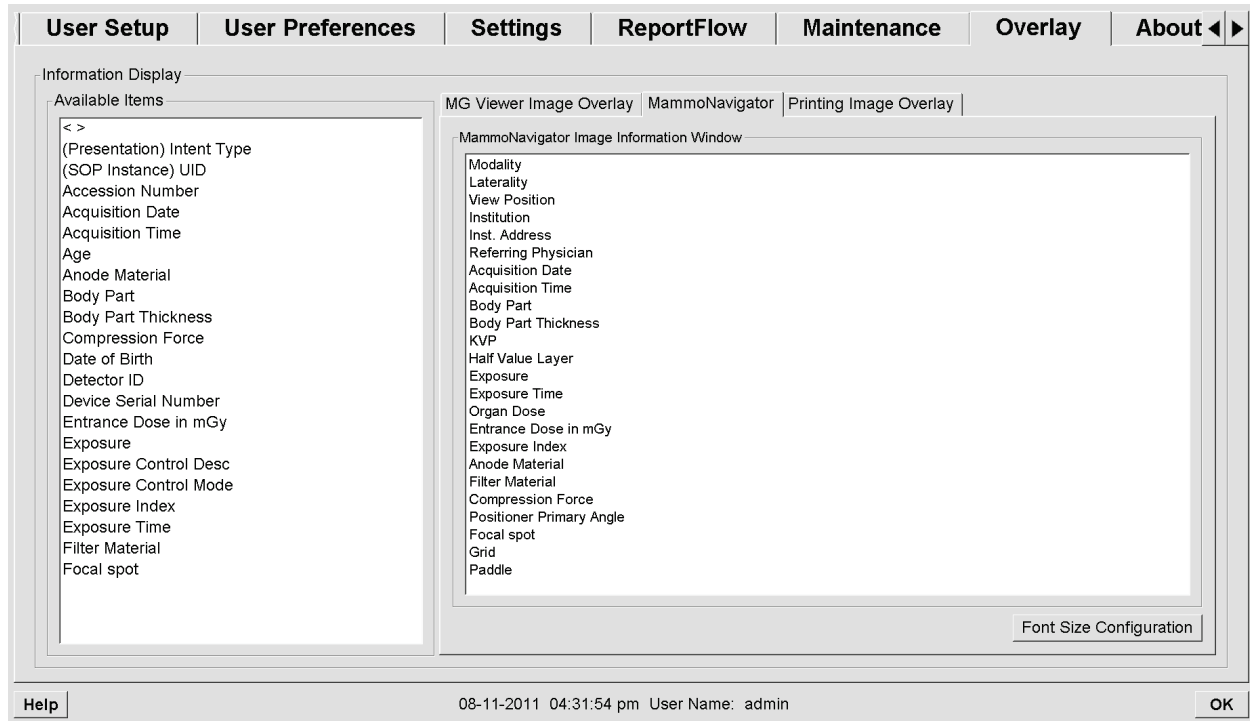Use the MammoNavigator tab to select which data fields appear in the MammoNavigator Image Information window. The Image Information window displays DICOM header data associated with the image. See [4.2.13. Image Information](#).

► **To customize the MammoNavigator overlay:**

- 1 In the Overlay tab, select the MammoNavigator tab.
- 2 Select an item from the Available Items list and drag into the Image Information Window.




- 3 To adjust font size, click **Font Size Configuration**. Select the font size for MammoNavigator Image Information and click **OK** to save your settings.
- 4 After configuring the overlay, click **OK** to save your settings.

### 8.7.3. Printing Image Overlay

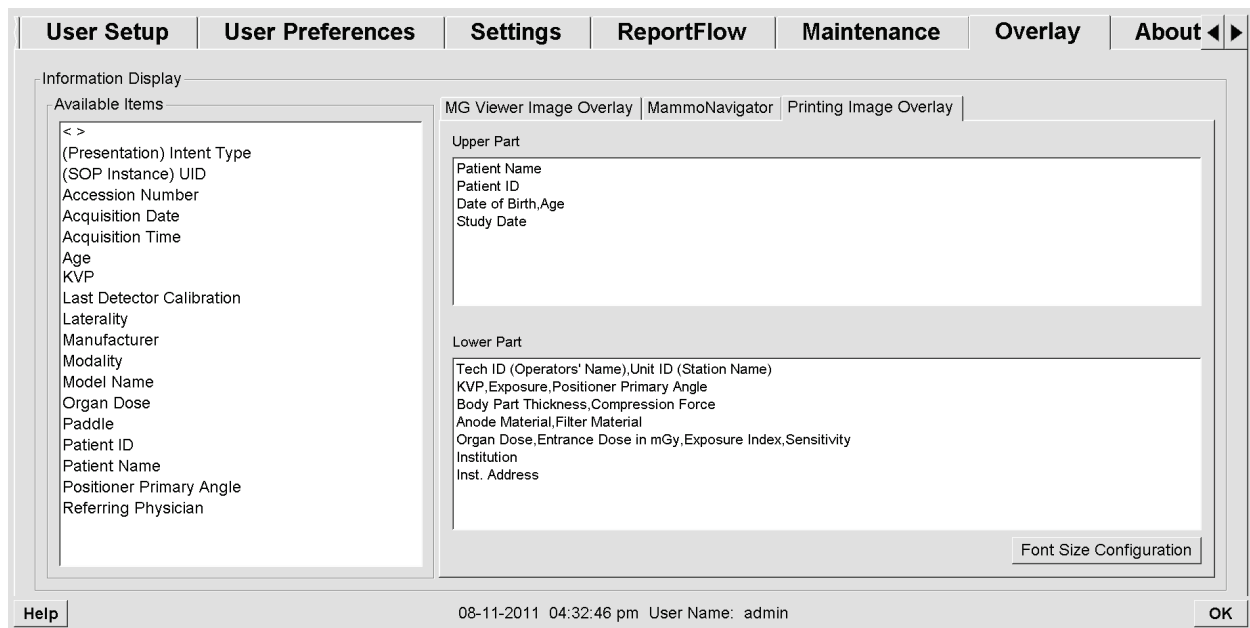You can use the Printing Image Overlay window to select which data fields appear in a printed image. A Radiologist user can print images, along with data (e.g., patient information, Annotations, etc.) on a DICOM film printer. The user selects areas to print (Upper Part, Lower Part or both) using the MG Viewer Print dialog. See [4.8. Printing Options](#).

► **To customize the Printing Image Overlay information:**

- 1 In the Overlay window, select the Printing Image Overlay tab.
- 2 Select an item from the Available Items list and drag into the Upper Part or Lower Part window.



- 3 To adjust font size, click **Font Size Configuration**. Select the font sizes for each tiling and click **OK** to save your settings.
- 4 After configuring the overlay, click **OK** to save your settings.

## 8.8. Maintaining the Database

The Maintenance tab provides options for backing up and restoring the database, and scheduling database maintenance.

- **8.8.1. Backing Up and Restoring the Database**
- **8.8.2. Scheduling Database Maintenance**

### 8.8.1. Backing Up and Restoring the Database

Back up the patient database onto a CD-R/DVD disk. During the backup/restore process, no other functions (such as accepting new images) are available until the process is complete. Schedule convenient time slots for maintenance and make sure that all configured senders have appropriate retry mechanisms.

**⚠ Important:** SecurView backs up the Patient List, ReportFlows, annotations, reading states, and configuration settings available on the user interface. It does not back up image data.

► **To back up the database:**

To avoid loss of data in the event of a system failure, back up the patient database monthly or when known configuration changes have occurred.

**⚠ Important:** In a multiworkstation environment, perform the database backup only on the Manager.

- 1 Select the Maintenance tab to display the following window:



- 2 Click **Backup**. The insert CD prompt appears.
  - 3 Insert disk into CD/DVD tray and click **OK**. When the backup process is complete, 'Backup Completed Successfully' appears.
- ⚠ Note:** SecurView displays a message if the database size exceeds the disk capacity. In such cases, use a DVD.

► **To restore the database:**

- 1 Select the Maintenance tab to display the System Configuration and Patient Database window.
- 2 Click **Restore**. The Insert Database Backup CD prompt appears.
- 3 Insert disk into the CD/ DVD tray and click OK. A dialog appears prompting you to click **OK** to restart the system, or click **Cancel** to stop the restore process.

### 8.8.2. Scheduling Database Maintenance

When the Database Maintenance function is 'Active', SecurView automatically restarts the workstation at the interval specified, and then analyzes and reindexes the database. Enable this function only if directed to do so by a Hologic Technical Support representative.

The **Reset** button changes all settings back to their default values.



---

## Chapter 9: Case Administrator Tasks

- ▶ **9.1. Opening the Administration Module**
- ▶ **9.2. Deleting Patients**

Case Administrators can merge multiple patient records for a single patient and delete patients from the Patient List. This chapter provides an overview of the Administration module for Case Administrators and describes how to delete patients.

## 9.1. Opening the Administration Module

When you log in as a Case Administrator, the Administration module includes the Patient Manager, User Preferences, and About tabs.

► **To open the Administration module:**

- 1 Log into SecurView.
- 2 Click the **Administration** tab to display the Patient List:

**Patient Manager** | **User Preferences** | **About**

**Patient List** | Sessions | Log

Reset Columns | Resend | Notices | Cancel Editing | Create Session | Review | Clear  
 Update Patient List | Merge Patients | Cancel Import | Import... | Suspend And Review | Reload

**Patient List**

Study Date	Name	Patient ID	Date of Birth	Modality	State	Note	Type	CAD	N ▲
05-18-2011	00300670	00300670	01-01-2007	MG+	Read			+	
04-28-2011	01_Multimodality, 001	01_Multimodality_001	04-06-1961	MG, US, MR	Read			+	
04-12-2011	02_Patient, 001_ScrDgnExt_Scr	02_Patient_001	10-29-1962	MG	Not Re Pend.			+	
04-12-2011	21400057	21400057	01-01-1940	MG+	Not Re Add.			+	
04-03-2011	Brown, James H	1336574	12-02-1936	CT, OT	Old				
09-07-2010	01_Multimodality, 006	01_Multimodality_006	01-10-1952	MG, US, MR	Old			+	
09-07-2010	01_Multimodality, 004	01_Multimodality_004	01-10-1928	MG, US, MR	Old				
12-11-2009	22400008	22400008	01-01-1961	MG+	Old			+	
12-10-2009	22400005	22400005	01-01-1951	MG+	Old			+	
12-10-2009	22400013	22400013	01-01-1950	MG+	Old			+	
12-10-2009	22400014	22400014	01-01-1954	MG+	Old			+	

Local Search | Search on PACS | 0 of 91 patient(s) selected

Help | Delete Patients | 05-25-2011 04:25:20 pm User Name: tzhang | OK

The Administration window displays three tabs:

- **Patient Manager** – Displays the Patient List with all patients, their studies, and series currently in the database. The Case Administrator does not have the option to create a new session or synchronize patients.
- **User Preferences** – Displays the Case Administrator's user profile.
- **About** – Displays software information. Refer to this information when contacting the Hologic Help Desk.



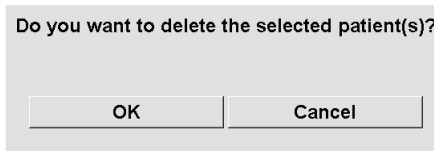
## 9.2. Deleting Patients

Case Administrators can manually delete patients and associated images from the Patient List. In general, you can delete patients with the reading states 'Read', 'Read Once', 'Not Read', and 'Old'. Patients that are part of a Session or in a locked state cannot be deleted.

► **To delete patients:**

- 1 From the Patient List, select one or more patients, then click **Delete Patients**. In response, SecurView:

- Deletes all selected patients with the reading state 'Old'.
- Posts a message for other selected patients such as:



- 2 Click **OK** to confirm or **Cancel**.

SecurView posts alternative confirmation messages for patients in other states. For example, if a patient is protected against Automatic Deletion, SecurView displays:



SecurView posts a similar message for patients that have unsent Annotations (if there is a configured destination).

SecurView does not allow deletion of:

- Patients with the reading state 'Changed' (new images arrived after the study was read)
- Patients that are locked (for example, 'Additional Images Required' – see [4.7. Closing a Study](#))
- Patients currently in use on another workstation connected to the same Manager
- Patients that are part of a Session
- Patients that are being printed

If you delete a merged patient, SecurView deletes the primary patient, secondary patient, and all associated objects.

**Note:** If a patient with non-local studies is deleted, the patient may reappear on the patient list after the next Study List Manager update. If such a patient was a merged patient prior to deletion, it will reappear as an unmerged patient.



---

## Chapter 10: Patient and ReportFlow Files

- ▶ 10.1. Exporting Currently Displayed Image Files
- ▶ 10.2. Exporting DICOM Files
- ▶ 10.3. Importing and Exporting ReportFlows

This chapter provides procedures to support the technologist in managing DICOM and ReportFlow files.

## 10.1. Exporting Currently Displayed Image Files

With this procedure, SecurView exports images that currently appear on the displays to a user-designated folder. By default, SecurView exports the images to F:\Exports. Be aware of the following:

- You can export Tiff images in two resolutions, depending upon the setting in your User Preferences (see [6.3. Tools and Overlays Preferences](#)).
- SecurView creates filenames for each image in the form: [Patient Name\_Examination Date\_View\_SOP Instance UID.ext], where 'ext' can be 'tif' or 'dcm', depending upon the file type.
- If a DICOM Mammography CAD Structured Report is available for any displayed image(s), SecurView exports it with the filename [Patient Name-CAD\_Examination Date\_SOP Instance UID.sr]. You cannot export CAD SR files in .tif format.

For tomosynthesis images:


- SecurView exports all tagged slices. If no slices are tagged, SecurView exports only the currently displayed image (conventional mammography or C-View 2D image, projection image, or reconstructed slice), not the entire stack.
- SecurView identifies the exported slice by appending the slice number to the end of the filename (e.g., '\_42' for slice 42).
- If you set the slab thickness to be greater than 1, then for Tiff images only, SecurView exports the slab view, not just the middle slice and appends the number of slices to the file name (e.g., '\_42(7)' for a view of seven slices).

► **To export currently displayed image files to a workstation folder:**


- 1 Display the images on the SecurView DX workstation. If you are exporting tomosynthesis images, tag them as explained in [5.4. Tagging Tomosynthesis Reconstructed Slices](#).

- 2 On the keyboard, press [E] (for **Export**).

- 3 When the dialog opens, select either **DICOM** or **Tiff** format.

 **Important:** Do not use exported Tiff files for diagnostic use. Use DICOM format instead.

- 4 Click **Export** and browse to the drive and folder where you wish to store the images. When you click **OK**, SecurView exports the images currently appearing on both displays to the designated folder. When all files have been exported, the dialog closes automatically.

 **Important:** Do not click **Cancel** until all files have been exported. If you click **Cancel** too soon, SecurView may export an incomplete data set.

## 10.2. Exporting DICOM Files


Use this procedure to transfer DICOM Part 10 files for one or more patients from SecurView to external media (USB drive or disk). If there is a Manager, it is best to use it when possible.

 **Note:** To import images, see [3.4. Importing DICOM Images](#).

- 1 Log into SecurView and click the **Administration** tab.
- 2 On the Patient List, highlight the patient or patients you want to export.
- 3 Right-click the highlighted patient (or patient group) and click **Export to media**. Then either:
  - Insert disk into CD/DVD tray and close tray, or
  - Click **Browse....** Under My Computer, find and click the USB or DVD drive. Highlight the destination folder and click **OK**.
- 4 Click **Export** to begin exporting all files for the selected patient(s), which can take several minutes (the DICOM files are large). At the selected destination, SecurView creates a FILES folder and copies the DICOM files to the folder. When done, SecurView displays 'Export was successful'.
- 5 If you exported the files to a disk, the CD/DVD tray opens automatically. Remove the disk and label it. Close the tray.
- 6 Click **Close**, and then click **OK**.

## 10.3. Importing and Exporting ReportFlows

If there is a Manager, it is best to use it when possible.

 **Note:** To import, the ReportFlows must have been created for SecurView 6-x or later. You cannot import ReportFlows that were created for SecurView 5-x or earlier.

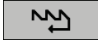


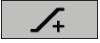

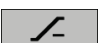
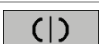

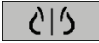







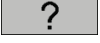

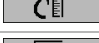



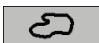













### 10.3.1. Importing ReportFlows from a USB drive to SecurView

- 1 Insert USB drive into a USB port on SecurView.
- 2 Log into SecurView using 'admin'.
- 3 Click the **Administration** tab, then the **Maintenance** tab.
- 4 Under Hanging Snapshots and ReportFlows, click **Import**. The Open dialog appears.
- 5 In the **Look in** drop-down list, find and click the **G:** drive. (If there is no G drive, look for the removable drive.)
- 6 Find, then click the ReportFlow or ReportFlow group you want to import.
- 7 Click **Open**. The Import of ReportFlows dialog appears.
- 8 Highlight the ReportFlow(s) you want to import.
- 9 Click **Import**. The Import of ReportFlows dialog closes and SecurView writes the ReportFlows to its database.

### 10.3.2. Exporting ReportFlows from SecurView to a USB Drive

- 1 Insert USB drive into a USB port on SecurView.
- 2 Log into SecurView using 'admin'.
- 3 Click the **Administration** tab, then click the **Maintenance** tab.
- 4 Under Hanging Snapshots and ReportFlows, click **Export**. The Export of ReportFlows dialog opens.
- 5 Highlight the ReportFlow(s) to export, then click **Export** to open the Save as window.
- 6 In the **Look in** drop-down list, click the **G:** drive. (If there is no G drive, look for the removable drive.)
- 7 If desired, rename the ReportFlow or the group of ReportFlows that you are exporting to your USB drive at the bottom of the window where it says File name. Do not use '/' as part of your ReportFlow name.
- 8 Click **Save**. The Save As window closes and SecurView writes the ReportFlows to the USB drive, which takes just a few seconds.
- 9 Remove the USB drive.
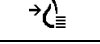
## Appendix A: Keyboard Shortcuts

Tool	Key	Function	Tool	Key	Function
	[1] or [End]	FlowBack		[F9]	Window / Level
	[4] or [Left Arrow]	FlowNext		[F3] or [N]	Increase Gamma
	[0] or [Ins]	Overview		[F1] or [M]	Decrease Gamma
	[-]	CC Current		[F2]	Reset Window/Level
	[+]	MLO Current		[F8]	Reset
	[/]	CAD		[Delete]	Previous Patient
	[7], [D], [Home] or [Up Arrow]	View Actual Pixels <b>Left</b>		[Enter]	Next Patient
	[9], [8], [F] or [Page Up]	View Actual Pixels <b>Right</b>		[Q]	Exit
	[Y]	Same Size		[?]	Help
		Right Size	—	[Backspace]	Delete Selected Marking
	[X]	True Size	—	[Z]	Undo Delete Marking
	[F5]	Fit to Viewport	—	[I]	Invert All Images
	[R]	Synchronize	—	[E]	Export Images
	[F10]	Arrow	—	[*]	Display/Hide Toolbar
	[F11]	Ellipse		[J]	Toggle MG / Tomo
	[F12]	Freehand		[F6]	Cine Start/Stop
	[2] or [Down Arrow]	Measurement	—	[Spacebar]	Tag Tomo Images
	[A]	Magnifier		[W]	Next 3D CAD Mark
	[6] or [Right Arrow]	MammoNavigator		[S]	Previous 3D CAD Mark
	[3] or [Page Down]	Patient Information	—	[B]	3D CAD Boundaries
	[G]	Patient List	—	[C]	3D CAD Clusters
	[5]	Close Study	—	[F4]	Display Mouse Cursor Position (if configured)
	[F7]	Continuous Zoom	—	[H]	Display full DICOM header content for an image
		Reset Continuous Zoom		[V]	Focus
—	[CTRL] and [t]	Toggle current/prior color indication			





---

## Appendix B: Technologist's Guide

This appendix provides a quick reference for procedures commonly used by radiologic technologists.

For this procedure...	See section...
To add a new user to SecurView	<b>8.2. Managing User Profiles</b>
To create and manage Sessions on SecurView DX	<b>3.3. Creating Sessions</b>
To send and view Notices	<b>4.6. Sending and Viewing Notices</b>
To close a study on SecurView RT	<b>4.7.2. Closing a Study as a Technologist</b>
To create and manage Hanging Snapshots	<b>7.3. Creating and Modifying Hanging Snapshots</b>
To create and manage ReportFlows	<b>7.4. ReportFlows</b> <b>7.5. Linking a ReportFlow to a Procedure</b> <b>7.6. Creating New ReportFlows</b> <b>7.7. ReportFlows Preferences</b>
To print standard mammography images	<b>4.8. Printing Options</b>
To print tomosynthesis images	<b>5.5. Printing Tomosynthesis Reconstructed Slices</b>
To export images to external media	<b>10.1. Exporting Currently Displayed Image Files</b> <b>10.2. Exporting DICOM Files</b>
To import images into SecurView	<b>3.4. Importing DICOM Images</b>
To import or export ReportFlows	<b>10.3. Importing and Exporting ReportFlows</b>



---

# Index

## A

- accession number, 36
- Active Directory, 136
  - setting up, 143
- Administration module, opening, 135
- Administrator tasks, 134–59
- Advanced Image Enhancement (AIE), 69
- Annotation User Filter, 53, 85
- Annotations, 82–86
  - configuring for double reading, 150
  - descriptions, 84
  - flow of, 20, 22, 26
  - setting close of study default for, 111
  - setting default format for, 113
  - storing, 89–91
  - third-party GSPS, 86
  - viewing, 85
- antivirus products, 14
- application event logging, 146–47
- Application Synchronization, 19
- Arrow tool, 53, 82, 84
- auto-completion, 142
- auto-deletion
  - configuring, 141
  - protecting against, 36, 38
- auto-fetching, 36, 38, 142

## B

- BACS calcium scoring, 79
- barcode reader, 46
- birth date, patient, 35
- brightness, image, 71

## C

- CAD, 75–81, 106–7
  - BACS, 79
  - displaying, 75, 76
  - Hologic Imaging Biomarkers, 79
  - ImageChecker 3D Calc CAD, 106–7
  - ImageChecker CAD, 76–78
  - on Patient List, 35
  - Quantra, 79
  - setting preferences for, 113–16
- CAD SRs
  - switching between, 81
  - where displayed, 27
- Calc mark, CAD, 76
- Case Administrator tasks, 160–62
- Cine mode, 101

- user preference for scrolling linked tiles, 113–16
- CLAHE Image Enhancement, 75
- Cleanup button, 142
- Close Study dialog, 89–91
  - ReportFlow icon, 132
- consecutive hanging ReportFlow icon, 132
- Continuous Zoom, 70
- contrast, image, 71
- C-View, 97, 98, 99
- cybersecurity, 14

## D

- database maintenance, 159
- database, backing up/restoring, 157
- date, configuring format for, 149
- Decrease Gamma tool, 71
- DICOM
  - conformance statements, 12
  - exporting files, 165, 166
  - importing files, 43
- DICOM 6000 overlays, 74
- DICOM 6000 Overlays, 74
- double reading, configuring, 150

## E

- Ellipse tool, 53, 82, 84
- EmphaSize CAD marks, 77
- Export Movie tool, 53
- exporting
  - image files, 38, 165, 166
  - ReportFlows, 167

## F

- Fit to Viewport, 59
- flipping images, 54
- Focus, 98, 103
- Freehand tool, 53, 82, 84

## G

- GSPS Notice. *See* Notices
- GSPS Reports, 34, 37, 38, 90, 150

## H

- Hanging Snapshots
  - adding to ReportFlow, 131
  - changing an icon, 127
  - copying and editing, 125
  - creating new, 123–27

- renaming, 126
- hard disk, monitoring, 140
- help to display user guides, 50
- Hologic Imaging Biomarkers, 79–80

**I**

- Image Notice, 88
- Image Tools menu, 53
- ImageChecker 3D Calc CAD, 106–7
- ImageChecker CAD, 76–78
- images
  - DICOM information for, 63
  - displaying, 50–65
  - displaying prior, 61
  - exporting, 38, 165, 166
  - flipping, 54
  - flow of, 20, 22, 25
  - importing, 43
  - inverting, 53, 68
  - magnifying, 52, 68–69
  - manual scrolling through
    - tomosynthesis, 99
  - marking, 82
  - MPE, 72–73
  - navigating tomosynthesis, 98
  - panning, 55
  - predefined hangings, 56
  - rotating, 53
  - scaling, 59
  - setting predefined hangings, 112
  - visualizing, 65–75
- importing
  - image files, 43
  - ReportFlows, 167
- Increase Gamma tool, 71
- institution name & address, 151
- Intelligent Roaming, 57
  - setting default behavior for, 114
- Invert Image tool, 53
- Inverted Magnifier tool, 53, 68

**K**

- keypad, 51

**L**

- LesionMetrics, 78
- Link Tile tool, 53
  - using, 104
- linking tiles, 113–16
- Local Cine mode, 102
- locking a study, 90
  - about, 55
  - indication on Patient List, 36
  - unlocking, 38

- logging in and out, 30
  - setting Auto Log-off, 117

**M**

- Magnifier tool, 52, 68–69, 69
  - scaling lines, 113–16
- maintaining the database, 157
- Malc mark, CAD, 76
- Mammography Prior Enhancement. *See* MPE images
- MammoNavigator, 61
  - ReportFlow icons, 132
- Manager-Client workstations, 21–22, 25–26
- marking an image, 82
- Mass mark, CAD, 76
- Measurement tool, 53, 82
- merging patients, 34, 39
  - undoing, 38
- MG Secondary Capture images, 34, 37, 38, 99
  - about, 65
  - configuring format for, 151
  - configuring local institution for, 151
  - configuring source of institution for, 151
  - storing, 90
- MG Viewer, 48–49
- missed view safety warning, 112
- MM ScreenCapture images, 34, 37, 38
  - about, 65
  - configuring local institution for, 151
  - configuring source of institution for, 151
  - setting close of study default for, 111
- mouse button, setting default tool, 113
- mouse wheel
  - setting scrolling default for, 113
  - with Stack and Prior indicators, 61
- MPE images, 72–73
- Multimodality option, 19
- Multiworkstations, 21–22, 25–26

**N**

- Notices, 34, 37, 38, 87–88, 151, *See also* Image Notice
  - sending all, 87–88, 90
  - sending for a single image, 88
  - viewed by Technologist user, 90

**O**

- overlays, 64
  - customizing, 154–59
  - customizing for display, 154

- customizing for printed image, 156
- customizing MammoNavigator, 155
- overview, SecurView, 10

## P

### PACS

- searching on, 40
- study date format when searching, 147

- panning an image, 55

- Patient Information overlays, 64

- configuring, 113

- Patient List, 33–41

- columns, 35

- Patient Manager, 32

- patients

- birth date, 35
  - deleting, 162
  - ID field, 35
  - merging, 34, 39
  - name field, 35
  - navigating, 50
  - protecting against auto-deletion, 36, 38
  - reviewing, 45–95
  - searching for, 34, 40
  - selecting, 33
  - synchronizing, 38
  - taking over, 38
  - undoing merged, 38
  - unlocking, 38

- PeerView CAD marks, 78

- Pie menu, 52–54

- Pixel Meter, 60

- setting zoom factor, 113–16

- precautions, 13–14

- printing

- customizing overlays, 156
  - MG images, 92
  - tagging tomosynthesis images for, 108
  - tomosynthesis images, 108

- Prior indicator, 61

- user preference for, 113–16

- procedure names, configuring, 152

- product complaints, 15

## Q

- quality control, displays, 12, 14

- Quantra breast density assessment, 79

## R

- reading states, 37

- changing on close of study, 89–91

- during patient review, 54

- ReportFlows, 128–29

- creating new, 131–32

- importing and exporting, 167

- linking to a procedure, 130

- preferences for, 133

- synchronizing with, 94

- using, 54

- viewing, 121

- Resend button, 34, 37

- Right Size, 59

- RightOn CAD marks, 76

- rotating images, 53

- ruler, 83

- user preference for, 113

## S

- Same Size, 59

- scaling lines, Magnifier, 113–16

- scaling modes, 59

- setting default, 112

- screen capture. *See* MM ScreenCapture

- scrolling tomosynthesis images

- manual, 99

- through linked tiles, 104

- with Cine mode, 101

- searching, 34

- close dialog automatically, 111

- for patients, 40

- second read studies, 47, 90

- secondary capture. *See* MG Secondary Capture

- SecurView DX, 10

- Client, 21

- description of, 19

- Manager, 21

- SecurView RT, 10

- description of, 23

- Send All Notices tool, 53, 87

- Send Image Notice tool, 53, 88

- Sessions, 32

- creating, 34, 42

- enabling, 140

- selecting, 47

- shortcut menu, 38

- shutdown, system, 29

- slider tool, tomosynthesis, 99–100

- use in tagging images for print, 108

- with 3D CAD, 106

- Stack indicator, 61

- user preference for, 113–16

- Standalone workstations, 20, 24

- startup, system, 29

- studies

- closing, 89–91
- configuring for double reading, 150
- deleting, 141
- displaying, 46–49
- locking, 90
- second read, 47, 90
- time limit for current, 152
- suspend and review, 34, 38
- synchronizing with external
  - application, 38, 94
  - configuring, 117
  - ReportFlow icon, 132
- system maintenance, 164–67
  - database, back up/restore, 157
- system-level settings, 139–51
  - auto-completion, 142
  - auto-fetching, 142
  - automatically generated worklists, 149
  - date/time format and units, 149
  - deleting studies, 141
  - double reading for studies, 150
  - enabling Sessions, 140
  - institution name and address, 151
  - local institution, 151
  - MG Secondary Capture format, 151
  - monitoring hard disk space, 140
  - PACS study date input, 147
  - sending Annotations, 150
  - time limit for current studies, 152

## T

- Tag Images for Print tool, 53
- technologist quick reference, 170
- tiff images
  - exporting, 165
  - setting resolution of, 113–16
- time, configuring format for, 149
- tomosynthesis imaging, 96–109
  - Cine mode, 101
  - exporting a movie, 105
  - navigating, 98
  - overview of, 97
  - printing, 108
  - scrolling linked tiles, 104
  - scrolling manually, 99
  - slab thickness, 100
  - slider tool, 99, 106
  - tagging images for printing, 108
- toolbar
  - hiding, 114
  - image visualization buttons, 65–75
  - patient navigation buttons, 50
  - predefined hanging buttons, 56

- training programs, 12
- True Size, 59

## U

- unlocking a patient, 38
- user accounts, 30, 136–37
- user groups, 28
- user preferences, 110–18
  - CAD display, 113–16
  - configuring overview hanging, 133
  - configuring synchronization, 117
  - displaying, 110
  - format for Annotations, 113
  - hiding the toolbar, 114
  - image overlays, 113
  - magnification scaling lines, 113–16
  - mouse button default, 113
  - patient search default focus (PACS/Local), 112
  - Pixel Meter, 113–16
  - predefined hangings, 112
  - Prior indicator, 113–16
  - ReportFlows, 133
  - roaming indicator, 114
  - scaling mode default, 112
  - sending Annotations, 111
  - Stack indicator, 113–16
  - tiff image resolution, 113–16
  - tile linking properties, 113–16
  - user profile, 117
  - worklist sorting, 111
- user profiles, 136–37

## V

- View Actual Pixels tool, 53, 59
- viewing Notices, 88
- VOI LUTs, applying, 72
- V-Split Cursor, 100

## W

- warnings, 13–14
- Window/Level tool, 53, 71
  - numeric, 53
- worklists, 30
  - automatic, 47
  - configuring auto-generated, 149
  - default sorting, 111
- workstations
  - description of, 17–30
  - Manager-Client, 21–22, 25–26
  - Manager-Client functional division, 27
  - overview, 18
  - SecurView DX, 19

SecurView RT, 23  
Standalone, 20, 24

**Z**

zoom factor, Pixel Meter, 113–16







**At Hologic, we turn passion into action, and action into change.**

Hologic is defining the standard of care in women's health. Our technologies help doctors see better, know sooner, reach further and touch more lives.

BREAST IMAGING SOLUTIONS • INTERVENTIONAL BREAST SOLUTIONS • BONE HEALTH  
PRENATAL HEALTH • GYNECOLOGIC HEALTH • MOLECULAR DIAGNOSTICS

**HOLOGIC®**

[www.hologic.com](http://www.hologic.com) | [info@hologic.com](mailto:info@hologic.com) | +1.781.999.7300

**North America / Latin America**

35 Crosby Drive  
Bedford, MA 01730-1401  
USA



**Europe**

Everest (Cross Point)  
Leuvensesteenweg 250A  
1800 Vilvoorde  
Belgium



**Asia Pacific**

7th Floor, Biotech Centre 2  
No. 11 Science Park West Avenue  
Hong Kong Science Park  
Shatin, New Territories  
Hong Kong

**Australia / New Zealand**

Suite 402, Level 4  
2 Lyon Park Road  
Macquarie Park NSW 2113  
Australia