S e t u p G u i d e

eviva®

Stereotactic Guided Breast Biopsy System

**Siemens MAMMOMAT 3000
Nova with Opdima® and Siemens
MAMMOMAT Novation DR with
Opdima®**

CE 2797

Hologic BVBA
Da Vincilaan 5
1930 Zaventem, Belgium
Tel: +32 2 711 46 80

PLEASE READ PRIOR TO PERFORMING PROCEDURE:

This setup guide is designed to be used in conjunction with, not replace, the Eviva® breast biopsy device instructions for use and/or the Siemens MAMMOMAT 3000 Nova with Opdima® operator manual and/or the Siemens MAMMOMAT Novation DR with Opdima® operator manual. Prior to performing the procedure, the physician must review and be familiar with the full operating instructions for the Eviva breast biopsy device and Siemens system(s). Consult the Eviva breast biopsy device instructions for use and the Siemens system(s) operator manual for full operational instructions, as well as any contraindications and safety information. Verification testing was conducted to ensure accuracy of the Eviva biopsy device on the Siemens system(s). However, the setup numbers provided herein are for guidance purposes only. The setup numbers must be verified onsite with the corresponding Eviva biopsy device prior to performing any procedure.

Please retain this setup guide. Note that while Hologic frequently updates the contents of this guide, medical information and technology can change rapidly. Therefore some of the information in this guide may be out of date. Use of these instructions with any product other than those specifically noted may result in injury to the patient and/or damage to the equipment.

Setup of Siemens MAMMOMAT 3000 Nova with Opdima® and Siemens MAMMOMAT Novation DR with Opdima® for Eviva® Biopsy Device


Program the Siemens system(s) for Eviva biopsy device according to the needle setup instructions provided in the Supplement to the Instructions for Use Mammomat 3000 Opdima manual, in conjunction with steps below.

- 1. Press the Needle Setup button in the service dialog
- 2. Press Add
- 3. Select needle type “Core” from the option menu
- 4. Enter needle length (L1) and aperture length (L2) in the text fields and press OK

Device	Eviva 0910-20 Eviva 1210-20	Eviva 0910-12	Eviva 0910-12T
Recommended Name in Software	EvivaXX1020v	EvivaXX1012	Eviva XX1012T
L1 (mm)	134	130	134
L2 (mm)	18	10	14

Performing a Stereotactic Biopsy Using the Eviva Biopsy Device on the Siemens MAMMOMAT 3000 Nova with Opdima and Siemens MAMMOMAT Novation DR with Opdima

1. Attach the Locating Bar to the Eviva Adapter
2. Install the Eviva Adapter with Locating Bar onto the Siemens system(s).

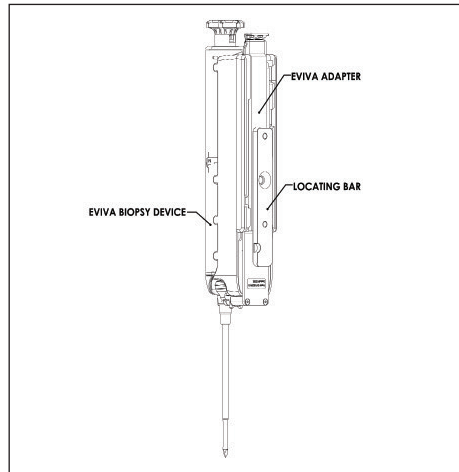3. Install the appropriate disposable needle guide, Eviva_NG09R or Eviva_NG12R, onto the Siemens system(s).
4. Ensure the proper Eviva biopsy device is selected in the Siemens system(s) software prior to sending the target to the stereotactic system.

NOTE: The selected Eviva biopsy device should match the biopsy device that is being used in the procedure.

5. Perform the biopsy procedure per the Eviva breast biopsy device instructions for use and per the Supplement to the Instructions for Use Mammomat 3000 Opdima Manual.
6. Perform the biopsy procedure per the Eviva breast biopsy device instructions for use.
7. Clean the *Eviva Adapter* per the cleaning instructions found on page 4 of this document.

Cleaning Instructions for *Eviva Adapter*

1. Upon completion of the biopsy procedure, immediately rinse the *Eviva Adapter* with warm water.
2. Spray all components with a cleaner

The following cleaning products are NOT RECOMMENDED as they may damage the exposed components:

- Bleach based cleaning agents
 - Hydrogen Peroxide
 - Any cleaners/disinfectants with a pH of less than 4 or greater than 10
3. Allow the necessary time for the chosen cleaner to disinfect the components.
 4. Rinse with warm water.
 5. Wipe dry with cloth or paper towel.

© 2014-2020 Hologic, Inc. All Rights Reserved. Hologic and Eviva are trademarks and/or registered trademarks of Hologic, Inc. and/or its subsidiaries in the United States and/or other countries. All other trademarks, registered trademarks and product names used with this document are the property of their respective owners.

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