

HOLOGIC[®]

Marker Wire Kit

CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.

Description	Catalogue Number
Hologic Marker Wire Kit	MLMW08

Product Description

The Hologic Marker Wire Kit is intended to aid in the identification of structures in radiographic images. The kit consists of two sets of four individual marker wires. One set is comprised of four differently-sized Nitinol wires ranging in increasing diameter from 0.019 inches to 0.031 inches. The second is a set of four distinctive radiopaque (RO) inked marker wires with unique ink patterns.

The Hologic Marker Wires are illustrated in Figure 1.

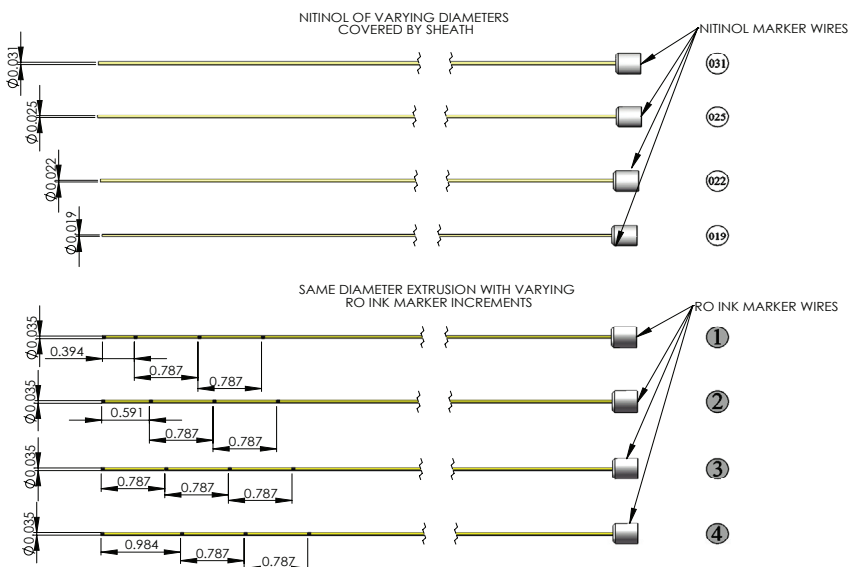


Figure 1. Illustration of HOLOGIC Marker Wires
AW-04103-001 Release Date: 2/12/2020

Intended Use/Indications

The Hologic Marker Wire Kit is intended to aid in the identification of structures in radiographic images.

Warnings

- This product should be carefully inspected by authorized medical personnel prior to each clinical use.
- Do not use a Hologic Marker Wire if any kinks or damage to the inked markings are observed.
- Do not use excessive force to insert a Hologic Marker Wire into a brachytherapy applicator.
- The Hologic Marker wire kit is provided non-sterile and is not intended to be sterilized by any means. Do not sterilize.

How Supplied

- The Hologic Marker Wire Kit, model MLMW08, contains two sets of four individual marker wires. One set is comprised of four differently-sized Nitinol wires ranging in increasing diameter from 0.019 inches to 0.031 inches. The second set is a set of four distinctive radiopaque (RO) inked marker wires with unique ink patterns.
- All Hologic Marker Wire Kits are supplied in non-sterile packaging. Each package should be examined carefully prior to opening.

Hologic Marker Wire Use

The Hologic Marker Wire may be inserted into any lumen sized to accept the diameter of the wire. Also, the marker wires can be used to provide markings for any radiographic identification needs.

If desired, the Hologic Marker Wires may be wiped with 70% isopropyl alcohol.

For More Information

For technical support or reorder information in the United States, please contact:









Hologic, Inc.
250 Campus Drive
Marlborough, MA 01752 USA
Phone: 1-877-371-4372
www.hologic.com

Warranty

Except as otherwise expressly stated in the Agreement: i) Equipment manufactured by Hologic is warranted to the original Customer to perform substantially in accordance with published product specifications for one (1) year starting from the date of shipment, or if Installation is required, from the date of Installation ("Warranty Period"); ii) digital imaging mammography x-ray tubes are warranted for twenty-four (24) months, during which the x-ray tubes are fully warranted for the first twelve (12) months and are warranted on a straight-line prorated basis during months 13-24; iii) replacement parts and remanufactured items are warranted for the remainder of the Warranty Period or ninety (90) days from shipment, whichever is longer; iv) consumable Supplies are warranted to conform to published specifications for a period ending on the expiration date shown on their respective packages; v) licensed Software is warranted to operate in accordance with published specifications; vi) Services are warranted to be supplied in a workman-like manner; vii) non-Hologic Manufactured Equipment is warranted through its manufacturer and such manufacturer's warranties shall extend to Hologic's customers, to the extent permitted by the manufacturer of such non-Hologic Manufactured Equipment. Hologic does not warrant that use of Products will be uninterrupted or error-free, or that Products will operate with non-Hologic authorized third-party products. These warranties do not apply to any item that is: (a) repaired, moved, or altered other than by Hologic authorized service personnel; (b) subjected to physical (including thermal or electrical) abuse, stress, or misuse; (c) stored, maintained, or operated in any manner inconsistent with applicable Hologic specifications or instructions, including Customer's refusal to allow Hologic recommended Software upgrades; or (d) designated as supplied subject to a non-Hologic warranty or on a pre-release or "as-is" basis.

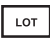Symbols Key

The following symbols may appear on the label or carton:

Symbol	Definition
	Batch Code
	Catalogue number
	Caution, consult accompanying documents
	Do not use if package is damaged.
	Manufacturer
	Non-sterile



Hologic, Inc.
250 Campus Drive
Marlborough, MA 01752 USA
Phone 1-877-371-4372
www.hologic.com

Hologic and associated logos are
trademarks and/or registered trademarks
of Hologic, Inc. and/or its subsidiaries in
the United States and/or other countries.

© 2009-2019 Hologic, Inc.

HOLOGIC®

AW-04103-001 Release Date: 2/12/2020

MAN-01570-001 Rev. 004