




# EU IBS PRODUCT RETURN KIT, INSTRUCTIONS FOR USE

This Return Kit Contains:

- 1 Clear Re-Sealable Ziploc Bag
- 1 Biohazard Bag
- 1 Sheet of Bubble wrap
- 1 Product Return Kit
- 1 Product Return Shipper

Package the opened and/or used device for return:

1. Empty fluid from the device and decontaminate the device as best as possible without further damaging the device.
2. Follow next steps for inserting the device into the Product Return Kit:

Open Product Return Kit	
Position foam target	
Insert sharp into center of foam target	

<p>Lay device and foam target flat, then pack tubing</p>	
<p>Close divider and close tray</p>	

3. Fill any empty space with bubble wrap.
4. Close Product Return Kit, assuring that the tabs of the kit are tucked into the proper location.
5. Insert the Product Return Kit into Clear Re-Sealable Ziploc Bag and seal shut.
6. Insert the Product Return Kit and Clear Re-Sealable Ziploc Bag into the Biohazard Bag and tie shut.
7. Place the Product Return Kit, Clear Re-Sealable Ziploc Bag, and Biohazard Bag into the Product Return Shipper as shown below:



8. Tape the Product Return Shipper closed.

9. Place return label -supplied by Hologic Technical Support- onto the Product Return Shipper in the proper location:



 **Hologic, Inc. 250 Campus Drive  
Marlborough, MA 01752 USA**

10. Complete the required fields of RMA number and Case number on the Product Return Shipper (note: RMA and Case numbers will be supplied from Technical Support once a complaint is filed):

## **Product Return**

**For assistance, please contact Hologic  
(800) 442-9892 or (508) 263-2900**

**RMA # \_\_\_\_\_**

**CASE # \_\_\_\_\_**

**!! DO NOT DISCARD !!**

**Contents**

**Used Medical Device**

*If you have any questions about this procedure, please contact Hologic:*

Hologic, Inc.  
250 Campus Drive, Marlborough, MA, 01752 USA  
(508) 263-2900  
[www.hologic.com](http://www.hologic.com)