# **Brevera**®

**Breast Biopsy System** 



User Guide
MAN-07980-002 Revision 001





## **User Guide**

Models BREV100, BREV200 Part Number MAN-07980-002 Revision 001 March 2023



## **Product Support**

USA: +1.877.371.4372

Europe: +32 2 711 4690

Asia: +852 37487700

Australia: +1 800 264 073

All Other: +1 781 999 7750

Email: BreastHealth.Support@hologic.com

© 2023 Hologic, Inc. Printed in the USA. This manual was originally written in English.

Hologic, Brevera, CorLumina, Affirm, ATEC, Eviva, MultiCare, and associated logos are trademarks and/or registered trademarks of Hologic, Inc., and/or its subsidiaries in the United States and/or other countries. All other trademarks, registered trademarks, and product names are the property of their respective owners.

This product may be protected by one or more U.S. or foreign patents as identified at www.Hologic.com/patent-information.

## **Table of Contents**

List	of Figures	X1
List	of Tables	xiii
1: Ir	ntroduction	1
1.1	Preface	1
1.2	Indications for Use	1
1.3	Contraindications	1
1.4	Patient Target Group	2
1.5	Clinical Benefits	2
1.6	Potential Adverse Effects	2
1.7	Essential Performance	2
1.8	System Capabilities	
1.9	User Profiles	
	1.9.1 Mammography Technologist	4
	1.9.2 Radiologists, Surgeons	
	1.9.3 Medical Physicist	
1.10	Where to Find the Installation Instructions	
1.11	Where to Find Technical Description Information	
1.12	Warranty Statement	
1.13	Technical Support	5
1.14	Product Complaints	
1.15	Hologic Cybersecurity Statement	
1.16	Antivirus Information	
1.17	Where to Obtain Copies of the Manuals	
1.18	Symbols	
1.19	Descriptions of Warnings, Cautions, and Notes	11
2: G	General Information	13
2.1	System Overview	
2.2	Safety Information	14
2.3	Warnings and Precautions	14
2.4	Interlocks	20
2.5	Compliance	20
	2.5.1 Compliance Requirements	20
	2.5.2 Compliance Statements	21
26	System Label Location	22

3: C	ompo	nents, Controls, and Indicators	23
3.1	Syster	m Components	23
	3.1.1	Imaging Cabinet Components	27
	3.1.2	Suction Canister Components	28
	3.1.3	Biopsy Device Components	29
3.2	Syster	m Connections	33
	3.2.1	Power and Network Connections	34
	3.2.2	Footswitch Connection	35
	3.2.3	Device Driver and Remote Control Connections	35
	3.2.4	Suction Canister Connections	36
	3.2.5	Biopsy Needle Connections	36
	3.2.6	Combination Device Information	39
3.3	Contr	ols and Indicators	39
	3.3.1	Imaging Panel Controls and Indicators	39
	3.3.2	Mode Controls and Biopsy Device Arm and Fire Controls	40
3.4	Lock a	and Unlock a Caster	42
3.5	Move	the Console	42
3.6	How t	to Start the System	42
3.7	How t	to Shut Down the System	44
	3.7.1	How to Remove All Power from the System	
4: U	ser In	terface - Imaging Display	45
4.1	About	t the Taskbar	45
4.2	Select	Patient Screen	47
	4.2.1	Open a Patient	48
	4.2.2	Add a Patient	48
	4.2.3	Edit the Patient Information	49
	4.2.4	Delete a Patient	50
	4.2.5	Filters for Patients	50
	4.2.6	Refresh the Worklist	52
	4.2.7	Query the Worklist	52
4.3	Procee	dure Screen	52
	4.3.1	Select a Procedure	54
	4.3.2	Add a Procedure	54
	4.3.3	Access the Review Mode Screen	54
	4.3.4	Access the Image Enhancement Tools	54
	4.3.5	Access the Filter Info	
	4.3.6	Access the Reimage Feature	55
	4.3.7	Close a Patient	
4.4		ıt Sets	
	4.4.1	Select an Output Set	
	4.4.2	Add or Edit an Output Set	
		1	

4.5	On-Demand Outputs	56
	4.5.1 Export	56
	4.5.2 Archive	57
	4.5.3 Print	58
4.6	Gain Calibration	60
5: L	User Interface - Technologist Display	
5.1	Technologist Display Screens and Modes	
	5.1.1 Setup	
	5.1.2 Test	
	5.1.3 Standby	
	5.1.4 Biopsy	
	5.1.5 Lavage	
	5.1.6 Aspirate	66
6: I1	maging System Modes	67
6.1	X-ray Mode	67
6.2	No X-ray Mode	
6.3	Single Chamber Mode	68
7. D	Pioner.	71
	Siopsy	
7.1	Set Up the Console	
7.2	Prepare for Image Acquisition	
7.3	Connect the Device Driver and the Biopsy Supplies	
7.4	Pre-Procedure System Check	
7.5	Attach the Introducer to the Biopsy Device	
7.6	Attach a Biopsy Device Adapter to the Biopsy Guidance System	
	7.6.1 Using the System with Stereotactic (STX) Guidance	
7.7	Attach the Biopsy Device to the Biopsy Device Adapter	
7.8	Performing a Biopsy and Acquiring Real-time Images	
	7.8.1 When the Twelve Chambers of the Tissue Filter Contain Cores	
	7.8.2 Lavage and Aspirate	
	7.8.3 Conclude Tissue Acquisition	
	7.8.4 Deploy a Biopsy Site Marker	
	7.8.5 Remove the Biopsy Device Adapter	
7.9	Conclude the Procedure	
7.10		
7 11	How to Change the Bionsy Needle During Troubleshooting	88

8: I1	mages		91
8.1	Introd	uction	91
8.2	Review	w the Images	91
	8.2.1	Tag an Image	92
	8.2.2	Review Mode Screen	94
	8.2.3	Image Enhancement Tools	95
	8.2.4	Filter Info and Comments Screen	95
	8.2.5	Reimage	97
8.3	Send t	he Images to the Output Devices	97
9: S	ystem	Administration Interface	99
9.1	Admii	n Screen	99
9.2	About	Screen	101
9.3	Chang	ge the User Language Preference	102
9.4	Chang	ge the Keyboard Layout	102
9.5	Accou	nt Security Settings	103
9.6	Systen	n Tools	104
	9.6.1	System Tools Screen	105
10:	Mainte	enance, Cleaning, and Disinfecting	107
10.1		al Information	
	10.1.1		
	10.1.2	C	
	10.1.3	ě	
10.2	Comp	onent Specific Cleaning and Disinfection	
	10.2.1	•	
	10.2.2	Clean and Disinfect the Device Driver and Remote Control	109
	10.2.3	Clean and Disinfect the Tissue Filter Drawer	110
	10.2.4	Clean the Imaging Display	110
	10.2.5	Clean the Technologist Display	111
	10.2.6	Clean the Footswitch	111
	10.2.7	Cleaning Schedule	112
10.3	Preve	ntive Maintenance	113
	10.3.1	Vacuum Line Assembly	113
	10.3.2	Device Driver	
	10.3.3	Footswitch	115
	10.3.4	Power Cord	115
	10.3.5	Preventive Maintenance Schedule for the User	
	10.3.6	Preventive Maintenance Schedule for the Service Engineer	117

App	pendix A System Specifications	119
A.1	Product Measurements	119
	A.1.1 Console	119
	A.1.2 Device Driver	120
	A.1.3 Biopsy Needle	120
A.2	Operation and Storage Environment	120
	A.2.1 General Conditions for Operation	120
	A.2.2 Storage Environment	120
A.3	Electrical Input	121
A.4	Classification	121
A.5	Console Technical Information	122
	A.5.1 General Information	122
	A.5.2 Network Environment	122
A.6	X-Ray Tube Technical Information	122
	A.6.1 X-Ray Generator	123
	A.6.2 Disposal	123
A.7	Imaging System Technical Information	123
	A.7.1 Image Receptor	123
App	pendix B Troubleshooting	125
B.1	Error Recovery and Troubleshooting	125
B.2	Types of Messages and Alert Messages	125
	B.2.1 Fault Levels	125
	B.2.2 System Messages	126
B.3	System Restart	126
B.4	Troubleshooting During Setup	
B.5	Troubleshooting During a Procedure	129
Apr	pendix C Brevera Compatible Components and Accessories	133
C.1	Hologic Factory-Verified Compatible Components and Accessories	
Anr	pendix D Owner's Record	135
	•	
Glo	ossary of Terms	137
Ind	av	130

## **List of Figures**

Figure 1: Brevera Breast Biopsy System Overview	13
Figure 2: Location of the System Label	22
Figure 3: System Components (Front, Right)	23
Figure 4: System Components (Back, Left)	24
Figure 5: Imaging Cabinet Components	27
Figure 6: Suction Canister Components	28
Figure 7: Device Driver Components	29
Figure 8: Metal Parts of the Device Driver	29
Figure 9: Biopsy Needle Components	30
Figure 10: Biopsy Needle Components - Detailed	31
Figure 11: Tissue Filter Components	32
Figure 12: System Connections	
Figure 13: Device Driver Connection	35
Figure 14: Suction Canister Connections	36
Figure 15: Biopsy Needle and Device Driver Connection	37
Figure 16: Tissue Filter Connections in the Tissue Filter Drawer	38
Figure 17: Tissue Filter Tubing Connection to the Suction Canister	
Figure 18: Imaging Panel Controls and Indicators	39
Figure 19: Mode, Arm and Fire Buttons	40
Figure 20: Windows 10 Login Screen	43
Figure 21: Startup Screen	43
Figure 22: The Taskbar	45
Figure 23: Select Patient Screen	
Figure 24: Add Patient Screens	48
Figure 25: Edit Patient Screens	49
Figure 26: Filter Criteria in the Patient Filter Screen	
Figure 27: An Example Procedure Screen	52
Figure 28: Add Procedure Dialog Box	
Figure 29: Two Images Selected for Archive or Export	56
Figure 30: Two Images Selected for Archive or Export	57
Figure 31: Example of the Print Screen	
Figure 32: Setup Screen	61
Figure 33: Test Screen	
Figure 34: Test Screen - Arm and Fire	62
Figure 35: Standby Screen	
Figure 36: Biopsy Screen	64
Figure 37: Lavage Screen	65
Figure 38: Aspirate Screen	
Figure 39: No X-ray Screen	
Figure 40: Menu Option for Single Chamber Mode	
Figure 41: Prompt for the Single Chamber Tissue Filter	
Figure 42: Single Chamber Tissue Filter	68

## **Brevera Breast Biopsy System User Guide** Table of Contents

Figure 43: Single Chamber Video	69
Figure 44: Procedure Example	72
Figure 45: Connect the Biopsy Needle to the Device Driver	74
Figure 46: Connect the Tissue Filter Tubing to the Suction Canister	75
Figure 47: Placement of the Saline Tubing in the Saline Pinch Valve	75
Figure 48: Seat the Tissue Filter in the Tissue Filter Drawer	76
Figure 49: Slide the Introducer onto the Biopsy Device	78
Figure 50: Use Standard or Petite Aperture	78
Figure 51: Biopsy Device Adapter	79
Figure 52: Install the Tissue Filter Cap	85
Figure 53: System Recognizes When the Device Driver Is Ready	87
Figure 54: System Recognizes That the Biopsy Needle Is Removed	88
Figure 55: System Recognizes When the Device Driver Is Ready	89
Figure 56: An Acquired Image	91
Figure 57: Image Tools in the Procedure Screen	91
Figure 58: An Example Tagged Image	92
Figure 59: Tag for Service Button on the Procedure Screen	93
Figure 60: Tag for Service Screen	93
Figure 61: Review Mode Screen	94
Figure 62: Image Enhancement Tools	95
Figure 63: Filter Info and Comments Screens	96
Figure 64: Admin Screen	99
Figure 65: System Tab of the About Screen	101
Figure 66: Accented Letters on the On-screen Keyboard	102
Figure 67: Account Security Screens	103
Figure 68: System Tools Button	104
Figure 69: System Tools Screen	105
Figure 70: Brevera System Measurements	119

## **List of Tables**

Table 1: Filter Criteria Options	51
Table 2: Admin Screen Functions	
Table 3: The System Tools	105
Table 4: Recommended Maintenance Schedule for the User	
Table 5: Service Engineer Preventive Maintenance	117
Table 6: Resolving Possible Issues During Setup	
Table 7: Resolving Possible Issues During a Procedure	

## **Chapter 1 Introduction**

#### 1.1 Preface

Read all this information carefully before installation and operation. Follow all warnings and precautions as stated in this manual. Keep this manual available during procedures. Physicians should tell patients about all potential risks and adverse events described in this manual with respect to the operation of the system.

#### 1.2 Indications for Use

 ${R\hspace{-0.2em}/}_{\hspace{-0.2em}X}$ Only

Caution: United States federal law restricts this device to sale by or on the order of a physician.

The Hologic® Brevera® breast biopsy system with CorLumina® imaging technology is intended to provide breast tissue samples for diagnostic sampling of breast abnormalities. The Brevera breast biopsy system excises targeted tissue and optionally delivers in-line radiographic images of the excised tissue. The Brevera breast biopsy system is intended to provide breast tissue for histologic examination with partial or complete removal of the imaged abnormality. In instances when a patient presents with a palpable abnormality that has been classified as benign through clinical and/or radiological criteria (for example, fibroadenoma, fibrocystic lesion), the Brevera breast biopsy system may also be used to partially remove such palpable lesions. The extent of histologic abnormality cannot be reliably determined from its mammographic appearance. Therefore, the extent of removal of the imaged evidence of an abnormality does not predict the extent of removal of histological abnormality, for example, malignancy. When the sampled abnormality is not histologically benign, it is essential that the tissue margins be examined for completeness of removal using standard surgical procedure.

#### 1.3 Contraindications

- The Brevera breast biopsy system with CorLumina imaging technology is not intended for therapeutic applications.
- The Brevera breast biopsy system with CorLumina imaging technology is contraindicated for those patients who, based on the physician's judgment, may be at increased risk or develop complications associated with core removal or biopsy.
   Patients receiving anticoagulant therapy or who may have bleeding disorders may be considered at increased risk of procedural complications.

## 1.4 Patient Target Group

The Brevera breast biopsy system is intended to be used on patients with suspicious breast tissue abnormalities that need to be histologically sampled via biopsy for the primary diagnosis of said abnormality.

#### 1.5 Clinical Benefits

The Brevera breast biopsy system performs biopsies in a time efficient manner by combining vacuum-assisted tissue acquisition, real-time imaging verification, and advanced post-biopsy handling in one integrated system. Real time imaging enables visual confirmation of tissue acquisition steps, allowing physicians to make informed clinical decisions with confidence thereby avoiding repeated biopsy.

#### 1.6 Potential Adverse Effects

The following potential adverse effects can occur or have been reported in association with the use of the Brevera breast biopsy system:

- Hematoma
- Perforation
- Blunt Trauma
- Infection
- Tissue Damage
- Pain
- Bleeding
- Inflammation
- Electric Shock
- Radiation Exposure, unintended
- Foreign Body Reaction

#### 1.7 Essential Performance

The essential performance of the Brevera breast biopsy system is to maintain compliance with the applicable requirements of IEC 61010-2-91:2012 to prevent exposure of the patient, operator, or other individuals to unacceptable levels of X-ray radiation.

## 1.8 System Capabilities

The Hologic Brevera breast biopsy system with CorLumina imaging technology is a vacuum-assisted biopsy device which is used to remove breast tissue in a minimally invasive manner using stereotactic or tomosynthesis imaging. The system is designed for biopsy and for specimen radiographic image acquisition and display only. The system is NOT intended to be used outside the standards, specifications, and limitations outlined in these instructions for use.

The Brevera biopsy needle is single-use and is disposable. The user connects the biopsy needle to a reusable device driver and connects the biopsy device components to the console. The biopsy needle primarily consists of a hollow needle with a side aperture and a sharpened inner cannula that, when connected to the Brevera breast biopsy system, rotates and extends across the aperture to acquire targeted tissue. The Brevera device driver contains mechanical and electrical components that drive needle rotation and advancement. During the biopsy process, vacuum created inside the biopsy device pulls tissue into the aperture. The cannula translates and rotates to cut the tissue. The tissue sample is then aspirated through a tubing line to a tissue filter. Saline is supplied through the biopsy device to lavage the cavity and deliver tissue to the tissue filter.

When the Brevera biopsy needle is connected to the Brevera device driver, the combination is referred to as the Brevera breast biopsy device.

An introducer, specifically designed for use with the Brevera breast biopsy system, is packaged with the biopsy needle. This introducer maintains access to the targeted area of interest and allows for deployment of a biopsy site marker. The introducer is used to control the variable aperture function of the device. The introducer also prohibits the user from administering medication through the Y-site in the tubing while the device is in the armed (pre-fire) position.

The Brevera breast biopsy system with CorLumina imaging technology acquires and displays the radiographic images of core specimens contained in the tissue filter. A tissue filter, specifically designed for use with the Brevera breast biopsy system, is provided with the biopsy needle. The Brevera breast biopsy system with CorLumina imaging technology also has the capability to display images as well as to transfer these images to external devices. The images acquired with this system are intended to confirm removal of tissue from a suspected lesion or pathology. The system is not intended for diagnostic purposes.

Radiology technologists, surgical personnel, surgeons, radiologists, and pathologists can use the specimen radiography equipment in the Brevera breast biopsy system with CorLumina imaging technology.

Hologic recommends that users receive training in basic ionizing radiation safety before using the system.

## 1.9 User Profiles

#### 1.9.1 Mammography Technologist

- Meets all requirements that apply to the location in which the Mammography Technologist operates.
- Completed training on the mammography system.
- Has training in mammography positions.
- Understands stereotactic breast biopsy procedures.
- Understands how to operate a computer and its peripherals.
- Understands sterile procedures.

### 1.9.2 Radiologists, Surgeons

- Meets all requirements that apply to the location in which the Physician operates.
- Understands stereotactic breast biopsy procedures.
- Understands how to operate a computer and its peripherals.
- Understands sterile procedures.
- Gives local anesthesia.
- Understands basic surgical procedures for core biopsy.

## 1.9.3 Medical Physicist

- Meets all requirements that apply to the location in which the Medical Physicist operates.
- Understands mammography.
- Has experience with digital imaging.
- Understands how to operate a computer and its peripherals.

#### 1.10 Where to Find the Installation Instructions

Contact Hologic Technical Support for installation instructions.

## 1.11 Where to Find Technical Description Information

Contact Hologic Technical Support for technical description information.

## 1.12 Warranty Statement

Except as otherwise expressly stated in the Agreement: i) Equipment manufactured by Hologic is warranted to the original Customer to perform substantially in accordance with published product specifications for one (1) year starting from the date of shipment, or if Installation is required, from the date of Installation ("Warranty Period"); ii) digital imaging mammography x-ray tubes are warranted for twenty-four (24) months, during which the x-ray tubes are fully warranted for the first twelve (12) months and are warranted on a straight-line prorated basis during months 13-24; iii) replacement parts and remanufactured items are warranted for the remainder of the Warranty Period or ninety (90) days from shipment, whichever is longer; iv) consumable Supplies are warranted to conform to published specifications for a period ending on the expiration date shown on their respective packages; v) licensed Software is warranted to operate in accordance with published specifications; vi) Services are warranted to be supplied in a workman-like manner; vii) non-Hologic Manufactured Equipment is warranted through its manufacturer and such manufacturer's warranties shall extend to Hologic's customers, to the extent permitted by the manufacturer of such non-Hologic Manufactured Equipment. Hologic does not warrant that use of Products will be uninterrupted or error-free, or that Products will operate with non-Hologic authorized third-party products.

These warranties do not apply to any item that is: (a) repaired, moved, or altered other than by Hologic authorized service personnel; (b) subjected to physical (including thermal or electrical) abuse, stress, or misuse; (c) stored, maintained, or operated in any manner inconsistent with applicable Hologic specifications or instructions, including Customer's refusal to allow Hologic recommended Software upgrades; or (d) designated as supplied subject to a non-Hologic warranty or on a pre-release or "as-is" basis.

## 1.13 Technical Support

Refer to the title page of this manual for contact information for product support.

## 1.14 Product Complaints

Report any complaints or problems in the quality, reliability, safety, or performance of this product to Hologic. If the device has caused or added to patient injury, immediately report the incident to the Hologic Authorized Representative and Competent Authority of the respective member state or country. The Competent Authorities for medical devices are usually the individual Member State's Ministry of Health or an agency within the Ministry of Health.

## 1.15 Hologic Cybersecurity Statement

Hologic continuously tests the current state of computer and network security to examine possible security problems. When necessary, Hologic provides the updates to the product.

For Cybersecurity Best Practices documents for Hologic products, refer to the Hologic Internet site at <a href="https://www.Hologic.com">www.Hologic.com</a>.

#### 1.16 Antivirus Information

Contact your service representative for antivirus installation.

## 1.17 Where to Obtain Copies of the Manuals

To obtain a PDF file of the User Guide, go to the support pages at <u>www.Hologic.com</u>. (Adobe Acrobat Reader version 5.0 or higher is required to view the PDF file.)

To order printed manuals, contact the Hologic Parts Support Group.

E-mail: *parts@hologic.com* 

Tel: 781-761-7003 Fax: 877-574-3244

## 1.18 Symbols

This section describes the Symbols on this system.

Symbol	Description	Standard
ڻ	"ON" and "OFF" (power) [variation from standard of "Stand-by"]	IEC 60417, Reference 5009
$\sim$	Alternating current (AC)	IEC 60417, Reference 5032
<b>∳•</b> ◆	Atmospheric pressure limitation	ISO 15223-1, Reference 5.3.9
EC REP	Authorized Representative in the European Community	ISO 15223-1, Reference 5.1.2
LOT	Batch code	ISO 15223-1, Reference 5.1.5
REF	Catalog number	ISO 15223-1, Reference 5.1.6

Symbol	Description Standard		
	Note	Hologic	
$\triangle$	Caution	ISO 15223-1, Reference 5.4.4	
<u> </u>	Warning	ISO 7010, Reference W001	
Ronly	Prescription use only	FDA 21 CFR 801.109	
	Caution—Radiation	Hologic	
	Mass; weight	ISO 60417, Reference 1321A	
www.hologic.com/package-inserts	Consult Instructions for Use	ISO 15223-1, Reference 5.4.3	
	Contents	N/A	
<u>~</u>	Date of Manufacture	ISO 15223-1, Reference 5.1.3	
<b>▼</b>	Device driver and remote control connection	Hologic	
8	Device driver cord management plate	Hologic	
STERNIZE	Do not re sterilize	ISO 15223-1, Reference 5.2.6	
<b>(2)</b>	Do not re-use	ISO 15223-1, Reference 5.4.2	
	Stacking limit by number	ISO 7000, Reference 2403	

Symbol	Description	Standard	
	Do not use if package is damaged	ISO 15223-1, Reference 5.2.8	
	Discard electrical and electronic equipment separately from standard waste. Send decommissioned material to Hologic or contact your service representative.	WEEE Directive 2012/19/EU	
	Refer to instruction manual or booklet	ISO 7010, Reference M002	
	Footswitch connection	Hologic	
•	Footswitch cord management plate	Hologic	
<u></u>	Humidity limitation	ISO 15223-1, Reference 5.3.8	
<del>*</del>	Keep dry	ISO 15223-1, Reference 5.3.4	
	Manufacturer	ISO 15223-1, Reference 5.1.1	
C UL US	Medical - General medical equipment as to electrical shock, fire, and mechanical hazards only in accordance with ANSI/AAMI ES 60601-1:2005 (AI:2012), CAN/CSA C22.2 No. 60601-1 (2014), and IEC 60601-1:2012.	UL Classification	
MR	Conditional use for magnetic resonance imaging	ASTM F2503 Reference no. Table 2; 7.4.6.1; Fig. 6,7	

Symbol	Description	Standard	
MR	Not safe for magnetic resonance imaging	ASTM F2503 Reference no. Table 2, Symbol 7.3.3; 7.4.9.1; Fig. 9	
QTY	Quantity	Hologic	
	Saline tubing management plate	Hologic	
SN	Serial Number	ISO 15223-1, Reference 5.1.7	
STERILE R	Sterilized using irradiation	ISO 15223-1, Reference 5.2.4	
	Single sterile barrier system with protective packaging inside	ISO 7000, Reference 3708	
	Single sterile barrier system	ISO 7000, Reference 3707	
1	Temperature limit	ISO 15223-1, Reference 5.3.7	
IPX8	The equipment or accessory is suitable for continuous immersion in water (up to 1 m of submersion for 1 hour).	IEC 60529	
<u>11</u>	This way up	ISO 7000, Reference 0623	
<b>*</b>	Type BF applied part	IEC 60417, Reference 5333	
$\subseteq$	Use-by Date	ISO 15223-1, Reference 5.1.4	
<u>A</u>	Warning electricity	ISO 7010, Reference W012	

Symbol	Description	Standard		
<b>?</b>	Wi-Fi	Hologic		
\$	Equipotentiality	IEC 60417, Reference 5021		
***	Computer network	IEC 60417, Reference 5988		
	Pinch point	ISO 7010		
n	Locking, general	IEC 60417, Reference 5569		
	Unlocking	IEC 60417, Reference 5570		
<b>●</b> H	USB	IEC 60417-1		
Made in USA	Made in United States of America	Hologic		
MD	Medical Device	ISO 15223-1, Reference 5.7.7		
Made in Costa Rica	Made in Costa Rica	Hologic		
<b>C</b> € xxxx	CE Mark with Notified Body reference number	MDR regulation (EU) 2017/745		
Translations in Box	Translations in box	Hologic		
Patents	Patents	Hologic		
CC	Country code for translation	ISO 3166		

## 1.19 Descriptions of Warnings, Cautions, and Notes

Descriptions of Warnings, Cautions, and Notes used in this manual:



#### **WARNING!**

The procedures that you must follow accurately to prevent possible dangerous or fatal injury.



### Warning:

The procedures that you must follow accurately to prevent injury.



#### Caution:

The procedures that you must follow accurately to prevent damage to equipment, loss of data, or damage to files in software applications.



#### Note

Notes show additional information.

## **Chapter 2 General Information**

## 2.1 System Overview

The Brevera breast biopsy system with CorLumina imaging technology has system components for vacuum-assisted breast tissue biopsy and system components for x-ray generation and image acquisition. The user interface for data entry, patient selection, and image acquisition and review is on the Imaging display. The user interfaces for controlling the modes of the biopsy device are buttons on the Technologist display and also on the remote control. The user interface for arming and firing the biopsy device is on the remote control. The user adjusts the rotation of the aperture on the biopsy device with the aperture knob on the disposable biopsy needle. The user adjusts the dimension of the aperture on the biopsy device using the introducer.

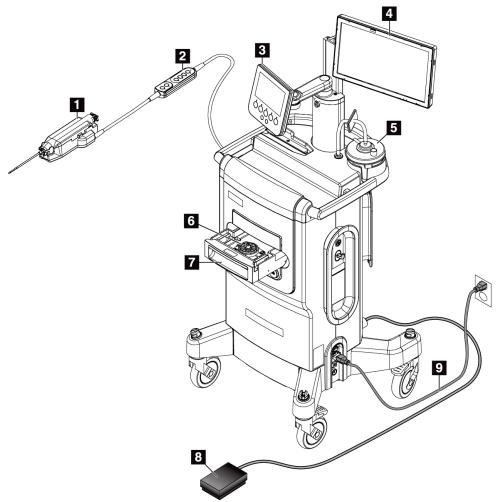


Figure 1: Brevera Breast Biopsy System Overview

#### Figure Legend

- 1. Biopsy device
- Remote control for mode control and arming or firing the biopsy device
- 3. Technologist display
- 4. Imaging display
- 5. Vacuum line assembly and suction canister
- 6. Tissue filter assembly
- 7. Tissue filter drawer
- 8. Footswitch
- 9. Power cord

## 2.2 Safety Information

Read and understand this manual before you use the system. Keep the manual available during the operation of the unit.

Always follow all the instructions in this manual. Hologic does not accept responsibility for injury or damage from incorrect system operation. For training options, contact your Hologic representative.

The system has safety interlocks, but the user must understand how to safely operate the system and be aware of the health hazards of x-ray radiation.

## 2.3 Warnings and Precautions



#### **WARNING!**

The Brevera breast biopsy system must be installed and put into service according to the guidance provided in these instructions to ensure its electromagnetic compatibility. Refer to the electromagnetic emissions and immunity tables.



#### **WARNING!**

Portable and mobile RF communications equipment can affect the Brevera breast biopsy system. Do not use portable RF communications equipment any closer than 30 cm (12 inches) to any part of the Brevera system, including cables.



#### **WARNING!**

The Brevera breast biopsy system console is designed to use the power cord provided. Do not use a different power cord. Using a different power cord can create an electrical and fire hazard. DO NOT remove the ground wire or ground prong from any power plug. DO NOT use an extension cord with this equipment. An adapter may be necessary depending on the outlet used.



#### **WARNING!**

Make sure that the power cord for the Brevera breast biopsy system console is in good condition. A damaged power cord can be an electrical shock hazard. When disconnecting the console from power, always grasp the plug at the insertion point and pull lightly. NEVER pull on the cord to unplug the unit.



#### **WARNING!**

To avoid the risk of electric shock, this equipment must only be connected to a supply mains with protective earth. Check grounding continuity regularly.



#### WARNING!

To prevent shock hazard, do not use the polarized plug for the console with a receptacle unless the blades can be fully inserted.



#### **WARNING!**

To prevent fire or shock hazard, do not expose the console to rain or moisture.



#### **WARNING!**

Only trained Service Engineers authorized through Hologic can open any of the panels. This system contains lethal voltages.



#### **WARNING!**

Electrical equipment used near flammable anesthetics can cause an explosion.



#### WARNING!

Do not use the system in an oxygen-rich environment.



#### **WARNING!**

The user must correct problems before the system is used. Contact an approved service representative for preventive maintenance.



#### **WARNING!**

Do not attempt to use any console that could present a shock hazard. Immediately contact Hologic or your distributor.



#### **WARNING!**

Do not touch the metal parts of the device driver and the patient at the same time.



#### **WARNING!**

Grounding reliability can only be achieved when this equipment is connected to a receptacle marked "hospital grade". Check grounding continuity regularly.



#### **WARNING!**

The console should not touch other electrical equipment during use.



#### **WARNING!**

Use caution when attaching sharp devices to the adapter.



#### **WARNING!**

Avoid operator or instrument contact with the sheathed needle part of the Brevera biopsy needle.



#### **WARNING!**

Dispose of all opened disposable instruments whether used or not.



#### **WARNING!**

Do not resterilize or reuse the Brevera biopsy needle or the introducer. Resterilization or reuse can compromise the integrity of the instrument. This can lead to potential risks of failure of the biopsy needle to perform as intended or to cross-contamination associated with using inadequately cleaned and sterilized devices.



#### **WARNING!**

No modification of this equipment is allowed.



#### **WARNING!**

The protection provided by the equipment is reduced if the equipment is used in a method not specified by Hologic.



#### Warning:

This device contains dangerous material. Send decommissioned material to Hologic or contact your service representative.



#### Warning:

This system is intended for use by healthcare professionals only and should be operated in a professional healthcare facility.



#### Warning:

Control the access to the equipment according to local regulations for radiation protection.



#### Warning:

This system can be dangerous to the patient and the user. Always follow the safety precautions for x-ray exposures.



#### Warning:

As with any medical procedure, make sure that users wear appropriate personal protective equipment to guard against potential contact with bodily fluids.



#### Warning:

The Brevera breast biopsy system with CorLumina imaging technology is not for use with MRI or Ultrasound.



#### Warning:

The Brevera system breast biopsy procedure should be performed only by persons with sufficient training and familiarity with this procedure. Consult medical literature relative to techniques, complications, and hazards before performing any minimally invasive procedure.



#### Warning:

The Brevera biopsy device should only be used by physicians trained in percutaneous biopsy procedures.



#### Warning:

Use sound professional judgment when using the Brevera breast biopsy device on patients with breast implants.



#### Warning:

Minimally invasive instruments and accessories manufactured or distributed by companies not authorized by Hologic may not be compatible with the Brevera breast biopsy system. Use of such products can lead to unanticipated results and possible injury to the user or patient.



#### Warning:

Instruments or devices that come into contact with bodily fluids can require special disposal handling to prevent biological contamination.



#### Warning:

Cables and cords can pose a tripping hazard. Position cords safely out of the way. When not in use, wind cords securely around the cord management plates.



#### Caution:

If the console will not be used for several days or an extended period of time, disconnect the console from the electrical outlet. Place the console in a location where it will not be damaged.



#### Caution:

The console can radiate radio frequency energy and, if not installed and used in accordance with the instructions, can cause harmful interference to other devices in the vicinity. There is no guarantee that interference will not occur in a particular installation. If the console does cause harmful interference to other devices, which can be determined by turning the console off and on, the user is encouraged to try to correct the interference by one or more of the following measures: reorient or relocate the receiving device; increase the separation between the equipment; connect the console to an outlet on a circuit different from that to which the other device is connected; or consult Technical Support.



#### Caution:

To prevent internal heat buildup, allow sufficient air circulation around the console. Do not place the console within .3 meters (1 foot) of any obstructive surface.



#### Caution:

Use and store the console in a location away from heat sources such as radiators or air ducts, direct sunlight, excessive dust, mechanical vibration, or shock.



#### Caution:

Do not operate the equipment at altitudes higher than 3,000 meters (9,842 feet). Do not store or operate the equipment in conditions outside of the stated equipment storage, transport, and operating environmental conditions.



#### Caution:

Allow the console to acclimate for 24 hours before use after removal from storage.



#### Caution:

Risk of data loss. Do not put any magnetic media near or on devices that create any magnetic fields.



#### Caution:

To prevent possible damage to the system, follow the recommended procedure to turn off the equipment.



#### Caution:

Never use strong solvents or abrasive cleansers to clean the console since these will damage the console casing. Clean the console after the power cord is disconnected from the power outlet.



#### Caution:

Use the least possible amount of cleaning fluids. The fluids must not flow or run.



#### Caution:

To prevent damage to the electronic components, do not use disinfectant sprays on the system.



#### Caution:

The weight and mobility of this system require the following precautions:

- Be careful when you move the system on uneven or sloped surfaces.
- Make sure that the wheels are locked when the system is stationary.
- Before you move the system, make sure that the cables are stored in a safe position and that the display screens are pushed down to their lowest heights. Make sure that all four casters are unlocked.
- Do not make quick stops. Do not move with excessive force.



#### Caution:

The system is a laboratory device and not a normal computer. Do not make changes to the hardware or software that are not authorized. Install this device behind a firewall for network security. The computer virus protection or network security for this laboratory device is not provided (for example, a computer firewall). The network security and anti-virus provisions are the responsibility of the user.



#### Caution:

If you are a Hologic customer in the United States, contact Hologic if you cannot correct a problem using these instructions for use. Outside of the United States, contact your distributor or local Hologic representative with questions, comments, or technical service issues.

#### 2.4 Interlocks

The x-ray generation stops automatically and a system message or alert appears when there is an interruption of an x-ray exposure. (Examples of x-ray interruptions include: the mAs timer or maximum exposure timer times out, the tissue filter drawer is opened, or the key switch is turned to the locked position.) To make another exposure:

- 1. Select the **OK** button in the system message dialog box.
- 2. Select the status icon (shaped like the Brevera breast biopsy console) in the taskbar at the bottom of the Imaging display screen.
- 3. Complete any action (such as "close the drawer") that shows in the system message area of the Imaging display screen.
- 4. Select the Clear All Faults option.

## 2.5 Compliance

This section describes the system compliance requirements and the responsibilities of the manufacturer.

## 2.5.1 Compliance Requirements

The manufacturer has the responsibility for the safety, reliability, and performance of this equipment with the following provisions:

- The equipment is used according to the instructions for use.
- The assembly operations, extensions, adjustments, changes, or repairs are performed only by authorized persons.
- The network and communication equipment must be installed to meet IEC Standards.



#### Caution:

This system is intended for use by healthcare professionals only. This system may cause radio interference or may disrupt the operation of nearby equipment. It may be necessary to take mitigation measures, such as re-orienting or relocating the equipment or shielding the location.



### Caution:

The emissions characteristics of this equipment make it suitable for use in industrial areas and hospitals (CISPR 11 class A). If it is used in a residential environment (for which CISPR 11 class B is normally required), this equipment might not offer sufficient protection to radio-frequency communication services. The user may need to take mitigation measures, such as relocating or re-orienting the equipment.



#### Caution:

The Medical Electrical (ME) Equipment or ME System should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, make sure that the ME Equipment or ME System operates correctly in this configuration.



#### Caution:

Changes or modifications not expressly approved by Hologic could void your authority to operate the equipment.

#### 2.5.2 Compliance Statements

The manufacturer states this device is made to meet the following requirements:

- IEC 60601-1: 2005/A1: 2012 ed. 3.1 Medical Electrical Equipment Part 1: General Requirements for Basic Safety and Essential Performance
- ANSI/AAMI ES 60601-1: 2005 A1: 2012, C1: 2009/(R)2012 and A2: 2010/(R)2012
   Medical Electrical Equipment -- Part 1: General Requirements for Basic Safety and Essential Performance
- CAN/CSA-C22.2 No. 60601-1 (2014) Medical Electrical Equipment Part 1: General Requirements for Basic Safety and Essential Performance
- EN 60601-1: 2006 / A1: 2013 Medical Electrical Equipment Part 1: General Requirements for Basic Safety and Essential Performance
- IEC 60601-1-2 ed. 4: 2015 Medical Electrical Equipment Part 1-2: General Requirements for Basic Safety and Essential Performance Collateral Standard: Electromagnetic Compatibility Requirements and Tests
- IEC 60601-1-6: 2010/AMD1: 2013 and IEC 62366: 2007/AMD1: 2014 Application of Usability Engineering to Medical Devices

#### 2.6 **System Label Location**

For the location of the system label and serial number, refer to the figure that follows.

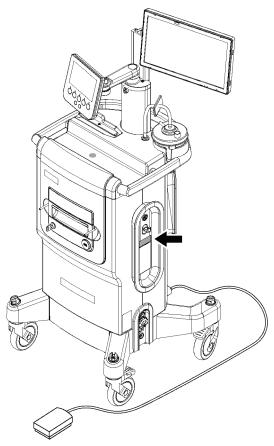


Figure 2: Location of the System Label

# **Chapter 3 Components, Controls, and Indicators**

#### **System Components** 3.1

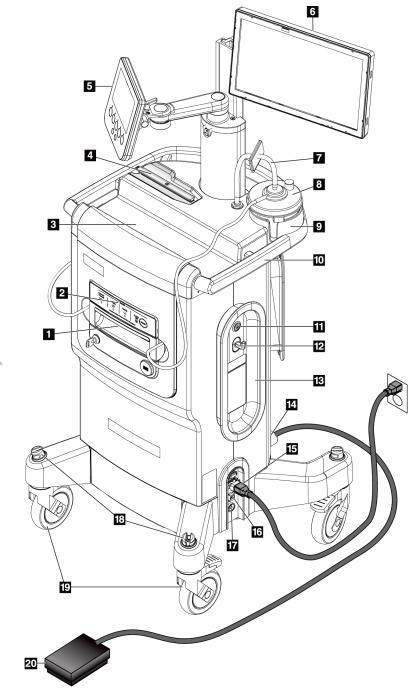


Figure 3: System Components (Front, Right)

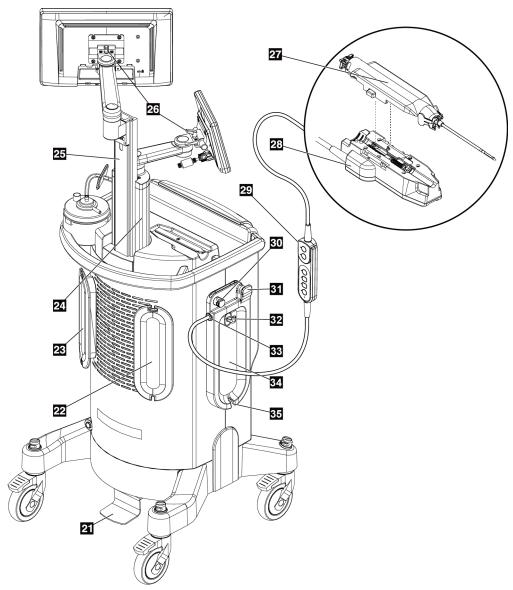


Figure 4: System Components (Back, Left)

_			
1	Tissue filter drawer	The compartment where biopsy cores are collected and imaged.	
2	Imaging panel	Includes status indicators and lockout key for the x-ray system.	
3	Console tray	Storage and staging area for the biopsy needle packaging.	
4	Device driver holder	Storage and setup area for the device driver.	
5	Technologist display	The display screen shows the current status of the system, and the buttons provide one of the two methods for selecting the system mode. The display has an articulating arm and is attached to the display mounting post.	
6	Imaging display	User interface touch screen display for viewing, annotating, saving, and exporting radiographic images. The display is secured to an adjustable slide in the display mounting post.	
7	Vacuum line assembly	Clear tubing attached to the console at one end. The other end has a blue connection that attaches to the suction canister lid at the port labeled "VACUUM". The vacuum line assembly includes a hydrophobic filter to prevent fluid backflow.	
8	Suction canister	Disposable canister used for the collection, retention, and disposal of liquid medical waste.	
9	Suction canister holder	Recessed area of the console tray that holds the suction canister.	
10	Console handle	Move the console by pushing this handle.	
11	Power button	Push-button switch that powers ON the system. The power button illuminates green when the console is powered on.	
12	Utility hook	Hook for hanging reference cards or patient charts.	
13	Power cord management plate	Provides safe storage for the power cord when moving or storing the console. Wind the power cord clockwise around the power cord management plate.	
14	Footswitch connection	Receptacle for the footswitch cord. The red dots align at the top when attached correctly.	
15	Power connection	AC inlet. Receptacle for the hospital grade power cord (provided).	
16	Ethernet connection	Port for Ethernet (network) cable. (Optional Wi-Fi available.)	
17	Potential equalization connection	Receptacle for equipotential ground.	
18	Cord management slots	Slots for cord management.	
19	Casters	All four casters allow for 360-degree swivel and have locking brakes to hold the unit in place.	
20	Footswitch	Depressing the footswitch activates the biopsy device.	
	ı	1	

21	Imaging display height- adjust pedal	Depressing the pedal releases the Imaging display mounting slide to allow for height adjustment. Release the pedal to lock the slide.	
22	Device driver cord management plate	Provides safe storage for the device driver cord when moving or storing the console. Wind the device driver cord clockwise around the device driver cord management plate.	
23	Footswitch cord management plate	Provides safe storage for the footswitch and the footswitch cord when moving or storing the console. Wind the footswitch cord counter-clockwise around the footswitch management plate.	
24	Display mounting post	Stationary post that the Technologist display is mounted on. The mounting post also houses and guides the Imaging display mounting slide.	
25	Imaging display mounting slide	Height-adjustable slide that the Imaging display is mounted on. The slide is released and locked with the Imaging display height-adjust pedal.	
26	Display tilt-adjust levers	Release the lever to adjust the tilt of the display. Engage the lever to lock the display into the desired tilt position.	
27	Biopsy needle	Single-use, disposable biopsy needle for acquiring biopsy specimens. Refer to <i>Biopsy Needle Components</i> on page 30.	
28	Device driver	This electromechanical component is powered by the console. When the device driver is connected to the biopsy needle, the device driver propels the rotation and the advancement of the cutting cannula in the biopsy needle. The device driver also arms and fires the biopsy needle.	
29	Remote control	This in-line, hand-held component allows the physician to arm and fire the biopsy device. The remote control also provides one of the two methods for selecting the system mode.	
30	Saline pinch valve	Area to insert the saline tubing to control the flow of saline.	
31	Saline tubing conduit	Slot used to guide the saline tubing toward the saline pinch valve.	
32	Saline bag hook	Hook for hanging the saline bag (250 cc recommended).	
33	Device driver and remote control connection	Receptacle for the cord for the device driver and remote control. The red dots align at the top when attached correctly.	
34	Saline tubing management plate	Wind the saline tubing counterclockwise around the saline tubing management plate.	
35	Saline tubing notch	Recess in the saline tubing management plate used to guide the saline tubing from the saline bag and around the saline tubing management plate.	

# 3.1.1 Imaging Cabinet Components

The Brevera console houses a cabinet where the biopsy cores are collected and imaged. The cores are collected in a disposable tissue filter that is placed in the drawer of the imaging cabinet. The imaging cabinet includes a panel that gives status information about the x-ray system.

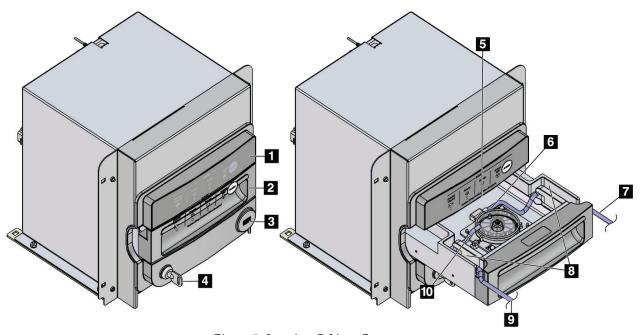


Figure 5: Imaging Cabinet Components

- 1. Imaging panel
- 2. Tissue filter drawer
- 3. USB port (for non-powered USB flash drive only)
- 4. Key switch
- 5. Detector plate
- 6. Tissue filter assembly
- 7. Tubing outlet to the suction canister
- 8. Tubing channels
- 9. Suction tubing inlet from the biopsy needle
- 10. Locator pin (each corner)

# 3.1.2 Suction Canister Components

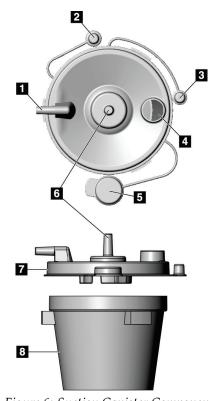


Figure 6: Suction Canister Components

# Figure Legend

- 1. "Patient" port receives the suction tubing (clear with a blue connector end) from the tissue filter
- 2. Cap for patient port
- 3. Cap for vacuum port
- 4. Large opening
- 5. Cap for large opening
- 6. "Vacuum" port receives the vacuum line assembly connection from the console
- 7. Suction canister lid
- 8. Suction canister base



#### Note

The suction canister is a single-use product with a recommended maximum volume of 400 cc.

# 3.1.3 Biopsy Device Components

The Brevera breast biopsy device consists of a sterile, disposable biopsy needle that locks into the non-sterile, Brevera reusable device driver. The biopsy needle includes a tissue filter assembly to collect the biopsy cores for imaging.

## **Device Driver Components**

The device driver is the electromechanical component of the biopsy device. When connected to the biopsy needle, the device driver rotates and advances the cutting cannula. The device driver also arms and fires the biopsy needle.

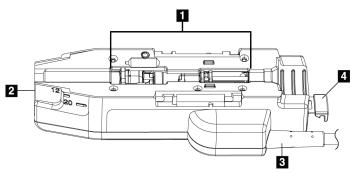


Figure 7: Device Driver Components

### Figure Legend

- 1. Connection to the biopsy needle
- Connection point to the biopsy device adapter (groove on bottom)
- 3. Device driver cord
- 4. Release latch for the biopsy device adapter



#### Warning:

Do not touch the metal parts of the device driver and the patient simultaneously.

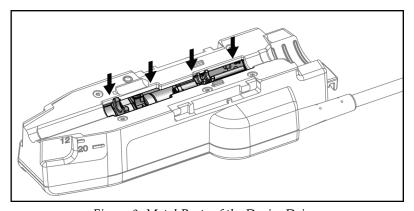


Figure 8: Metal Parts of the Device Driver



#### Note

Store the device driver on the top of the console in the device driver holder. Wrap the device driver cord (with the remote control) clockwise around the corresponding cord management plate on the back of the console.

### **Biopsy Needle Components**

The biopsy needle includes the disposable needle itself and the tubing for the vacuum and saline lines. The biopsy needle also includes a tissue filter assembly to collect the biopsy cores for imaging. The biopsy needle is packaged with the tissue filter assembly and the introducer.

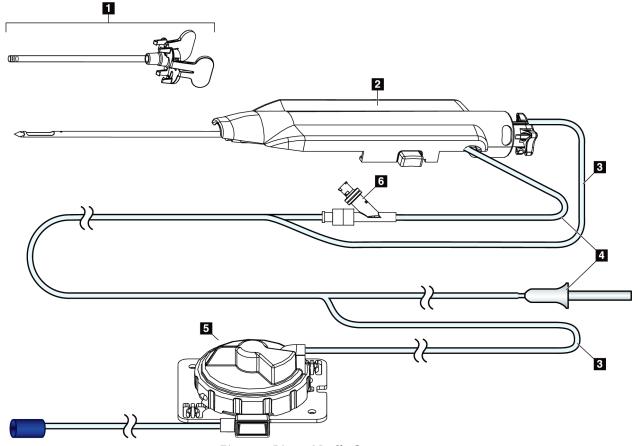


Figure 9: Biopsy Needle Components

#### Figure Legend

- 1. Introducer
- 2. Biopsy needle (attaches to the device driver)
- 3. Suction line
- 4. Saline line
- 5. Tissue filter assembly (Component may appear different in markets where product is not yet available.)
- 6. Y-valve



#### Note

The biopsy needle comes packaged with a tissue filter. Refer to <u>Tissue Filter Components</u> on page 32 for details on the tissue filter.

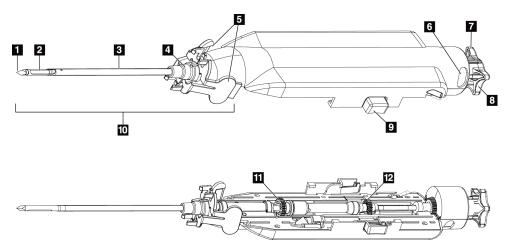


Figure 10: Biopsy Needle Components - Detailed

- 1. Trocar tip
- 2. Outer cannula aperture
- 3. Outer cannula/introducer
- 4. Introducer hub
- 5. "Wing" tabs on introducer hub
- "Clock" position indicators for outer cannula aperture orientation
- 7. Tactile flat area on the thumbwheel that indicates the outer cannula aperture orientation (flat at 12 o'clock position)

- 8. Thumbwheel
- 9. Button for engaging and disengaging the biopsy needle and device driver (both sides)
- 10. Introducer, with lilac-colored hub (shown with protective sheath removed)
- 11. Outer cannula gears
- 12. Inner cannula gears

### **Tissue Filter Components**

The tissue filter is a twelve-chambered container where the biopsy cores are collected for imaging. During the biopsy process, the tissue samples are aspirated through the biopsy needle tubing to the in-line tissue filter. A tissue filter cap is provided to cover and seal the tissue filter for storage in formalin.

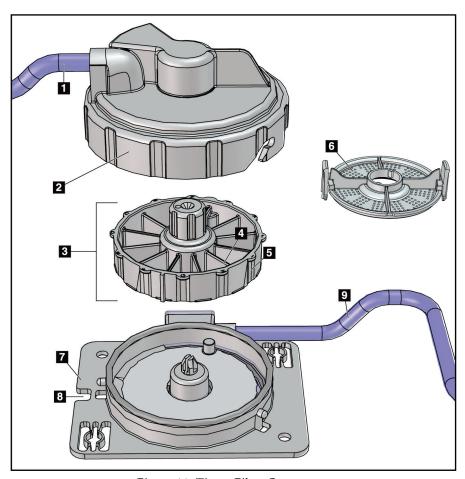


Figure 11: Tissue Filter Components

### Figure Legend

- 1. Tubing inlet from the biopsy needle
- 2. Tissue filter housing cover
- 3. Tissue filter
- 4. Tissue filter chamber (12 chambers)
- 5. Tissue filter chamber indicator (12 letter designations)
- 6. Tissue filter cap
- 7. Tissue filter base
- 8. Notch
- 9. Tubing outlet to the suction canister (waste)

\*Component may appear different in markets where product is not yet available.

# 3.2 System Connections

This section provides information on how to make all of the necessary connections to set up the Brevera breast biopsy system for use. The following connections will be explained:

- 1. Power and network connections
- 2. Footswitch connection
- 3. Device driver and remote control connection
- 4. Suction canister connections
- 5. Biopsy needle (needle, tubing, and tissue filter) connections

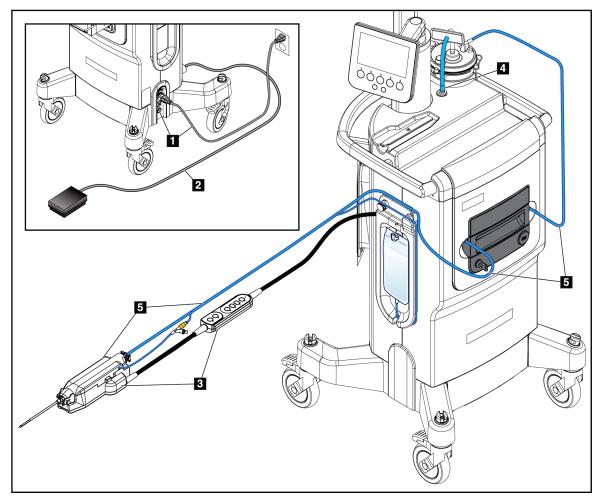


Figure 12: System Connections



#### **WARNING!**

The Brevera breast biopsy system console is designed to use the power cord provided. Do not use a different power cord. Using a different power cord can create an electrical and fire hazard. DO NOT remove the ground wire or ground prong from any power plug. DO NOT use an extension cord with this equipment. An adapter may be necessary depending on the outlet used.



#### **WARNING!**

Make sure that the power cord for the Brevera breast biopsy system console is in good condition. A damaged power cord can be an electrical shock hazard. When disconnecting the console from power, always grasp the plug at the insertion point and pull lightly. NEVER pull on the cord to unplug the unit.



#### Warning:

The use of any cables or accessories other than those specified in these instructions can result in increased emissions or decreased immunity of the Brevera breast biopsy system.



#### Caution:

Grounding reliability can only be achieved when this equipment is connected to a receptacle marked "hospital grade". Check grounding continuity regularly.

### 3.2.1 Power and Network Connections

- 1. Put the Brevera breast biopsy system in a location where you can easily access power connections and network connections.
- 2. Connect the system AC power cord to the electrical outlet.



#### Note

Position the console so that you can easily access the AC power cord connection to the electrical outlet.

3. (Optional) Connect the network cable to the Ethernet connector.

#### 3.2.2 Footswitch Connection



Warning:

Position the footswitch and power cords to prevent accidental tripping.



Warning:

Position the footswitch to prevent accidental operation by a patient or wheelchair.

The footswitch is initially installed by a Service Engineer. If the footswitch is disconnected from the console, insert the footswitch cord into the footswitch receptacle (on the back of the console). The red dots align at the top when attached correctly.

#### 3.2.3 Device Driver and Remote Control Connections

The device driver cord has an in-line remote control. The device driver cord attaches to the device driver receptacle on the left side of the saline tubing management plate, near the saline pinch valve. The red dots will align at the top when attached correctly.



### Warning:

Do not place fingers on the metal parts of the device driver. These parts can move during system use.



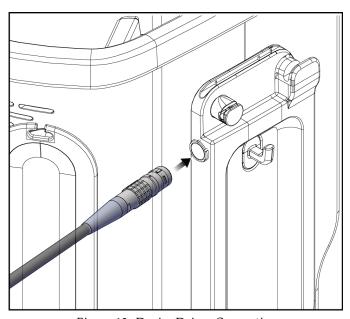


Figure 13: Device Driver Connection

#### 3.2.4 Suction Canister Connections

The system will generate vacuum when the proper modes are selected. The vacuum connections require a suction canister. The suction canister connects to the suction tubing from the tissue filter and to the tubing of the vacuum line assembly. The vacuum line assembly includes a hydrophobic filter and is required to prevent backflow into the console.

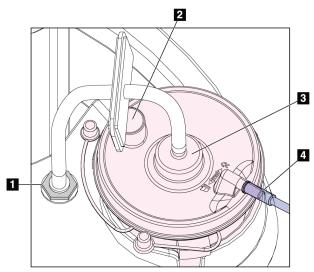


Figure 14: Suction Canister Connections

### Figure Legend

- Connection where the vacuum line assembly attaches to the console
- 2. Large opening sealed with the large opening cap
- 3. Connection where the vacuum line assembly attaches to the top port on the suction canister lid labeled "VACUUM"
- 4. Connection where the clear suction tubing from the tissue filter attaches to the horizontal side port on the suction canister lid labeled "PATIENT"

# 3.2.5 Biopsy Needle Connections



#### Warning:

Before use, inspect the protective packaging and the needle to confirm that neither has been damaged during shipment. If the packaging or the needle appear to be compromised, do not use the needle.



#### Warning:

Do not connect the biopsy needle until the system is powered on.



#### Warning:

To maintain sterility, leave the protective sheath on the tip of the biopsy needle.

The biopsy needle is packaged with the suction and saline tubing, the tissue filter, and the introducer. The biopsy needle is already connected to the suction and saline lines. The suction tubing includes an in-line tissue filter assembly. The tissue filter comes preassembled and is placed in the tissue filter drawer. Refer to <u>Tissue Filter Connections</u> on page 38 for more information on connecting the tissue filter.

The biopsy needle is powered by the device driver. The biopsy needle is connected to the device driver by carefully lining up and moving the needle down over the driver until it engages. The needle is then slid forward until it locks into position. The connected combination of the biopsy needle and the device driver is referred to as the biopsy device. For complete instructions on connecting the biopsy needle to the device driver, refer to <u>Connect the Device Driver and the Biopsy Supplies</u> on page 71.

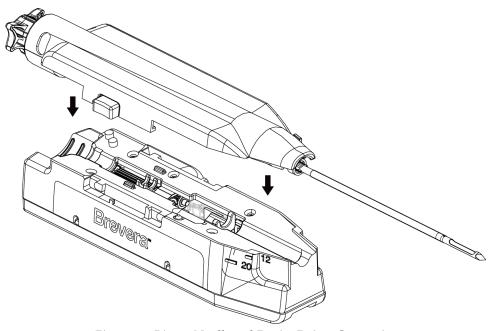


Figure 15: Biopsy Needle and Device Driver Connection



Warning:

Make sure that the biopsy needle is fully latched onto the device driver.

### **Tissue Filter Connections**

The biopsy core travels through the suction tubing to the tissue filter in the tissue filter drawer when the proper modes are selected.



#### Note

Each Brevera disposable biopsy needle package includes a tissue filter cap. Save the tissue filter cap until the biopsy acquisition is complete. The tissue filter cap covers the tissue filter when the biopsy cores are placed in formalin.

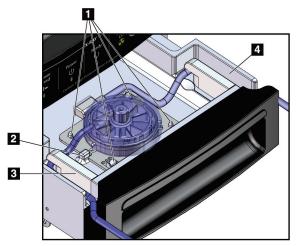


Figure 16: Tissue Filter Connections in the Tissue Filter Drawer

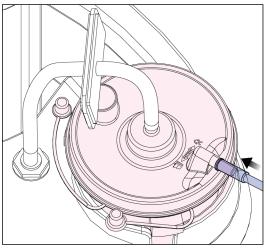


Figure 17: Tissue Filter Tubing Connection to the Suction Canister

### Figure Legend

- 1. Locator pin holes align with locator pins
- 2. Tissue filter base notch aligns with the alignment tab
- 3. Guide for the gray tubing from the biopsy needle
- 4. Guide for the suction tubing (clear tubing with a blue connector end) to the suction canister

\*Component may appear different in markets where product is not yet available.

#### 3.2.6 Combination Device Information

Brevera breast biopsy system is compatible with Hologic's Affirm® prone biopsy system and Affirm® breast biopsy guidance system, MammoTest, and GE Senographe® Stereotaxy guidance systems.

Refer to the following setup guides for Adapter setup information:

- Brevera System Setup Guide for Hologic Affirm and MultiCare® Platinum Products
- Brevera System Setup Guide for MammoTest
- Brevera System Setup Guide for GE Senographe DS Stereotaxy and GE Senographe Essential Stereotaxy

# 3.3 Controls and Indicators

# 3.3.1 Imaging Panel Controls and Indicators

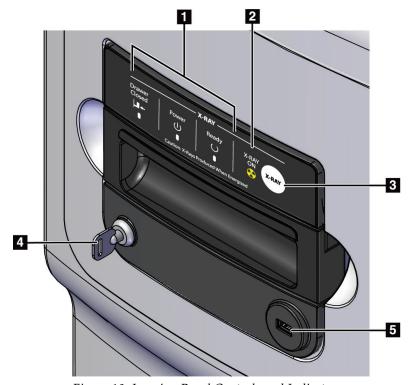


Figure 18: Imaging Panel Controls and Indicators

1.	Status indicators	<b>Drawer Closed</b> = LED is green when the tissue filter drawer is fully closed.	
		<b>Power</b> = LED is green when the power is on.	
		<b>Ready</b> = LED is green when the drawer is ready to receive cores.	
2.	X-ray On indicator	<b>X-RAY ON</b> = LED is amber when an x-ray is in progress.	
3.	X-ray button	Button used for Manual Gain Calibration and to abort current x-ray exposure	
4.	Key switch	Locked: X ray is disabled (orientation as shown). Unlocked: X ray is enabled.	
5.	USB port	Connection for non-powered USB device (optional)	

#### 3.3.2 **Mode Controls and Biopsy Device Arm and Fire Controls**

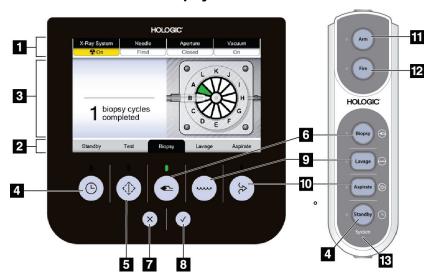


Figure 19: Mode, Arm and Fire Buttons

1.	System status	At the top of the Technologist display screen, four tabs indicate the x-ray status, the needle position, the aperture size, and the vacuum status.	
2.	Mode indicator	At the bottom of the Technologist display screen, the black highlight and green light indicate the current mode.	
3.	Message area	At the center of the Technologist display screen, the system provides instructions and other notifications.	
4.	Standby button	This push-button control turns the vacuum off and closes the aperture on the needle.	
5.	Test button	This push-button control allows for the self-priming of the system with saline. When placed in Test mode, the saline pinch valve is opened and the vacuum is turned on. When placed in Test mode, the user is guided through a series of tests to confirm the flow of saline and the completion of a cutting cycle.	
6.	Biopsy button	When placed in Biopsy mode, the biopsy device is ready for tissue acquisition. Footswitch input will initiate a biopsy cycle.	
7.	X button	Reject the message on the Technologist display.	
8.	<b>✓</b> button	Accept the message on the Technologist display.	
9.	Lavage button	When placed in Lavage mode, the saline pinch valve is opened and the inner cutting cannula is retracted. In Lavage mode, the vacuum is turned on.	
10.	<b>Aspirate</b> button	When placed in Aspirate mode, the saline pinch valve is closed and the inner cutting cannula is retracted. In Aspirate mode, the vacuum is turned on.	
11.	Arm button	This push-button control retracts the outer cannula on the biopsy needle. The user must press and hold the <b>Arm</b> button to arm the needle.	
12.	Fire button	This push-button control advances the inner and outer cannulas by the stroke noted on the product label. The position and orientation of the introducer determine whether the needle is in its petite or standard orientation. The user must press and hold the <b>Fire</b> button to fire the needle.	
13.	System power indicator	Green light illuminates when the console is powered on. Red light illuminates when there is a device driver or biopsy needle issue.	

### 3.4 Lock and Unlock a Caster

- To lock a caster, step on the locking lever on the wheel until the lever locks.
- To unlock a caster, lift the locking lever on the wheel to the UP position.

### 3.5 Move the Console



#### Note

The power cord and footswitch cord wrap on the right side of the console to provide safe storage of the cords during console movement.



#### Note

The device driver cord wraps on the left side of the console to provide safe storage of the cord during console movement.

- 1. Wrap all cords on the corresponding cord management plates.
- 2. Make sure all components and cables are in a safe position on the console.
- 3. Depress the Imaging display height adjust release pedal and push the Imaging display to slide down to the lowest height. Release the pedal to lock the height.
- 4. Rotate both the Imaging display and the Technologist display so that they are as compact as possible.
- 5. Unlock all the caster locks.
- 6. Move the console by pushing the console handle.
- 7. Make sure to lock the casters when the console is at the correct location.

# 3.6 How to Start the System



### Warning:


Do not place fingers on the metal parts of the device driver. These parts can move when the system powers on.



# **∧** Note

The console beeps three times before resetting the position of the metal parts of the device driver.

- 1. Lock the console casters.
- 2. Connect the system to power and the network.
- 3. Press the **Power** button on the side of the console.

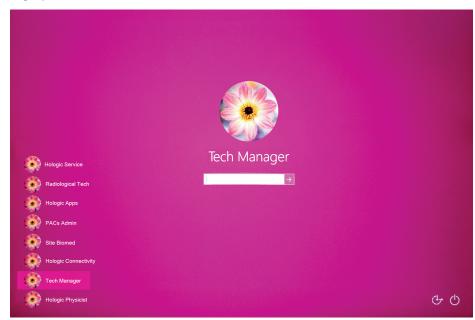

4. The system powers on and the Windows 10 *Login* screen opens on the Imaging display.

Figure 20: Windows 10 Login Screen

- 5. Select your user name from the user list.
- 6. Enter your password then select the **arrow** icon.



#### Note

To show or hide the virtual keyboard, tap the background.

7. The *Startup* screen for the Brevera breast biopsy system opens on the Imaging display. On the *Startup* screen, select the **Patient List** button to go to the *Select Patient* screen.



Figure 21: Startup Screen

#### Chapter 3: Components, Controls, and Indicators



#### Note

To log out of the Windows 10 operating system, select the **Log Out** button.



#### Note

The control panel LEDs can blink when the system starts.



#### Note

The *Startup* screen includes a **Reboot** button that restarts only the computer.



#### Note

To change the system language or other preferences, go to the *Admin* screen.

# 3.7 How to Shut Down the System

- 1. From the *Procedure* screen, select the **Close Patient** button.
- 2. From the *Select Patient* screen, select the **Close** button.
- 3. Select the **Shutdown** button on the Imaging display or press the console Power button.
  - If the system is not ready for shutdown, a prompt displays the option to continue or cancel the system shutdown. Select the **Check** button on the Technologist display to stop all pending jobs and continue shutdown. Select the **X** button on the Technologist display to keep the system running and finish pending jobs.

# 3.7.1 How to Remove All Power from the System

- 1. Make sure that the computer system and console are completely turned OFF.
- 2. Disconnect the power cord from the electrical outlet.

# **Chapter 4 User Interface - Imaging Display**

# 4.1 About the Taskbar


The taskbar at the bottom of the screen displays additional icons. For some of the icons, you can select the icon to access information or perform system tasks.



Figure Legend

	Menu	
1.	Information Icon Select the Information icon to display the Alarms menu. This section of the taskbar flashes a yellow color if an alarm exists. Select Acknowledge All to remove the flashing indication. Select the Manage Alarms option to display and close any open alarms.	No Alarms Acknowledge All Manage Alarms
2.	Aperture Status Indicator  When a biopsy device is connected to the system, this section indicates if the aperture on the biopsy needle is standard or petite.	
3.	Needle Status Indicator When a biopsy device is connected to the system, this section indicates if the biopsy needle is armed or fired. Note: Arming and firing of the biopsy device is controlled with the remote control.	
4.	Mode Indicator This section indicates the current mode for the biopsy system. Note: The mode is changed with the buttons on the remote control or the buttons on the Technologist display.	

<ul> <li>6. Output Device Icons</li> <li>Select this icon to display a menu.</li> <li>Manage Queues displays the status of jobs in the queue and job information for the selected output, and allows you to filter the quedisplay.</li> <li>7. System Status Icons  If the yellow exclamation mark icon is displayed and this taskbar set flashes yellow, select the icon for more information about the error. If the green check mark icon is displayed, select the icon to display Faults menu.</li> <li>Clear All Faults deletes all error messages that can be cleared by the</li> </ul>		
If the yellow exclamation mark icon is displayed and this taskbar see flashes yellow, select the icon for more information about the error. If the green check mark icon is displayed, select the icon to display Faults menu.		
Camera opens the camera screen to view inside the closed tissue fil drawer.  System Diagnostics accesses subsystem settings.  About displays information about the workstation.	ror.  System Dia System Def About  y the user.	ignostics

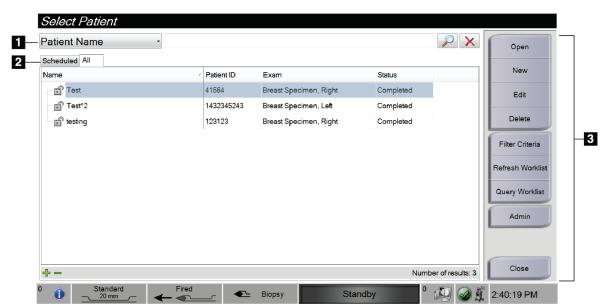

# 4.2 Select Patient Screen

Figure 23: Select Patient Screen

1. Quick Search	Search the local database for the Patient Name, Patient ID, or Accession Number.
2. Tabs	Two tabs display at the top of the screen. These tabs are configurable. A Manager can delete tabs and create new tabs (see <i>The Filter Criteria and Columns Tabs in the Patient Filter Screen</i> on page 51).  • The <b>Scheduled</b> tab displays the scheduled procedures.  • The <b>All</b> tab displays all procedures on this CorLumina system.
3. Button Functions	<ul> <li>You can perform many functions from this screen using the buttons in this area.</li> <li>Open: Open the procedure for the selected patient. Refer to Open a Patient on page 102.</li> <li>New: Add a new Patient. Refer to Add a Patient on page 48.</li> <li>Edit: Edit the patient information. Refer to Edit the Patient Information on page 49.</li> <li>Delete: Delete a patient from the worklist. Refer to Delete a Patient on page 50.</li> <li>Filter Criteria: Use a Patient Filter. Refer to Filters for Patients on page 50.</li> <li>Refresh Worklist: Update the patient list.</li> <li>Query Worklist: Search for a patient in the Modality Worklist. Refer to Query the Worklist on page 52.</li> <li>Admin: Access the administration functions. Refer to System Administration Interface on page 99.</li> <li>Close: Exit and return to the Startup screen.</li> </ul>

# 4.2.1 Open a Patient

To open a patient and start a procedure, select a patient from the list, then select the **Open** button.

# 4.2.2 Add a Patient

- 1. On the Select Patient screen, select the **New** button.
- 2. Type in new patient information (fields with an asterisk are required). Use the arrows to access the next or previous screen. Select a procedure.

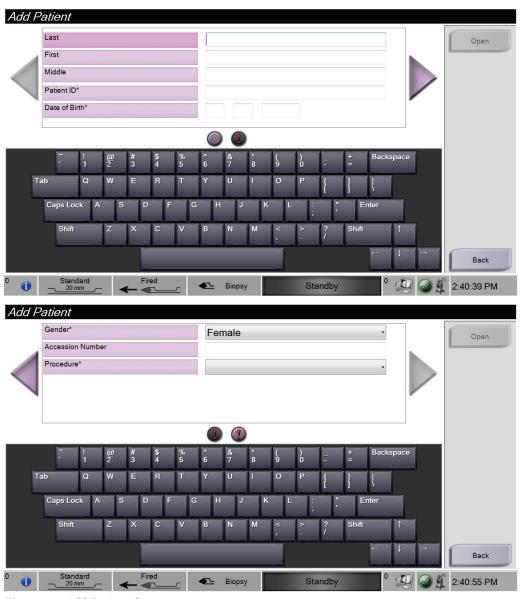


Figure 24: Add Patient Screens

3. Select the **Open** button. A *Procedure* screen for the new patient opens.

### 4.2.3 Edit the Patient Information

- 1. On the *Select Patient* screen, select the patient name, then select the **Edit** button.
- 2. On the *Edit Patient* screens, make the changes. Use the arrows to access the next or previous screen.

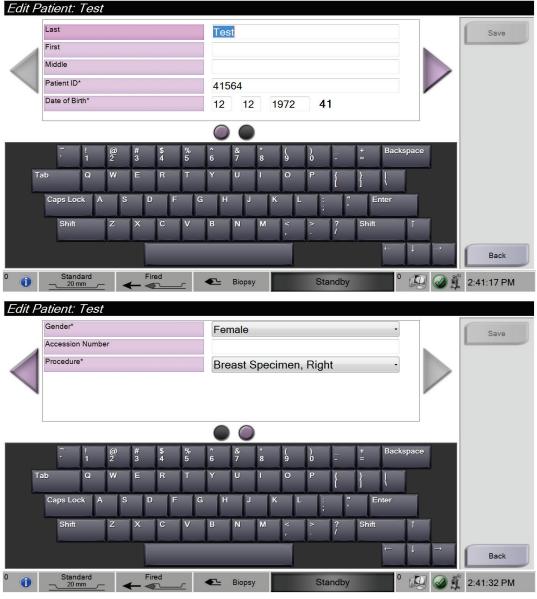


Figure 25: Edit Patient Screens

- 3. Select Save.
- 4. When the Update Successful message is displayed, select **OK**.

#### 4.2.4 Delete a Patient

- 1. On the *Select Patient* screen, select one or more patients.
- 2. Select the **Delete** button.
- 3. To the Confirmation Required prompt, select **Yes**.



#### Note

Only a Manager can delete patients.



#### Note

Reclamation normally removes the requirement to delete patients.

#### 4.2.5 Filters for Patients

When you select the **Filter Criteria** button on the *Select Patient* screen, the *Patient Filter* screen opens.

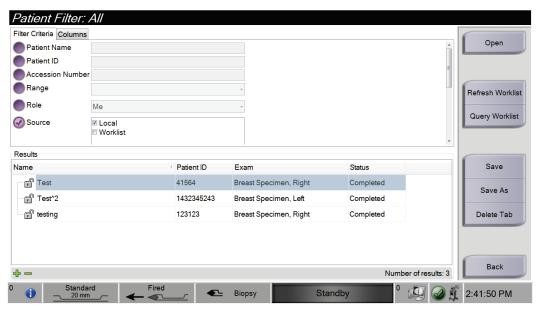


Figure 26: Filter Criteria in the Patient Filter Screen

#### Filter Criteria and Columns Tabs in the Patient Filter Screen

The **Filter Criteria** tab allows you to change the filtering options for the patient list. When you select or deselect an option, the change shows in the results area of the screen.



#### Note

A Manager can save these new filters to the selected tab on the *Select Patient* screen (refer to *Other Functions of the Filter Criteria* on page 51).

The **Columns** tab allows you to add more search options (for example, Age, Gender, Date of Birth) to the filtered list. The options display as columns in the results area. To add more columns to a filtered list, select the **Columns** tab, and then select the options.



#### Note

When you select the **Open** button, the *Procedure* screen for the selected patient opens.

#### Other Functions of the Filter Criteria

The **Save**, **Save As**, and **Delete Tab** buttons on the *Patient Filter* screen allow Managers to add, change, or delete tabs on the *Select Patient* screen.

Table 1: Filter Criteria Options

Option		Step
Change the current patient filter	1.	Select a tab on the Select Patient screen.
parameters.	2.	Select the Filter Criteria button.
	3.	Select the filter options.
	4.	Select the <b>Save</b> button.
	5.	Make sure the name of the tab you selected is in the name field.
	6.	Select <b>OK</b> .
Create a new tab for the Select Patient	1.	Select a tab on the Select Patient screen.
screen.	2.	Select the Filter Criteria button.
	3.	Select the filter options for the tab.
	4.	Select the <b>Save As</b> button.
	5.	Enter a new name for the tab.
	6.	Select <b>OK</b> .
Delete a tab from the Select Patient	1.	Select a tab on the Select Patient screen.
screen.	2.	Select the Filter Criteria button.
	3.	Select the <b>Delete</b> button.
	4.	Select <b>Yes</b> to the confirmation prompt.

#### 4.2.6 Refresh the Worklist

Select the **Refresh Worklist** button to update the patient lists.

# 4.2.7 Query the Worklist

Use the Query Worklist feature to search for a patient or a list of patients. Use one or more fields to query the Modality Worklist Provider. All fields to query are configurable. The default fields are as follows: Patient Name, Patient ID, Accession Number, Requested Procedure ID, Scheduled Procedure Date. The scheduled procedure is displayed and the patient is added to the local database.

# 4.3 Procedure Screen

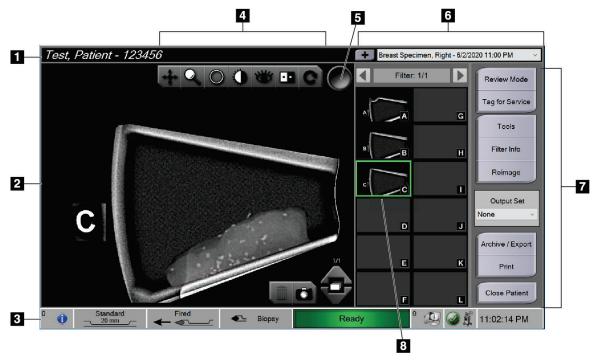


Figure 27: An Example Procedure Screen

1.	Patient information	Shows the patient name and ID.	
2.	Image	Real-time image of the specimen in the tissue filter. The letter-designation on the tissue filter chamber is visible in the image.	
3.	Taskbar	Shows the status of the console, needle, and imaging functions.  A green name in the imaging status area of the taskbar indicates that function is in a ready status. A gray name in the imaging status area indicates that function is not ready. A yellow/orange in the imaging status indicates that an x-ray is being taken. Refer to About the Taskbar on page 45 for more information on the taskbar.	
4.	Image enhancement tools	Use the image enhancement tools (refer to <i>Image Enhancement Tools</i> on page 95) to enhance the whole image or areas of interest in the image.	
5.	Tag tool	Selects a particular image or images. Tagged images show a green check mark in a circle in the upper right corner of the image as well as the image thumbnail.	
6.	Procedure information	View the list of procedures for the current patient. Use the + button to add a procedure for the current patient.	
7.	Other functions	Review and annotate images, reimage specimens already in the tissue filter chambers, select the output destination for acquired images, or close the patient.	
8.	Image thumbnails	Shows small icons of images related to the selected tissue filter. A thumbnail image with a camera icon is a snapshot image. Use the Filter bar at the top of the image thumbnails ("Filter: 1/1" shown) to navigate through multiple tissue filters.	

#### 4.3.1 Select a Procedure

Select an entry from the list in the Procedure information area of the screen.

#### 4.3.2 Add a Procedure

1. To add another procedure for the active patient, select the + button on the upper right of the *Procedure* screen to display the Add Procedure dialog box.

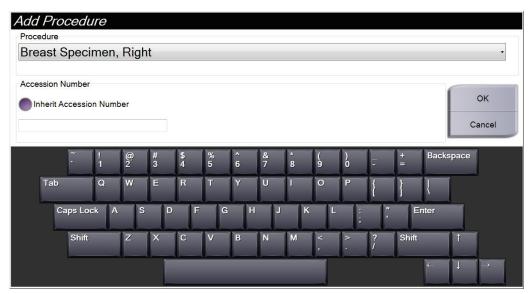


Figure 28: Add Procedure Dialog Box

- 2. Use the drop-down list to select the type of procedure to add.
- 3. Enter an Accession Number (optional).
- 4. Select the **OK** button. A new number displays in the Procedure Information area of the *Procedure* screen.

#### 4.3.3 Access the Review Mode Screen

Select the **Review Mode** button on the *Procedure* screen to access the *Review Mode* screen and the image review features. Refer to <u>Review Mode Screen</u> on page 93 for more information.

# 4.3.4 Access the Image Enhancement Tools

Select the **Tools** button on the *Procedure* screen to access the tools bar and image enhancement features. Refer to *Image Enhancement Tools* on page 95 for more information.

#### 4.3.5 Access the Filter Info

Select the **Filter Info** button to access filter information such as laterality, accession number, procedure information, and filter comments. Refer to *Filter Info and Comments Screen* on page 95 for more information.

### 4.3.6 Access the Reimage Feature

Select the **Reimage** button to acquire a new image of a specimen already in the tissue filter chambers. Refer to <u>Reimage</u> on page 97 for more information.

#### 4.3.7 Close a Patient

Select the **Close Patient** button. Select **Yes** to the confirmation prompt. The system returns to the *Select Patient* screen and automatically saves to the configured output devices any images that were not sent previously.

# 4.4 Output Sets

The images are sent automatically to the output devices in the selected Output Set when a patient is closed.

# 4.4.1 Select an Output Set

Select an output device set like PACS, CAD devices, and printers from the Output Set drop-down list in the *Procedure* screen.



#### Note

Images are not sent if an Output Set is not selected.

# 4.4.2 Add or Edit an Output Set



#### Note

The configuration of Output Sets occurs during installation, but you can edit existing groups or add new groups.

To add a new Output Set:

- 1. Go to the Admin screen.
- 2. Select the **Manage Output Groups** button.
- 3. Select the **New** button, type the information, then select the output device(s).
- 4. Select the **Add** button. When the Update Successful message is displayed, select **OK**.
- 5. You can select any group to set as the default by selecting the **Set As Default** button.

### To edit an Output Set:

- 1. Go to the Admin screen.
- 2. Select the Manage Output Groups button.
- 3. Select the group to be edited.
- 4. Select the **Edit** button, then make the changes.
- 5. Select the **Save** button. When the Update Successful message is displayed, select **OK**.

# 4.5 On-Demand Outputs

The On-Demand Outputs are: Archive, Export, or Print. You can manually Archive, Export, or Print an image until the patient is closed. When you select an On-Demand Output button, you have the option to send the image to any of the configured Outputs.

### **4.5.1** Export

The export function sends data to a mapped drive such as a local hard drive, a USB device connected to the console, or a network drive.

- 1. Select the **Archive/Export** button. The images from the current procedure appear in the main window.
  - The archive function sends data to a PACS device.
- Touch one or more images to select the images to archive or export. Touch again to
  deselect an image. Use the Select All button to select all of the images for the current
  procedure. Use the Select Tagged button to select only the images marked with a
  green check mark.

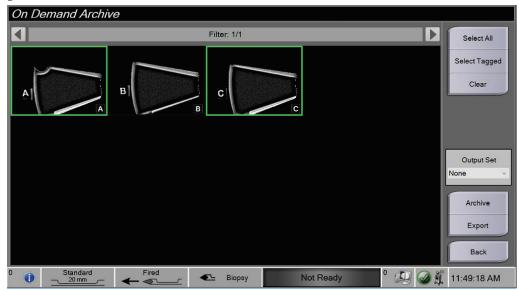


Figure 29: Two Images Selected for Archive or Export

- 3. Select the **Export** button. The Export dialog box opens.
- 4. In the Export dialog box, select the destination from the options in the drop-down list.
  - To make the patient-related information not known during the export, select the Anonymized check box.
  - To automatically eject the removable media storage device when the export is complete, select **Eject USB device after write**.
  - To select a folder on your local systems for storage of your selections and to select the Export Type for the image, select Advanced.
- 5. Select the **Start** button to export the images, or select the **Cancel** button to cancel the export.

#### 4.5.2 Archive

The archive function sends data to a PACS device.

- 1. Select the **Archive/Export** button. The images from the current procedure appear in the main window.
- Touch one or more images to select the images to archive or export. Touch again to
  deselect an image. Use the Select All button to select all of the images for the current
  procedure. Use the Select Tagged button to select only the images marked with a
  green check mark.

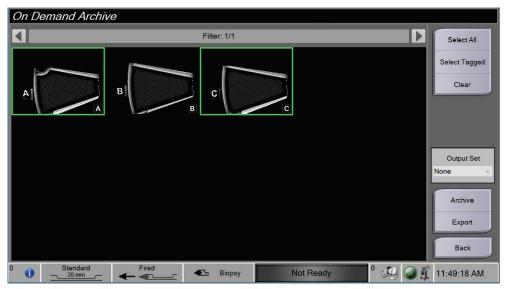


Figure 30: Two Images Selected for Archive or Export

- 3. Select the **Archive** button.
- 4. Select a storage device from an existing Output Group, or create an Output Group. (See *Output Sets* on page 55 for more information.)
- 5. Select the **Send** button to copy all selected images from the open case study to the selected device.



#### Note

Use the Manage Queue utility in the taskbar to review the archive status.

Standard 20 mm

Fired

#### 4.5.3 **Print** 6 7 4 5 Print 8 Tray: 1/1 Auto All 9 Auto Tagged 1 10 Reset III Properties Options 12 Printer Info Printer DryStar (1001) Size Print 13 2 Automatic 14 Paper Print Film Orientation Portrait 15 3 1/1

Figure 31: Example of the Print Screen

Biopsy

Standby

# Figure Legend

1.	Film format	Select one, two, or six images per film (number of tiles per film). A green rectangle indicates the position that the active image will have on the film.
2.	New film, Delete film, and Delete image from film	Select the "+" button to create a new film.  Select the "x" button to delete the current film selection.  Select the <b>Delete image from film</b> button to delete only the active image from the film.
3.	Step	When more than one film is being printed, use these arrows to step through the print preview of the film output.
4.	Print preview area	Shows a preview of the final film.
5.	Printer Options	Select the printer options. The Automatic setting will automatically determine a suitable size for the printed images.
6.	Patient information	Activate or deactivate the patient information display. The check mark indicates that the patient information display is active and patient information will appear on a film. Touching the <b>Patient Information</b> button will deactivate the patient information only on the current film for the same patient.

2:43:34 PM

7.	Image thumbnails	Choose the images to output by touching the image thumbnail. The selected image thumbnail is surrounded by a green box in the thumbnail area and the position of that image appears as a green box in the film format area.	
8.	Auto All	Automatically select all of the images for that procedure to be printed.	
9.	Auto Tagged	Automatically select the images that are tagged on the <i>Procedure</i> screen for that procedure to be printed, and automatically create a film or films for them.	
10.	Reset	Return the <i>Print</i> screen to its previous settings and remove any images from a film or films.	
11.	Properties	Open the <i>Properties</i> screen for selection of the printer defaults.	
12.	Printer Info	Show the printer IP address, AE Title, Port and capability for True Size print.	
13.	Print	Output the current films to the print preview area and start the printing process. To allow for subsequent printing and printing to another device, the images remain in the print preview until the user removes them.	
14.	Paper Print	Output the current films to the print preview area and start the process for printing to paper. To allow for subsequent printing and printing to another device, the images remain in the print preview until the user removes them.	
15.	Back	Cancel the print job and return to the <i>Procedure</i> screen. If print jobs have been created but not yet printed, the system will ask the user to confirm the cancelation.	

- 1. On the *Procedure* screen, select the **Print** button. The *Print* screen opens.
- 2. Select the film format from the Options area of the screen.
- 3. Select an image thumbnail from the right side of the screen.
- 4. Select the Print preview on the left side of the screen to put the selected image on the film.
- 5. To put other images on the film, repeat steps 3 and 4.
- 6. To print a different film format of the same images, select the **New Film** button and repeat steps 2 to 4.
- 7. Select the **Print** button or the **Paper Print** button to print your films.



#### Note

When you choose Paper Print, the Windows Print dialog box opens. Select your desired print and printer options.

## 4.6 Gain Calibration

Do the Gain Calibration monthly and when artifacts are seen.



#### Note

When the system shows the Gain Calibration message, do the calibration before you proceed with using the system. If you do the Gain Calibration at a different time, access the procedure through the *Admin* screen.

- 1. Select **Yes** to do the calibration now. If you select **No**, the system shows the *Select Patient* screen.
- 2. If you select to do the calibration now, you are prompted to select an Automatic Gain Calibration mode. Select **Yes** for Automatic Gain Calibration or **No** for Manual Gain Calibration.
- 3. Make sure that there are no items in the tissue filter drawer and that the surface of the x-ray imaging area is clean. Close the tissue filter drawer. Select **OK** to begin.
- 4. For Automatic Gain Calibration, the system takes four images.



#### Note

In the event that the system is unable to complete an Automatic Gain Calibration, the user can continue with Manual Gain Calibration. To continue calibration, use the manual x-ray function until four images are acquired. The system can complete Gain Calibration using the combination of automatically acquired and manually acquired images.

5. After four images have been acquired, select the **Complete Calibration** button to start the calibration. When the Gain Calibration is finished, a message about successful completion is displayed. Select **OK** to return to the previous screen.

#### For Manual Gain Calibration:

- 1. Make sure that there are no items in the tissue filter drawer and that the surface of the x-ray imaging area is clean. Close the tissue filter drawer. Press **OK** to begin.
- 2. Press the **X-ray** button.
- 3. Wait for the system to return to the Ready state.
- 4. Repeat steps 2 and 3 until you acquire four images.
- 5. After four images have been acquired, select the **Complete Calibration** button to start the calibration. When the Gain Calibration is finished, a message about successful completion is displayed. Select **OK** to return to the previous screen.

# Chapter 5 User Interface - Technologist Display

The system has five biopsy system modes controlled by the Technologist display: Standby, Test, Biopsy, Lavage, and Aspirate.

# 5.1 Technologist Display Screens and Modes

When the Brevera breast biopsy system powers on, it requires setup and a test cycle. Setup and testing are completed by following the prompts on the Technologist display.

## 5.1.1 **Setup**

After powering on the system, the *Setup* screen opens on the Technologist display. The system will automatically detect and confirm the connections for the device driver, biopsy needle, tissue filter assembly, and x-ray system. Follow the prompts to manually finish setup. The X-ray system will not show a Ready status until patient information is entered on the Imaging display.

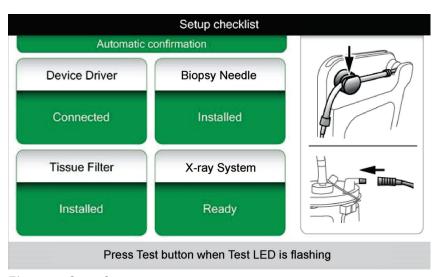


Figure 32: Setup Screen

#### 5.1.2 Test

After setup is successful, the system requires testing. The device driver, biopsy needle, and tissue filter must be connected and installed to run Test mode. It is not necessary for the X-ray system to be ready to run Test mode. When the system is in Test mode, it tests the vacuum, prompts the user to confirm saline flow at the tip of the needle, then tests the cutting cycle. The system cannot use the x-ray function in Test mode.

When the system is ready for testing, the green light above the **Test** button flashes. Press the **Test** button to begin the testing procedure. Follow the prompts to complete testing. The Arm and Fire functions can also be tested. After testing is complete, the system can be put into Standby mode.

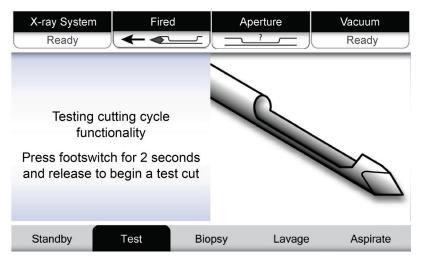


Figure 33: Test Screen



## Warning:

Keep fingers and hands away from the biopsy device during Arm and Fire testing.



## Warning:

Leave the protective sheath on the biopsy device tip during Arm and Fire testing.

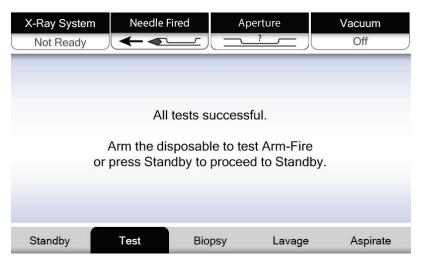


Figure 34: Test Screen - Arm and Fire

## 5.1.3 Standby

When the system is in Standby mode, the vacuum is off and, if a biopsy device is connected, the aperture on the needle is closed. The system cannot arm or fire the biopsy needle when it is in Standby mode. The user must change the console from Standby mode to Biopsy mode to use the biopsy device and x-ray function.

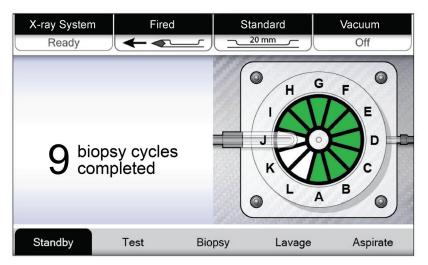


Figure 35: Standby Screen

## 5.1.4 Biopsy

When the system is in Biopsy mode, depressing the footswitch activates the vacuum and operation of the biopsy device. In Biopsy mode, the biopsy device can be armed and fired with the remote control. As part of the normal cutting cycle, the system performs a brief aspiration with each core acquisition. Removing pressure from the footswitch inactivates or stops the device *after* completion of the current cycle.

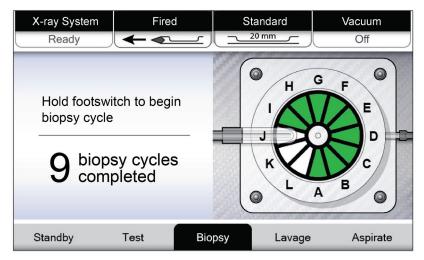


Figure 36: Biopsy Screen

# **5.1.5** Lavage

When the system is in Lavage mode, the vacuum system is activated and will pull saline through the system. The aperture on the needle is open during Lavage mode. Depressing the footswitch does not control the flow of saline in Lavage mode.

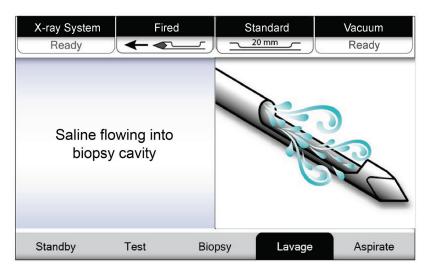


Figure 37: Lavage Screen

## 5.1.6 Aspirate

When the system is in Aspirate mode, the vacuum system is continuously on and the aperture on the needle is open. Aspirate mode allows for continuous vacuum to aspirate the biopsy cavity. Saline does not flow in Aspirate mode. Depressing the footswitch does not control the flow of air or the vacuum in Aspirate mode.

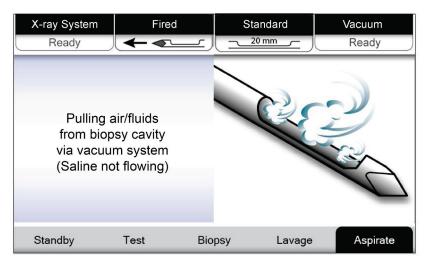


Figure 38: Aspirate Screen

# **Chapter 6 Imaging System Modes**

# 6.1 X-ray Mode

X-ray mode is the default mode of the Imaging system. To use the X-ray mode, start up the system and log in. Make sure the key switch on the imaging panel is in the unlocked position.

# 6.2 No X-ray Mode

The system also has a No X-ray mode that allows the user to perform a biopsy without taking any images. To enter No X-ray mode, select the **No X-ray** button on the *Startup* screen on the Imaging display.

When the system is in No X-ray mode, the tissue filter moves one position after each biopsy cycle and the number of biopsy cycles completed is shown on the Technologist display (same as in X-ray mode). Standby, Test, Biopsy, Lavage, and Aspirate modes operate in normal fashion. The user interface screens on the Imaging display are absent. In place of the X-ray image, a real-time video of the tissue filter is displayed.

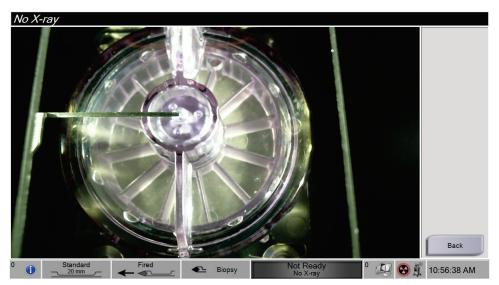


Figure 39: No X-ray Screen

# 6.3 Single Chamber Mode

The system has a specialized mode that enables the user to perform a biopsy without taking any images when the Imaging system cannot detect or index the tissue filter. The Single Chamber tissue filter does not move or change position during the biopsy acquisition cycle. The number of biopsy cycles/acquisitions completed is displayed on the Technologist display (same as the other system modes). Standby, Test, Biopsy, Lavage, and Aspirate modes operate in normal fashion.

To enter Single Chamber mode, select the System Status icon on the taskbar, then select **Single Chamber** from the menu.



Figure 40: Menu Option for Single Chamber Mode

In Single Chamber mode, only a Single Chamber tissue filter can be used. The system prompts the user to install the Single Chamber tissue filter.



#### **WARNING!**

Do not proceed with the biopsy until the single chamber tissue filter is installed. Failure to install the single chamber tissue filter can result in lost cores.



## Warning:

Using a twelve chamber tissue filter during single chamber mode causes all cores to collect in one chamber and can cause system damage.



Figure 41: Prompt for the Single Chamber Tissue Filter

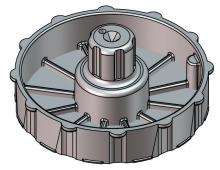


Figure 42: Single Chamber Tissue Filter

The Imaging display does not show the usual user interface screens. In place of the x-ray image, the Imaging display shows a real-time video of the tissue filter.

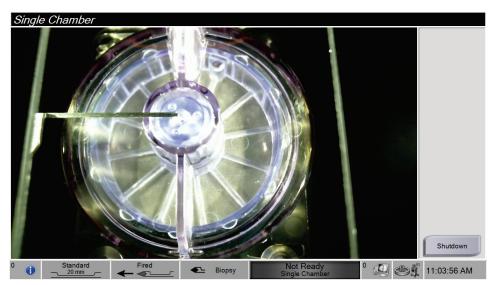


Figure 43: Single Chamber Video

# **Chapter 7 Biopsy**

# 7.1 Set Up the Console

- 1. Move the console into position and lock the casters.
- 2. Make sure the footswitch is connected.
- 3. Connect the console to power.
- 4. Connect the network cable to the Ethernet port (optional).
- 5. Unwind the footswitch and place it on the floor in front of the console.



#### Warning:

Position the footswitch and power cords to prevent accidental tripping.

6. Power on the console.



# Warning:

Do not place fingers on the metal parts of the device driver. These parts can move when the system powers on.



# 7.2 Prepare for Image Acquisition

Refer to <u>Select Patient Screen</u> on page 47 for detailed information on the screen components.

- 1. On the Windows 10 *Login* screen, log in with your user name and password.
- 2. On the *Startup* screen, select the **Patient List** button to go to the *Select Patient* screen.
- 3. Select a patient from the worklist on the *Select Patient* screen, or manually add a new patient (refer to *Add a Patient* on page 48).



4. On the *Procedure* screen, select the output devices.

Figure 44: Procedure Example

- 5. Confirm that the tissue filter drawer is closed and that the system is free of any error messages.
- 6. Turn the Key switch to the unlocked position.

# 7.3 Connect the Device Driver and the Biopsy Supplies

## **Suction Canister**

- 1. Attach the canister lid to the canister base.
- 2. Secure the large opening cap onto the large opening.
- 3. Place the suction canister in the canister holder on the console with the patient port pointing to the right.
- 4. Plug the vacuum line assembly into the top port labeled "VACUUM" on the suction canister lid.
- 5. Make sure that all connections are secure and sealed to avoid vacuum leaks.

#### **Device Driver**



#### Warning:

Do not place fingers on the metal parts of the device driver. These parts can move before a biopsy needle is installed.



- 1. Slide the device driver onto the device driver holder on the console until it clicks.
- 2. Confirm that the device driver cord is connected to the receptacle. The receptacle is on the left side of the console, near the saline pinch valve.



#### Note

The console beeps three times before resetting the position of the metal parts of the device driver.

- 3. Wait for the device driver to home before attaching the biopsy needle.
- 4. Press the **Check** button on the Technologist display if the device driver needs to be rehomed.

## **Biopsy Needle and Tubing**



## Warning:

Before use, inspect the protective packaging and needle to verify that neither has been damaged during shipment. If it appears that the packaging or needle has been compromised, do not use the needle.



#### Warning:

Do not connect the biopsy needle until the system is powered on and the device driver is homed and ready.

- 1. Place the sealed packaging for the biopsy needle on the console tray.
- Peel open the sterile packaging of the biopsy needle. If the needle starts to lift out of the tray, place your hand above the needle and hold the needle in place until the top of the packaging is removed.
- 3. Remove the biopsy needle from the tray, leaving the tubing and the tissue filter in the tray. Make sure the biopsy needle gears are fully forward, toward the needle tip.



#### Warning:


To maintain sterility, leave the protective sheath on the tip of the biopsy needle.

4. Hold the biopsy needle in one hand, with the gears facing down and the sheathed needle tip facing to the right. Line up the tab on the biopsy needle with the notch in the device driver.



## Warning:

Do not place fingers on the metal parts of the device driver. These parts can move before a biopsy needle is installed.

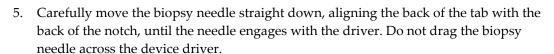

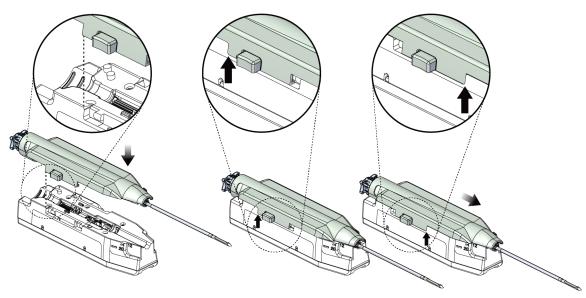


Figure 45: Connect the Biopsy Needle to the Device Driver



## Warning:

Do not place fingers on the device driver when system power is on. Parts can move unexpectedly.


6. Slide the biopsy needle forward (to the right) until it locks into position with a click.



#### Warning:

Make sure that the biopsy needle is fully latched onto the device driver.

7. Remove the clear vacuum tubing from the biopsy needle packaging.

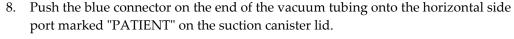

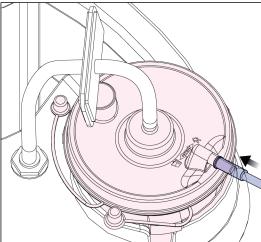


Figure 46: Connect the Tissue Filter Tubing to the Suction Canister

- 9. Get a saline bag and remove the protective cap. (A 250 cc saline bag is recommended.)
- 10. Remove the saline tubing from the biopsy needle packaging.
- 11. Insert the spike into the saline bag.



## **WARNING!**

Make sure you use an aseptic technique when spiking the saline bag to prevent contamination.

- 12. Place the saline bag on the saline bag hook on the left side of the console.
- 13. Insert the larger diameter section of the saline tubing into the saline pinch valve. Make sure the saline tubing is fully seated.

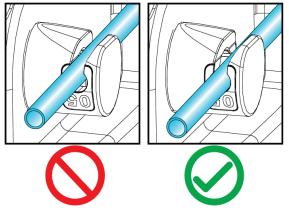


Figure 47: Placement of the Saline Tubing in the Saline Pinch Valve

#### **Tissue Filter**

- 1. Pull open the tissue filter drawer.
- 2. Remove the tissue filter assembly from the biopsy needle packaging.
- 3. Confirm that the tissue filter housing is securely in place. (If needed, refer to <u>Tissue Filter Components</u> on page 32 for component assembly instructions.)
- 4. Put the tissue filter assembly in the center of the tissue filter drawer. Align the notch on the left-hand side of the tissue filter with the tab in the drawer. Seat the tissue filter on the four locator pins. Keep the tissue filter housing cover on the tissue filter as shown in the following figure. (\*Component may appear different in markets where product is not yet available.)

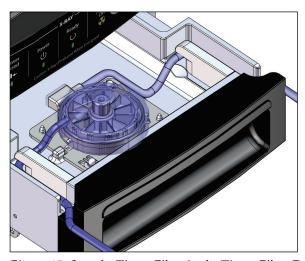


Figure 48: Seat the Tissue Filter in the Tissue Filter Drawer



#### Note

Each Brevera disposable biopsy needle includes a tissue filter cap. Save the tissue filter cap until the biopsy acquisition is complete. The tissue filter cap covers the tissue filter when the biopsy cores are placed in formalin.

- 5. Feed the gray tubing through the guide on the left side and feed the clear tubing through the guide on the right side of the drawer so that the drawer is able to close.
- 6. Close the tissue filter drawer.

# 7.4 Pre-Procedure System Check

- 1. With the console in the desired location and powered on, the default mode is Standby. In Standby mode, the green LED indicator flashes above the **Test** button.
- 2. Assemble the Brevera breast biopsy device. Refer to <u>Connect the Device Driver and the Biopsy Supplies</u> on page 71 for information on assembling the biopsy device.



#### Warning:

Leave the protective sheath on the biopsy device tip to maintain sterility.

- 3. Confirm that the suction canister and tissue filter are assembled and placed in the console. Confirm that the vacuum line assembly and tubing are correctly connected. See <u>Components, Controls, and Indicators</u> on page 23 for more information on individual component assembly and connections.
- 4. Confirm that the silicone section of the saline tubing line is properly positioned in the saline pinch valve.
- 5. Confirm that the saline spike is correctly positioned in the saline bag and that the saline bag is hanging from the saline hook. Confirm that the saline flows freely from the bag.
- 6. Press the **Test** button on the Technologist display to switch the mode to Test mode. Follow the prompts on the Technologist display. Test mode will prime the system and generate vacuum.
- 7. Visually verify the flow of saline into the aperture of the biopsy device. Press the **Check** button on the Technologist display to confirm the saline flow.



#### **WARNING!**

Make sure the saline flows into the biopsy device to prevent lost cores and patient injury.



# Warning:

Moving parts.

8. Step on the footswitch for two seconds and release the footswitch to initiate a test cutting cycle. Verify that the aperture opens and closes smoothly during the cutting test.



#### warning:

If the aperture does not open and close smoothly, remove the biopsy needle and install a new one.

- 9. After the test, the system gives two options: Go to Standby mode or test the Arm and Fire functions of the biopsy needle. (The console returns to Standby mode after the Arm and Fire test is completed.)
- 10. After completion of a successful test cycle, the system is ready for tissue acquisition.

# 7.5 Attach the Introducer to the Biopsy Device

- 1. With the system in Standby mode, remove the protective sheath from the biopsy needle.
- 2. Carefully slide the introducer over the needle of the biopsy device. Advance the hub of the introducer up to the device driver.

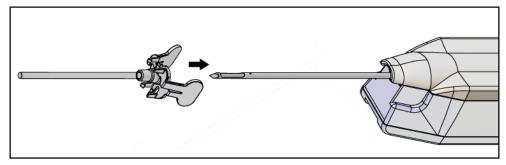


Figure 49: Slide the Introducer onto the Biopsy Device



## **WARNING!**

Make sure to use an aseptic technique when attaching the introducer to prevent contamination.

3. Orient the introducer to select a petite or standard aperture. For an aperture of 20 mm, the 20 mm mark on the introducer lines up with the long mark on the device driver. Rotate the introducer 180° for an aperture of 12 mm, where the 12 mm mark on the introducer lines up with the short mark on the device driver.

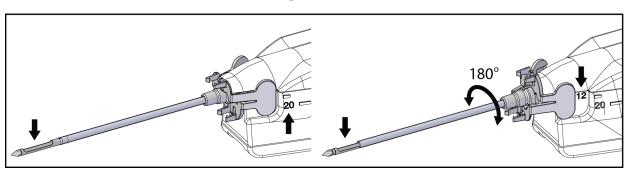


Figure 50: Use Standard or Petite Aperture

# 7.6 Attach a Biopsy Device Adapter to the Biopsy Guidance System

A biopsy device adapter is required to hold the biopsy device in place on a compatible biopsy guidance system. Attach the correct adapter to the biopsy guidance system.

Hologic offers biopsy device adapters for the Brevera breast biopsy system. Refer to the instructions for use provided with the Brevera biopsy needle and with the biopsy device adapter for additional information.



Figure 51: Biopsy Device Adapter



#### Warning:

Make sure the needle guide is installed correctly before use.



#### Note

Use only biopsy device adapters approved by Hologic.

# 7.6.1 Using the System with Stereotactic (STX) Guidance



#### Note

Complete training and understanding of your stereotactic guidance system should be provided by the vendor of your stereotactic system.

For additional training or questions on specific stereotactic applications, contact Product Support at 877-371-4372 or your local Hologic representative.

# 7.7 Attach the Biopsy Device to the Biopsy Device Adapter

- 1. Place the system in Biopsy mode.
- 2. Press the **Arm** button on the remote control for at least two seconds to arm the biopsy device.
- 3. Align the biopsy device with the biopsy device adapter.
- 4. Slide the biopsy device forward until the introducer on the distal end of the Brevera breast biopsy device engages with the biopsy device adapter and the release latch of the driver engages with the proximal end of the biopsy device adapter. There is an audible click when the pieces are properly engaged, and the biopsy device will no longer slide on the adapter.

# 7.8 Performing a Biopsy and Acquiring Real-time Images

Perform all steps per standard interventional technique.

- 1. Identify the targeted area and prepare the site as required.
- 2. Assemble the biopsy device and advance it to the desired coordinates provided by the imaging system.



## Warning:

If there is a problem advancing the needle into the breast, check the sharpness of the trocar needle tip. Replace the needle if necessary.



#### Note

X-ray is enabled when the drawer is closed. If the drawer is opened during an exposure, the exposure ends automatically.

- 3. Confirm that the system is ready to perform a biopsy procedure.
  - Confirm that the system is free of any error messages.
  - Confirm that all patient entry steps are complete and that the *Procedure* screen is displayed on the Imaging display.
  - Confirm the aperture size and that the biopsy device is armed.
  - Confirm that the Key switch is in the unlocked position.
  - Confirm that the system is in Biopsy mode.
- 4. Fire the biopsy device by pressing and holding the **Fire** button on the remote control for at least 2 seconds.
- 5. Administer anesthetic (optional).
  - To initiate automatic delivery of anesthetic: Attach a 10 cc needleless syringe of anesthetic to the Y-valve. Inject 1–2 cc of anesthetic via the needleless syringe.
  - To manually administer anesthetic to the area of interest before tissue acquisition: (the biopsy device has been fired, the footswitch has not been depressed) Manually inject anesthetic via syringe and rotate the thumbwheel to allow 360 degrees of access.
  - To manually administer anesthetic to the area of interest during tissue
    acquisition: (the biopsy device has been fired, the footswitch has been depressed)
    place the system in Standby mode to close the aperture. Manually inject
    anesthetic via syringe and rotate the thumbwheel to allow 360 degrees of access.
     Once complete, return the system to Biopsy mode to continue tissue acquisition.



#### Note

The device does not allow the Y-valve injection when the device is in the armed (prefire) position.

- 6. Begin tissue acquisition. Press down on the footswitch and hold it in the depressed position throughout the biopsy procedure.
  - Removing your foot from the footswitch will inactivate or stop the device **after** completing the current cycle and the x-ray for the core in that cutting cycle.
  - The system beeps to indicate that it has completed a cutting cycle. A tissue filter icon appears in the thumbnail area to confirm that the cutting cycle is complete and ready for x-ray.
- 7. Acquire a real-time image.
  - The core handling system beeps to indicate that it is taking an x-ray. This beep is significantly softer in volume and tone than the beep that signals the end of the cutting cycle.
  - The Imaging display updates as each x-ray is available. A letter appears to indicate which core sample the image is associated with. The letters A–L in each image indicate the tissue filter chamber.
  - Touch the thumbnail image on the Imaging display to review the image at full size on the Imaging display.
- 8. Rotate the aperture on the outer cannula of the biopsy device by turning the thumbwheel to the next desired position immediately after the system beeps to indicate that it has completed a cutting cycle. The numbers in the window on the biopsy device and the line on the thumbwheel clock dial indicate the position of the aperture.
- 9. Continue tissue and image acquisition until the desired target area has been sampled and imaged. The system beeps two times after twelve cores are taken.
- 10. Review the images and make necessary enhancements. Refer to <u>Review the Images</u> on page 91.

#### 7.8.1 When the Twelve Chambers of the Tissue Filter Contain Cores

The Brevera breast biopsy system with CorLumina imaging technology can acquire more than twelve biopsy cores per tissue filter. For these cases, when each of the twelve chambers in the tissue filter contains a biopsy core, the Brevera breast biopsy system presents two options: to continue using the existing filter or to continue with a new filter.

#### **Continue with the Existing Tissue Filter**

- 1. Release the footswitch.
- 2. Press down on the footswitch and hold it in the depressed position throughout the biopsy procedure.
- 3. Continue to acquire biopsy specimens and to view their x-rays on the Imaging display.
- 4. At the end of the tissue acquisition or after the twenty-fourth core specimen, whichever comes first, continue with the instructions for *Lavage and Aspirate* on page 83.

#### Continue with a New Tissue Filter

- 1. Release the footswitch.
- 2. Open the tissue filter drawer.
- 3. Remove the tissue filter cover from the tissue filter assembly.
- 4. Place a dry tissue filter cap on the tissue filter. (The tissue filter cap is provided with the biopsy needle.)
- 5. Remove the used filter from the tissue filter assembly and the tissue filter drawer.
- 6. Place the used tissue filter in a specimen jar on the surface of the formalin.



#### Note

The specimen jar should have a minimum inner diameter of 6.1 cm (2.4 inches) and a minimum height of 5.7 cm (2.25 inches) in order to fit a Brevera tissue filter.

- 7. Slowly submerge the tissue filter to allow the formalin to enter the chambers and air to escape. (Do not allow the tissue filter to float on top of the formalin.) Prepare the specimens for pathology per facility policy.
- 8. Install a new tissue filter.
- 9. Replace the cover on the tissue filter assembly, and close the tissue filter drawer. The system is ready when the indicators on the Imaging display return to green.
- 10. Select **Yes** on the Imaging display when asked if a new filter was inserted into the drawer.
- 11. Depress the footswitch.
- 12. Continue to acquire biopsy specimens and to view their x-rays on the Imaging display.
- 13. At the end of tissue acquisition or after the twenty-fourth core specimen, whichever comes first, continue with the instructions for *Lavage and Aspirate* on page 83.

## 7.8.2 Lavage and Aspirate

#### Lavage

Select Lavage mode to irrigate the cavity and to clear the biopsy device of tissue. The system will turn the vacuum on, open the aperture, open the saline pinch valve, and close the aspiration valve in order to push saline through the aperture. It is not necessary to depress the footswitch.

#### **Aspirate**

Select Aspirate mode to vacuum the cavity. The system will turn the vacuum on, open the aperture, close the saline pinch valve, and open the aspiration valve in order to push air through the aperture. It is not necessary to depress the footswitch.

## 7.8.3 Conclude Tissue Acquisition

- 1. Release the footswitch. Select Standby mode to close the aperture prior to removal of the biopsy device from the biopsy cavity.
- 2. Remove the biopsy needle and device driver as a unit from the biopsy device adapter while leaving the introducer in place for marker deployment. (Use the release latch on the back of the device driver to detach the biopsy device from the adapter.)
- 3. After removing needle from the breast, use the aspirate mode to clear the needle aperture.
- 4. Press the buttons on the sides of the biopsy needle to disengage it from the device driver. Detach the biopsy needle from the device driver and discard in a sharps container. (The tubing and tissue filter are removed from the console later.)



#### Warning:

Do not place fingers on the metal parts of the device driver. These parts can move during system use.



#### Note

The console beeps three times before resetting the position of the metal parts of the device driver.

5. Slide the device driver onto the device driver holder on the console until it clicks.

## 7.8.4 Deploy a Biopsy Site Marker

To deploy a biopsy site marker, refer to the biopsy site marker instructions for use.



#### Note

Make sure that the correct biopsy marker is selected for the needle length used.

After deploying an optional biopsy site marker, retract the introducer and stereotactic adapter from the patient (refer to *Remove the Biopsy Device Adapter* on page 84).

## 7.8.5 Remove the Biopsy Device Adapter

- 1. After deploying an optional biopsy site marker, pull back the introducer and the biopsy site marker deployment device from the breast as one unit by moving the biopsy device adapter away from the patient.
- 2. After everything is retracted from the patient, remove the introducer and biopsy site marker deployment device as a unit from the biopsy device adapter by pinching the wing tabs on the introducer. Slide the introducer back from the adapter. Dispose of the introducer and marker deployment device per facility policy.
- 3. Refer to the instructions for use provided with the biopsy device adapter for additional information regarding removal of the adapter from the biopsy guidance system.
- 4. Immediately following the procedure, properly clean and disinfect the biopsy device adapter as recommended in the section <u>Maintenance, Cleaning, and Disinfecting</u> on page 107.



#### Note

Complete training and understanding of your biopsy guidance system should be provided by your biopsy guidance system vendor.

## 7.9 Conclude the Procedure

- 1. Confirm that the console is in Standby mode.
- 2. Open the tissue filter drawer.
- 3. Remove the tissue filter cover from the tissue filter assembly.
- 4. Place a dry tissue filter cap on the tissue filter. (The tissue filter cap is provided with the biopsy needle.) To install, gently lower the tissue filter cap onto the tissue filter until the two tabs snap into place.



# Warning:

Make sure that the protrusions in the center of the tissue filter cap align with the grooves on the tissue filter spindle. Make sure both tabs are fully latched.

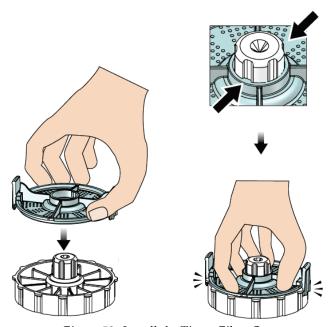


Figure 52: Install the Tissue Filter Cap

- 5. Remove the used filter from the tissue filter assembly and the tissue filter drawer.
- 6. Place the used tissue filter in a specimen jar on the surface of the formalin.



#### Note

The specimen jar should have a minimum inner diameter of 6.1 cm (2.4 inches) and a minimum height of 5.7 cm (2.25 inches) in order to fit a Brevera tissue filter.

7. Slowly submerge the tissue filter to allow the formalin to enter the chambers and air to escape. (Do not allow the tissue filter to float on top of the formalin.) Prepare the specimens for pathology per facility policy.

- 8. Replace the cover on the tissue filter assembly.
- 9. Release the tissue filter assembly from the locator pins and temporarily leave in the drawer.
- 10. Remove the suction tubing from the tubing guides on the two sides of the drawer.
- 11. Remove the suction tubing from the suction canister.
- 12. Remove the saline tubing from the saline pinch valve. Remove the spike from the saline bag.
- 13. Gather all the tubing and the tissue filter assembly and discard in the sharps container with the previously discarded biopsy needle.
- 14. Dispose of the saline bag per facility policy.
- 15. Dispose the needle guide per facility policy.
- 16. Close the tissue filter drawer.
- 17. Disconnect the vacuum line assembly from the suction canister, and close the suction canister ports. Dispose of the suction canister per facility policy for biohazardous waste.
- 18. After logout and shutdown, perform the recommended cleaning and disinfection procedures. Refer to *Maintenance, Cleaning, and Disinfecting* on page 107 for more information.

# 7.10 How to Change the Biopsy Needle Between Procedures

Follow these instructions to change the biopsy needle for a new procedure.

- 1. Confirm that the console is in Standby mode.
- 2. Press the buttons on the sides of the biopsy needle to disengage it from the device driver. Detach the needle from the driver and place it in a sharps container.



## Warning:

Do not place fingers on the metal parts of the device driver. These parts can move before a biopsy needle is installed.



#### Note

The console beeps three times before resetting the position of the metal parts of the device driver.

- 3. Slide the device driver onto the device driver holder on the console until it clicks.
- 4. Wait for the device driver to automatically home in preparation for a new biopsy needle.
- 5. Follow the instructions in <u>Conclude the Procedure</u> on page 85 to disconnect and remove the biopsy supplies.
- 6. Install the new biopsy supplies and the new biopsy needle. Refer to <u>Connect the Device Driver and the Biopsy Supplies</u> on page 71 for detailed instructions.

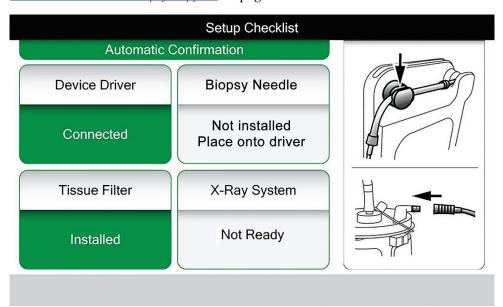


Figure 53: System Recognizes When the Device Driver Is Ready

# 7.11 How to Change the Biopsy Needle During Troubleshooting

Follow these instructions to change the biopsy needle to troubleshoot issues during setup and testing.

- 1. Slide the device driver onto the device driver holder on the console until it clicks.
- 2. Press the buttons on the sides of the biopsy needle to disengage it from the device driver. Detach the needle from the driver and place it in a sharps container.

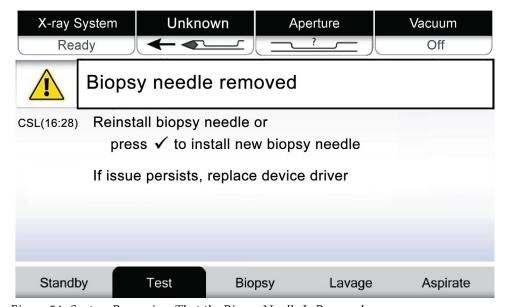


Figure 54: System Recognizes That the Biopsy Needle Is Removed

- 3. Refer to the instructions in *Conclude the Procedure* on page 85 to disconnect and remove the biopsy supplies.
- 4. Press the **Check** button on the Technologist display to home the device driver.



#### Warning:

Do not place fingers on the metal parts of the device driver. These parts can move before a biopsy needle is installed.



#### Note

The console beeps three times before resetting the position of the metal parts of the device driver.

- 5. Wait for the device driver to home.
- 6. Install the new biopsy supplies and the new biopsy needle. Refer to <u>Connect the Device Driver and the Biopsy Supplies</u> on page 71 for detailed instructions.

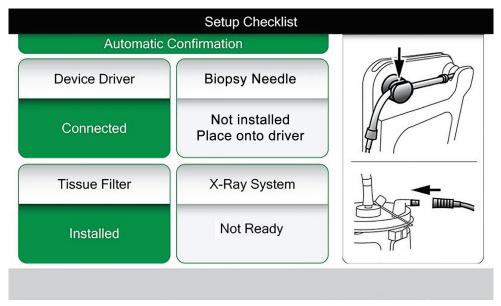


Figure 55: System Recognizes When the Device Driver Is Ready

# **Chapter 8 Images**

## 8.1 Introduction

After you make an exposure, the acquired image displays and is accepted automatically. A thumbnail image appears on the screen, with the corresponding letter designation for the tissue filter chamber. Use the image functions to tag, review, enhance, and annotate the images or reimage the cores.

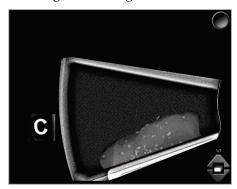


Figure 56: An Acquired Image

# 8.2 Review the Images



Figure 57: Image Tools in the Procedure Screen

Select a thumbnail image.

- A preview image shows in the large window on the screen.
- Use the image tools (refer to <u>Image Enhancement Tools</u> on page 95) to enhance the whole image or areas of interest in the image.

# 8.2.1 Tag an Image

The Tag tool is the round button in the upper right corner of the image. Pressing this button selects or deselects the active image. Tagged images show a green check mark in the upper right of the image and the image thumbnail. The check mark is burned into the tagged images that are exported, archived, or printed.



Figure 58: An Example Tagged Image

## Tag an Image for Service

You can tag an image to allow Hologic service personnel to access it easily.

1. In the right panel of the *Procedure* screen, select the **Tag for Service** button.

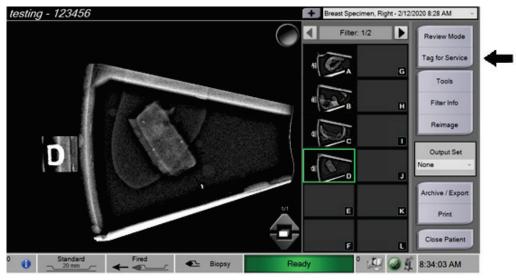


Figure 59: Tag for Service Button on the Procedure Screen

2. On the *Tag for Service* screen, select the image of concern.



Figure 60: Tag for Service Screen

- 3. Enter a description of the concern.
- 4. Select the **Back** button when complete.

## 8.2.2 Review Mode Screen

The *Review Mode* screen allows for a quick review of the images. The *Review Mode* screen no longer shows an active image and instead shows larger thumbnails of the images. The images can be quickly tagged by selecting them from the group of images instead of tagging them individually from the *Procedure* screen. The *Review Mode* screen includes a Tag All and a Clear All feature.

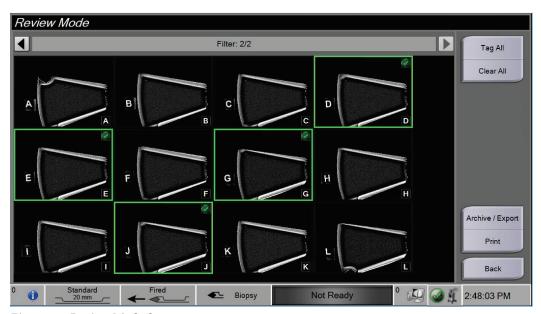


Figure 61: Review Mode Screen

### 8.2.3 Image Enhancement Tools

When you select the **Tools** button, the Image Enhancement Tools bar displays on the active image. Select the tool icon to enable it. The active tool appears to be pressed into the tools bar.

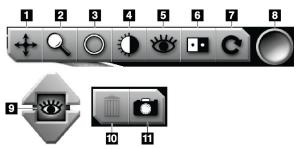


Figure 62: Image Enhancement Tools

#### Figure Legend

- 1. The Pan tool is used to move the view to a different portion of the image by dragging.
- 2. The Zoom tool magnifies the whole image. Drag up to zoom in and make the image larger. Drag down to zoom out and make the image look smaller.
- 3. The Circle tool is used to draw a circle on the image. Drag up or down to make the circle larger or smaller. Once the circle is placed its size cannot be changed, but it can be moved by dragging or deleted by double-tapping.
- 4. The Window/Level tool is used to change the brightness and contrast.
- 5. The Enhanced Visualization tool adjusts the image sharpness.
- 6. The Invert tool changes blacks to white and whites to black.
- 7. The Reset tool returns an image to the original state. Any annotations, panning, zooming, etc. are removed.
- 8. The Tag tool selects a particular image or images. Tagged images show a green check mark in the upper right of the thumbnail image. The check mark is burned into the tagged images that are exported, archived, or printed off the system.
- 9. The Enhancement Level is displayed when the Enhanced Visualization tool is active. The level of enhancement can be adjusted from greater (5) to lesser (1) in real time.
- 10. The **Delete** button deletes a snapshot.
- 11. The **Camera** button captures a snapshot of the image display area and adds the snapshot to the current procedure.



#### Note

The Enhanced Visualization tool is inactive for snapshot images.

#### 8.2.4 Filter Info and Comments Screen

From the *Procedure* screen, click on the **Filter Info** button to make notes or comments about image content. One set of comments can be added per tissue filter. Comments are saved with the images from that tissue filter and are exported with the image or images.

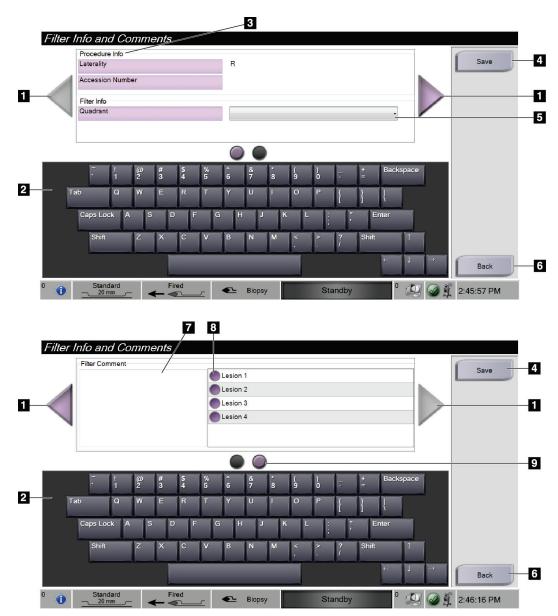


Figure 63: Filter Info and Comments Screens

#### **Figure Legend**

- 1. Scroll between the two screens using the arrows.
- 2. Use the on-screen keyboard to type, edit, and remove Filter Comments.
- 3. The Procedure Info is the information already in the system for the procedure. It cannot be edited on the *Filter Info and Comments* screen.
- 4. Select the Save button to save the Filter Info and Comments.
- 5. Indicate the quadrant for the specimens in that particular tissue filter.
- 6. Select the **Back** button to exit the *Filter Info and Comments* screen without saving and return to the previous screen. If there are unsaved changes, a prompt displays the option to save or continue without saving the comments.
- 7. Enter, edit, and remove additional Filter Comments using the on-screen keyboard.
- 8. Select from a predefined list describing the specimens in that particular tissue filter.
- 9. Use these circles to scroll between the two screens. A red exclamation point in a circle indicates that there is an error that needs to be fixed on that screen.

## 8.2.5 Reimage

Select the **Reimage** button to acquire a new image of the core shown in the active image.

# 8.3 Send the Images to the Output Devices

You can send the images to output devices or use the Export function to copy images to a temporary storage device. Refer to *Output Sets* on page 55 for instructions.

# **Chapter 9 System Administration Interface**

## 9.1 Admin Screen

This section describes the functions available in the *Admin* screen. To access this screen:

- 1. Log in to the system.
- 2. In the *Startup* screen, select the **Patient List** button to go to the *Select Patient* screen.
- 3. In the right panel of the *Select Patient* screen, select the **Admin** button. The *Admin* screen opens.



#### Note

The figures in this User Guide show the *Admin* screen with Manager level access. A Radiological Tech (Primary User) has access to some functions on the *Admin* screen. A Manager system user has access to more functions on the *Admin* screen. A Hologic Service User has access to all functions on the *Admin* screen.

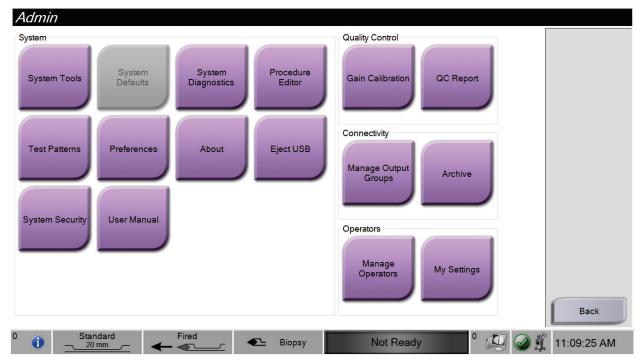


Figure 64: Admin Screen

	Table 2: Admin Screen Functions			
Section	Screen Function			
System	System Tools	The interface for service for the configuration of and identification of problems.		
	System Diagnostics	Shows the status of all subsystems including the x-ray detector and the camera above the tissue filter drawer.		
	Procedure Editor	Add, edit, or delete the procedures.		
	Test Patterns	Select and send the test patterns to output devices.		
	Preferences	Set the system preferences.		
	About	Tells about the system.		
	Eject USB	Eject a media storage device connected to the USB port.		
	System Security	Access Account Security settings and the Windows OS Tools.		
	User Manual	Access the User Guide and other user documents.		
Quality Control	Gain Calibration	Select the Gain Calibration procedure.		
	QC Report	View information on the most recent Gain Calibration.		
Connectivity	Manage Output Groups	Add, delete, or edit output groups.		
	Archive	Export or archive one or more procedures for one or more patients all at one time.		
Operators	Manage Operators	Add, delete, or change Operator information.		
	My Settings	Change the information for the current Operator. All user permission levels allow a user to edit fields in the My Settings screens.		

You must have permissions to access all features. The permission level controls the functions you can change.

### 9.2 About Screen

The *About* screen provides information about the machine, such as system level, IP address, and serial number. This type of data can be useful when working with Hologic to configure the system or resolve a system issue.

The screen is accessed in two ways:

- From the *Select Patient* screen select the **System Status** icon (on the taskbar) then select **About...**
- From the *Admin* screen select **About** (in the System group)

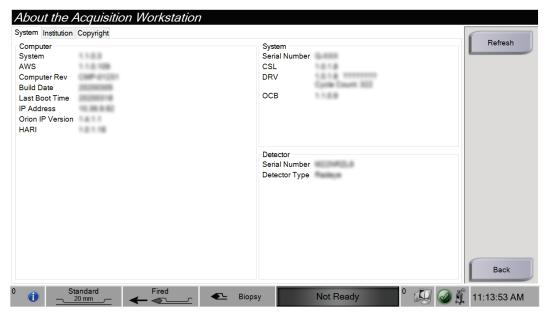


Figure 65: System Tab of the About Screen

There are three tabs on the *About* screen:

- System Tab (default) lists system configuration information
- **Institution Tab** lists the name and address of the organization assigned to this machine
- Copyright Tab lists the copyrights of Hologic and third-party software installed on this machine

## 9.3 Change the User Language Preference

- 1. In the Operators group of the *Admin* screen, select the **My Settings** button.
- 2. Go to the second page. In the Locale field, select a language from the drop-down list.
- 3. Select **Save**, then select **OK in** the *Update Successful* message. The user interface changes to the selected language.

# 9.4 Change the Keyboard Layout

The system supports both the QWERTY keyboard and the AZERTY keyboard.

- 1. In the Operators group of the *Admin* screen, select the **My Settings** button.
- 2. Go to the second page. In the **Keyboard** field, select a keyboard from the drop-down list.
- 3. Select **Save**, then select **OK in** the *Update Successful* message. The user interface changes to the selected keyboard.

#### **Letters with Accents**

Both keyboard layouts support accented letters. To display the accent options for a letter, press and hold (long press) the on-screen key.



Figure 66: Accented Letters on the On-screen Keyboard

# 9.5 Account Security Settings

Users with Windows Administrator permissions can change some of the Windows 10 password and account policy settings. To update the settings:

- 1. Log in as a Manager.
- 2. Go to the *Admin* screen.
- 3. In the Operators group of the *Admin* screen, select the **System Security** button.
- 4. The *System Security* screen opens. Select the **Account Security** button.
- 5. Make the desired changes, then select **Save**.



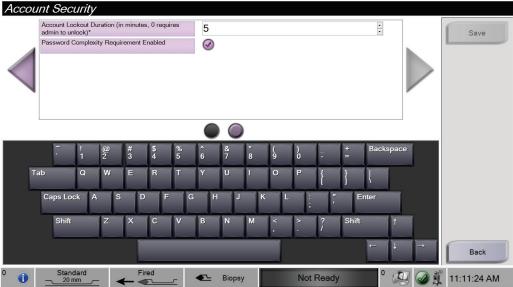


Figure 67: Account Security Screens

# 9.6 System Tools

The Managers (and Hologic Service) can access the System Tools function. This function contains the configuration information about the system.

To access the System Tools function:

- 1. Log on as a Manager.
- 2. On the *Startup* screen, select the **Patient List** button to go to the *Select Patient* screen.
- 3. In the right panel of the *Select Patient* screen, select the **Admin** button.
- 4. From the System group of the *Admin* screen, select the **System Tools** button.

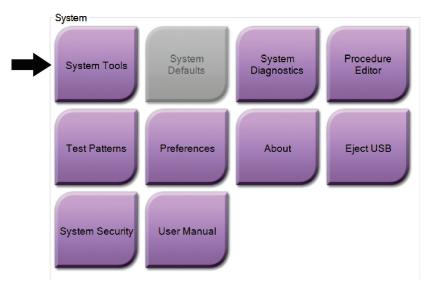


Figure 68: System Tools Button

#### **System Tools Screen** 9.6.1

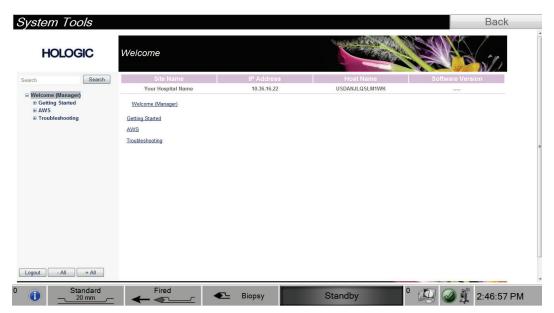


Figure 69: System Tools Screen

*Table 3: The System Tools* 

Section	Description		
Getting Started	About	The introduction to the service tool.	
	FAQ	List of common questions.	
	Glossary	List of terms and descriptions.	
	Platform	List of directories, software version numbers, and system software statistics.	
	Shortcuts	List of Windows® shortcuts.	
AWS	Connectivity	List of Installed Devices.	
(Acquisition Workstation)	Film and Image Information	Set of tools and options related to film and images.	
, vonstation,	Licensing	List of installed licenses.	
	User Interface	Change the options in the Software application.	
	Internationalization	Select the local language and culture.	
Troubleshooting	Acquisition Workstation	Allows for download of images.	
	Computer	Reboot or Shutdown; Current IP address; Change Host name.	
	Log	Change the event record options.	
	Backups	Control the backups for the system.	

# Chapter 10 Maintenance, Cleaning, and Disinfecting



#### Warning:

Do not perform maintenance, cleaning procedures, or disinfection while the system is being used on a patient.

### 10.1 General Information

### 10.1.1 For General Cleaning

This procedure includes cleaning the unit and the surfaces that the specimens do not contact.

Use a lint-free cloth and apply a diluted, mild detergent to clean components and surfaces. To remove more difficult soils, use a soft bristle brush as needed.



#### Caution:

Use the least possible amount of cleaning fluids. The fluids must not flow or run.

## 10.1.2 For Disinfecting

To disinfect the system's components and surfaces, Hologic recommends the following disinfecting solutions:

- 10% chlorine bleach solution and water with one part commercially available chlorine bleach solution (normally 5.25% chlorine and 94.75% water) and nine parts water. Mix this solution daily for best results.
- Commercially available isopropyl alcohol solution (70% isopropyl alcohol by volume, not diluted)
- Commercially available wipes containing less than or equal to 2% quaternary ammonium compounds by weight in water.

Apply disinfecting solutions to the system using a clean, lint-free cloth or wipe. Wipe the system with the wet cloth, observing appropriate contact (wet) times. Do not spray cleaning solutions directly on the console or other parts of the Brevera breast biopsy system.



#### Caution:

To prevent damage to the electronic components, do not use disinfectant sprays on the system.

### 10.1.3 To Prevent Possible Injury or Equipment Damage

Do not use a corrosive solvent, abrasive detergent, or polish. Select a cleaning/disinfecting agent that does not damage the plastics, aluminum, or carbon fiber.

Do not use strong detergents, abrasive cleaners, high alcohol concentration, or methanol at any concentration.

Do not expose equipment parts to steam or high temperature sterilization.

Do not let liquids enter the internal parts of the equipment. Do not directly apply cleaning sprays or liquids to the equipment. Always use a clean, lint-free cloth and apply the spray or liquid to the cloth. If liquid enters the system, disconnect the electrical supply and examine the system before returning it to use.



#### Caution:

Wrong cleaning methods can damage the equipment, decrease imaging performance, or increase the risk of electrical shock.

Always follow instructions from the manufacturer of the product you use for cleaning and disinfection. The instructions include the directions and precautions for the application and contact time, storage, wash requirements, protective clothing, shelf life, and disposal. Follow the instructions and use the product in the most safe and effective method.

# 10.2 Component Specific Cleaning and Disinfection



#### **WARNING!**

The console must be disconnected from electrical power source prior to cleaning. Failure to do so may cause electrical shock or death.



#### **WARNING!**

Do not immerse the console in water. Immersion in water will cause damage and may cause electrical shock or death.

#### 10.2.1 Clean and Disinfect the Console Exterior Surfaces

- Disconnect the console from the electrical power source.
- After each use, clean the console using a soft, damp cloth and mild detergent. Wipe dry.
- After cleaning, use a pre-moistened disinfectant wipe or apply a disinfectant solution
  to a clean cloth. Wipe all surfaces with the wipe or wet cloth, observing appropriate
  contact (wet) times. Refer to <u>General Information</u> on page 107 for recommended
  disinfecting solutions.
- Do not spray cleaning or disinfectant solutions directly on the console or other parts of the Brevera breast biopsy system.

#### 10.2.2 Clean and Disinfect the Device Driver and Remote Control

- Disconnect the console from the electrical power source.
- After each use, clean the reusable device driver using a soft, damp cloth and mild detergent. Wipe dry.
- After cleaning, use a pre-moistened disinfectant wipe or apply a disinfectant solution
  to a clean cloth. Wipe all surfaces with the wipe or wet cloth, observing appropriate
  contact (wet) times. Refer to <u>General Information</u> on page 107 for recommended
  disinfecting solutions.
- Do not spray cleaning or disinfectant solutions directly on the reusable device driver, the remote control, or other parts of the Brevera breast biopsy system.
- Do not immerse the reusable device driver in liquid.

#### 10.2.3 Clean and Disinfect the Tissue Filter Drawer

- Disconnect the console from the electrical power source.
- After each use, clean the tissue filter drawer using a soft, damp cloth and mild detergent. Wipe dry.
- After cleaning, use a pre-moistened disinfectant wipe or apply a disinfectant solution to a clean cloth. Wipe all surfaces with the wipe or wet cloth, observing appropriate contact (wet) times. Refer to <u>General Information</u> on page 107 for recommended disinfecting solutions.
- Do not spray cleaning or disinfectant solutions directly in the drawer or other parts of the Brevera breast biopsy system.

### 10.2.4 Clean the Imaging Display

- Disconnect the console from the electrical power source.
- Use care when you clean the outer surface of the display area.
- Always use a clean, soft, lint-free cloth to clean the display area. Microfiber cloths, available at most stores, are recommended.
- Use a pre-moistened disinfectant wipe or apply disinfectant solution to a clean cloth.
  Wipe all surfaces with the pre-moistened wipe or wet cloth, observing appropriate
  contact (wet) times. Refer to <u>General Information</u> on page 107 for recommended
  disinfecting solutions.
- Strong chemicals and abrasives can damage the display window and should never be used.
- Never apply pressure to the display window.
- Never use a spray or flow a liquid onto the display.
- Never use steel wool.
- Never use a sponge with abrasives.



#### Note

There are many commercially available products to clean LCD displays. Any product that is free of the ingredients or abrasives described above and used according to the directions of the manufacturer can be safely used.



#### Note

There are commercially available disposable monitor drape products to drape LCD touch screen displays. Always follow instructions from the manufacturer of the product you use.

## 10.2.5 Clean the Technologist Display

- Disconnect the console from the electrical power source.
- Do not touch the display when working with specimens.
- Use care when you clean the outer surface of the display area.
- Always use a clean, soft, lint-free cloth to clean the display area. Microfiber cloths, available at most stores, are recommended.
- To clean the button area of the technologist display, use a pre-moistened disinfectant wipe or apply a disinfectant solution to a clean cloth. Wipe all surfaces with the pre-moistened wipe or wet cloth, observing appropriate contact (wet) times. Refer to *General Information* on page 107 for recommended disinfecting solutions.
- Strong chemicals and abrasives can damage the display window and should never be used.
- Never apply pressure to the display window.
- Never use a detergent with fluorides, ammonia, alcohol, bleach, or abrasives to clean the display window.
- Never use a spray or flow a liquid onto the display.
- Never use steel wool.
- Never use a sponge with abrasives.



#### Note

There are many commercially available products to clean LCD displays. Any product that is free of the ingredients or abrasives described above and used according to the directions of the manufacturer can be safely used.

#### 10.2.6 Clean the Footswitch



### **WARNING!**

Do not immerse the footswitch in water. Immersion in water will cause damage and may cause electrical shock or death.

- 1. Disconnect the footswitch from the console.
- 2. Periodically clean the footswitch using a soft, damp cloth and mild detergent.
- 3. Wipe the footswitch dry.

# 10.2.7 Cleaning Schedule

Action	Recommended Frequency	
Action	Each Use	As Required
Clean and disinfect the console unit	✓	
Clean and disinfect the device driver	✓	
Clean and disinfect the tissue filter drawer interior	✓	
Clean and disinfect the tissue filter drawer handle and the Imaging panel	✓	
Clean and disinfect the Imaging display screen	✓	
Clean and disinfect the Technologist display screen	✓	
Clean the footswitch		✓

### 10.3 Preventive Maintenance

Every twelve months, the preventive maintenance service is to be performed only by Hologic Technical Services or a Hologic-certified technician.

Contact Hologic Technical Support or your local Hologic representative for more information on preventive maintenance and biomedical training.

### 10.3.1 Vacuum Line Assembly

### Inspect the Vacuum Line Assembly

- 1. Once a week, visually inspect the tubing for any fluid or moisture.
- 2. Visually inspect the white side of the in-line hydrophobic filter for any discoloration.
- 3. If the tubing is wet or the filter is discolored, replace the vacuum line assembly.

### Replace the Vacuum Line Assembly

- 1. Power off the system.
- 2. Locate the white lock collar where the vacuum line assembly attaches to the console.
- 3. Lift up the vacuum line assembly approximately 3 mm (1/8 inch) to expose a gap between the white collar and the base of the black barb.
- 4. Hold down the white collar and pull the vacuum line assembly up and out of the connector. Discard as biohazardous waste.
- Press a new vacuum line assembly into the vacuum line assembly connector.
- Pull up to check that the new vacuum line assembly is locked into the connector. The vacuum line assembly will have a small amount of vertical movement but should not pull free of the connector.



#### Note

Do not twist the vacuum line assembly as you pull up.

- 7. Power on the system.
- 8. Plug the vacuum inlet line using your thumb or finger.
- 9. Press the **Test** button on the console.
- 10. Check that the system displays an indicator that the vacuum system is functioning correctly.
- 11. Power off the system if desired.

#### 10.3.2 Device Driver

#### Inspect the Device Driver

- 1. Once a week, visually inspect the cable and connectors. The cable should not be cracked or damaged.
- 2. Once a week, visually inspect the area where the biopsy needle connects with the device driver.
- 3. If the device driver or its cable appears worn or damaged, replace the device driver.

#### Replace the Device Driver

- 1. Power off the system.
- 2. Completely unwrap the device driver cord and remove it from the cord management plate.
- 3. Unplug the device driver from the console.
- 4. Using a new device driver, insert the cord into the receptacle. (The red dots align at the top when attached correctly.)
- 5. Wrap the cord around the cord management plate.
- 6. Power on the system. The console beeps, and the *Setup* screen opens on the Technologist display.
- 7. Install a suction canister, biopsy needle, saline, and tissue filter assembly as if performing a biopsy.
- 8. Check that the system displays an indicator that the device driver, biopsy needle, and tissue filter are connected and installed. Complete the Setup check list as prompted.
- 9. When the system is ready for Test mode, the green **Test** button will flash. Press the **Test** button to begin the testing procedure.
- 10. Follow the prompts on the Technologist display until testing is complete.
- 11. When testing is successful, the system goes into Standby mode.
- 12. Power off the system if desired.
- 13. To return the device driver, contact your local Hologic representative or refer to the "Important Replacement Information" and "Customer Instructions for Returning Brevera Biopsy Driver" instructions.

#### 10.3.3 Footswitch

#### Inspect the Footswitch Cord

- 1. Once a week, visually inspect the cord and connector. The cord should not be cracked or damaged.
- 2. Verify that the footswitch cord is wrapped counter-clockwise (CCW) direction around the cord management plate.
- 3. If the footswitch or its cord appears worn or damaged, replace the footswitch.

#### Replace the Footswitch

- 1. Power off the system.
- 2. Completely unwrap the footswitch cord and remove it from the cord management plate.
- 3. Unplug the footswitch cord from the console.
- 4. Using a new footswitch, plug the cord into the console.
- 5. Wrap the footswitch cord counter-clockwise around the cord management plate.
- 6. Power on the system. The console beeps, and the *Setup* screen opens on the Technologist display. Install a suction canister, biopsy needle, saline, and tissue filter assembly as if performing a biopsy.
- 7. Check that the system displays an indicator that the device driver, biopsy needle, and tissue filter are connected and installed. Complete the Setup check list as prompted.
- 8. When the system is ready for Test mode, the green **Test** button will flash. Press the **Test** button to begin the testing procedure.
- 9. Follow the prompts on the Technologist display until testing is complete.
- 10. When testing is successful, the system goes into Standby mode.
- 11. Power off the system if desired.
- 12. Contact your local Hologic representative for product return instructions.

#### 10.3.4 Power Cord

#### Inspect the Power Cord

- 1. Once every quarter, visually inspect the power cord for cuts, damage to the cover, or strain relief.
- 2. If the power cord appears damaged, contact Hologic for replacement of the power cord.

# 10.3.5 Preventive Maintenance Schedule for the User

Table 4: Recommended Maintenance Schedule for the User

	Recommended Frequency			
Action	Weekly	Monthly	Quarterly	Annually
Inspect cable for the device driver	✓			
Inspect the footswitch cord	✓			
Inspect the vacuum line assembly	<b>√</b>			
Perform Gain Calibration		✓		
Inspect the power cord			✓	
Preventive maintenance by Hologic Technical Service				✓

# 10.3.6 Preventive Maintenance Schedule for the Service Engineer

Table 5: Service Engineer Preventive Maintenance

Maintanana Tarl Description	Recommended Frequency		
Maintenance Task Description	Semiannually	Annually	
Obtain previous copy of PM		✓	
Check with customer for any questions or concerns that they may have		✓	
Check error logs		✓	
Upgrade to current software / firmware configuration if under warranty / contract		✓	
Check covers and drawers for mechanical integrity, missing hardware, clean vents		✓	
Input wiring inspected for safety and integrity		✓	
Inspect all ground wiring for integrity		✓	
Inspect specimen drawer for integrity and sensing		✓	
Verify operation of all interlocks and bypass functions		✓	
Verify exposure parameters		✓	
Check motor accuracy		✓	
Evaluate image quality		✓	
Verify completion of Gain Calibration		✓	
Perform backup timer procedure		✓	
Clear error log		✓	
Perform monitor check		✓	
Test the current leakage on the system		✓	
Test the resistance on the power cord		✓	

# **Appendix A System Specifications**

# **A.1** Product Measurements

### A.1.1 Console

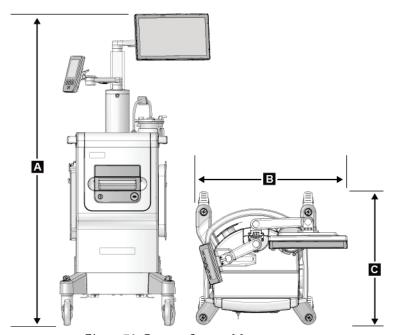


Figure 70: Brevera System Measurements

~66 cm Hg at sea level (~26 in. Hg)

 A. Height
 1647 mm (64.8 in.) nominal

 B. Width
 766 mm (30.2 in.) nominal

 C. Depth
 630 mm (24.8 in.) nominal

 Weight
 90.7 kg (200 lbs.) nominal

Vacuum Generated

### A.1.2 Device Driver

Height 4 cm (1.6 in.) nominal

Width 5.6 cm (2.2 in.) nominal

Depth 19.6 cm (7.7 in.) nominal

Weight 949.7 g (33.5 oz.) with remote nominal

### A.1.3 Biopsy Needle

Height 3 cm (1.2 in.) nominal

Width 5.6 cm (2.2 in.) nominal

Depth 22.4 cm (8.8 in.) nominal

Weight 269.3 g (9.5 oz.) with tubing set nominal

# A.2 Operation and Storage Environment

## A.2.1 General Conditions for Operation

This equipment is designed for safe and effective operation under the following conditions for Indoor Use:

Altitude 3000 meters

Degree of Pollution N/A

Temperature Range  $10 \, ^{\circ}\text{C} \, (50 \, ^{\circ}\text{F}) \text{ to } 40 \, ^{\circ}\text{C} \, (104 \, ^{\circ}\text{F})$ 

Relative Humidity Range 20% to 75% without condensing moisture

# A.2.2 Storage Environment

Temperature Range  $-10 \, ^{\circ}\text{C} \, (14 \, ^{\circ}\text{F}) \, to \, 60 \, ^{\circ}\text{C} \, (140 \, ^{\circ}\text{F})$ 

 $Maximum\ rate\ of\ temperature$  N/A

change

Relative Humidity Range 10% to 80% without condensing moisture

# A.3 Electrical Input

The system is designed for temporary overvoltages on the Mains supply.

Brevera 100

Line Voltage Range100 V-120 VLine Current10 AmpsOperating Frequency50 Hz-60 Hz

Brevera 200

Line Voltage Range220 V-240 VLine Current5 AmpsOperating Frequency50 Hz-60 Hz

Line Connection Supplied Cord

Duty Cycle Continuous

# A.4 Classification



 $Type\ of\ protection\ against\ electric\ shock:\ Class\ I$ 

Degree of protection against electric shock: Type BF

Degree of protection against ingress of water:

 $System\ (unless\ specifically\ identified):\ Ordinary$ 

Footswitch: IPX 8

Mode of operation: Continuous

APPLIED PARTS Disposable biopsy needle (when mounted on the reusable device driver)

### A.5 Console Technical Information

### A.5.1 General Information

Operating System Windows 10
Hard Disk Drive Capacity 500 GB

Imaging Display 38.1 cm (15 in.) 1366 x 768 wide screen, touch screen monitor

Technologist Display 18 cm (7 in.) 800 x 480 screen

### A.5.2 Network Environment

Network Interface 100/1000 BASE-T Ethernet, Wi-Fi 802.11 b/g/n (optional)

# A.6 X-Ray Tube Technical Information

Source-to-Image Distance (SID) 7.20" (18.3 cm)

Focal Spot 50  $\mu$ m

Tube Voltage 20-35 kVp

Tube Beam Angle 25°/+0°, -2.5°

Beam Output Angle 15°  $\pm$  1°

Anode Material Tungsten
X-Ray Window Beryllium

Physical Mass 6.5 lbs (maximum)

Duty Cycle 1:4 (Continuous)

Input Power +24 volts DC,  $\pm$  10%, 2.4 A (nominal)

Reference Axis Integrated automatic tissue handling system with fixed imaging

window and reference axis.

Reference Axis Position Accuracy  $\ \ zero$ -angle  $\ reference \pm 1.25 \ degrees$ 

### A.6.1 X-Ray Generator

kV Range 20-35 kVp adjustable

kV Accuracy  $\pm 1 \%$ 

mA 1 ma Fixed

mA Accuracy ± 2 %

# A.6.2 Disposal



Equipment to be disposed in compliance with European Directive 2002/96/EC on Waste Electrical and Electronic Equipment (WEEE).

# A.7 Imaging System Technical Information

## A.7.1 Image Receptor

Active Imaging Area 33 x 24.9 mm Min.

Pixel Pitch 20 µm Max.

Modulation Transfer Function >60% @ 2 lp/mm

 $<\!\!20\% @ 6 lp/mm$ 

Detector Dynamic Range 2000:1
Detector Digitization 12 bits

# Appendix B Troubleshooting

# **B.1** Error Recovery and Troubleshooting

Most faults and alert messages are cleared without result to your workflow. Follow the instructions on the screen or fix the condition then clear the status from the Taskbar. Some conditions require a system restart or indicate that more action is necessary (for example, to call Hologic Technical Support). This appendix describes the message categories and your actions to return the system to normal operation. If errors repeat, contact Hologic Technical Support.

# **B.2** Types of Messages and Alert Messages

#### B.2.1 Fault Levels

There are five fault levels: Warning, Minor, Major, Critical, and Alerts.

#### **Warning Faults**

The Warning faults are not displayed to the user. These faults are recorded in the log files.

Warning fault designs:

- Resolve through the software or communications commands.
- Do not cancel an exposure that is in progress.
- Do not prevent the start of a new exposure.

#### **Minor Faults**

Minor fault designs:

- Resolve through the software or communications commands.
- Do not cancel an exposure that is in progress.
- Require a response before a new exposure can start.

#### **Major Faults**

Major fault designs:

- Resolve through the software or communications commands.
- Cancel an exposure that is in progress.
- Prevent the start of a new exposure.

### **Critical Faults**

Critical fault designs:

- Not resolved through the software or communications commands.
- Cancel an exposure that is in progress.
- Prevent the start of a new exposure.

### **Alert Messages**

The Alert Messages are routine messages that can prevent an exposure. An Alert message remains active until the required action is complete or the condition no longer exists.

## **B.2.2 System Messages**

Select the system status icon on the Taskbar to get information about the cause and correction of a system problem. When you correct the problem, the System Message area displays a Ready status.

# **B.3** System Restart

- To restart the computer system after a critical fault, select the **Reboot** button.
- If you select the **Exit** button, a message advises that this option shuts down only the computer.

# **B.4** Troubleshooting During Setup

Table 6: Resolving Possible Issues During Setup

Problem	Possible Cause	Suggested Action
Console will not turn on.	Power cord is not plugged in at the console and/or at the power outlet.	Check power cord for proper engagement to console and power outlet.
"Vacuum system not at required pressure" message	Protective sheath is not fully on the tip of the biopsy needle cannula.	Reinstall the protective sheath.
	Suction canister lid is not sealed onto the canister.	Properly seal the lid of the suction canister.
	Suction canister is cracked.	Replace the suction canister with a new one.
	The large port on the suction canister lid is not capped with the large plug.	Cap the large port with the large plug.

Table 6: Resolving Possible Issues During Setup

Problem	Possible Cause	Suggested Action
	The biopsy needle suction fitting is not connected to the suction canister lid.	Connect the suction fitting to the canister.
	The biopsy needle fitting is not connected to the correct port on the suction canister lid.	Connect the biopsy needle suction fitting to the horizontal port marked "PATIENT" on the suction canister lid.
	The console vacuum line is not connected to the suction canister.	Connect the console vacuum line to the port marked "VACUUM" on the suction canister lid.
	The spike on the biopsy needle tubing is not inserted into the saline bag.	Install the spike into the saline bag.
	Defective biopsy needle.	Retain the biopsy needle. Record the lot number and contact your local Hologic representative.  Retry with a new biopsy needle.
Biopsy device connection error.	The biopsy needle is not properly connected or difficult to attach to the device driver.	Properly assemble the biopsy needle to the device driver.
		Make sure that the device driver is homed by pressing the button on the Technologist Display when prompted to do so. Make sure that the needle gears are pushed all the way forward before reinstalling the needle.
		If this does not work, try a new needle.
		If this does not work, try a new driver.
	The device driver is not properly connected to the console.	With the biopsy needle removed, disconnect and reconnect the cord on the device driver into the receptacle on the console.
	Defective biopsy needle.	Retain the biopsy needle. Record the lot number and contact your local Hologic representative.  Retry with a new biopsy needle.
	Defective device driver.	Retain the device driver. Record the lot number and contact your local Hologic representative.  Retry with a new reusable device driver.

Table 6: Resolving Possible Issues During Setup

Problem	Possible Cause	Suggested Action
Biopsy device makes unusual noises during test	The biopsy needle is not properly connected to the device driver.	Properly attach the biopsy needle to the device driver.
mode.		Make sure that the device driver is homed by
		pressing the button on the Technologist Display when prompted to do so. Make sure that the needle gears are pushed all the way forward before reinstalling the needle.
		If this does not work, try a new needle.
		If this does not work, try a new driver.
"Device driver loss of communication"	The biopsy needle is not properly connected to the device driver.	Properly attach the biopsy needle to the device driver.
message "Device driver error" message		With the biopsy needle removed, disconnect and reconnect the cord on the device driver into the receptacle on the console. Repeat 2 or 3 times until the device driver is homed.
		Make sure that the needle gears are pushed all the way forward before reinstalling the needle.
		If this does not work, try a new driver.

#### **Troubleshooting During a Procedure B.5**

Table 7: Resolving Possible Issues During a Procedure		
Problem	Possible Cause	Suggested Action
System does not respond to the Arm Button.	Incorrect mode selected.	Select Biopsy Mode.
	Remote malfunction.	Replace the device driver.
System does not respond to the footswitch.	Incorrect mode selected.	Select Biopsy Mode.
	Footswitch malfunction.	Replace the footswitch.
Biopsy device makes unusual noises during procedure.	The biopsy needle is not properly connected to the device driver.	Remove the biopsy device from the patient and disassemble it (refer to <i>Conclude Tissue Acquisition</i> on page 83). Place the device driver on the console.  If cores are present, place the cap on the tissue filter, and place the tissue filter in formalin.  Make sure that the device driver is homed by pressing the button on the Technologist Display when prompted to do so. Install a new needle. Make sure the needle is properly attached to the device driver.
		If this does not work, try a new driver.

Table 7: Resolving Possible Issues During a Procedure		
Problem	Possible Cause	Suggested Action
"Device driver loss of communication" message	The biopsy needle is not properly connected to the device driver.	If the biopsy needle is clean: Remove the biopsy needle. Disconnect and reconnect the cord on the device driver into the receptacle on the console.
"Device driver error" message		Make sure that the needle gears are pushed all the way forward before reinstalling the needle.  If this does not work, try a new driver.
		If the biopsy needle is not clean: Remove the biopsy device from the patient and disassemble it (refer to <i>Conclude Tissue Acquisition</i> on page 83). Place the device driver on the console.
		If cores are present, place the cap on the tissue filter, and place the tissue filter in formalin.
		Make sure that the device driver is homed by pressing the button on the Technologist Display when prompted to do so. Install a new needle. Make sure the needle is properly attached to the device driver.
		If this does not work, try a new driver.

Table 7: Resolving Possible Issues During a Procedure				
Problem Possible Cause		Suggested Action		
Poor quality biopsy cores or no cores.	The biopsy needle fitting is not connected to the port on the suction canister lid. The biopsy needle fitting is not connected to the correct port on the suction canister lid.	Connect the biopsy needle suction fitting to the horizontal port marked "PATIENT" on the suction canister lid.		
	Suction canister is cracked.	Replace the suction canister with a new one.		
	Reduced or no vacuum at the tip of the biopsy needle cannula.	Retain the biopsy needle. Record the lot number and contact your local Hologic representative.		
	Biopsy needle occlusion.	Retain the biopsy needle. Record the lot number and contact your local Hologic representative.  Retry with a new biopsy needle.		
	The tissue filter is occluded by blood.	Replace with a new tissue filter or retry with a new biopsy needle.		
	The biopsy needle tubing is twisted.	Reposition the biopsy needle tubing. Select Aspirate mode, Select Biopsy mode, and retry the biopsy.		
No image is acquired for sample. Thumbnail marked with red chamber letter.	Detector readout failure.	After tissue acquisition is complete, select the chamber and reimage.		
		If this does not work: Restart the detector. Select the <b>System Status</b> icon on the taskbar. Select System Diagnostics from the pop-up menu.		
		Select the <b>Restart Detector</b> button.		
		Select the <b>Back</b> button.		
		Use the Reimage feature to capture an image for the chamber that previously failed.		

Table 7: Resolving Possible Issues During a Procedure			
Problem Possible Cause		Suggested Action	
"Unable to position tissue filter" message "Indexing core	Broken belt or indexer motor fault. Unable to move the tissue filter for any reason.	Enter Single Chamber mode: Select the <b>System Status</b> icon on the taskbar. Select <b>Single Chamber</b> from the pop-up menu.	
handler fault" message		Follow the prompts on the Technologist display.	
"No filter found" message	Filter not in tissue filter drawer.	Insert the tissue filter into the tissue filter drawer.	
"No filter found" message	Broken belt.  No magnet in filter assembly.  Magnet sensor is broken.	Replace the tissue filter.  If the error does not clear, enter Single Chamber mode:	
"Filter sensor fault" message		Select the <b>System Status</b> icon on the taskbar. Select <b>Single Chamber</b> from the pop-up menu.  Follow the prompts on the Technologist display.	

# Appendix C Brevera Compatible Components and Accessories

## C.1 Hologic Factory-Verified Compatible Components and Accessories

The Brevera system is compatible with the following components and accessories:

Catalog Number	Description
BREV100	Brevera 100 System
BREV200	Brevera 200 System
BREVDRV	Driver
BREVDISP09	Standard Stereotactic Biopsy Needle
BREVADPTR	Adapter
BREVADPTRM	Adapter for MammoTest
BREVSTYLBRKT	Stylet Bracket
EVIVA CALIBRATE 13CM	Calibration Handpiece 13 cm
EVIVA NG HOLDER	Needle Guide Holder for MammoTest
BREVADPTRG	Adapter for GE Senographe® Stereotaxy
EVIVA BUSHING GE	Bushing for GE Senographe® Stereotaxy
BREVTF01	Single Chamber Tissue Filter
BREVTF12	12 Chamber Tissue Filter
EVIVA_NG09L	Needle Guide
EVIVA_NG09R	Needle Guide
ATEC CANISTER	Suction canister with lid
SMark-Eviva-13	Titanium Biopsy Site Marker
SMark-Eviva-2S-13	Titanium Biopsy Site Marker
SMark-E13-ss1	Stainless Steel Biopsy Site Marker
SMark-E13-ss2	Stainless Steel Biopsy Site Marker
SMark-E13-ss3	Stainless Steel Biopsy Site Marker
TriMark-Eviva-13	Titanium Biopsy Site Marker
TriMark-Eviva-2S-13	Titanium Biopsy Site Marker

### **Appendix D Owner's Record**

## D.1 Brevera Breast Biopsy System with CorLumina Imaging Technology

The serial number of the Brevera breast biopsy system console is located on the panel on the left side of the console. Record this number in the space provided below. Refer to this serial number whenever you contact your local Hologic representative regarding your Brevera system.

REF	Model Number: BREV100 or BREV200
SN	Serial Number:

### **Glossary of Terms**

#### **Annotations**

Graphic or text marks on an image to show an area of interest

#### DICOM

Digital Imaging and Communications in Medicine

#### **Image Receptor**

Captures and digitizes the x-ray images

#### **PACS**

Picture Archiving and Communications System. A computer and network system that transmits and archives digital medical images.

#### **ROI**

Region of Interest

#### SID

Source to Image Distance

Index	• 87
A	change the biopsy needle during troubleshooting • 88
about screen • 101	buttons • 40
access	С
about screen • 101	
image enhancement tools • 95	cable connections • 34
adobe requirements • 6	capabilities, system • 3
alert messages • 126, See faults	cart • See console
aspirate mode ● 83	change the biopsy needle between procedures • 87
attach • See connections	change the biopsy needle during troubleshooting  • 88
В	cleaning • 107
hionov	cleaning schedule • 112
biopsy	cleaning the components • 109
aspirate mode • 83	disinfecting • 107
biopsy mode • 61	general cleaning • 107
conclude the procedure • 83, 85	preventing possible injury or equipment
deploying a biopsy site marker • 84 lavage mode • 83	damage • 108
~	clinical procedures • 71
mode control and arming/firing the device • 40	complaints, product • 5
<del>-</del> -	compliance • 20, 21
performing a biopsy and acquiring real-time images • 80	compliance requirements • 20
performing a biopsy using stereotactic	components • 13, 23
guidance • 79	biopsy device components • 29
test cycle • 77	biopsy needle components • 30
biopsy device	compatible components and accessories • 133
attach the biopsy device to the biopsy device	imaging cabinet components • 27
adapter • 79	suction canister components • 28
attach the introducer to the biopsy device • 78	system components • 23
biopsy device components • 29	tissue filter components • 30, 32, 76
connect the biopsy device to the console • 35	tissue filter drawer components • 27
biopsy device adapter	conclude the procedure • 83, 85
attach a biopsy device adapter to the biopsy	conditions ● 120
guidance system • 79	configuration information of system, where to find
remove the biopsy device adapter • 84	it in About screen • 101
biopsy device adapter components	connections • 33, 34
attach the biopsy device to the biopsy device	attach a biopsy device adapter to the biopsy
adapter • 79	guidance system • 79
retracting the stereotactic adapter • 84	attach the biopsy device to the biopsy device
biopsy needle • 120	adapter • 79
biopsy needle components • 30	attach the introducer to the biopsy device • 78
biopsy needle connections • 36, 72	biopsy needle connections • 36, 72
Diopsy ficcule conficctions • 30, 72	cable connections • 34

connect the biopsy device to the console • 35 connect the device driver and the biopsy supplies • 72 device driver and remote control connections	error messages • See faults error recovery and troubleshooting • 125 essential performance • 2
• 35, 72	F
footswitch connection • 35 suction canister connections • 36, 72 system connections • 38 tissue filter connections • 38, 72 console • 119 cleaning the components • 109 connect the biopsy device to the console • 35 system components • 23 system connections • 33 contraindications • 1 controls and indicators • 39 imaging panel controls and indicators • 39 mode control and arming/firing the device • 40	faults • 125  alert messages • 126, See faults  critical faults • 126  error recovery and troubleshooting • 125  major faults • 125  minor faults • 125  system messages • 126  warning faults • 125  filter • See tissue filter, See tissue filter drawer  other functions of the filter criteria • 51  footswitch • 35, 115  footswitch connection • 35
taskbar • 45	
critical faults • 126	general cleaning • 107
cybersecurity statement • 6	general conditions for operation • 120
D	glossary warnings, cautions, and notes, defined • 11
definitions	I
warnings, cautions, and notes, defined • 11 deploying a biopsy site marker • 84 device driver • 120    biopsy needle connections • 36, 72    device driver and remote control connections         • 35, 72 disinfecting • 107 display    imaging display • 45    technologist display • 40, 61 disposal • 123    conclude the procedure • 83, 85	images image enhancement tools • 95 image receptor • 123 performing a biopsy and acquiring real-time images • 80 prepare for image acquisition • 71 send the images to the output devices • 97 imaging cabinet components • 27 imaging display • 45 imaging panel controls and indicators • 39 imaging system imaging panel controls and indicators • 39
E	imaging system modes • 67 imaging system technical information • 123
electrical input • 121 environment general conditions for operation • 120 network environment • 122 storage environment • 120	indications for use • 1 Institution tab of About screen, description • 101 intended use • 1 interlocks • 20 international symbols • 6

L	power		
	power button • 23		
lavage mode • 83	power connections • 34		
Licensing tab of About screen, description • 101	reboot ◆ 126		
M	system restart • 126		
1 <b>V1</b>	power cord ● 35, 115		
maintenance • 107, See cleaning	pre-procedure system check • 77		
maintenance schedule • 116	preventive maintenance • 113		
preventive maintenance • 113	preventing possible injury or equipment		
major faults • 125	damage • 108		
manuals, copies • 6	procedures		
messages • 125, See faults	change the biopsy needle between procedures		
system messages • 126	• 87		
minor faults • 125	conclude the procedure • 83, 85		
modes • 61	pre-procedure system check • 77		
aspirate mode • 83	troubleshooting during a procedure • 129		
biopsy mode • 61	0		
imaging system modes ● 67	Q		
lavage mode • 83	quality control requirements • 20		
mode control and arming/firing the device •			
40	R		
no x-ray mode ● 67	reboot • 126		
single chamber mode • 68	recovery • 125, See faults, See troubleshooting		
standby mode • 61	system restart • 126		
technologist display modes • 61	release level of system, where to find it • 101		
test cycle ● 77	remove the biopsy device adapter • 84		
test mode • 61	remove the propsy device adapter 1 04		
x-ray mode • 67	S		
N	and other trade are a state of the state of		
1	safety information • 14 interlocks • 20		
network connections • 34			
network environment • 122	warnings and cautions • 14 screens • 45		
no x-ray mode • 67	about screen • 101		
	imaging display • 45		
O	technologist display • 40, 61		
outputs	send the images to the output devices • 97		
send the images to the output devices • 97	serial number • 135		
overview, system • 13	single chamber mode • 68		
owner's record • 135	standby mode • 61		
	stereotactic (STX) guidance		
P	attach a biopsy device adapter to the biopsy		
nationt filter agreen	guidance system • 79		
patient filter screen	performing a biopsy using stereotactic		
other functions of the filter criteria • 51	guidance • 79		

using the system with stereotactic (STX)	tissue filter connections • 38, 72
guidance • 79	tissue filter handling • 85
storage environment • 120	when the chambers of the tissue filter contain
suction canister • 28, 36, 72	cores ● 82
suction canister components • 28	tissue filter drawer • 27
suction canister connections • 36, 72	tissue filter drawer components • 27
vacuum line ● 36	tools
symbols • 6	image enhancement tools • 95
system	tools bar • 95
imaging panel controls and indicators • 39	troubleshooting • 125
network connections • 34	change the biopsy needle during
overview, system • 13	troubleshooting • 88
power connections • 34	error recovery and troubleshooting • 125
pre-procedure system check • 77	system restart • 126
reboot • 126	troubleshooting during a procedure • 129
system capabilities • 3	troubleshooting during setup • 126
system components • 23	• • •
system configuration information, where to	U
find it in About screen • 101	Hairus Donies Identifier (IDI) tab of About
system connections • 33	Unique Device Identifier (UDI) tab of About
system description • 13	screen, description • 101
system messages • 126	user interface
system modes • 61	imaging display • 45
system restart • 126	technologist display • 40, 61
system specifications • 119	user profiles • 4
system status • 45	V
System tab of About screen, description • 101	·
using the system with stereotactic (STX)	vacuum canister • See suction canister
guidance • 79	vacuum line • 36
· ·	version level of system, where to find it • 101
T	W
taskbar ● 45	**
	warning faults • 125
technical information • 119	warnings and cautions • 14
imaging system technical information • 123 workstation technical information • 122	warnings, cautions, and notes, defined • 11
	workstation technical information • 122
x-ray tube technical information • 122	•
technologist display • 40, 61	X
technologist display modes • 61	v rav
test cycle • 77	x-ray no x-ray mode • 67
test mode • 61	x-ray generator • 123
tissue filter • 30, 32, 76	x-ray mode • 67
continue with a new tissue filter • 82	x-ray tube technical information • 122
continue with the existing tissue filter • 82	x-ray tube technical information ▼ 122
single chamber mode • 68	
tissue filter components • 30, 32, 76	

## **HOLOGIC®**

Hologic, Inc. 250 Campus Drive

Marlborough, MA 01752 USA 1-800-442-9892 • 1-508-263-2900

AU Sponsor Hologic Pty Ltd (Australia and New Zealand)

Suite 302, Level 3 2 Lyon Park Road

Macquarie Park, NSW 2113

Tel: 02 9888 8000

EC REP Hologic BV

Da Vincilaan 5 1930 Zaventem

Belgium

Tel: +32 2 711 46 80 Fax: +32 2 725 20 87

