Stereotactic Adapter and Needle Guide Holder

Cleaning Instructions

Please follow these instructions when cleaning Hologic® Stereotactic Adapters and Needle Guide Holders:

1. Upon completion of biopsy procedure, **immediately** rinse with **warm** water.
2. Spray adapter with a cleaner.
3. Allow the necessary time for the chosen cleaner to disinfect the adapter.
4. Rinse with **warm** water.
5. Wipe dry with cloth or paper towel.
6. **For ATEC® STX adapters only**, cycle hub retainer clamps, main clamp cock and fire adapter.
7. **For ATEC STX adapters only**, re-clean device if any component does not perform properly.

Monthly Recommendation - Completely submerge adapter assembly in warm soapy water for 30 minutes to break up and remove any possible cleaning solution build-up. Rinse with warm running water while exercising all moving components to remove any remaining build-up and allow adapter to air dry before storing.

Caution:

Some cleaning products may damage the exposed components.

The following cleaning products are **NOT RECOMMENDED** as they may damage the exposed components:

- Bleach based cleaning agents.
- Hydrogen Peroxide.
- Any cleaners/disinfectants with a pH of less than 4 or greater than 10.

****Please contact the Hologic product support with any questions****

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