


10 Review the image.

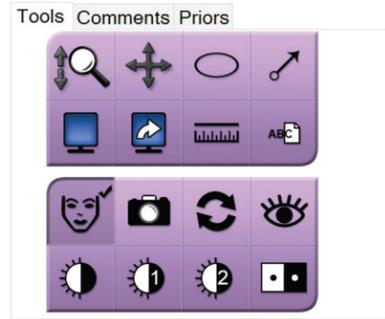


Faxitron™
Trident® HD
Specimen Radiography System

11


Archive or export the images then close the patient.



12 Remove the specimen tray from the imaging cabinet. Follow facility policy for specimen handling.

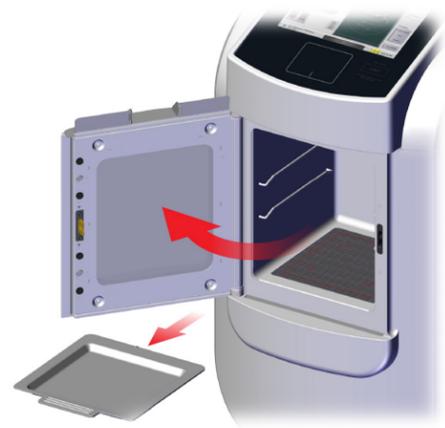

13 Turn off the system.

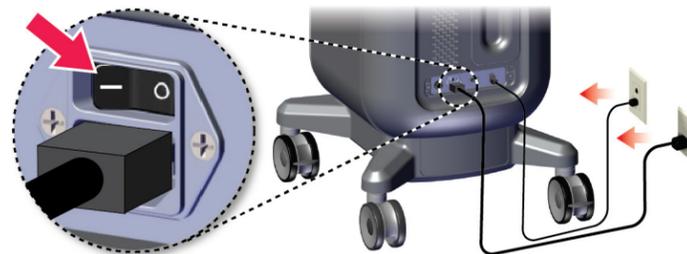

Faxitron™
Trident® HD
Specimen Radiography System

*Quick
Reference
Guide*



- 1 Image Display Monitor
- 2 Control Display with Integrated Touch Pad and X-RAY Activation Button
- 3 Imaging Cabinet

⚠ Refer to the User Guide for safety information and complete instructions.


1 Move the console to the correct location.



2 Lock the casters.



3 Connect the system to power and the network.

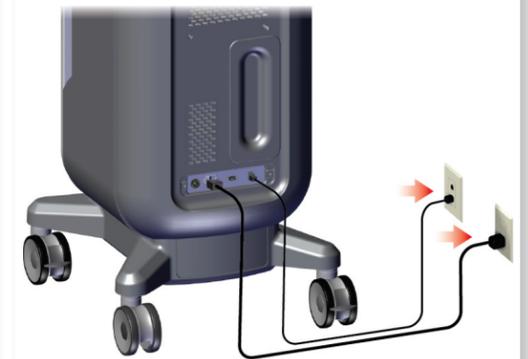

Hologic Inc.
600 Technology Drive
Newark, DE 19702
USA
Tel: +1.800.447.1856

EC REP **Hologic BV**
Da Vincilaan 5
1930 Zaventem
Belgium
Tel: +32 2 711 46 80





HOLOGIC®

Refer to the corporate website for more facilities worldwide.
© Copyright Hologic 2025. All rights reserved. Printed in USA. This manual was originally written in English.

Hologic, Faxitron, Trident and the Hologic Logo are trademarks or registered trademarks of Hologic, Inc. Any other product and company names mentioned herein are the trademarks or registered trademarks of their respective owners.

This product may be protected by one or more U.S. or foreign patents as identified at www.Hologic.com/patents.

4 Start the system and log in.

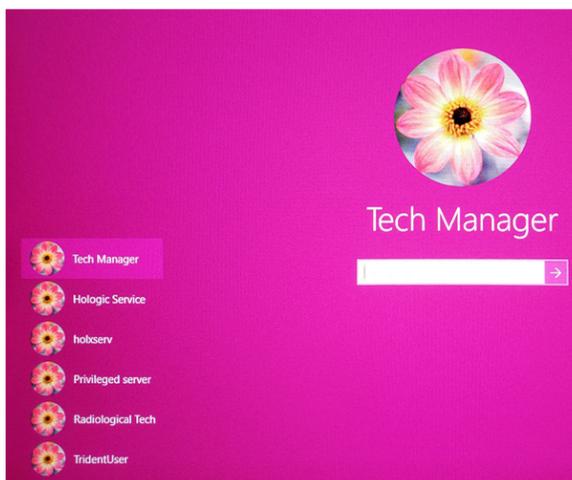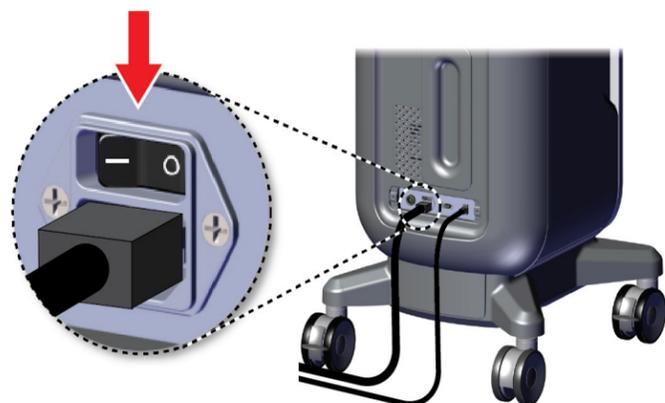


5 Select the patient and procedure.

Faxitron™
Trident® HD
Specimen Radiography System




Name	Date of Birth	Exam
Patient^Test	2/3/1979	Specimen, Rig...
Test^Test Patient	3/4/1972	Specimen, Un...
Patient^Patient Test	4/5/1957	Specimen, Left...
Test^Patient	12/17/1972	Specimen, Rig...

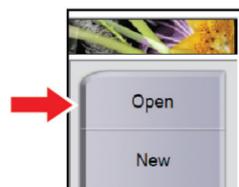


Name	Date of Birth	Exam
Patient^Test	2/3/1979	Specimen, Rig...
Test^Test Patient	3/4/1972	Specimen, Un...
Patient^Patient Test	4/5/1957	Specimen, Left...
Test^Patient	12/17/1972	Specimen, Rig...



6 Prepare the specimen per facility policy and center on the specimen tray.

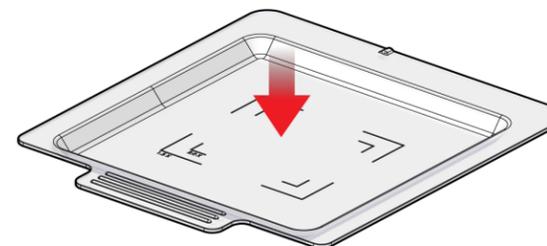

7 Position the specimen tray in the imaging cabinet for the required magnification level.

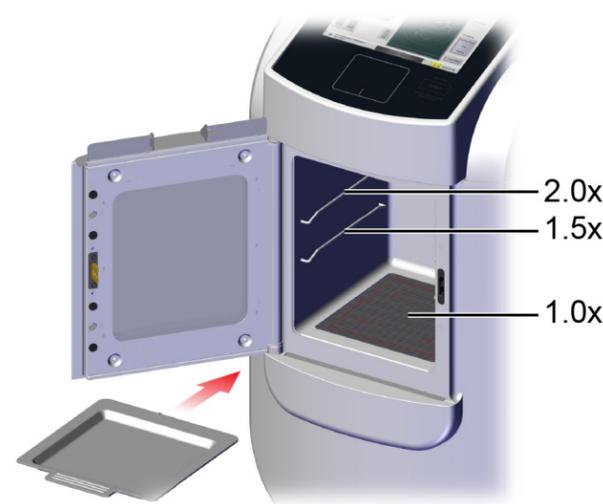

8 Close the door.



9 Acquire the image.

