Espiner Tissue Retrieval System

Instructions for Use: EcoSac

Fannin (UK) Ltd
Yeo Bank 3, Kenn Road, Clevedon, Somerset,
BS21 6TH, United Kingdom

EC REP DCC Vital Limited
Fannin House, South County Business Park
Leopardstown, Dublin 18

T: +44 (0) 1275 878 801
E: espiner@fannin.eu
W: www.espinermedical.com

E333/7

CE
0459

Illustration A: ECO 60 - 160

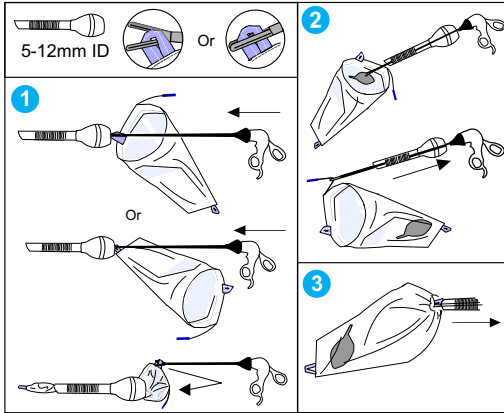
Illustration B: ECO 170 - 600

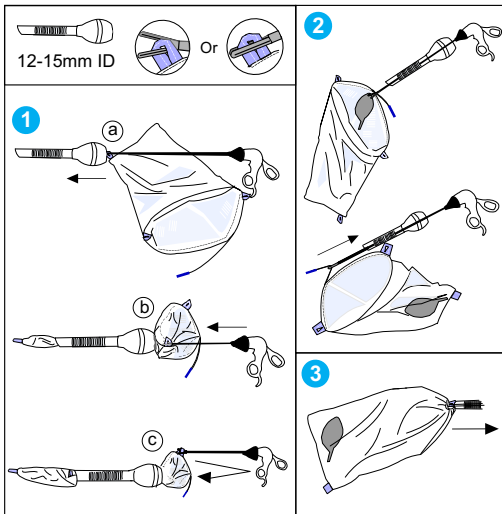
Illustration C: ECO-R


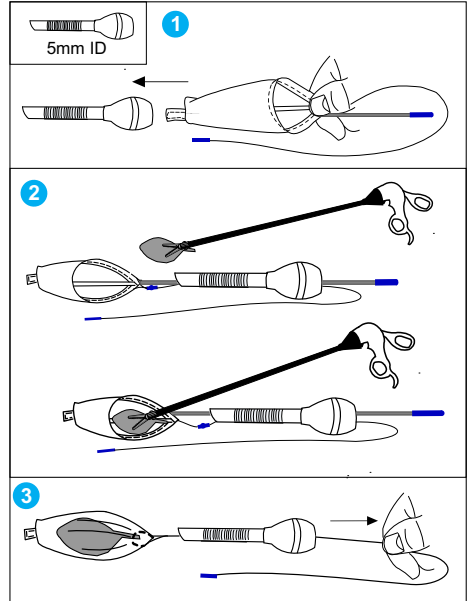
Illustration D: ECO-W

5mm ID	10mm ID	12mm ID	15mm ID
50 - 200 ECO-R	60 - 100 ECO	140 - 230 ECO	600 ECO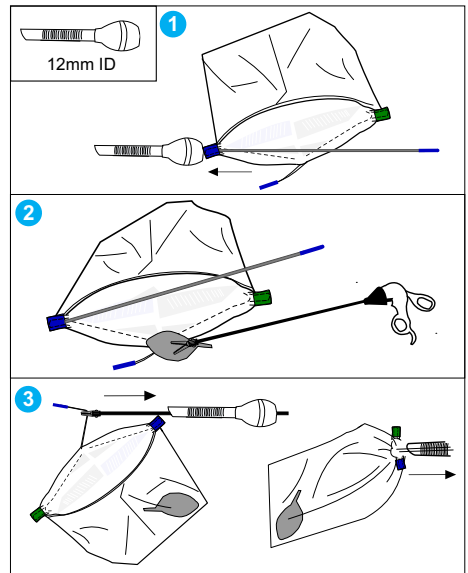

ENGLISH (GB)

DEVICE DESCRIPTION:

Device Name: The Espiner Tissue Retrieval System

The Espiner Tissue Retrieval System is designed to facilitate the safe and secure removal of tissues or organs in the process of laparoscopic surgery.

They are supplied sterile (ISO 11135) with a 5 year shelf life.

INDICATIONS FOR USE:

Retrieval System consists of a family of impervious sacs, which are sterile single use devices, that can be used alone, or with a dedicated introducer system, for the capture and removal of an organ, tissue or fluid from the body cavity during laparoscopic surgery.

CONTRAINDICATIONS:

There are no Contraindications for this device.

WARNINGS & PRECAUTIONS:

The device is only to be used by surgeon who are experienced with the specific techniques required by these medical devices.

Caution: Federal (USA) and European Union law restricts the device to sale, distribution and use by or on the order of a physician.

Use the drawstring by gripping the monofilament/suture approximately 15mm from the heatshrink. The drawstring is designed for sac closure and retrieval of the sac mouth from the port site. Once outside the abdomen, continue to pull the mouth of the sac.

Caution: Do not cut, puncture, or scrape the Espiner Tissue Retrieval System.

Use tabs only for deployment of the device. Do not attempt to deploy by grasping the fabric, or by placing instruments inside the sac.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the member state.

DISPOSAL

Dispose of all used products using standard hospital practices for biohazard control.

INSTRUCTIONS FOR USE – EcoSac

*For ECO 70 - 160 Sacs: Refer to Illustration A.

- 1. Insertion:** Deploy the sac through the Trocar, using the tab at either the sac mouth or sac base first. Once deployed, manoeuvre the sac to locate the opening of the sac mouth. This is shown by the black stitching on the inside of the sac mouth.
- 2. Tissue Capture:** Place the tissue into the sac. Once contained, pull the drawstring to close the sac mouth.
- 3. Retrieval:** Continue to pull the drawstring and sac up into the trocar and ease the mouth out of the incision. If necessary, the sac can be reopened outside the incision and the tissue retrieved.

*For ECO 170 - 600: Refer to Illustration B.

- 1. Insertion:** Deploy the sac base first using the tab, and continue to use the tabs at the mouth of the sac for insertion of the larger sacs.
- 2. Tissue Capture:** Once inserted, re-position the sac to ensure the mouth is open. Ease the tissue into the sac using forceps.
- 3. Retrieval:** When the tissue is fully contained, pull the drawstring to close the sac mouth. Continue to pull the drawstring and sac up into the trocar. If necessary, the mouth of the sac can be reopened outside the incision to retrieve the tissue.

*For ECO-R (with rod): Refer to Illustration C.

- 1. Insertion:** With the rod in the bottom pocket, hold the rod and sac securely together to push through the trocar. The long drawstring should remain outside of the abdomen.
- 2. Tissue Capture:** Once inside, the rod or forceps can be used to coordinate the sac. The rod can be used throughout retrieval and withdrawal, or can be removed. Place the tissue inside the sac mouth and allow it to slide into the sac, until fully contained.
- 3. Retrieval:** When the tissue is contained, pull the drawstring to close the sac mouth. After Sac closure, the rod can be removed. Continue to pull the drawstring and sac mouth up into the trocar, where the sac mouth can then be eased out of the incision. The sac mouth can be re-opened outside the incision, if necessary, to retrieve the tissue and the sac removed.

*For ECO-W: Refer to Illustration D.

- 1. Insertion:** Place the rod into the blue pocket on the sac mouth. Push the rod and blue pocket into the trocar. The sac mouth will open when inside the abdomen, ready for tissue capture.
- 2. Tissue Capture:** Identify the tissue and ensure the mouth of the sac is open. The rod and forceps can be used to coordinate the sac using the blue and green tabs. Place the tissue inside the sac mouth. Use the forceps to ease the tissue into the sac, ensuring the tissue is fully contained.
- 3. Retrieval:** When tissue is contained, pull the drawstring using forceps to close the sac mouth. Continue to pull the drawstring up through the trocar and the sac mouth can be eased out of the incision. The sac mouth can be re-opened outside the incision, if necessary, to retrieve the tissue and the sac removed.

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