Espiner Tissue Retrieval System

Instructions for Use: ECO-US

Fannin (UK) Ltd
Yeo Bank 3, Kenn Road, Clevedon, Somerset, BS21
6TH, United Kingdom

EC REP

DCC Vital Limited
Fannin House, South County Business Park
Leopardstown, Dublin 18

T: +44 (0) 1275 878 801
E: espiner@fannin.eu
W: www.espinermedical.com

E346/4

Illustration A: ECO-US: 70 - 160


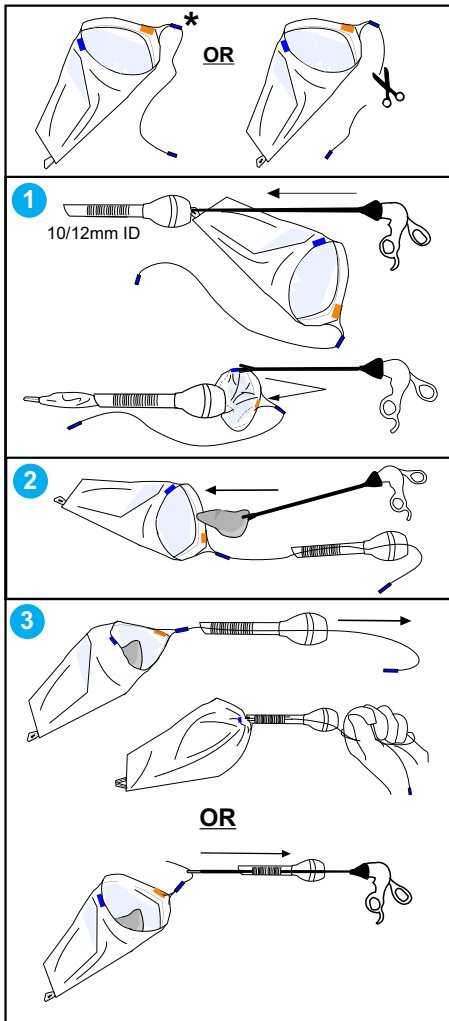
Illustration B: ECO-US: 170 - 230

10mm ID	12mm ID
70 - 100 ECO-US	140 - 230 ECO-US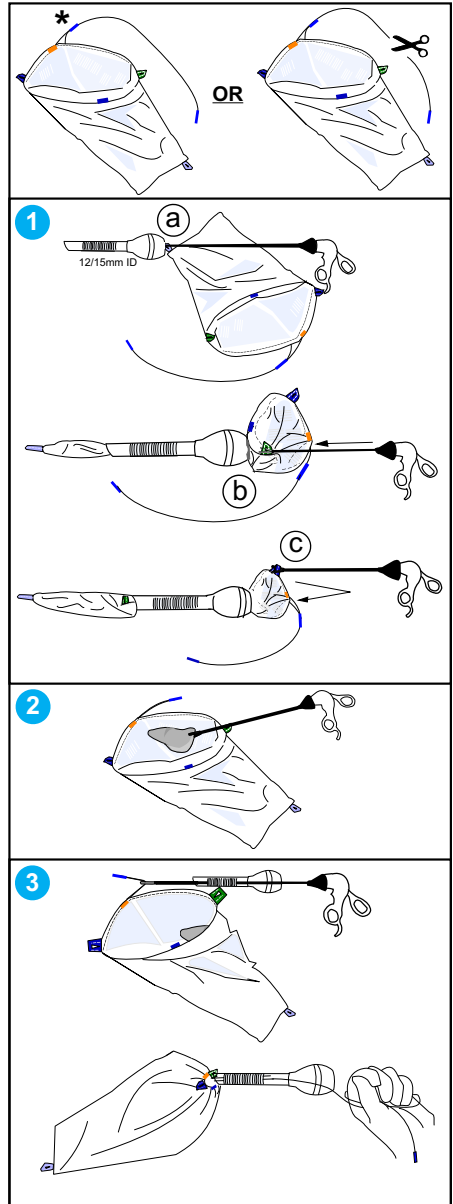

ENGLISH (GB)

DEVICE DESCRIPTION:

Device Name: The Espiner Tissue Retrieval System

The Espiner Tissue Retrieval System is designed to facilitate the safe and secure removal of tissues or organs in the process of laparoscopic surgery. They are supplied sterile (ISO 11135) with a 5 year shelf life.

INDICATIONS FOR USE:

Retrieval System consists of a family of impervious sacs, which are sterile single use devices, that can be used alone, or with a dedicated introducer system, for the capture and removal of an organ, tissue or fluid from the body cavity during laparoscopic surgery.

CONTRAINDICATIONS:

There are no Contraindications for this device.

WARNINGS & PRECAUTIONS:

The device is only to be used by surgeon who are experienced with the specific techniques required by these medical devices.

Caution: Federal (USA) and European Union law restricts the device to sale, distribution and use by or on the order of a physician.

Use the drawstring by gripping the monofilament/suture approximately 15mm from the heatshrink.

The drawstring is designed for sac closure and retrieval of the sac mouth from the port site. Once outside the abdomen, continue to pull the mouth of the sac.

Caution: Do not cut, puncture or scrape the Espiner Tissue Retrieval System. Use tabs only for deployment of the device. Do not attempt to deploy by grasping the fabric, or by placing instruments inside the sac.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the member state.

DISPOSAL

Dispose of all used products using standard hospital practices for biohazard control.

INSTRUCTIONS FOR USE -

The ECO-US sac has been designed to permit either a long drawstring for external control and closure of the sac, or a shortened drawstring for laparoscopic closure of the sac drawstring. Simply cut the monofilament to the desired length above the lower blue shrink-wrap*, before inserting sac through the trocar.

EcoSac 70 - 160 ECO-US (Refer to Illustration A)

- Insertion:** Using forceps, grip the Tab at the base of the Sac to deploy through the Trocar. Once deployed, manoeuvre the sac to locate the Sac mouth. This is shown by the black stitching on the inside of the Sac mouth.
- Tissue Capture:** Place the tissue inside the Sac mouth. Use forceps to ease the tissue into the Sac, ensuring the tissue is fully contained.
- Retrieval:** Once the tissue is fully contained, prepare to remove by closing the Sac mouth with the monofilament and bring the closed Sac mouth up through the incision. The Sac can be re-opened outside the incision to remove the tissue using manual morcellation.

EcoSac 170 - 230 ECO-US (Refer to Illustration B)

- Insertion:** The following three stages should be used: A) Using forceps, deploy the Sac through the Trocar by gripping the Tab at the base of the Sac. B) Remove the forceps and grip the Green Tab to insert through the Trocar. C) Remove the forceps and grip the Blue Tab to insert through the Trocar. Once deployed, manoeuvre the sac to locate the Sac mouth. This is shown by the black stitching on the inside of the Sac mouth.
- Tissue Capture:** Place the tissue inside the Sac mouth. Use forceps to ease the tissue into the Sac, ensuring the tissue is fully contained.
- Retrieval:** Once the tissue is fully contained, prepare to remove by closing the Sac mouth with the monofilament and bring the closed Sac mouth up through the incision. The Sac can be re-opened outside the incision to remove the tissue using manual morcellation.

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