

Faxitron® Pro

Specimen Radiography System



QUICK START GUIDE

1. POWER UP

- Ensure the power switch on the back of the Faxitron Pro cabinet is set to the **ON** position
- Turn the **key** on the front of the Faxitron Pro cabinet to **on**
- Turn the computer and monitor **on**

2. START SOFTWARE

- Verify there is nothing in the Faxitron Pro x-ray chamber
- The software should automatically start, if not Double Click the **VisionLS** icon in the center of the desktop

3. CALIBRATE

- You will be prompted to calibrate once you log in
Press the **“Start Calibrate”** button to proceed

4. SAMPLE DATA

- Click the **“Start Procedure”** button
- Click **“Select from Database”** (or **“Manual Entry”**)
- Enter any missing required fields
- Click **“Accept”**

5. X-RAY IMAGING

- Place specimen(s) in the Faxitron Pro x-ray chamber - use magnification tray if desired
- Press the green **“X-ray”** button to activate x-rays - image will appear after a few seconds

6. IMAGE MANIPULATION

- Hold down the **left mouse button** and drag the mouse to change brightness and contrast for window/level function
- **Adjust zoom** to enlarge image for viewing

7. SHUT DOWN

- Click the **“Shut Down”** button on the left side (or in the Home Menu) and then click **“OK”**
- Turn **key** on Faxitron Pro cabinet to **off**



Hologic, Inc.
600 Technology Drive
Newark, DE 19702 USA
1.800.447.1856
www.hologic.com



Faxitron Pro system also known as UltraFocus and UltraFocus100 System.

MAN-10244 Rev. 001 (7/23) Hologic Inc. ©2023 All rights reserved. Hologic, Faxitron Pro, UltraFocus, UltraFocus100, Faxitron and The Science of Sure are trademarks and/or registered trademarks of Hologic, Inc., and/or its subsidiaries in the United States and/or other countries. This information is intended for medical professionals in the U.S. and other markets and is not intended as a product solicitation or promotion where such activities are prohibited. Because Hologic materials are distributed through websites, eBroadcasts and tradeshow, it is not always possible to control where such materials appear. For specific information on what products are available for sale in a particular country, please contact your local Hologic representative.