


SecurView®

Breast Imaging Workstation

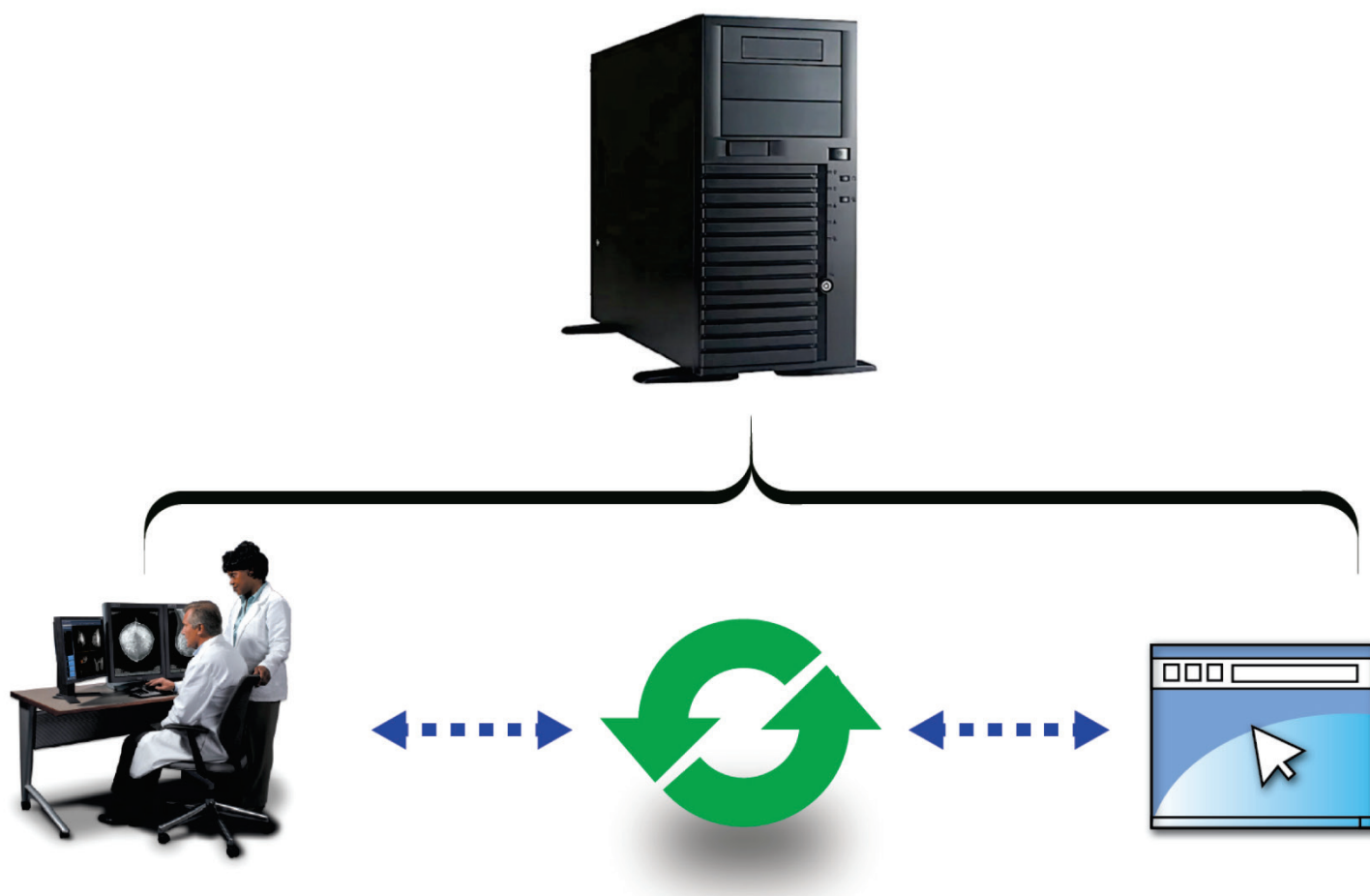

Application Synchronization v5.0

Customer Release Notes

MAN-12322 Revision 001

HOLOGIC®

Application Synchronization

Between Hologic and Third-Party Applications

Release Notes
For Software Version 5.0

Part Number MAN-12322
Revision 001
January 2026

Technical Support

Country	Contact Information
USA	Phone: +1 877-371-4372 Email: BreastHealth.Support@hologic.com
Canada	Phone: +1 877-209-7305 Email: Canada2@hologic.com
Europe	Email: TSbsh@hologic.com Please visit the following website to locate the country-specific contact information: https://www.hologic.com/support/emea
Asia / Pacific	Please visit the following website to locate the country-specific contact information: https://www.hologic.com/support/apac
All other regions	Please visit www.Hologic.com/support

© 2026 Hologic, Inc. Printed in the USA. This manual was originally written in English.

Hologic, SecurView, MultiView, Selenia, Aegis, Unifi, and associated logos are trademarks and/or registered trademarks of Hologic, Inc., and/or its subsidiaries in the United States and/or other countries. All other trademarks, registered trademarks, and product names are the property of their respective owners.

This product may be protected by one or more U.S. or foreign patents as identified at www.Hologic.com/patent-information.



Hologic Inc.
600 Technology Drive
Newark, DE 19702 USA
1.800.447.1856
www.hologic.com

Asian Pacific
Sponsor
Hologic Asia Pacific Ltd.
Unit Nos. 01-03A, 13/F
909 Cheung Sha Wan Road
Cheung Sha Wan
Kowloon, Hong Kong

HOLOGIC®

Australian
Sponsor
Hologic (Australia & New Zealand) Pty Ltd.
Level 3, Suite 302
2 Lyon Park Road
Macquarie Park NSW 2113
Australia

Table of Contents

1: Introduction **1**

2: Notes for Version 5.0 **2**

2.1 Enhancements in Version 5.0 2

2.1.1 Windows 11 Support..... 2

2.1.2 PowerScribe One SP3 Support..... 2

2.1.3 Epic Radiant RIS Hyperdrive XML Enhanced Encryption..... 2

2.1.4 Hologic TCP/IP Partner, Hologic Proxy Partner Security Update..... 2

2.2 Problems Corrected in Version 5.0 2

2.2.1 Application Synchronization License Failure..... 2

2.3 Known Issues in Version 5.0 2

2.3.1 GE Centricity RIS-IC driven workflow is not supported 2

2.4 Supported Applications in Version 5.0 3

3: Configuration Information **16**

3.1 Using Synchronization on SecurView Workstation..... 16

3.2 Using Synchronization on MultiView Application..... 16

Chapter 1 Introduction

The Release Notes provide an overview of the most recent release of the Application Synchronization option for the SecurView® DX diagnostic workstation and MultiView™ application. This document provides a listing of:

- Enhancements
- Supported Applications
- Configuration Information

When an application supports synchronization protocols with a Hologic workstation, a patient is selected only once, and both the Hologic workstation and the other application will synchronize to the same patient automatically. The interface can be unidirectional or bidirectional, depending on the capabilities of the third-party application.

The third-party application can run on a separate computer or as a thin client on the Hologic workstation. Section [Supported Applications in Version 5.0](#) on page 3 provides a list of currently supported third-party applications.

The table below lists SecurView versions supported by different versions of Application Synchronization.

Application Synchronization Versions	SecurView Versions
5.0	10.3 and later (SecurView workstation running Microsoft Windows 10)
4.6	
4.5	
4.4	
4.3	
2.1 to 4.2	7.3 to 10.3 (SecurView workstation running Microsoft Windows 7)

Application Synchronization versions 3.0 and later support MultiView.

Chapter 2 Notes for Version 5.0

2.1 Enhancements in Version 5.0

2.1.1 Windows 11 Support

Application Synchronization 5.0 is validated to run on Windows 11 and Windows 10 computers.

2.1.2 PowerScribe One SP3 Support

PowerScribe One SP3 introduced a new encryption dll that is not backward compatible with the original encryption dll used with the PowerScribe bi-directional application type. Application Synchronization 5.0 supports both the original and the new encryption dll.

2.1.3 Epic Radiant RIS Hyperdrive XML Enhanced Encryption

Application Synchronization 5.0 includes a new encryption type for the Epic Radiant RIS Hyperdrive XML application type, to support enhanced encryption of XML file content with web based Hyperdrive client applications.

2.1.4 Hologic TCP/IP Partner, Hologic Proxy Partner Security Update

Application Synchronization 5.0 includes a newer, more secure version of the log4net dll used in the AS Adapter interface of the Hologic TCP/IP Partner and Hologic Proxy Partner application types.

2.2 Problems Corrected in Version 5.0

2.2.1 Application Synchronization License Failure

Due to Windows OS updates, the Application Synchronization software license could become invalidated on a Windows 10 computer other than the SecurView workstation. This issue is resolved.

2.3 Known Issues in Version 5.0

2.3.1 GE Centricity RIS-IC driven workflow is not supported

Application Synchronization cannot receive OpenPatient messages from GE Centricity RIS-IC 10.7 and later due to stricter user context switching. GE Centricity RIS-IC requires Application Synchronization to reinitialize listening for OpenPatient messages each time a user logs into GE Centricity RIS-IC, but currently Application Synchronization does not have a way to detect GE Centricity RIS-IC user login.

2.4 Supported Applications in Version 5.0

Hologic Partners are applications that have implemented one of the three available Hologic partner interfaces (Command Line Proxy, SharedFile, or TCP/IP) and use the corresponding general application type.

Each non-Hologic partner interface has its own specific application type, with a default configuration that matches the implementation.

Hologic Proxy Partner

A Hologic Proxy Partner is one that uses the Hologic Command Line Proxy interface.

Hologic SharedFile Partner

A Hologic SharedFile Partner is one that uses the Hologic XML SharedFile interface.

Hologic TCP/IP Partner

A Hologic TCP/IP Partner is one that has implemented the Hologic TCP/IP AS Adapter interface.

- 3M Radiology Link Software (Hologic SharedFile Partner)
 - 3M Radiology Link Software version 2.0.0.15 or higher for VoiceScript Recorder and ChartScriptMD Document Builder Platform 7, feature pack 2 with 3rd service pack or higher
 - 3M Radiology Link Software, VoiceScript, and ChartScriptMD can run only on a separate computer.
- Agfa TalkStation (Custom: COM API)
 - Agfa TalkStation version 3.2 or higher
 - TalkStation can be run on the SecurView workstation or on a separate computer.
- Allscripts (Eclipsys) Sunrise Radiology (Hologic TCP/IP Partner)
 - Sunrise Radiology version 5.6 w/SP3 or higher
 - Sunrise Radiology can be run on the SecurView workstation or on a separate computer.
- Amicas Vision Series v4.0-5.1 (Custom: XML file)
 - Amicas Vision Series PACS version 4.0 - 5.1
 - Vision Series can run only on a separate computer.
- Amicas Vision Series v5.5 (Custom: XML file)
 - Amicas Vision Series PACS version 5.5 or higher
 - Vision Series can run only on a separate computer.
- Avreo InterWorks PACS (Hologic TCP/IP Partner)
 - Avreo InterWorks PACS version 4.6 or higher
 - Avreo InterWorks can run only on a separate computer.

- Candelis ImageGrid RIS (Hologic SharedFile Partner)
 - Candelis ImageGrid RIS version 2.0.3 or higher
 - ImageGrid RIS can run only on a separate computer.
- Carestream RIS (Hologic TCP/IP Partner)
 - Carestream RIS (2010) version 10.x, 11.x
 - Carestream RIS can be run on the SecurView workstation or on a separate computer.
- CDI Multifondacs (Hologic TCP/IP Partner)
 - CDI Multifondacs MF-SIS version 4.3.2 or higher
 - CDI Multifondacs can be run on the SecurView workstation or on a separate computer.
- Clario zVision Worklist (Hologic TCP/IP Partner)
 - Clario zVision Worklist version 3.5.7.11523.20 or higher
 - May also be known as Intelrad InteleOrchestrator
 - Clario zVision Worklist can run only on a separate computer.
- Commit; Screening MIS (Hologic TCP/IP Partner)
 - Commit; Screening MIS version 2.2.4 or higher
 - Commit; Screening MIS can be run on the SecurView workstation or on a separate computer.
- Compressus VM2 (Hologic TCP/IP Partner)
 - Compressus VM2 version 2.0.0 or higher
 - VM2 can be run on the SecurView workstation or a separate computer.
- Crescendo CSCIntegration (Hologic SharedFile Partner)
 - Crescendo CSCIntegration version 3.8.2134.0 or higher for DigiDictate-IP version 3.6.287.12 or higher
 - CSCIntegration and DigiDictate-IP can run only on a separate computer.
- Dataworxs AudioWav Mic (Hologic SharedFile Partner)
 - Dataworxs AudioWav Mic version 2.00.017 or higher
 - AudioWav Mic can be run on the SecurView workstation or on a separate computer.
- Dolby FusionDictate (Custom: XML file)
 - Dolby FusionDictate version 1.4.28 (a.k.a. version 8) or higher
 - FusionDictate can be run on the SecurView workstation or on a separate computer.

- **Dolbey FusionExpert** (Custom: XML file)
 - Dolbey FusionExpert version 5.1.30 or higher
 - FusionExpert can be run on the SecurView workstation (with or without front-end speech recognition module) or on a separate computer.
 - Fusion Text version 8.6.13 or higher is required for Windows 10 (Pro, Enterprise, Enterprise LTSB).
- **DR Systems Send ID XML Interface** (Hologic SharedFile Partner)
 - DR Systems Send ID XML Interface version 8.2 or higher and AppLink version 10.0 or higher
 - DR Systems web application can be run on the SecurView workstation or on a separate computer.
- **Epic Radiant RIS** (Custom: COM API)
 - Epic Radiant RIS version Spring 2006 or higher
 - Radiant RIS can be run on the SecurView workstation or on a separate computer.
 - Version 2015 or higher is required for Windows 10 with IE 11.
- **Epic Radiant RIS** (Custom: Encrypted XML file)
 - Epic Radiant RIS Hyperdrive XML version 2022 or higher
 - Radiant RIS can be run on the SecurView workstation or on a separate computer.
- **Euromed MedSpeech** (Hologic TCP/IP Partner)
 - Euromed MedSpeech version 3.0 Rev 10 or higher
 - MedSpeech can be run on the SecurView workstation or on a separate computer.
- **G2 Speech MediSpeech** (Custom: COM API)
 - G2 Speech MediSpeech version 5.0 or higher
 - MediSpeech client can be run on the SecurView workstation or on a separate computer.
- **GE Centricity PACS** (Custom: COM API)
 - GE Centricity PACS version 3.2
 - GE Centricity RIS-i all versions, international
 - Centricity PACS can run only on a separate computer.
- **GE Centricity RIS-IC** (Custom: COM API)
 - GE Centricity RIS-IC version 10.5 SP25 or higher
 - Centricity RIS-IC client can be run on the SecurView workstation or on a separate computer.
- **GE/IDX Imagecast** (Custom: COM API)
 - GE/IDX Imagecast (a.k.a. Centricity RIS-IC) version 10.3.9 or higher
 - Imagecast client can be run on the SecurView workstation or a separate computer.

- HSS CRIS (Hologic TCP/IP Partner)
 - HSS CRIS version 1.06.12n or higher
 - HSS CRIS can be run on the SecurView workstation (with or without Speech Recognition or Digital Dictation) or on a separate computer.
- Ikonopedia Breast Imaging Suite (Hologic SharedFile Partner)
 - Ikonopedia Breast Imaging Suite version 1.0 or higher
 - Breast Imaging Suite can be run on the SecurView workstation or on a separate computer.
- Infinitt G3 PACS (Hologic TCP/IP Partner)
 - Infinitt G3 PACS version 3.0.11.4 BN2 or higher
 - Infinitt G3 PACS web viewer can be run on the SecurView workstation or on a separate computer.
- Insight OmniCare (Hologic SharedFile Partner)
 - Insight OmniCare version 7.5.2 or higher
 - OmniCare can be run on the SecurView workstation or on a separate computer.
- Intelrad IntelViewer (Hologic SharedFile Partner)
 - Intelrad IntelViewer version 4.6.1 or higher
 - IntelViewer can run only on a separate computer.
- Lanier Serial (Custom: Serial)
 - MedQuist/Lanier VoiceWriter Digital Stations
 - Application runs on separate hardware, with a serial cable connecting the SecurView workstation and the Lanier station.
 - MedQuist DocQvoice Workstation (formerly DocQment Ovation) version 1.1.5 or higher
 - DocQvoice can be run on the SecurView workstation or on a separate computer.
- M*Modal (Solventum) Fluency for Imaging Reporting (Hologic SharedFile Partner)
 - M*Modal (Solventum) Fluency for Imaging Reporting version 3.3.0 or higher
 - Fluency for Imaging Reporting can be run on the SecurView workstation or on a separate computer.
 - Version 4.0 or higher runs on Windows 10 with IE 11.
- MagView (Hologic SharedFile Partner)
 - MagView version 6.3 or higher
 - MagView can be run on the SecurView workstation or on a separate computer.
 - MagView 6.x or higher runs on Windows 10.

**Note**

This interface is preferred to use instead of the MagView TCP/IP interface.

- MagView (Custom: TCP/IP)
 - MagView version 5.3.500 or higher
 - MagView can be run on the SecurView workstation or on a separate computer.
- MammaSoft (Custom: Proxy TCP/IP)
 - KVB MammaSoft (Germany) version 1.0 or higher
 - MammaSoft can be run on the SecurView workstation or on a separate computer.
- MaSc RIS (Custom: HTTP)
 - KVWL MaSc RIS (Germany) version 2.0 or higher
 - MaSc (Mammography Screening Software) can be run on the SecurView workstation or on a separate computer.
- Medicalis Professional (Hologic TCP/IP Partner)
 - Medicalis Professional version 2.4.2.15 or higher
 - Professional can be run on the SecurView workstation or on a separate computer.
- MedOutlook RIS (Custom: TCP/IP)
 - Keymind Computing MedOutlook RIS (Norway) version 1.0.0.1 or higher
 - MedOutlook can run only on a separate computer.
- MedQ Dictation (Hologic SharedFile Partner)
 - MedQ Dictation version 5.0.x or higher
 - Dictation can be run on the SecurView workstation or on a separate computer.
- MedQuist SpeechQ (Custom: COM API)
 - MedQuist SpeechQ for Radiology version 1.2 or higher
 - SpeechQ can be run on the SecurView workstation or on a separate computer.
- MRS (Custom: TCP/IP)
 - MRS Mammography Reporting System version 6.6.46 or higher
 - MRS Aspen version 8.0 or higher
 - May also be known as Volpara Patient Hub
 - MRS client can be run on the SecurView workstation or on a separate computer.
 - MRS version 7.1 or higher is required for Windows 10 (Pro).
 - MRS Aspen runs on Windows 10 (Pro).
- MultiView or Aegis (Custom: TCP/IP)
 - Hologic MultiView version 4.0 or higher and Hologic Aegis version 3.2
 - MultiView can be run on the SecurView workstation or on a separate computer.
 - The current released version is 4.1.

- NBSS British Screening RIS (Hologic Proxy Partner)
 - NHSBSP NBSS British Screening RIS can be run on the SecurView workstation or on a separate computer.
- peerVue qiVue (Hologic TCP/IP Partner)
 - peerVue qiVue version 6.3.3 or higher and QICS version 7.0.2 or higher
 - qiVue can run only on a separate computer.
- PenRad (Custom: XML file or Hologic SharedFile Partner)
 - PenRad version 5.1.84 or higher
 - PenRad can be run on the SecurView workstation or on a separate computer.
 - Version 5.7.76 or higher runs on Windows 10 (Pro) with IE 10 or 11, Edge, Chrome.

**Note**

Intelerad acquired PenRad August 2022.

- Philips iSite PACS (Custom: HTTP plug-in)
 - Philips iSite PACS plug-in for iSite Radiology Client version 3.6 and 4.1
 - Philips IntelliSpace version 4.4
 - iSite PACS can run only on a separate computer.
- PowerScribe or EWS (Custom: XML file)
 - Nuance/Dictaphone PowerScribe version 4.6 or higher with PACSBridge 4.7 1009
 - Nuance Enterprise Workstation (EWS) version 3.0 with PACSBridge 6.5.1
 - PowerScribe or EWS client can be run on the SecurView workstation or on a separate computer.
- PowerScribe bi-directional (Custom: XML file)
 - Nuance/Dictaphone PowerScribe version 4.7 or higher with PACSBridge 4.7 1021 supports bi-directional open patient
 - PowerScribe client can be run on the SecurView workstation or on a separate computer.
 - Nuance RadWhere version 3.0.18.14 or higher
 - RadWhere client can be run on the SecurView workstation or a separate computer.
 - Nuance PowerScribe 360 version 1.0 or higher
 - Nuance/Microsoft PowerScribe One version 2019 or higher
 - PowerScribe 360 (including Dragon SDK client edition 10) can be run on the SecurView workstation or on a separate computer.
 - PowerScribe 360 Version 4.0 or higher runs on Windows 10.
 - The current released version is PowerScribe One 2023.1 SP3.

- QDoc StudyViewer (Custom: COM API)
 - Agfa QDoc StudyViewer version 5.5.1 or higher
 - StudyViewer client can be run on the SecurView workstation or on a separate computer.
- RadNet VDI (Custom: COM API)
 - Cerner Millenium RadNet VDI version 2007.18 or higher
 - RadNet VDI can be run on the SecurView Workstation or on a separate computer.
 - Version 2015.01.22 or higher is required for Windows 10.
- RadNet VDI prior to 2007.18 (Custom: COM API)
 - Cerner Millenium RadNet VDI versions prior to 2007.18
 - RadNet VDI can be run on the SecurView Workstation or a separate computer.
- ScImage Picom Client (Hologic SharedFile Partner)
 - ScImage Picom Client version 3.03.059 or higher
 - Picom Client can be run on the SecurView workstation or a separate computer.
- SecurView (Custom: TCP/IP)
 - SecurView DX Diagnostic Workstation version 5-0-0 or higher and SecurView RT Technologist Workstation version 6-0-0 or higher
- SecurView Serial Partner (Custom: Serial)
 - Prior to newer interface implementations, these partners included PenRad, MRS, MagView
 - Third-party application runs on a separate computer, with a serial cable connecting the SecurView workstation and the third-party application computer.
- Selenia® AWS (Custom: TCP/IP)
 - Hologic Selenia Acquisition Workstation version 3.3 or higher
 - Selenia AWS can run only on a separate computer.
- SERGAS SIDI (Hologic TCP/IP Partner)
 - SERGAS SIDI (Spain) version 2.10 or higher
 - SERGAS SIDI can run only on a separate computer.
- SESCOAM Dulcingest RIS (Hologic TCP/IP Partner)
 - SESCOAM Dulcingest RIS version 7.8.3.1 or higher
 - Dulcingest RIS can be run on the SecurView workstation or on a separate computer.
- Softek Illuminate InSight (Hologic SharedFile Partner)
 - Softek Illuminate InSight version 2.12 or higher
 - Illuminate InSight can be run on the SecurView workstation or on a separate computer.

- Spantel Audio Recorder (Custom: COM API)
 - Spantel Audio Recorder version 2.0 or higher
 - Audio Recorder can be run on the SecurView workstation or on a separate computer.
- syngo Workflow MLR (Europe) (Hologic Proxy Partner)
 - Siemens syngo Workflow MLR (Europe) browser version VB30 or higher
 - Siemens syngo Workflow MLR (Europe) can be run on the SecurView workstation or on a separate computer.
- syngo Workflow Portal Radiologist (Custom: HTTP)
 - Siemens syngo Workflow Portal Radiologist (with optional Voice) version 30 or higher
 - Portal Radiologist can be run on the SecurView workstation or on a separate computer.
- syngo Workflow ReadExam (Custom: XML file)
 - Siemens syngo Workflow (formerly Novius RIS) ReadExam (with optional Voice) version 27.1 or higher
 - ReadExam can be run on the SecurView workstation or on a separate computer.
- Therapixel MammoScreen (Hologic SharedFile Partner)
 - Therapixel MammoScreen V1.0.0 or higher
 - Hologic Genius AI Detection PRO v3.1.0 or higher, for SecurView workstation synchronization with the Widget
 - Therapixel MammoScreen Widget can be run on the SecurView workstation or on a separate computer
- Unifi Workspace (Custom: WebAPI)
 - Hologic Unifi Workspace version 1.0 or higher (no longer available)
- WinScribe Internet Author (Hologic SharedFile Partner)
 - WinScribe Internet Author version 3.8 or higher
 - WinScribe Internet Author can be run on the SecurView workstation or on a separate computer.

Partner	Partner Type	Receives					Sends			
		Login	Logout	Open Patient	Update Patient		Login	Logout	Open Patient	Update Patient
Hologic Proxy Partner+	Proxy	-	-	-	-		✓	✓	✓	-

Application Synchronization 5.0 Release Notes

Chapter 2 Notes for Version 5.0

Partner	Partner Type	Receives					Sends			
		Login	Logout	Open Patient	Update Patient		Login	Logout	Open Patient	Update Patient
Hologic SharedFile Partner+	SharedFile	✓	✓	✓	✓		✓	✓	✓	✓
Hologic TCP/IP Partner+	TCP/IP	✓	✓	✓	✓		✓	✓	✓	✓
3M Radiology Link Software	SharedFile	✓	✓	✓	-		-	-	-	-
Agfa TalkStation*	COM API	✓	✓	✓	-		-	-	-	-
Allscripts (Eclipsys) Sunrise Radiology	TCP/IP	-	-	-	-		✓	✓	✓	-
Amicas Vision Series (v4.0-5.1, v5.5)	XML file	-	-	-	-		-	-	✓	-
Avreo InterWorks PACS	TCP/IP	-	-	-	-		✓	✓	✓	-
Candellis ImageGrid RIS	SharedFile	-	-	✓	-		-	-	✓	-
Carestream RIS	TCP/IP	-	-	-	-		✓	✓	✓	-
CDI Multifondacs	TCP/IP	-	-	-	-		-	-	✓	-
Clario zVision Worklist (Intelerad InteleOrchestra tor)	TCP/IP	-	-	-	-		-	-	✓	-
Commit; Screening MIS	TCP/IP	-	-	-	-		-	-	✓	-
Compressus VM2	TCP/IP	-	-	-	-		✓	✓	✓	✓
Crescendo CSCIntegration	SharedFile	✓	-	✓	-		-	-	-	-

Application Synchronization 5.0 Release Notes

Chapter 2 Notes for Version 5.0

Partner	Partner Type	Receives					Sends			
		Login	Logout	Open Patient	Update Patient		Login	Logout	Open Patient	Update Patient
Dataworxs AudioWav Mic	SharedFile	-	-	✓	-		-	-	-	-
Dolbey FusionDictate	XML file	-	-	✓	-		-	-	-	-
Dolbey FusionExpert	XML file	-	-	✓	-		-	-	-	-
DR Systems Send ID	SharedFile	-	-	-	-		-	-	✓	-
Epic Radiant RIS*	COM API	✓	✓	✓	-		-	-	✓	✓
Epic Radiant RIS Hyperdrive	Encrypted XML file	✓	✓	✓	-		✓	✓	✓	✓
Euromed MedSpeech	TCP/IP	-	-	✓	-		-	-	-	-
G2 Speech MediSpeech*	COM API	✓	✓	✓	-		-	-	-	-
GE Centricity PACS, RIS-i	COM API	-	-	-	-		-	-	✓	-
GE Centricity RIS-IC*	COM API	✓	✓	✓	-		-	-	✓	-
GE/IDX Imagecast*	COM API	✓	✓	✓	-		-	-	-	-
HSS CRIS	TCP/IP	✓	✓	✓	-		✓	✓	✓	-
Ikonopedia Breast Imaging Suite	SharedFile	-	-	✓	-		-	-	✓	-
Infinitt G3 PACS	TCP/IP	✓	✓	✓	-		✓	✓	✓	-
Insight OmniCare	SharedFile	-	-	-	-		-	-	✓	-
Intelerad InteleViewer	SharedFile	-	-	-	-		✓	✓	✓	-
Lanier Serial	Serial	-	-	✓	-		-	-	-	-

Application Synchronization 5.0 Release Notes

Chapter 2 Notes for Version 5.0

Partner	Partner Type	Receives					Sends			
		Login	Logout	Open Patient	Update Patient		Login	Logout	Open Patient	Update Patient
MedQuist DocQvoice	Serial	-	-	✓	-		-	-	-	-
M*Modal (Solventum) Fluency for Imaging Reporting	SharedFile	✓	✓	✓	-		-	-	✓	✓
MagView	SharedFile	✓	-	✓	-		-	-	✓	✓
MagView	Cust TCP/IP	-	-	✓	-		-	-	✓	-
MammaSoft	Cust Proxy	-	-	-	-		-	-	✓	-
MaSc RIS	Cust HTTP	-	-	-	-		-	-	✓	-
Medicalis Professional	TCP/IP	-	✓	✓	✓		✓	✓	✓	✓
MedOutlook RIS	Cust TCP/IP	-	-	✓	-		-	-	✓	-
MedQ Dictation	SharedFile	-	-	✓	-		-	-	✓	-
MedQuist SpeechQ*	COM API	✓	✓	✓	-		-	-	✓	-
MRS (Volpara Patient Hub)	Cust TCP/IP	✓	✓	✓	-		✓	✓	✓	✓**
MultiView or Aegis	Cust TCP/IP	✓	✓	✓	✓		✓	✓	✓	-
NBSS British Screening RIS	Proxy	-	-	-	-		-	-	✓	-
peerVue qiVue	TCP/IP	✓	✓	✓	-		-	-	✓	-
PenRad	SharedFile	-	-	✓	-		-	-	✓	✓
PenRad	XML file	-	-	✓	-		-	-	✓	-
Philips iSite (IntelliSpace) PACS	Cust HTTP plug-in	-	-	-	-		-	-	✓	-

Application Synchronization 5.0 Release Notes

Chapter 2 Notes for Version 5.0

Partner	Partner Type	Receives					Sends			
		Login	Logout	Open Patient	Update Patient		Login	Logout	Open Patient	Update Patient
PowerScribe or EWS	XML file	✓	✓	✓	-		-	-	-	-
PowerScribe bi-directional	XML file	✓	✓	✓	-		-	-	✓	✓
- Nuance RadWhere	XML file	✓	✓	✓	-		-	-	-	-
- PowerScribe 360	XMLfile	✓	✓	✓	-		-	-	✓	✓
- PowerScribe One	XML file	✓	✓	✓	-		-	-	✓	✓
QDoc StudyViewer	COM API	✓	✓	✓	-		✓	✓	✓	-
RadNet VDI*	COM API	✓	✓	✓	-		-	-	-	-
RadNet VDI prior to version 2007.18*	COM API	✓	✓	✓	-		-	-	-	-
ScImage Picom Client	SharedFile	-	-	-	-		✓	✓	✓	-
SecurView	Cust TCP/IP	✓	✓	✓	✓		✓	✓	✓	✓
SecurView Serial Partner	Serial	-	-	✓	-		-	-	-	-
Selenia AWS	Cust TCP/IP	-	-	-	-		-	-	✓	-
SERGAS SIDI	TCP/IP	-	-	-	-		✓	-	✓	-
SESCAM Dulcingest RIS	TCP/IP	-	-	-	-		✓	✓	✓	-
Softek Illuminate InSight	SharedFile	✓	✓	✓	-		-	-	-	-
Spantel Audio Recorder*	COM API	-	-	✓	-		-	-	-	-
syngo Workflow (Europe)	Proxy	-	-	-	-		-	-	✓	-

Partner	Partner Type	Receives					Sends			
		Login	Logout	Open Patient	Update Patient		Login	Logout	Open Patient	Update Patient
syngo Workflow Portal Radiologist*	Cust HTTP	-	-	-	-		-	-	✓	-
syngo Workflow ReadExam	XML file	-	-	-	-		-	-	✓	-
Therapixel MammoScreen / Genius AI Detection PRO	SharedFile	-	-	✓	-		-	-	-	-
Unifi Workspace	Cust WebAPI	-	✓	✓	-		✓	✓	✓	-
WinScribe Internet Author	SharedFile	✓	✓	✓	-		-	-	-	-

+ Partners may not implement all supported message types

* Application Synchronization must be installed on the same computer as this third-party application. Application Synchronization can be installed and configured on a computer other than SecurView or MultiView if necessary.

** If MRS 7.3 or later (Aspen Breast or Volpara Patient Hub) can send UpdatePatientState messages, Application Synchronization will act on them even though the Sent Messages check box is unchecked on the *Message Type* page.



Note

Third-party application versions listed are the earliest versions of software validated by Hologic, usually the earliest version of the third-party application that supported the interface.

While synchronization with newer versions of third-party applications is expected to work, their usage on the Hologic workstation may result in performance degradation, or there may be interface incompatibilities. If you have questions regarding support of a particular third-party application version, contact Hologic service.



Note

Consult your Sales Account Manager to check on a third-party application that is not shown above.

Chapter 3 Configuration Information

3.1 Using Synchronization on SecurView Workstation

For complete information on configuring synchronization features within the SecurView workstation, see the latest *SecurView DX/RT Workstation User Guide*. For information on configuring synchronization features within a third-party application, see the appropriate third-party documentation.

When an approved third-party application is running on a SecurView workstation and displayed on an additional monitor that is shared with the SecurView Patient List and/or a SecurView Multimodality Viewer or MultiView Multimodality, use [Alt] + [Tab] to toggle through the applications.

3.2 Using Synchronization on MultiView Application

For complete information on configuring synchronization features within the MultiView application, refer to the latest *MultiView Multimodality User Guide*. For information on configuring synchronization features within a third-party application, see the appropriate third-party documentation.