

## Aptima Auto Detect Kit (Panther™ System)

For *in vitro* diagnostic use.

Rx only

### Intended Use

The Aptima Auto Detect Kit is for use on the Panther System.

### Materials Provided

#### Aptima Auto Detect Kit (Cat. No. 303013)

(Store at 15°C to 30°C upon receipt)

Symbol	Component	Quantity
R1	Aptima Auto Detect 1 <i>Aqueous solution containing 0.1% hydrogen peroxide and 0.001 N nitric acid.</i>	1 x 245 mL
R2	Aptima Auto Detect 2 <i>1.6 N sodium hydroxide.</i>	1 x 245 mL

### Warnings and Precautions

- A. Avoid contact of Auto Detect 2 with skin, eyes, and mucous membranes. If this fluid comes into contact with skin or eyes, wash with water. If this fluid spills, dilute the spill with water before wiping dry.
- B. Use routine laboratory precautions.

### Storage and Handling Requirements

- A. Store the Aptima Auto Detect Kit at 15°C to 30°C.
- B. Fluids must be at 15°C to 30°C before use. Once opened, fluids are stable for 60 days.
- C. If precipitate has formed, heat the fluids at 37°C to 60°C to facilitate dissolution of the precipitate. Cool the fluids to room temperature and thoroughly mix prior to use.
- D. Do not use after the expiration date.
- E. Some reagents of this kit are labeled with risk and safety symbols.

**Note:** For hazard communications information, refer to the Safety Data Sheet Library at [www.hologicsds.com](http://www.hologicsds.com).

### US Hazard Information



#### **Auto Detect 2**

**SODIUM HYDROXIDE 5 – 10%**

#### **DANGER**

H314 - Causes severe skin burns and eye damage  
 P260 - Do not breathe dust/fume/gas/mist/vapors/spray  
 P264 - Wash face, hands and any exposed skin thoroughly after handling  
 P280 - Wear protective gloves/protective clothing/eye protection/face protection  
 P310 - Immediately call a POISON CENTER or doctor/physician  
 P305 + P351 + P338 - IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing  
 P310 - Immediately call a POISON CENTER or doctor/physician  
 P303 + P361 + P353 - IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower  
 P363 - Wash contaminated clothing before reuse  
 P304 + P340 - IF INHALED: Remove victim to fresh air and keep at rest in a position comfortable for breathing  
 P310 - Immediately call a POISON CENTER or doctor/physician  
 P301 + P330 + P331 - IF SWALLOWED: rinse mouth. Do NOT induce vomiting

P405 - Store locked up

### EU Hazard Information


**Auto Detect 2**
**SODIUM HYDROXIDE 5 – 10%**
**DANGER**

H314 - Causes severe skin burns and eye damage

P280 - Wear protective gloves/protective clothing/eye protection/face protection

P260 - Do not breathe dust/fume/gas/mist/vapours/spray

P303 + P361 + P353 - IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower

P280 - Wear eye protection/ face protection

P305 + P351 + P338 - IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to

do. Continue rinsing

P310 - Immediately call a POISON CENTER or doctor/physician

## Procedure

Replace bottles in the Universal Fluids Drawer when notified by the system. Refer to the *Panther System Operator's Manual*.



**Hologic N.V.**  
 Da Vincilaan 5  
 1930 Zaventem  
 Belgium

Hologic, Inc.  
 10210 Genetic Center Drive  
 San Diego, CA 92121 USA

Customer Support: +1 800 442 9892  
[customersupport@hologic.com](mailto:customersupport@hologic.com)

Technical Support: +1 888 484 4747  
[molecularsupport@hologic.com](mailto:molecularsupport@hologic.com)

For more contact information visit [www.hologic.com](http://www.hologic.com).

Hologic, Aptima, and Panther are trademarks and/or registered trademarks of Hologic, Inc. and/or its subsidiaries in the United States and/or other countries.

All other trademarks that may appear in this package insert are the property of their respective owners.

This product may be covered by one or more U.S. patents identified at [www.hologic.com/patents](http://www.hologic.com/patents).

©2001–2018 Hologic, Inc. All rights reserved.

502180EN Rev. 003  
 2018-05