## Aptima analysevæskesett (Panther™-systemet)

Til *in vitro*-diagnostisk bruk.

Rx only

### Beregnet bruk

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Fareinformasjon for USA	
	<p><b>Buffer For Deactivation Fluid</b>  <b>SODIUM HYDROXIDE 1 – 5%</b>  <b>WARNING</b>            H315 – Causes skin irritation            H319 – Causes serious eye irritation            P264 – Wash face, hands and any exposed skin thoroughly after handling            P280 – Wear protective gloves/protective clothing/eye protection/face protection</p>

	<p>P305 + P351 + P338 – IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing</p> <p>P337 + P313 – If eye irritation persists: Get medical advice/attention</p> <p>P302 + P352 – IF ON SKIN: Wash with plenty of soap and water</p> <p>P332 + P313 – If skin irritation occurs: Get medical advice/attention</p> <p>P362 – Take off contaminated clothing and wash before reuse</p>
	<p><b>Aptima Oil</b>  <b>POLYDIMETHYLSILOXANE 95 – 100%</b>  <b>WARNING</b></p> <p>H315 – Causes skin irritation</p> <p>H319 – Causes serious eye irritation</p> <p>P264 – Wash face, hands and any exposed skin thoroughly after handling</p> <p>P280 – Wear protective gloves/protective clothing/eye protection/face protection</p> <p>P305 + P351 + P338 – IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing</p> <p>P337 + P313 – If eye irritation persists: Get medical advice/attention</p> <p>P302 + P352 – IF ON SKIN: Wash with plenty of soap and water</p> <p>P332 + P313 – If skin irritation occurs: Get medical advice/attention</p> <p>P362 – Take off contaminated clothing and wash before reuse</p>
<b>Fareinformasjon for EU</b>	
	<p><b>Buffer For Deactivation Fluid</b>  <b>NATRIUMHYDROKSID 1 – 5 %</b>  <i>Sodium Hypochlorite &lt; 1 %</i>  <b>ADVARSEL</b></p> <p>H315 - Irriterer huden</p> <p>H319 - Gir alvorlig øyeirritasjon</p>
	<p><b>Aptima Oil</b>  <b>POLYDIMETHYLSILOXANE 100 %</b>  <b>ADVARSEL</b></p> <p>H315 - Irriterer huden</p> <p>H319 - Gir alvorlig øyeirritasjon</p>

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©2001–2018 Hologic, Inc. Med enerett.
Hologic, Inc.