

Aptima Assay Fluids Kit (Panther™ System)

For *in vitro* diagnostic use.

Intended Use

The Aptima Assay Fluids Kit is for use on the Panther System.

Materials Provided

Aptima Assay Fluids Kit (Cat No. 303014)

(Store at 15°C to 30°C upon receipt)

Symbol	Component	Quantity
W	Aptima Wash Solution <i>10 mM HEPES buffered solution containing less than 2% preservative.</i>	1 x 2.9 L
O	Aptima Oil Reagent <i>Silicone oil.</i>	1 x 260 mL
DF	Aptima Buffer for Deactivation Fluid <i>800 mM Bicarbonate buffered solution.</i>	1 x 1.4 L

Warnings and Precautions

- A. Use routine laboratory precautions.
- B. For professional use.
- C. Avoid microbial and ribonuclease contamination of Aptima Assay Fluids.

Storage and Handling Requirements

- A. Store unopened Aptima Assay Fluids at 15°C to 30°C.
- B. Once opened, Aptima Assay Fluids are stable for 60 days.
- C. Do not use after the expiration date.
- D. Some reagents of this kit are labeled with risk and safety symbols.

Note: For hazard communication information, refer to the Safety Data Sheet Library at www.hologicsds.com.

US Hazard Information



Buffer For Deactivation Fluid

SODIUM HYDROXIDE 1 – 5%

WARNING

H315 – Causes skin irritation

H319 – Causes serious eye irritation




P264 – Wash face, hands and any exposed skin thoroughly after handling

P280 – Wear protective gloves/protective clothing/eye protection/face protection

P305 + P351 + P338 – IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing

P337 + P313 – If eye irritation persists: Get medical advice/attention

P302 + P352 – IF ON SKIN: Wash with plenty of soap and water

	P332 + P313 – If skin irritation occurs: Get medical advice/attention P362 – Take off contaminated clothing and wash before reuse
	<p>Aptima Oil POLYDIMETHYLSILOXANE 95 – 100% WARNING</p> <p>H315 – Causes skin irritation H319 – Causes serious eye irritation P264 – Wash face, hands and any exposed skin thoroughly after handling P280 – Wear protective gloves/protective clothing/eye protection/face protection P305 + P351 + P338 – IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing P337 + P313 – If eye irritation persists: Get medical advice/attention P302 + P352 – IF ON SKIN: Wash with plenty of soap and water P332 + P313 – If skin irritation occurs: Get medical advice/attention P362 – Take off contaminated clothing and wash before reuse</p>
EU Hazard Information	
	<p>Buffer For Deactivation Fluid SODIUM HYDROXIDE 1 – 5% <i>Sodium Hypochlorite < 1%</i> WARNING</p> <p>H315 – Causes skin irritation H319 – Causes serious eye irritation</p>
	<p>Aptima Oil POLYDIMETHYLSILOXANE 100% WARNING</p> <p>H315 – Causes skin irritation H319 – Causes serious eye irritation</p>

Procedure

Replace bottles in the Universal Fluids Drawer when notified by the system. Refer to the *Panther System Operator's Manual*.



Hologic, Inc.
10210 Genetic Center Drive
San Diego, CA 92121 USA

Hologic BVBA
Da Vincilaan 5
1930 Zaventem
Belgium

Customer Support: +1 800 442 9892
customersupport@hologic.com

Technical Support: +1 888 484 4747
molecularsupport@hologic.com

For more contact information visit www.hologic.com.

Hologic, Aptima, and Panther are trademarks and/or registered trademarks of Hologic, Inc. and/or its subsidiaries in the United States and/or other countries. All other trademarks that may appear in this package insert are the property of their respective owners.

This product may be covered by one or more U.S. patents identified at www.hologic.com/patents.

©2001–2019 Hologic, Inc. All rights reserved.

502181EN Rev. 005
2019-09