



Date: 02/06/2020

Author: Software Engineering

System: Apex

Subsystem: Microsoft Windows OS 10

Subject: Microsoft Windows 10 OS Patch release January 2020

Purpose:

To release the list of validated Microsoft critical patches on the Apex systems.

Note:

1. Installation of the January updates enabled the Microsoft Compatibility Appraiser.

Follow the below steps to disable it:-

1. Open the Task Scheduler. Type the taskschd in the Run prompt.
2. Go to Task Scheduler Library / Microsoft / Windows / Application Experience.
3. Select the Microsoft Compatibility Appraiser, right click and disable it.

2. If you are using the following versions of APEX, if you are required to update to the latest Hologic validated Windows updates, you must ensure your system(s) have been patched with the "Sentinel Run-Time 7.80.0.0" which can be located in the Windows Control Panel "Program and Features". If needed you can install the patch by ordering "APEX Service Dongle Patch Release" (ASY-12610). Please contact Hologic Support if you have any questions.

| System Type | Software Versions |
|--------------------|--------------------------|
| Horizon | APEX v5.6.0.7 |
| Workstation | APEX v5.6.0.7 |

List of patches/KB's validated for January 2020:

KB4534271

List of all patches/KB's validated on the Apex System:

KB4534271
KB4520724

KB4516115
KB4494175

KB4132216
KB4022619

KB2565063

KB2467173