

## ThinPrep® Stain EA Solution

## Instructions for Use

CE

Rx only

**INTENDED USE**

ThinPrep® EA Solution is intended to be used in a Papanicolaou staining procedure in conjunction with any ThinPrep Nuclear Stain, ThinPrep Rinse Solution, ThinPrep Bluing Solution, and ThinPrep Orange G Solution.

**SUMMARY AND EXPLANATION**

ThinPrep EA Solution is an alcoholic solution that serves to stain the cytoplasm of cells on slide preparations for cytologic evaluation.

**PRINCIPLES OF PROCEDURE**

When used in conjunction with the ThinPrep staining solutions and cytologic slide preparations, ThinPrep EA Solution will stain the cytoplasm of cells without interfering with nuclear staining and morphology.

**COMPOSITION**

An alcoholic solution of Ethanol, Isopropyl Alcohol, Methanol, and proprietary biological dyes. CAS-64-17-5, CAS 67-56-1, CAS 67-63-0

**WARNINGS**

**Danger.** Flammable. Contains Ethanol, Isopropyl Alcohol, Methanol.

H302 - Harmful if swallowed.

H312 - Harmful in contact with skin.

H319 - Causes serious eye irritation.

H371 - May cause damage to organs.

H225 - Highly flammable liquid and vapor.

For In Vitro Diagnostic Use. Not for external or internal use in humans or animals. Cannot be made non-poisonous.

**PRECAUTIONS**

P260 - Do not breathe vapors.

P210 - Keep away from heat/sparks/open flames/hot surfaces.

P280 - Wear protective gloves/protective clothing/eye protection/face protection.

Wash hands thoroughly after handling. Keep container tightly closed when not in direct use. Use with adequate ventilation. Results with non-ThinPrep cytology samples have not been evaluated. Do not use if primary packaging is damaged.

**PRETREATMENT**

No reconstitution, mixing or dilution is required.

## ThinPrep® Stain EA Solution

### **STORAGE**

Store ThinPrep EA Solution at 15°C (59°F) to 30°C (86°F) in the container provided. Do not use ThinPrep EA Solution beyond the expiration date marked on the container.

### **APPEARANCE AND INTEGRITY**

Opaque, non-sterile solution.

### **SPECIMEN COLLECTION AND PREPARATION**

For use with gynecologic cytology samples that have been processed on any ThinPrep Processor.

### **PROCEDURE**

See the ThinPrep Stain User's Manual for specific staining protocols.

### **LIMITATIONS OF PROCEDURE**

Must be used according to the instructions provided in the ThinPrep Stain User's Manual. ThinPrep EA Solution cannot be substituted with any other solution.

### **PERFORMANCE CHARACTERISTICS**

When used as directed, ThinPrep EA Solution will stain the cytoplasm of cells on cytologic slide preparations.

### **DISPOSAL**

Dispose in accordance with all applicable regulations.

### **FIRST AID MEASURES**

IF SWALLOWED: Call a POISON CENTER or doctor/physician if you feel unwell. See [www.hologicds.com](http://www.hologicds.com) for the entire Safety Data Sheet.



Manufacturer



Catalogue number



Flammable



Consult instructions for use



Use by



Irritant, Respiratory Tract Irritation, Dermal Sensitizer



*In vitro* diagnostic medical device



Batch code



Respiratory Sensitizer, Target Organ Toxicity



Authorised representative in the European Community



Temperature limitation



Hologic, Inc. • 250 Campus Drive • Marlborough, MA 01752 USA  
Tel: +1 508 263-2900 • [www.hologic.com](http://www.hologic.com)

Hologic BVBA • Da Vinciaan 5 • 1930 Zaventem • Belgium

Australian Sponsor: Hologic (Australia) Pty Ltd • Suite 302, Level 3 • 2 Lyon Park Road  
Macquarie Park NSW 2113 • Australia • Tel: 02 9888 8000