



Date: 01/22/2020

Author: Software Engineering

System: Apex

Subsystem: Microsoft Windows OS 10

Subject: Microsoft Windows 10 OS Patch release January 2020

Purpose:

To release the list of validated Microsoft critical patches on the Apex systems.

Note 1: If you are using the following versions of APEX, if you are required to update to the latest Hologic validated Windows updates, you must ensure your system(s) have been patched with the “Sentinel Run-Time 7.80.0.0” which can be located in the Windows Control Panel “Program and Features”. If needed you can install the patch by ordering “APEX Service Dongle Patch Release” (ASY-12610). Please contact Hologic Support if you have any questions.

System Type	Software Versions
Horizon	APEX v5.6.0.7
Workstation	APEX v5.6.0.7

Note 2: After installing the January 2020 or later patches, ensure that the Microsoft Compatibility Appraiser disabled.

This is done by bringing up the Task Scheduler (Type “Task Scheduler” in the Windows Search bar) and then navigating to Task Scheduler Library -> Microsoft → Windows → Application Experience.

Right click “Microsoft Compatibility Appraiser” and select “Disable”.

List of patches/KB’s validated for January 2020:

KB4534271

List of all patches/KB’s validated on the Apex System:

KB4534271

KB4520724

KB4516115

KB4494175
KB4132216

KB4022619
KB2565063

KB2467173