



Date: August 7, 2024

Author: Software Engineering

System: SecurView Manager – Windows Server 2016

Subsystem: Windows 10 Server 2016 & Software Security Patches

Subject: Windows 10 Server 2016 & Software Security Patch release July 2024

Purpose:

To release the list of validated Microsoft critical patches on the SecurView Manager – Windows Server 2016 System.

Note: Hologic would like to bring attention to the following known issue statement released by Microsoft related to the Windows cumulative security patch released on July 9, 2024. Devices that have been configured for BitLocker encryption, particularly those configured for Device Encryption, may require entering the BitLocker recovery key during restart of Windows after installing the latest security patch. If the recovery key is requested by Windows, the device will not be able to start until the key is provided. Standard BitLocker (not Device Encryption) is an optional configuration that may be enabled on Hologic SecurView and Horizon product devices.

BEFORE installing patches to a BitLocker enabled device, please ensure that you have access to the required BitLocker recovery key. For SecurView devices, BitLocker recovery key(s) can be found within E:\BitLocker and should also be stored securely, off device, by the Health Delivery Organization (HDO). For Horizon devices, a USB flash drive should have been created during the encryption process. If the BitLocker recovery key cannot be located, contact Hologic Technical Support for assistance.

Microsoft Known Issue Statement

After installing the July 9, 2024, Windows security update, you might see a BitLocker recovery screen upon starting up your device. This screen does not commonly appear after a Windows update. You are more likely to face this issue if you have the Device Encryption option enabled in Settings under Privacy & Security > Device encryption. Because of this issue, you might be prompted to enter the recovery key from your Microsoft account to unlock your drive.

List of patches/KB's validated for July 2024:

KB5040562 KB5040434

List of all patches/KB's validated on the SecurView Manager – Windows Server 2016 System:

KB890830	KB5034119	KB5023788
KB5040562	KB5033373	KB5023697
KB5040434	KB5032391	KB5022838
KB5039334	KB5032197	KB5022289
KB5039214	KB5031362	KB5017396
KB5037763	KB5030504	KB5012170
KB5037016	KB5030213	KB5005698
KB5036899	KB5029242	KB4589210
KB5035962	KB5028169	KB4494175
KB5035855	KB5027219	KB4132216
KB5034862	KB5026363	
KB5034767	KB5025228	