



Date: July 16, 2025

Author: Software Engineering

System: SecurView

Subsystem: Windows 10 LTSC 2016 (v1607) & Software Security Patches

Subject: Windows 10 LTSC 2016 (v1607) & Software Security Patch release June 2025

Purpose:

To release the list of validated Microsoft critical patches on the SecurView System.

Latest .NET Versions Validated for the Product:

Note: Security updates should only be installed for existing .NET installations already present on the product. For example, .NET 8.0.14 update can be installed only if an earlier version of .NET 8.0 is already part of the product installation.

- **.NET Framework 3.5 / 4.x**

The latest security updates are included in the Windows Cumulative Update.

List of patches/KB's validated for June 2025:

KB5061010 KB5060954

List of all patches/KB's validated on the SecurView System:

KB5061010	KB5054006	KB5044293
KB5060954	KB5053594	KB5043124
KB5058524	KB5052006	KB5043051
KB5058383	KB5050109	KB5041773
KB5055661	KB5049993	KB5041576
KB5055521	KB5048671	KB5040562
KB890830	KB5046612	KB5040434

KB5039334	KB5019964	KB5001402
KB5039214	KB5018411	KB5001347
KB5037763	KB5017396	KB5001078
KB5037016	KB5017305	KB5000803
KB5036899	KB5017095	KB4601393
KB5035962	KB5016622	KB4601392
KB5035855	KB5016058	KB4601345
KB5034862	KB5015808	KB4601318
KB5034767	KB5014702	KB4601060
KB5034119	KB5014026	KB4598243
KB5033373	KB5013952	KB4593226
KB5032391	KB5012596	KB4589210
KB5032197	KB5012170	KB4589208
KB5031362	KB5011570	KB4586830
KB5030504	KB5011495	KB4580346
KB5030213	KB5010359	KB4580325
KB5029242	KB5009546	KB4577586
KB5028169	KB5008207	KB4577015
KB5027219	KB5007192	KB4576750
KB5026363	KB5006669	KB4571694
KB5025228	KB5005698	KB4565912
KB5023788	KB5005573	KB4565511
KB5023697	KB5005112	KB4562561
KB5022838	KB5005043	KB4561616
KB5022286	KB5004238	KB4561600
KB5021235	KB5003638	KB4557397
KB5020866	KB5003197	KB4556813

KB4550994

KB4550949

KB4550929

KB4540723

KB4540670

KB4539571

KB4537764

KB4537759

KB4535680

KB4534271

KB4530689

KB4525236

KB4521858

KB4520724

KB4519998

KB4516115

KB4516044

KB4512574

KB4512517

KB4509091

KB4503308

KB4494175

KB4494174

KB4346087

KB4132216

KB2565063

KB2467173