



Date: July 06, 2026

Author: Software Engineering

System: SELENIA DIMENSIONS / 3DIMENSIONS

Subsystem: Windows 10 LTSC 2019 (v1809) & Software Security Patches

Subject: Windows 10 LTSC 2019 (v1809) & Software Security Patch release June 2026

Purpose:

To release the list of validated Microsoft critical patches on the SELENIA DIMENSIONS / 3DIMENSIONS System.

Latest .NET Versions Validated for the Product:

Note: Security updates should only be installed for existing .NET installations already present on the product. For example, .NET 8.0.24 update can be installed only if an earlier version of .NET 8.0 is already part of the product installation.

- **.NET Framework 3.5 / 4.x**

The latest security updates are included in the Windows Cumulative Update.

- **.NET Core 2.1 / 2.2**

Hosting Bundle 2.1.30

Latest SQLSERVER Versions Validated:

Note: Security updates should only be installed for existing SQLSERVER installations already present on the product.

- **SQL Server 2017:**

The latest security updates are included in the Windows Cumulative Update.

List of patches/KB's validated for June 2026:

KB5094123

**List of all patches/KB's validated on the SELENIA DIMENSIONS /
3DIMENSIONS System:**

KB5094123	KB890830	KB5032196
KB5087061	KB5053596	KB5031990
KB5087538	KB5052000	KB5031361
KB5082123	KB5050008	KB5030214
KB5082413	KB5049608	KB5029925
KB5068402	KB5048661	KB5029376
KB5078752	KB5046615	KB5029247
KB5075904	KB5044277	KB5028168
KB5066143	KB5044016	KB5027222
KB5073723	KB5043050	KB5027124
KB5071544	KB5042215	KB5026362
KB5065225	KB5041578	KB5025229
KB5068791	KB5040430	KB5023702
KB5066137	KB5039879	KB5022840
KB5066586	KB5039217	KB5022504
KB5065428	KB5037765	KB5022286
KB5063877	KB5036896	KB5021237
KB5062557	KB5036604	KB5021126
KB5060531	KB5035849	KB5020874
KB5058392	KB5034768	KB5020615
KB5055519	KB5034127	KB5019966
KB5050533	KB5033904	KB5018419
KB5049615	KB5033371	KB5017315

KB5016884	KB5003646	KB4566424
KB5016623	KB5003243	KB4565349
KB5015811	KB5003171	KB4562562
KB5014692	KB5001407	KB4561608
KB5014553	KB5001404	KB4561600
KB5014164	KB5001342	KB4558998
KB5013941	KB5000859	KB4558997
KB5013641	KB5000822	KB4557397
KB5013626	KB5000685	KB4552924
KB5012647	KB4601393	KB4551853
KB5012170	KB4601345	KB4549949
KB5012119	KB4601060	KB4549947
KB5011503	KB4598480	KB4539571
KB5010351	KB4598230	KB4537759
KB5009557	KB4592440	KB4535680
KB5008873	KB4589208	KB4514366
KB5008218	KB4587735	KB4505225
KB5007206	KB4586793	KB4494174
KB5006672	KB4580325	KB4022619
KB5005568	KB4577668	KB4018073
KB5005112	KB4577667	KB2565063
KB5005030	KB4577586	KB2467173
KB5004244	KB4570333	
KB5003711	KB4570332	