

Aptima Auto Detect Kit (Panther™ System)

For *in vitro* diagnostic use.

Intended Use

The Aptima Auto Detect Kit is for use on the Panther System.

Materials Provided

Note: For information on any hazard and precautionary statements that may be associated with reagents, refer to the Safety Data Sheet Library at www.hologic.com/sds.

Aptima Auto Detect Kit (Cat. No. 303013)

(Store at 15°C to 30°C upon receipt)

Symbol	Component	Quantity
R1	Aptima Auto Detect 1 <i>Aqueous solution containing 0.1% hydrogen peroxide and 0.001 N nitric acid.</i>	1 x 245 mL
R2	Aptima Auto Detect 2 <i>1.6 N sodium hydroxide.</i>	1 x 245 mL

Warnings and Precautions


- Avoid contact of Auto Detect 1 and 2 with skin, eyes, and mucous membranes. If these fluids come into contact with skin or eyes, wash with water. If these fluids spill, dilute the spill with water before wiping dry.
- Use routine laboratory precautions.

Storage and Handling Requirements

- Store the Aptima Auto Detect Kit at 15°C to 30°C.
- Fluids must be at 15°C to 30°C before use. Once opened, fluids are stable for 60 days.
- If precipitate has formed, heat the fluids at 37°C to 60°C to facilitate dissolution of the precipitate. Cool the fluids to room temperature and thoroughly mix prior to use.
- Do not use after the expiration date.

Procedure

Replace bottles in the Universal Fluids Drawer when notified by the system. Refer to the *Panther System Operator's Manual*.

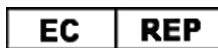

Hologic, Inc.
10210 Genetic Center Drive
San Diego, CA 92121 USA

EMERGO EUROPE
Molenstraat 15
2513 BH, The Hague
The Netherlands

Customer Support: +1 844 Hologic (+1 844 465 6442)
customersupport@hologic.com

Technical Support: +1 888 484 4747
molecularsupport@hologic.com

For more contact information visit www.hologic.com.

Hologic, Aptima, and Panther are trademarks and/or registered trademarks of Hologic, Inc. and/or its subsidiaries in the United States and/or other countries.

All other trademarks that may appear in this package insert are the property of their respective owners.

©2001–2014 Hologic, Inc. All rights reserved.

502180EN Rev. 001
2014-11