


# Protocol: endocervical brush/spatula


## Quick reference guide

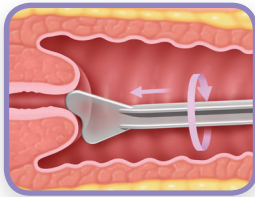

**Obtain** an adequate sample from the ectocervix using a plastic spatula. If desired, use lukewarm water to warm and lubricate the speculum. Apply water-soluble, carbomer-free gel lubricant sparingly to the posterior blade of the speculum if necessary.<sup>1,2</sup> Select the contoured end of the plastic spatula and rotate it 360 degrees around the entire ectocervix, while maintaining tight contact with ectocervical surface.



**Rinse** the spatula as quickly as possible into the PreservCyt<sup>®</sup> Solution vial by swirling the spatula **vigorously** in the vial 10 times. Discard the spatula.

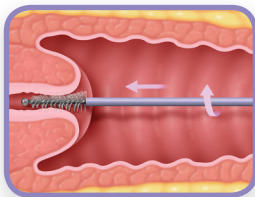

**Obtain** an adequate sampling from the endocervix using an endocervical brush device. Insert the brush into the cervix until only the bottom-most fibers are exposed. Slowly rotate 1/4 or 1/2 turn in one direction. **DO NOT OVER-ROTATE THE BRUSH.**



**Rinse** the brush as soon as possible in the PreservCyt Solution by rotating the device in the solution 10 times while pushing it against the PreservCyt vial wall. Swirl the brush **vigorously** to further release material. Discard the brush.



**Tighten** the cap so that the torque line on the cap passes the torque line on the vial.




**Record** the patient's name and ID number on the vial.  
**Record** the patient information and medical history on the cytology requisition form.



**Place** the vial and requisition in a specimen bag for transport to the laboratory.

# Protocol: broom-like device

## Quick reference guide

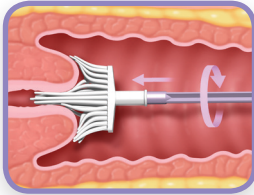

**Obtain** an adequate sampling from the cervix using a broom-like device. If desired, use lukewarm water to warm and lubricate the speculum. Apply water-soluble, carbomer-free gel lubricant sparingly to the posterior blade of the speculum if necessary.<sup>1,2</sup> Insert the central bristles of the broom into the endocervical canal deep enough to allow the shorter bristles to fully contact the ectocervix. Push gently and rotate the broom in a clockwise direction for five complete, 360 degree turns.



**Rinse** the broom as quickly as possible into the PreservCyt<sup>®</sup> Solution vial by pushing the broom into the bottom of the vial 10 times, forcing the bristles apart. Swirl the broom **vigorously** to further release material. Do not leave the head of the broom in the vial. Discard the collection device.



**Tighten** the cap so that the torque line on the cap passes the torque line on the vial.



**Record** the patient's name and ID number on the vial.

**Record** the patient information and medical history on the cytology requisition form.



**Place** the vial and requisition in a specimen bag for transport to the laboratory.

**Refer to the instructions provided with the collection device for warnings, contraindications, and limitations associated with specimen collection.**

[www.thinprep.com](http://www.thinprep.com)

Hologic provides this quick reference guide ("guide") as a general informational tool only and is not an affirmative instruction or guarantee. While the information provided in this guide may describe a particular technique or protocol, it is not intended as an endorsement and/or requirement by Hologic to utilize such technique or protocol. It is the sole responsibility of the laboratory to determine which specific technique and/or protocol to employ. Laboratories using ThinPrep or PreservCyt products must read and understand each product's Instructions for Use and comply with applicable local, state and federal rules and regulations.

1. Cervicovaginal Cytology Based on the Papanicolaou Technique; Approved Guideline – Third Edition (Clinical and Laboratory Standards Institute GP15-A3). 2. Hologic internal study, data on file.

**hologic.com | diagnostic.solutions@hologic.com | +1.781.999.7300**

DS-05867-001 Rev. 002 © 2017 Hologic, Inc. All rights reserved. Hologic, Science of Sure, PreservCyt, ThinPrep and associated logos are trademarks and/or registered trademarks of Hologic, Inc. and/or its subsidiaries in the United States and/or other countries. All other trademarks are the property of their respective owners. This information is intended for medical professionals in the U.S. and other markets and is not intended as a product solicitation or promotion where such activities are prohibited. Because Hologic materials are distributed through websites, eBroadcasts and tradeshows, it is not always possible to control where such materials appear. For specific information on what products are available for sale in a particular country, please contact your Hologic representative or write to [diagnostic.solutions@hologic.com](mailto:diagnostic.solutions@hologic.com).

*ThinPrep*<sup>®</sup>  
PAP TEST